

Cura Air Chair User Manual.





Operation of the chair


Electric Version

Manual Version



Legrest Adjustment

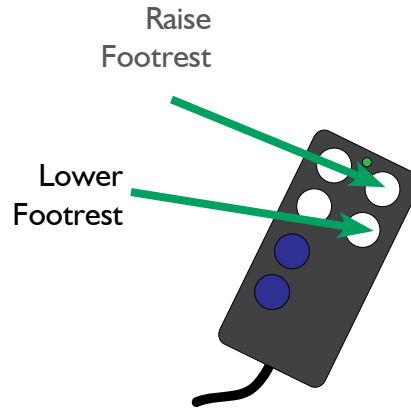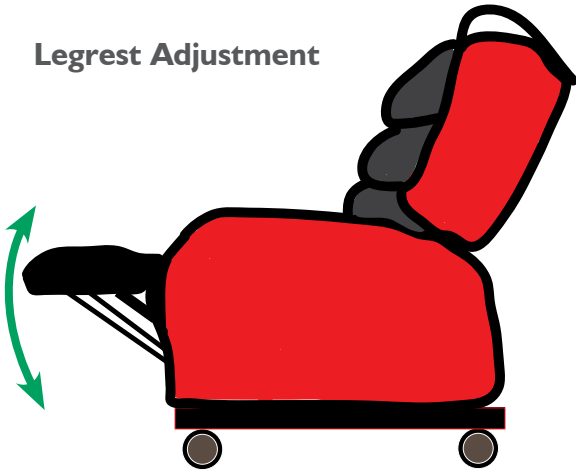

Pull lever & pull footrest at the same time to raise footrest. Pull lever and push footrest to lower footrest.

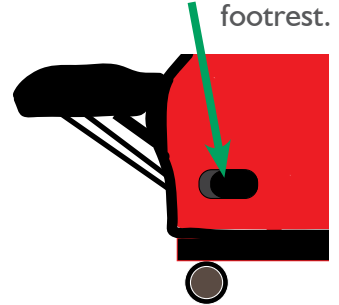


Tilt-In-Space Adjustment

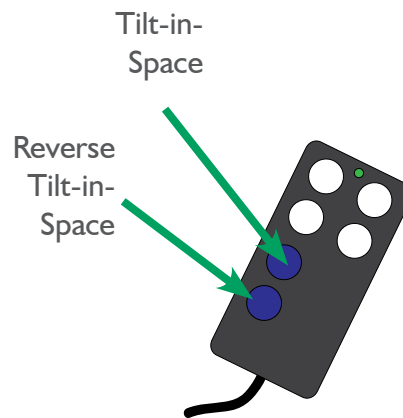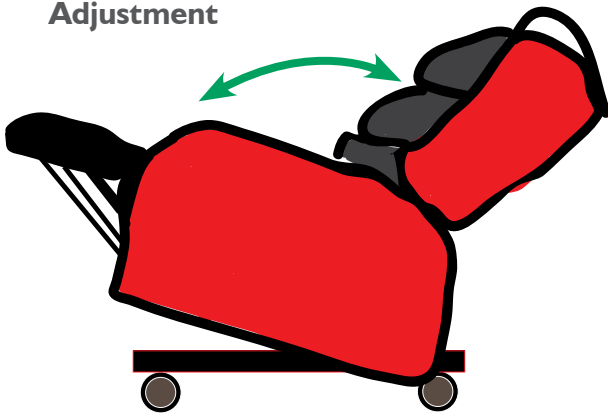


Tilt-in-Space Button



Backrest Adjustment

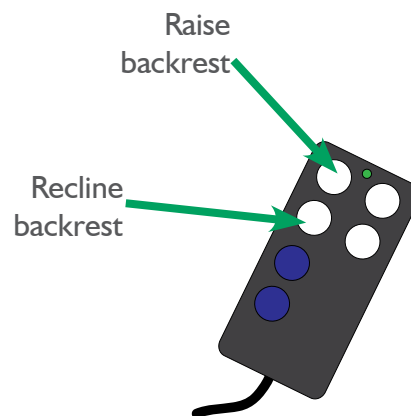

Backrest Recline Button

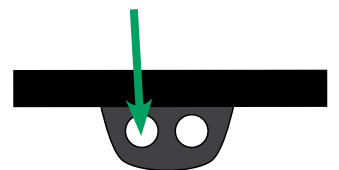

If the chair is an electric version and has a safety key lock handset, the chair is inoperable if the key is removed from the key slot as a safety precaution to prevent inappropriate use of chair.

To charge the chair's battery, insert the charger lead into the socket located at the back of the chair, and plug the charger lead into a normal wall socket.

Setting up the chair's pressure relief

The Cura Air chair incorporates a static air pressure relief system. This consists of three air chambers across the seat within the overlay. One cell is behind the knees, one cell is across the front half of the seat, and one cell is across the back half of the seat.


The Cura Air chair comes complete with a hand pump which can easily be attached to the air cell valves, which can be seen between the overlay and the arms of the chair. The cells can then be inflated until it feels comfortable for the client.

Steps:

1. Looking at the chair put your hand between the armrest and seat to locate the pipes and move them up so they're in view.
2. With the 3 pipes now in view twist and pull the caps off.
3. Push the seat with your hands or sit on the chair to force all of the air out of the cells.
4. Attach the pump and do 14 full pumps.
5. When releasing the pump off the pipe it is important to place the cap on immediately so no air is lost.
6. Complete the process for the other 2 aircells.

If the user finds the chair too firm then release some of the air until it feels comfortable for them. Over inflation of the aircells will lead to the chair feeling very firm and uncomfortable.

Once the air cells have been setup correctly there should be no need to alter the amount of air in them unless the user's needs change.

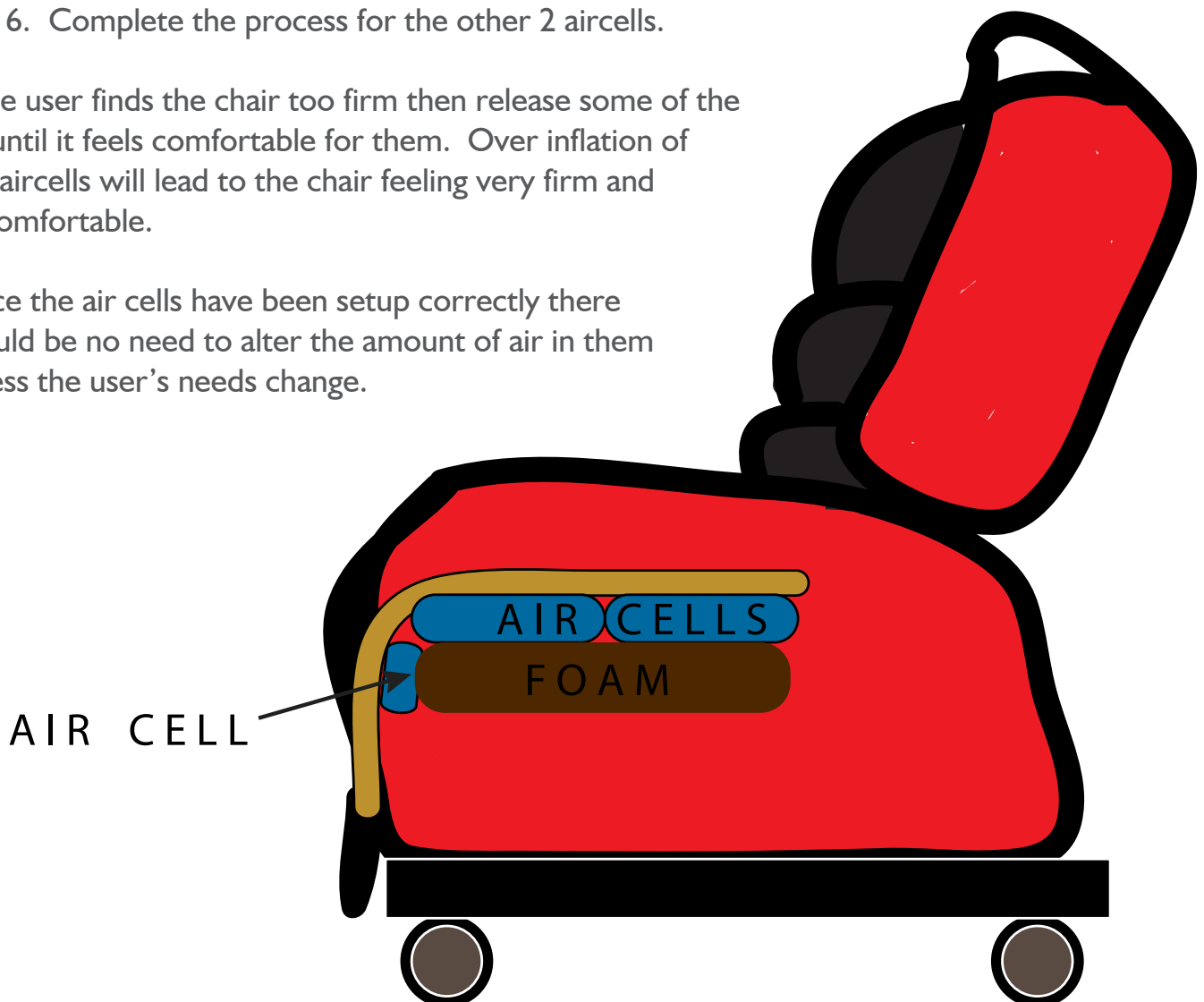

BATHING ○
BEDS ○
FURNITURE ○
HOUSEHOLD ○
MOBILITY ○
MOVING & HANDLING ○
SEATING ○
STAIRLIFTS ○
TOILETING ○

Taking Special Care of You.

Linkside House, Forest Lane Head, Harrogate HG2 7TE

Tel: 01423 799960

Fax: 01423 202116

6 Over Lane, Rawdon, Leeds LS19 6DY

Tel: 0113 250 3250

Fax: 0113 250 7433

enquiries@yorkshirecare.com