

## KNOW YOUR THERMOMETER

## INSTALL THE BATTERIES

1

### BOX CONTENTS

1 USER GUIDE



1 USER MANUAL



1 DIGITAL EAR THERMOMETER  
WITH TOP COVER




*Batterie not included*

[www.NantLife.com](http://www.NantLife.com) - 1-855-626-8227 - [support@nantlife.com](mailto:support@nantlife.com)  
Copyright© Revision 09 2012

2



3

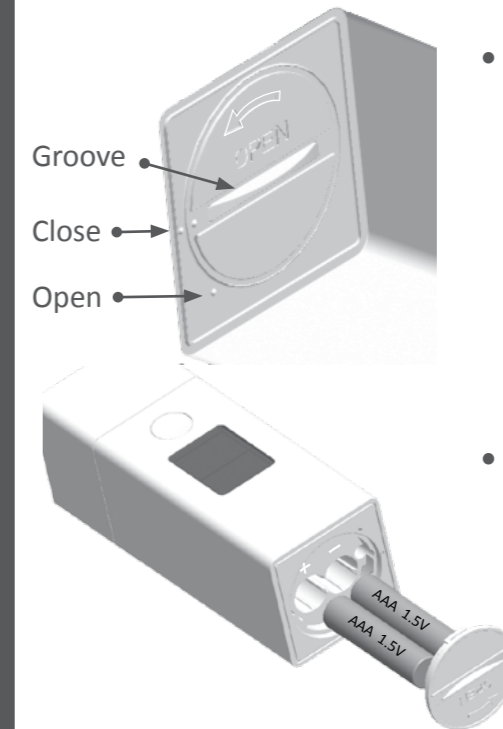

- To open, place a coin in the groove of the battery cover and following the arrow, turn the cover towards dot Open

- Install 2 "AAA"1.5V batteries in the compartment

- Follow the polarity signs in the compartment when placing the 2 "AAA" batteries

- To close the compartment, line up the groove dot with dot 2 and turn towards dot Close until secure

## TURN THE THERMOMETER ON

## TAKE YOUR MEASUREMENT

## READ YOUR MEASUREMENT

4

- Turn the Ear Thermometer on by pushing the button once
- The screen turns on and the NantLife logo displays
- Then the screen shows as in the illustration below
- By default, the thermometer will display the temperature in Fahrenheit, you can refer to the user manual to change the unit to Celsius



5

- Insert gently the thermometer ear probe into the ear canal
- Press and release the button
- Do not remove the probe from the ear until it beeps TWICE
- After the 2 beeps, remove gently the probe from the ear canal



**Warning:**


*If you experience any discomfort:*

- Remove the ear probe from the ear canal immediately.
- Stop using the device and contact our customer support at [support@NantLife.com](mailto:support@NantLife.com)

6

- After a successful measurement, the display shows like the example below:

Battery Level Indicator  
(will only show if batteries are empty)

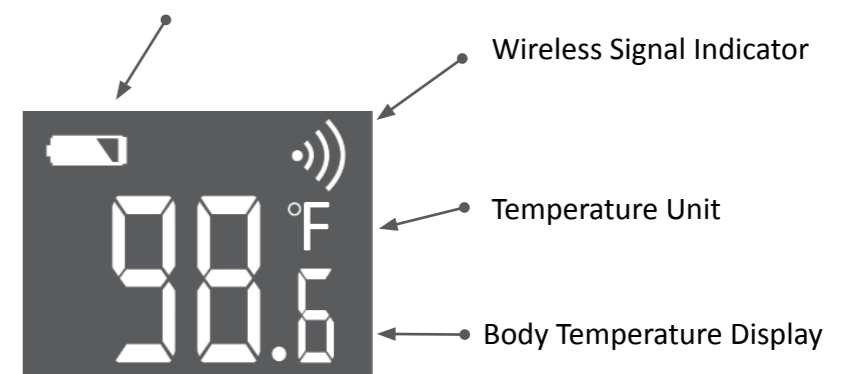

- The thermometer shuts down automatically

Please refer to your user manual section 1.4 for more general information about body temperature.