

BB-306

Oxygen Jet Plus



Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

B_B BEAUTY BEAUTY USA GROUP }

Before using the machine, be sure to read through this instruction manual.

A Declaration :

The user manual introduces the operations. All the material and charts are the latest, but the manufacturer reserve the authority to change its content with another other notices.

Cautions: User could not disassemble any accessories or use together with products not produced by us. They will induce out of warranty. We will not be responsible for damages on it.

- a) Please read the user manual carefully before use.
- b) The user manual is one part of the machine.
- c) Be sure to keep the user manual carefully for future check.

B Product Generalization :

Oxygen Jet Plus in the normal temperature, through transforming pressures and vacuum theory, it makes the gas ionization in air to produce 98% pure oxygen without any dirt and bacteria. And matches with the special parts of the infusing jet, mist sprayer, suction mask, together with vacuum pen for lymph drainage of face and body, to be carried out the oxygen infusion, spray and suction therapies completely. Four ways integrate perfectly, effectively enhance cells rejuvenation, accelerate metabolism and deeply care every skins of body.

Oxygen therapy has been largely known by people for the recently ten years. Not only in medicine, but also facial, it contributes much, popular favors by every estate. In medicine, it is over limit of minor wounded technique, without pain, without needle,


Oxygen Jet Plus supplies facial therapy without pain, without needle, it uses oxygen, infuse the special certain rejuvenate elements like serum into skin bottom, attain the natural spaces of cuticle cells, and produce right impaction.

BB BEAUTY BEAUTY USA GROUP }

Technology imported from Japan, stronger pressure by 2BAR, better precision, 98% of pure oxygen for your every inch of skins.

C Oxygen Importance for Human Body :

Oxygen is closed link with the human body. Except the less of oxygen in air, office work for a long time, bad habit of biting and supping, which induce lack of nutrition, skins dull, color speckles and etc. As the researches, every person will absorb 1.44 gallons oxygen through skin surface layer everyday, which can keep 60 trillions cells grow-up in good condition and produce new cells. In dinosaur era of remote antiquity, there is 38% to 50% oxygen volume in air, 30% at the beginning of eighteen century, and then 23% at the beginning of twenty century. By now, it is not over 20%.

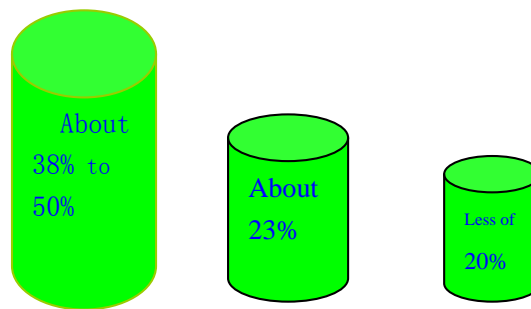

Remote antiquity At the beginning of 20 century At present

Serious lack of oxygen in air and daily busy in cities, facing to computer for a long time, less oxygen movement, all these induce our body oxygen in short supply. Healthy skins need enough oxygen. If skins lack of it, it will slow down metabolism, speed up skin aging, bring many problems like face yellow, black speckle, wrinkles, even destroy the balance of grease secretion, skin color turn dull and produce acne like an overtime yellow apple. The people lack of oxygen for a long time will produce the normal diseases in breath system, high blood pressure and others.

The Oxygen Jet Plus plus aims at skin with lack of oxygen, through no pain beauty therapy, uses oxygen to take the certain active elements including Hyaluronic acid and oxygen serum and etc into deep skin. All of them can be plus into skin bottom into

BB BEAUTY BEAUTY USA GROUP

the natural spaces of cuticle cells and produce positive impaction simultaneously.

The Oxygen Jet Plus plus, imported advanced technique of the world, can input 98% pure oxygen and active essence into deep skins, effectively speed up cells metabolism, recover healthy, tender and fresh. We can observe directly from the chart, after six weeks treating courses, skin surface is obviously improved and renewal, smoother, finer and firmer. Skin is like a fresh apple with moisture and shining.


Human body is a pollution source by itself, cell if in less of oxygen,

It will be decreased or stop its metabolism and induce face yellow,

ull, black speckle, pores pile-up, bad memory, breathless, eyestrain, cataract, sensitive coryza, bronchitis, high blood pressure, brain congest, clog of blood vessels in heart, hemorrhoids, astriction, psoriasis, emphysema, cirrhosis of blood vessels, cold with motion, rheumatism and even cancer.

D Illness chart of less of oxygen in cells:


We conclude all the illness owing to less of oxygen into the below chart

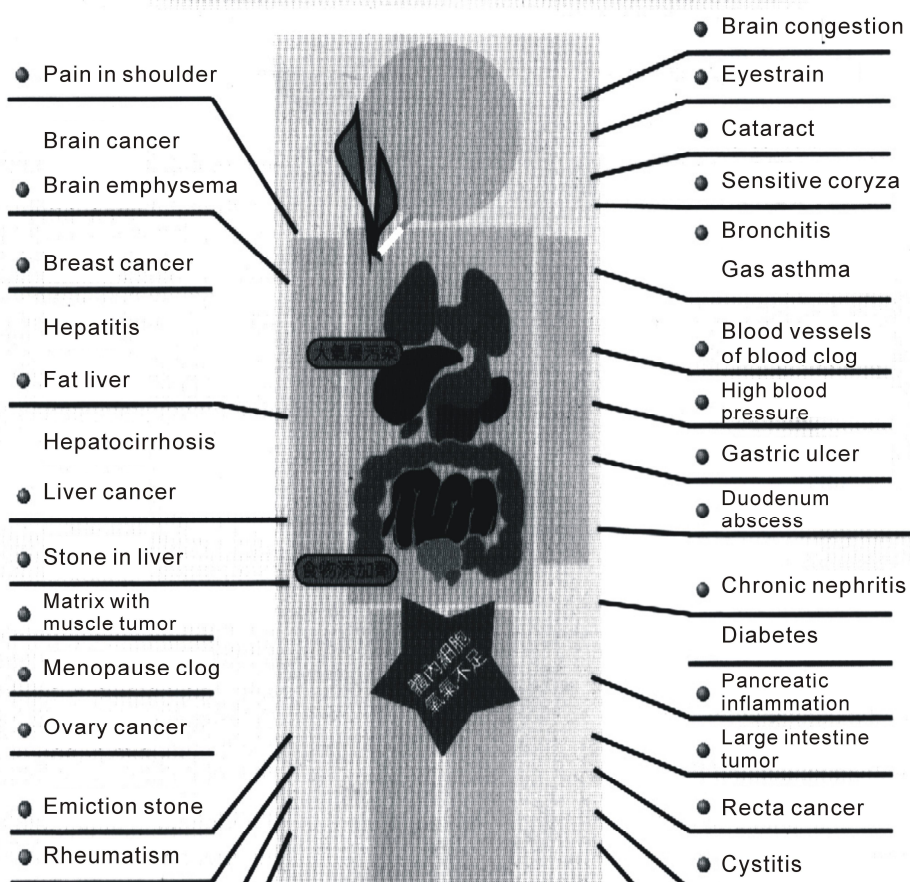

E The Operating Panel :



Parts

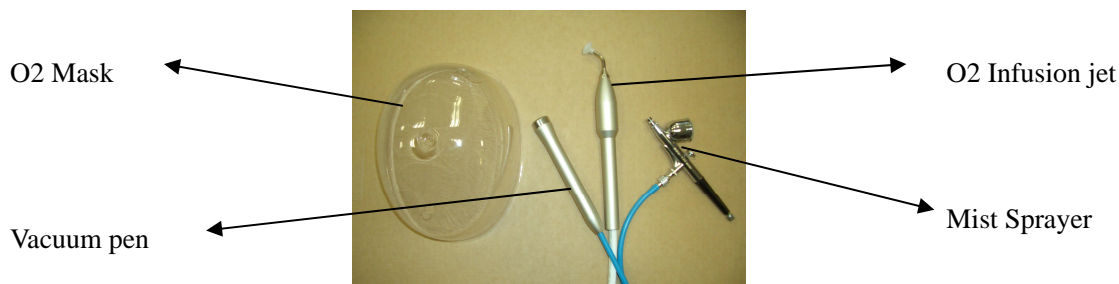

F Effect and usage :

A Sprayer -----Press the control button to produce the oxygen. Pull back to produce the extract moist, or produce mix mist of extract and oxygen after adjust, it has a container device.

B Vacuum Pen -----Lymph drainage use for the face and neck.

C O2 Infusion jet -----Use when infusing oxygen to skin

D Oxygen Mask -----For oxygen scent and Aroma therapy.

G Work Theory

Oxygen Jet Plus, through high pressure separate way to make gas ionization in air, to produce 98% pure oxygen, matched with oxygen infusion jet, oxygen sprayer and oxygen scent, also with vacuum pen to do lymph drainage, total four ways integrated perfectly. Even effectively enhance cells rejuvenation, accelerate metabolism, deeply care every parts of skins.

H Main functions :

1. Oxygen Spray : Cure all kinds of itch, sensitive skin, neurodermatitis, touch dermatitis and etc through massage with pure oxygen and extracts with rejuvenation

2. Lymph Drainage : through suction, make lymphatic pipes and blood circulation smooth and remove rubbish and toxins

3. Oxygen infusion : Through oxygen infusion jet without needle pressure to infuse the pure oxygen and special facial extracts into cuticle bottom to

B_B BEAUTY BEAUTY USA GROUP }

strengthen cells, enhance resistance, make the dirt easily out, promote metabolism, improve skins dull, wrinkles, slack, melanin and etc.

4 Oxygen scent : Face with the mask, to match oxygen with one hundred percent pure aroma exacts, through the breath system to be functioned as following: :

Breath system: coryza, faucitis, fever.

Neural system: neurasthenic, insomnia, migraine.

Blood: leucopenia.

Digestive system: astriction, intestines and stomach with function dysfunction.

Metabolism and incretion: diabetes, syndrome in menopause, menses maladjustment, promote wound heal up, enhance immunity and adjust incretion.

Skin problems: Revitalize cells, delay cells aging, improve blood circulation and metabolism, tight skins, remove wrinkles, improve speckles, whiten skins, balance grease and improve sensitive skins.

It is more suitable for the person with lung weak to smell oxygen 15minutes and have a rest.

5. Plump chests and loose weight with oxygen: Through the vacuum of infusion A, to plump chests, shape and do lymph drainage all of the body

I Suitable people :

The oxygen infusion is especially suitable for the smoke females in city, or who with drooping chests, the bad bodyline, skin dull, sensitivity, wrinkles, speckles and acne. It can increase cells rejuvenation, improve and promote blood circulation, accelerate metabolism, improve sensitive skins, adjust bodyline in good status.

After oxygen infusion, you can feel very relax, health. It also has whitening effects. It will be better if matching with special extracts.

J Attest :

B_B BEAUTY BEAUTY USA GROUP }

After test the oxygen with high pressure and rejuvenation in the science lab, we can conclude the results as following:

1. It infuses the high-pressure oxygen into cuticle through electronic technique, without any harm to skins.
2. The rejuvenated oxygen can be directly penetrated into bottom skins.
3. The rejuvenated oxygen forms a big pile in the bottom skins.
4. The optimal is to supply rejuvenated oxygen and nutrition to skins.
5. All the treatments are controlled by the electro, without by hand, it is very safety.

K Basic steps of treatment :

Face

1. Cleaning : After cleaning the face, to wait for the skins dry and start spraying the oxygen. The procedure should be matched with the diamond peel better.

2. Lymph Drainage : Use the **Vacuum Pen** to do lymph drainage with following the lymph way. (About 15 minutes)

3. Oxygen spray : Use **Mist Sprayer** to spray oxygen on the face to make skins accommodate with the pure oxygen status, to strengthen cells of face rejuvenate and metabolism. (About 10 minutes)

4. Oxygen infusion : Spread the extracts completely, adopt the **O2 Infusion Jet** with pressure and follow the face skins textures to massage, lift and remove wrinkles. (About 30 minutes)

5. Oxygen scent : Wear the **Oxygen Mask** to do Oxygen scent treatment, 15 minutes better.

6. Mask : Suggestion is to use some **Mask** that can make up water and

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

BB BEAUTY BEAUTY USA GROUP }

nutrition.

7. Suggestion treatment : 12 times one course, the first 8 times should be two times one week, the latter 4 times should be one time one week.

L Importance of lymph care :

It is a large industrial society, everything like wearing, eating, living, moving all can not avoid **Outer Toxins** spreading. And the **Inner Toxins** also produce gradually from metabolism at the human body. Both the toxins are accumulated steadily, and form harms to our health. They directly reflect changes in our skin, body, and impact our health and beauty.

Oxygen Jet Plus, according to the lymph place at neck, gesture at face, adopts the infusion jet and vacuum pen, to stimulate venation and points, push blood circulation, let lymph vessels and blood circulation free, and effectively remove rubbish, unwanted water, toxins from body through lymph system.

Three Important Toxins in Body

	Place of toxins	Sources of toxins	Symptoms
1	Alimentary canal, skins	Industry pollution, wearing, eating, living, moving	Halitosis, windy, astringent, skin slack, acne, speckles
2	Cells	Residual poison overspread	Fat, cells canceration, incretion maladjustment
3	Centrum of Nerve	Pressure from living and work	Swirl, headache, insomnia, bad memory, immunity dropping

M Lymph drainage steps :

•Lymph drainage by vacuum (Use Infusion B on face)

1. Let head one side, start from down lymph node of Mandible, to do lymph rolling from up and down.
2. From Lymph node of cheek, cheek, neck, to lymph of clavicle.
3. Form wrinkles on both sides of nose-wing pull down and outer to down lymph node of Mandible, along Deep lymph node of neck to lymph of clavicle.
4. From the root of nose, along bottom of cheekbone to Lymph node of fistula at ear front, down to neck, lymph of clavicle.


B_B BEAUTY BEAUTY USA GROUP }

5. From the bottom of eye socket bone, pull to Lymph node of fistula at ear front, down to ear back, along neck to lymph of clavicle.
6. From the upside of eye socket bone, pull to Lymph node of fistula at ear front, down to ear back, along neck to lymph of clavicle.
7. From the forehead top, pull out and down to Lymph node of fistula at ear front, then to ear back, along down neck to lymph of clavicle. .

Remarks: If face is with acne and serious red blood silks, please evade the area. The total therapy time is 15 minutes

N Operation chart of lymph drainage :



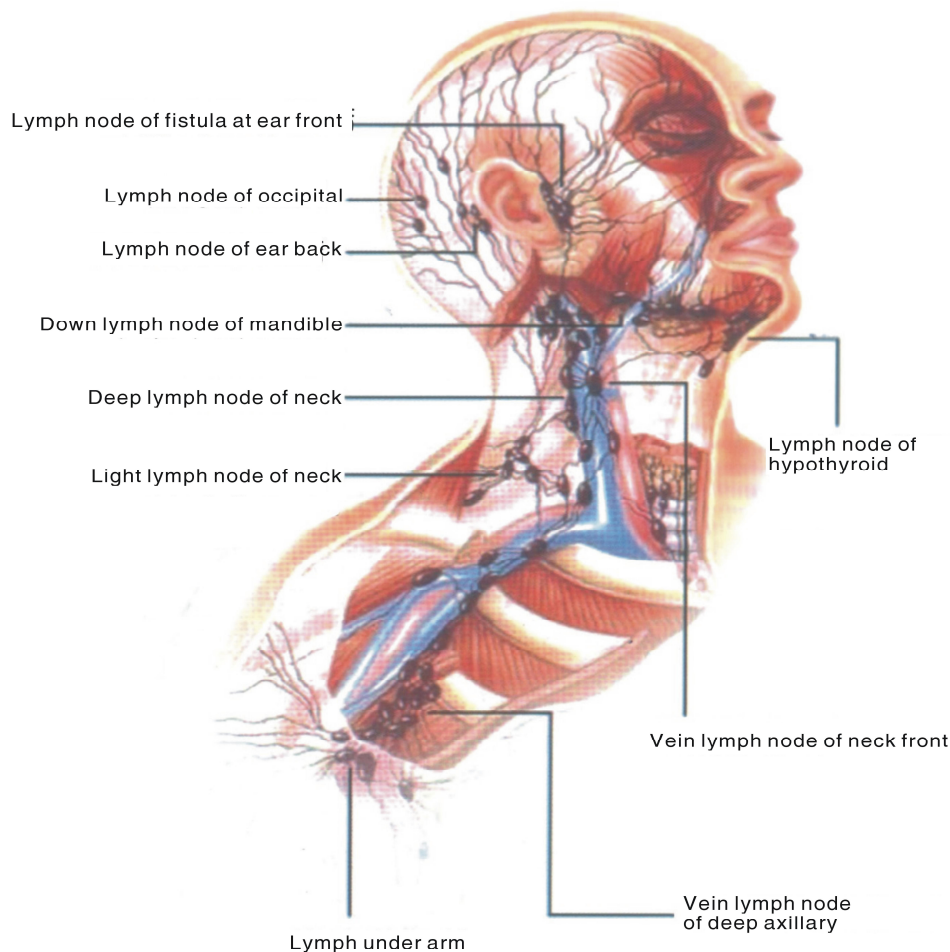

O Importance of oxygen therapy on face :

Oxygen therapy is the healthiest facial. There are no blood vessels to transport oxygen at cuticle, all that should be relied on the micro circulation system at fats of dermis. The micro circulation turns less and less by ages increasing. Scientist found, when people go into middle age, the oxygen volume just is half of their youth. If it can directly transport oxygen into skin bottom, it can effectively enhance cells rejuvenation, promote metabolism. In virtue of oxygen device, oxygen therapy can infuse the pure oxygen into skin bottom, if matched with other facial treatments, it will make skin younger.

P Steps of oxygen infusion, spray and suction therapy :

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

B_B BEAUTY BEAUTY USA GROUP }

1 Oxygen Spray (Use Sprayer)

Add proper aqueous skin care products that can be chosen by the actual needs, by pull back and down to spray mist, by press down to spray oxygen. Adjust the end knob of the sprayer to adjust the mist volume, Left is for big, right is for small.

Remarks: Please adjust the air pressure at 5LPM during facial oxygen therapy.

2 Oxygen Infusion (Use Infusion)

Aiming at different skins, to choose the proper extracts, it can moisten and whiten the dry skin, shrink pores, control oil of oily skins and make skin refreshing,

P.S.: During oxygen infusion, it should adjust the air pressure to 2LPM.

With 30 minutes to infuse at all the face, please evade eyelid because it is very thin during operation from forehead up and down. The infusion B has three muzzles for choice, usually, the Max is for face, Middle is for eye around, the Small is for two sides of nose-wing.

3 Oxygen Scent (Use Oxygen Mask)

Open Oxygen Mask function and place the mask on the nose to do oxygen scent. After it, please apply mask for skin care.

If less of oxygen, how is the people going?

Less oxygen at brain—Swirl, headache, tingle, nausea, nervous, lose heart, consciousness and to be death.

If less oxygen in skin—It will make skin bad ability of renew, rough pores, wrinkle, speckle, black orbit, skin color dull and yellow. Because of it, many young girls' skin loses blare, flexibility.


Remarks: during the therapy, please adjust the air pressure to 3LPM.

Q Areas and operations of oxygen therapy on face :

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

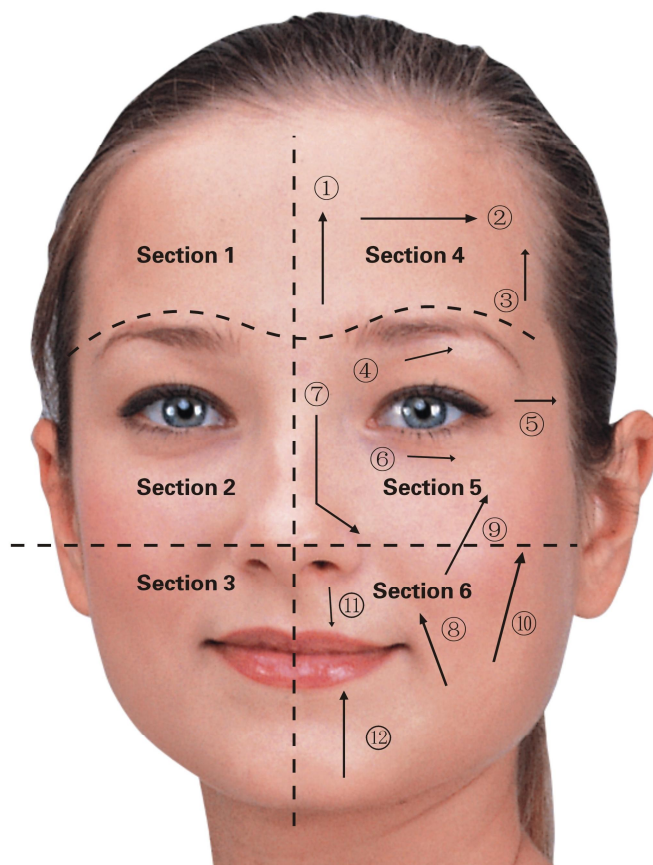

Suggest operate by the arrows way and gradation to do skin care.

R Assistant care :

When using Oxygen Jet Plus to do the therapy , it can be matched with soft light with rapid heat, color light, laser, irish diamond peeling, treatment before and after operation, skin change and etc, it will be better effects.

S Main Features:


1. Mist oxygen spray, oxygen infusion and oxygen scent, 3 functions together.
2. It can supply continuous high-pressure oxygen, 98% pure oxygen.
3. It makes skins easily inhale nutrition.
4. If matching with our special extracts liquid, it will receive remarkable effects.
- 5 Sprayer is made of stainless steel, it with extracts liquid container at the end.

T Analysis chart for steady courses within six weeks :

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

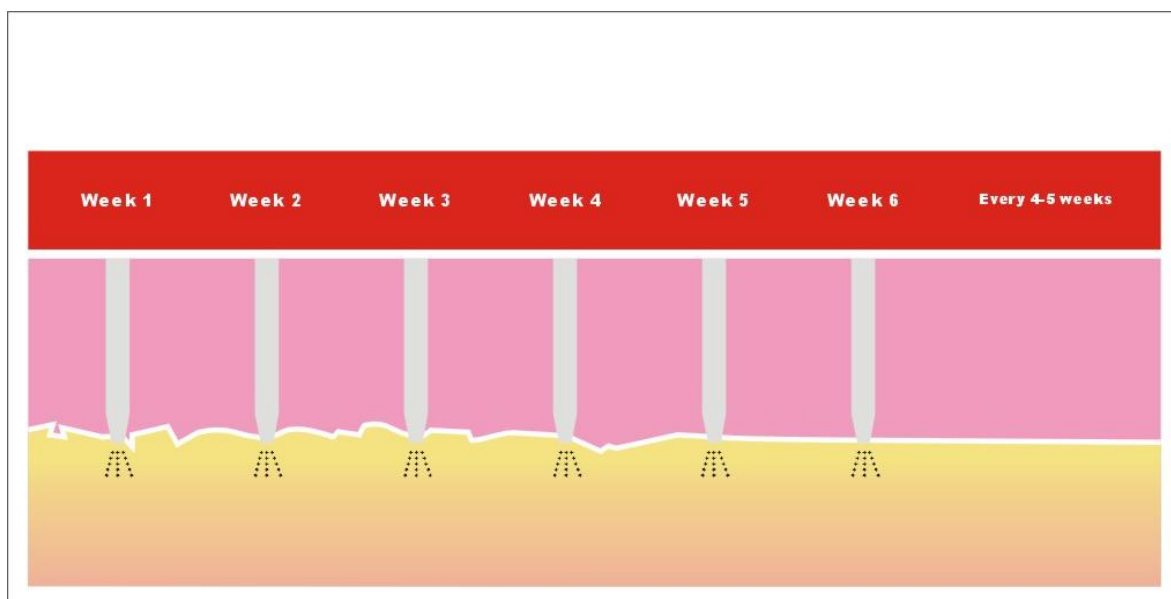

From the chart, we can directly observe, skin after six courses, its surface receives obviously renew and improvement, turns more tender and decent.

U Notices of operation :

1. Terms and circumstance

- (1) Please be not put in moisture or high temperature, uneven places to avoid creepage and operation troubles.
- (2) Voltage: 110V or 220V

2. Cautions before operation

- (1) Plug in
- (2) Be sure all the interfaces are placed right and connected.
- (3) Please operate the button right, be sure switches if in normal work status.
- (4) Choose proper function and way, be sure each fitting if in normal work status.
- (5) No use for some time, please be sure if the machine can start safety besides all the above cautions

V Tips after use :

The Oxygen Jet Plus is made up of precision electronic component. It

B_B BEAUTY BEAUTY USA GROUP }

cannot be at moisture, please notice to choose the keep place, especially pay attention to keep the output power lines

1. Shut off the power of main machine, and plug out of power plug.
2. Avoid the main machine at sun or in moisture places.
3. After finish, please clean all fittings, and return the original places.
4. After finish, please separate to place the main machine and output lines, be sure the output line in drafty place.
5. Be sure not to place the output lines on the main machine.
6. Be sure to avoid using or keeping the machine in high temperature and bathroom.
7. Be sure not moisten or splash water on the main machine to avoid short circuit and electric shock.

W Post Service and Warranty :

Post service

We will guarantee the quality of our equipment, normally there is no quality problem.

1. If there is a deficiency, please email or call us on the reasons, it would be best if this can be dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician, and our technician will give a satisfactory answer within 1-7 working days.
3. If the problem comes from the spare part, we could consider in changing the spare part for our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
4. If the problem comes from the main device, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new one to our client. If the cost of posting the problematic device back to JIMY is more than the value of a new device, JIMY would take it into consideration, according to the actual status; apply to re-send a completely new device back to our client.
5. If the inferior equipment very big or expensive, not convenient to send back by post office, we will give you an appointed Hong Kong place, you can send Hong Kong first. Jimy will ask a cooperated forwarder to take it back. Jimy will pay all charges from Hong Kong to our office. Once we check the equipment is the quality inferior, we also agree, in your next order, to deduct the freight

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com

B_B BEAUTY BEAUTY USA GROUP

from the customer's office to Hong Kong if you fax or email us relevant shipping receipts and invoice.

6. Please make sure, if you send the inferior one to us by air etc. without any advanced notice, Jimy will not pay relevant air freight and custom fee.

All in one, we will try our best to serve you and also need your fully cooperation. Thanks.

Remarks:

1. The term of maintaining guarantee is a year from the date of purchasing according to the receipt. Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchase the parts and easily expendable products.
2. We won't supply maintaining guarantee if troubles as below:
 - A. Disassembling the machine by self;
 - B. Fall it without caution;
 - C. Lack of reasonable keeping and protect;
 - D. Not operate according to the user manual.
3. If changes of the design and specification of the machine, we will not notice you in addition. If any queries, please feel free to contact with the local marketing company.

Beauty Beauty USA Group

442 Broadway 2nd Floor, New York, N.Y. 10013 Toll: 1(877)343-1983 Fax: 1(212)343-1984

Web: <http://www.beautybeautyusa.com> Email: beautybusa@aol.com