

MAX-Ability 12-10

Armedica Changing Table User Manual

Make sure the power cord is plugged into a wall outlet. There should be no obstructions near or under the table. Apply the wheel brakes before transferring a person on or off of the table.

Adjust the height of the table by pressing the up or down button on the hand or foot control. The height adjustment provides a proper height for wheelchair transfers and an ergonomic working height for caregivers.

Raise or lower the safety rails by pulling sharply on the lock release handle. It is recommended that when the table is occupied both safety rails be in the up position.

To move the table, release the wheel brakes. It is not recommended that the table be used to transport an occupant. Take care to secure the foot or hand control for transport with the table.

Protect the vinyl with a waterproof cover-pad as needed during changing. The table is not to be used in a shower.

Clean the vinyl after each use with a soft cloth with soap and water. A hospital grade disinfectant may be used. Do not use bleach or coarse cleaning pads. Clean the underside of the table daily with all-purpose cleaner and a soft cloth.

Preventative maintenance is recommended quarterly. Check the operation of the product; check all fasteners; check the power cord and up/down control. Lubrication and adjustments should not be necessary.

For technical support call MAX-Ability: 800-577-1555.