




QUILTED COMFORT PROFILED MATTRESS USER MANUAL QC63

The packaging of this mattress is sealed prior to despatch and the mattress itself will be clean prior to closure. If the mattress has become exposed or damaged during transit, please contact us at the address below

CMHR castellated foam mattress. Quilted multi-stretch knitted removable cover, sewn seams and zipped closure



FOAM FOR CARE
Azalea Close, Clover Nook Industrial Park,
Alfreton, Derbyshire DE55 4QX
Tel 01773 838800 Fax 01773 838870

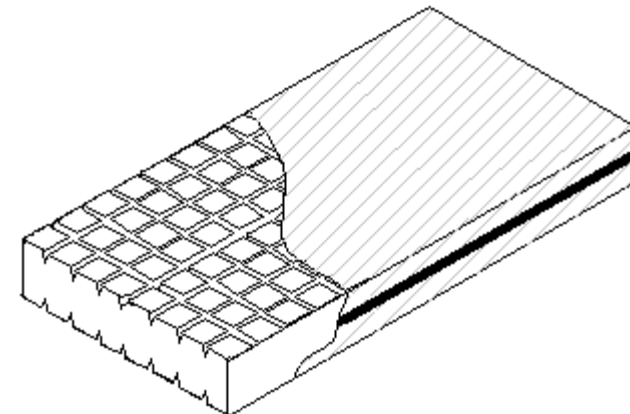

The manufacturing process operates under Class 1 of the Medical Devices Directive by fully complying with the essential requirements for CE marking this product.

Flammability

The Quilted Comfort profiled mattress meets the requirements of BS7177:2008 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

User Instructions

The Quilted Comfort profiled mattress has been designed for use on solid / mesh, profiling / flat bed bases, by patients in the high risk user category♣ with a weight guide of up to 213 Kg / 34 stone.

Ensure that the surface of the bed is clear from debris before placing the mattress – debris can damage the coating of the fabric.

Frequent rotation is strongly recommended for 4-way turning mattresses as all foam grades fatigue during use and turning will ensure loading is evenly spread across both ends of the mattress.

The cover on this mattress is removable for cleaning purposes. Always ensure that, if removed, the cover is returned to the same inner. Using this cover on a different mattress core MAY render the warranty invalid.

The label on this mattress has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

♣Indication of use based upon interface pressure readings– not a prescriptive criteria. Always defer to clinician's judgement. Safe working load based on test methods devised at the Recticel International Development Centre, Wetteren, Belgium.

Warranty

Covered against faulty manufacture and materials as replacements goods.

Normal wear and tear expected would not be included.

All warranties subject to terms and conditions of trading.

Please refer to our warranty schedule for terms.

In order to claim under warranty refer to the batch number printed on the product label.

Cleaning

Removable, contaminated covers should be laundered at 65°C for not less than 10 minutes or at 95°C, for not less than 3 minutes. Do not mangle or iron.

Do not bleach

Do not mangle or iron

Thoroughly rinse the cover with clean water and allow to dry before re-use or storage

Spin & tumble dry at no more than 60°C.

Storage & Care

- Store the mattress flat – do not fold
- Do not place heavy objects on the mattress surface when not in use
- Do not allow sharp objects to penetrate the cover coating
- Keep the mattress clean
- Regularly inspect the interior and exterior of the mattress
- Do not store in damp conditions

To aid storage and handling this Quilted Comfort profiled mattresses may be compact rolled. Whilst this does not affect the performance and lifespan of the mattress, the mattress should be unrolled within 21 days of receipt.

When removing packaging use great care to ensure the cover is not damaged.

This mattress can be used one hour after unrolling. However, it may take longer before the mattress is fully restored to its original shape.

Please ensure that once unrolled the cover seams sit correctly to the edges of the foam mattress and that the zip is fully closed before use. Some seam slippage may occur in the rolling process but can easily be re aligned over the mattress core.

Some degree of creasing may occur to the mattress cover; this is a normal result of the rolling process and cannot be avoided.