

NEW PEN SETUP
DO THIS ONE TIME ONLY

NOTE: This is only a quick reference guide and is not intended to replace the BYETTA[®] (exenatide) pen user manual. Refer to the pen user manual and the Consumer Information sheet that comes with the BYETTA pen carton for detailed instructions and commonly asked questions. The instructions to use both the 5 µg and 10 µg pens are the same.

For additional information, contact AstraZeneca at 1-800-668-6000 or visit www.mybyettacare.com.

STEP A Check the Pen




- ❑ Wash hands prior to use.
- ❑ Check pen label to make sure it is the right dose – 5 µg or 10 µg as prescribed.
- ❑ Pull off the blue pen cap.



- ❑ Check BYETTA in the cartridge. The liquid should be clear, colourless and free of particles. If it is not, do not use.

Note: A small air bubble in the cartridge is normal.

STEP B Attach the Needle

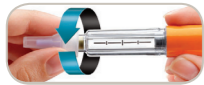

- ❑ Remove paper tab from outer needle shield.
- ❑ **Push** outer needle shield containing the needle **straight** onto the pen, then **screw** needle on until secure.





- ❑ Pull off outer needle shield. **Do not throw away.**




- ❑ Pull off inner needle shield and throw away. A small drop of liquid may appear. This is normal.

STEP C Dial the Dose



- ❑ Check that the  is in the dose window. If not, turn dose knob clockwise **until it stops** and the  is in the dose window.



- ❑ **Pull dose knob out until it stops** and the  is in the dose window.



- ❑ **Turn dose knob clockwise until it stops at 5 or 10.** Make sure that the 5 or 10 with the line under it is in the centre of the dose window.

STEP D Prepare the Pen




- ❑ Point the needle of the pen up and away from you.



PUSH and HOLD


- ❑ Use **thumb to firmly push the injection button in until it stops**, then continue holding the injection button in while slowly counting to 5.
- ❑ If you do not see a stream or several drops come from the needle tip, repeat Steps C and D.



- ❑ Pen preparation is complete when the  is in the centre of the dose window AND you have seen a stream or several drops come from the needle tip.

STEP E Complete New Pen Setup



- ❑ **Turn dose knob clockwise until it stops** and the  is in the dose window.

- ❑ New Pen Setup is now done. Do not repeat New Pen Setup for routine use; if you do, you will run out of BYETTA before 30 days of use.

- ❑ You are now ready for your first dose of BYETTA.
- ❑ **Go to ROUTINE USE Section, Step 3, for instructions on how to inject your first routine dose.**

BYETTA is used to improve blood sugar control in adults with type 2 diabetes in combination with metformin and/or a sulfonylurea when metformin and/or a sulfonylurea plus diet and exercise have failed to adequately control blood sugar levels. Continue to follow your diet and exercise plan.³

BYETTA helps your body release more insulin when your blood sugar is high. This helps to improve your blood sugar control.³

The most common side effects with BYETTA include nausea, vomiting, diarrhea, feeling jittery, dizziness, headache, acid stomach and heartburn.³

ROUTINE USE

STEP 1 Check the Pen




- Wash hands prior to use.
- Check pen label to make sure it is the right dose pen.
- Pull off the blue pen cap.



- Check BYETTA in the cartridge.
- The liquid should be clear, colourless and free of particles. If it is not, do not use.

Note: A small air bubble will not harm you or affect your dose.

STEP 2 Attach the Needle

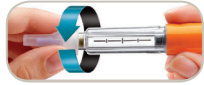

- Remove paper tab from outer needle shield.
- Push** outer needle shield containing the needle **straight** onto the pen, then **screw** needle on until secure.



- Pull off outer needle shield. **Do not** throw away.



- Pull off inner needle shield and throw away. A small drop of liquid may appear. This is normal.

STEP 3 Dial the Dose



- Check that the **5** is in the dose window. If not, turn dose knob clockwise **until it stops** and the **5** is in the dose window.



- Pull dose knob out until it stops** and the **5** is in the dose window.



- Turn dose knob clockwise until it stops** at **5** or **10**. Make sure that the **5** or **10** with the line under it is in the centre of the dose window.

STEP 4 Inject the Dose



- Grip pen firmly.
- Insert needle into skin using the **hygienic** injection technique recommended by your healthcare professional.



PUSH and HOLD

- Use thumb to firmly push the injection button in until it stops**, then continue holding the injection button in while **slowly counting to 5** in order to get a full dose.
- Remove needle from skin.



- Injection is complete when the **5** is in the centre of the dose window.
- The pen is now ready to be reset.

Note: If you see several drops of BYETTA leaking from the needle after the injection, the injection button was not pushed in all the way.

STEP 5 Reset the Pen



- Turn dose knob clockwise until it stops** and the **5** is in the dose window.

Note: If you cannot turn the dose knob or if your pen leaks, your full dose has not been delivered.

STEP 6 Remove and Dispose of the Needle



- Remove the needle after each injection.**
- Carefully put the outer needle shield back over the needle.



- Unscrew the needle.
- Replace blue cap on pen before storage.



- Throw away needles in a puncture-resistant container or as recommended by your healthcare professional.

STEP 7 Store Pen for Next Dose

- Store your BYETTA pen properly.
- When it is time for your next routine dose, go to **ROUTINE USE Section, Step 1**, and repeat Steps 1-7.

References: 1. BYETTA 1.2 mL Pre-filled Pen: User Manual. Eli Lilly Canada Inc. September 21, 2011. 2. BYETTA 2.4 mL Pre-filled Pen: User Manual. Eli Lilly Canada Inc. September 21, 2011. 3. BYETTA® Product Monograph, Bristol-Myers Squibb Canada and AstraZeneca Canada Inc., July 31, 2013.

BYETTA® is a registered trademark of the AstraZeneca group of companies.
The AstraZeneca logo is a registered trademark of AstraZeneca AB.
© 2014 AstraZeneca Canada Inc.