


Reizen Mouse Electronic Magnifier Wireless Version for TV (#807432)

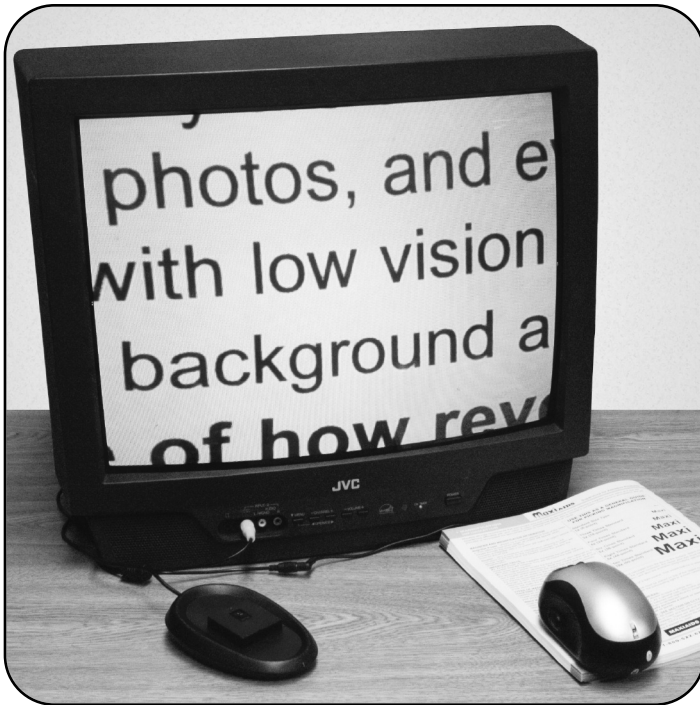

**REIZEN, Inc.
P.O BOX 3312
Farmingdale, NY 11735
www.ReizenUSA.com**



User's Manual

Congratulations on your purchase of the Reizen Mouse Electronic Magnifier. This magnifier was designed to enable those with low vision to read comfortably and easily. By simply rolling the mouse magnifier over newspapers, magazines or other reading material, you'll be able to view text magnified on your TV screen

With adjustable 2x to 32x magnification, variable viewing modes and an Image Lock (Freeze Frame) feature, you'll have the flexibility and functionality you need to make reading easier than ever.

Important Safety Precautions

To keep the unit in good condition and to ensure safe usage, please refer to the following guidelines:

- Read the instructions in this user's guide before starting to use the unit.
- Do not attempt to repair the unit or open its case. This product contains no user serviceable parts.
- Keep the unit away from wet areas, rain, and other sources of liquids or high humidity. Do not submerge the unit.
- Always operate the unit in a place where the temperature is between 10° and 40° C (50° and 104° F).
- Always store where the temperature is between -20° and 65° C (-4° and 149° F).
- Always use the unit on a stable, level, hard surface such as a desk or table.
- Never force connectors into the unit's ports. All appropriate connectors should fit easily into the ports. If the connector does not fit, turn it around, and try again.
- Use only the power cable that came with your unit. Other adapters may damage unit.
- When cleaning the unit, make sure it is unplugged. To clean it, lightly dampen a soft, clean cloth with water or mild detergent.
- Do not bend or twist the cable excessively as this could cause fatigue and lead to unit failure.
- Do not leave the power cable or product cable where someone might trip over it.

Technical Specifications:

Item	Description
Display Resolution	Multiple Options
Display Mode	Full Color, B/W, W/B, High Contrast
Image Sensor	CMOS, VGA resolution
Power Output	5V DC; 250ma (Max)
Battery Life	3-4 hours
Battery Type	Lithium
Weight	100 grams adapter excluded
Dimensions: Mouse	5" x 2.75" x 2.50"
Dimensions: Receiver	5.5" x 3.5" x 1.375"
Magnification Range:	32 times depending on display screen size

One Year Manufacturer's Limited Warranty

WARRANTY TERMS:

1. This unit is covered for one year from the date of purchase for manufacturer's defects only.
2. The following are not covered:
 - A) damage due to improper handling or use
 - B) normal wear and tear
 - C) accessories: power transformers, cords, etc.
3. If repairs are needed after the warranty period expires or in case of improper use or handling of the item, the customer will be responsible for paying the appropriate parts and labor costs.
4. In case repair is necessary in the future, please record the date of purchase below for your own records:

Date of Purchase: _____

Mouse Magnifier Button Functions:

1. "M": Mode selection button (Full Color - Black on White - White on Black - High Contrast)
2. "+": Zoom in
3. "-": Zoom out
4. "*": Image Lock (Freeze Frame)
5. Camera On/Off Switch

Installing & Using the Mouse Electronic Magnifier - (Wireless Version for TV):

1. First, charge the wireless mouse magnifier using the supplied cable (Disconnect AC cable near yellow Video Out plug and connect to jack on mouse.) Mouse fully charges in 3-4 hours. After mouse is charged, re-connect cable.
2. Plug the Receiver's Video Output (yellow plug) into your TV's Video In jack
3. Flip the receiver switch on (straight line). Red indicator light will illuminate.
4. Slide the switch on the mouse to "On" and you're ready to go!
5. Just place the Mouse on top of the reading material and view it magnified on your TV screen. Gently slide the mouse across the page as you read.
6. Use the "Plus" and "Minus" keys to adjust the magnifying power (2x to 32x)
7. Use "M" to choose Color, Black on White, White on Black or High Contrast mode
8. Use the "*" (Image Lock) button to 'freeze' an image for extended viewing

What's in the Box?

- Reizen Mouse Electronic Magnifier
- Charger & Adapter
- Receiver (w/Video Output Plug)
- Drawstring Carry/Storage Pouch
- User's Manual

Main Features of Mouse Electronic Magnifier:

- 8 adjustable magnification levels (2x-32x) to accommodate varying user needs
- Multiple viewing modes: full color, black on white, white on black, high contrast
- Built-in LED lighting system allows for operation in different lighting environments
- Ergonomically designed and easy to use
- Stable, reliable wireless receiver and transmitter ensure strong signal quality
- Image Lock/Freeze Frame: Freeze images on screen for extended viewing

