

RollER

Owner's Manual



Marketed in Australia and New Zealand by Bzooma Pty Ltd ABN 37 640 907 507

TABLE OF CONTENTS

Table of Contents.....	2
Bzooma PTY LTD.....	2
Manufacturing Company Profile.....	2
Operating Procedures.....	3
Repair & Maintenance.....	3
Charging the Battery.....	4
Product Specifications	4

BZOOMA PTY LTD

This User Manual is written and published according to the latest BZOOMA PTY LTD specifications. We will continue the policy of improvement and may make some changes in the design and specifications without notice. If you have any enquiries, please contact your local dealer or buy directly from this web site bzooma.com.au. Delivery will be via your nearest Bzooma dealer.

This manual is published by Bzooma Pty Ltd

MANUFACTURING COMPANY PROFILE

Your Bzooma Roller is manufactured by AVIC Guangzhou Aviation Industry Group Company Ltd, founded in 1951. The company was founded to conduct scientific, production and management research. The enterprise belongs to China Aviation Industry Corp, and has a staff of more than 10,000 people. Development of electric cars is a focus of the company's efforts.


OPERATING PROCEDURES

1. Make sure the new scooter is fully charged upon first use.
2. Check the controller is in the neutral position.
3. When seated fasten your seat belt.
4. Insert the key to the power switch and turn ON (at night use Night mode P). An alarm will sound to indicate that power is on. Speed level setting and battery level will be indicated.
5. Operate the joystick controller to move forward/back/left/right.
6. Release the joystick controller and the electronic brakes will stop the scooter. A quick stop is possible by moving the joystick in the reverse direction. Never turn the key off while moving because the brakes are electrically operated and power must remain available to the brakes while in operation.
7. Collision avoidance is employed by Intelligent Function on the left arm rest. When collision is imminent (obstacle or person within 2.2 meters) the Roller will stop and a voice warning will sound. To continue, it must be switched off and back on again before the joystick is active again. Please note that this function is a rider assistance only and driver diligence is vital at all times.
8. When the scooter is not in use, turn off the power and remove the key.


REPAIR & MAINTENANCE

The Roller is a low maintenance machine, however a 6 monthly inspection by your Bzoooma dealer is advised to check tyres and brakes.

1. Insert charger into the controller charge point, then insert power port to the AC outlets, then switch on the power.

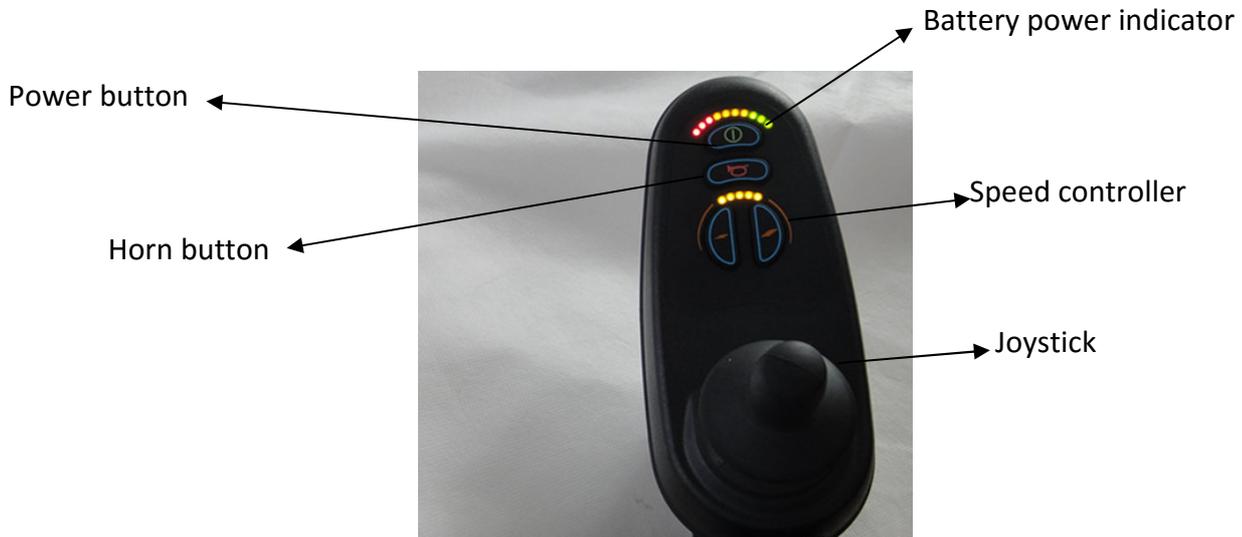

2. Now turn key to 'POWER ON' to start charging.

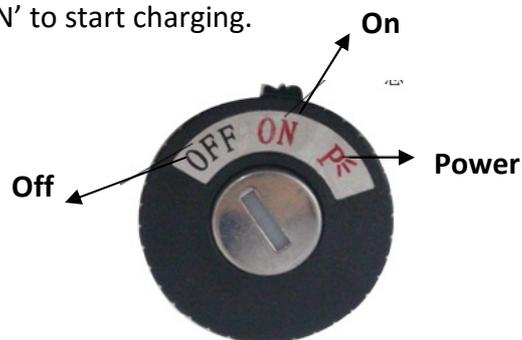

3. Do not press the button to turn on the controller. Fully charge the new battery for at least 7 hours in a well ventilated area before use.



4. The LED light of the charger will turn RED while charging.
5. When fully charged the LED light will turn GREEN. The charger is a smart charger which will not continue charging when the chair is full charged.
6. After charging, first turn off power and pull the AC power plug, then pull out the charger port.

7. The battery is a high quality unit, and with normal use and care it is capable of approximately 2000 cycles. It will only gradually lose range and power but remain functional, and should give a long serviceable life if this simple advice is followed.

PRODUCT SPECIFICATIONS

Dimensions	1060 x 650 x 1200
Maximum Speed (km/h)	6 km/h
Maximum Endurance	25 km/h
Driving Mode	Front Wheel Drive
Brake Mode	Electro-Magnetic Brake
Turning Radius (mm)	700
Maximum Gradient	10
Controller	PG controller imported from UK
Motor Specification	200w X 2
Battery Specification (V)	24v Lithium Battery
Battery Capacity (Ah)	30 amp
Charge Time	5-6hrs
Special Feature	Intelligent Obstacle Avoidance

WARRANTY

Bzooma Pty Ltd Limited Warranty

Bzooma Pty Ltd provides this Limited Warranty to the original purchasers of new Bzooma machines in Australia and New Zealand.

Bzooma Pty Ltd guarantees the machine, power base including Motor, Battery and charger, against defects in materials and workmanship for a period of one (1) year, from date of delivery to the original purchaser. Damage caused by accident or obvious abuse is not covered by this warranty.

To make a claim covered by this warranty the purchaser must contact an authorised BZOOMA Pty Ltd Dealer for inspection and service. Indications of abuse, accident, or unauthorised tampering with the machine power base mechanical, or electrical equipment, will void any claim. Bzooma Pty Ltd is not responsible for transport costs associated with obtaining service or repairs under this warranty.

