



Kingkraft

ROPOX
Better ways to better days



Kingkraft Ltd, 26D Orgreave Crescent, Dore House Industrial Estate, Sheffield S13 9NQ
phone 0114 2690697 fax 0114 2695145 email info@kingkraft.co.uk web www.kingkraft.co.uk



KingKraft Shower & Changing Bed User Manual

General information



It is important to read this manual before mounting and daily use.

Product : Ropox Shower & Changing Bed, height adjustable

Producer : Ropox, DK-4700 Naestved, Phone +45 55 75 05 00


Usability – The height adjustable shower bed can be used to nurse children and adults (up to 150Kg). The shower bed can be used at home, in day care centre or institutions to obtain good and correct working conditions for the carer.

When not in use, the shower tray can be folded up against the wall to save space.

By using the up and down button on the handheld control switch, the height adjustment of the shower bed can be performed.

Part Numbers

Length of shower bed	Part No. (excl. back support)	Part No. (incl. back support)
146cm	40-25023	40-25073
178cm	40-25026	40-25076
210cm	40-25029	40-25079

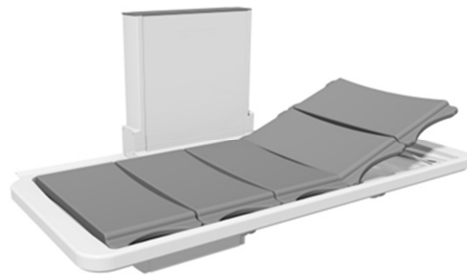

Technical data

Electrical connection:	230V, 50Hz, max 2.0A
Standby consumption:	7W
Duty cycle:	Max 10%, 2 min/18 min
Height:	Work surface 30 – 100 cm Unit, B=60 cm, H=116cm, D=13cm
Material – Frame: and aluminium	Welded steel tubes St.37, Stainless
Shower tray:	ABS
Surface treatment: matt, Chromate	Powder coating, white RAL 9010
Maximum load:	150 kg



This product has CE-marking according to "Medical equipment, Low Voltage and EMC directives. Has been tested according to the European standards DS/EN ISO 12182 and DS/EN ISO 60601-1. This product is in risk group 1.

Daily use

Before start using this product, be sure that "main power" has been connected as described in electrical connections.

By using the up ▲ down ▼ bottom on the handheld control switch, the height adjustment of the shower bed can be performed. The control switch has a spiral cord, which can be pulled out to a length of 200cm.


**Maximum load onto the Shower & Changing Bed
150 kg.**

The slat/cushion

The slats have two sides with different hardness and can be turned around in the Shower tray.

The soft side is used, when the person on the Shower bed needs to be washed.

The hard side is used, when the person on the Shower bed needs to be changed.

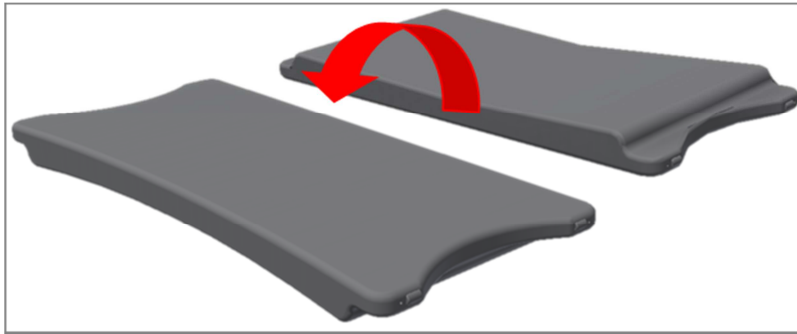

After use

When not using the shower bed, the shower tray can be folded up against the Lifting unit. This way it will only take up the space of approx. 30cm from the wall.

Inside the lifting unit, a gas damper is mounted, to reduce the power needed, when folding the tray up and down.

Recommendations in use

The height adjustable shower bed helps the caretaker to obtain the most optimal working height.

Height adjustment allows you to customize the working height to the individual carer, and can also save unnecessary lifting, when transferring from e.g. wheelchair.

To ensure optimum utilization of the shower bed features and avoid accidents, it is important to observe some basics safety procedures

- NEVER leave a person on the shower bed unattended - not even for a very short period.
- The shower bed must be operated only by adults.
- Pay particular attention to other people in the room, when using the shower bed. Make sure that no children are playing under or with the shower bed.
- Keep the hand control out of the reach of children. Make sure that they cannot operate the shower bed unattended. A bracket for wall mounting is included with delivery.
- Make sure that there is free space above and below the shower bed to allow height adjustment. The shower bed is up foldable, but please be aware of obstacles above, below and around the shower bed to prevent risk of trapping.

- If the control switch has been damaged or stopped working, it has to be replaced immediately. The up and down buttons are "push and hold". The motor stops when the button is being released. This is a very important safety element.
- If two shower beds are mounted next to each other, we recommend to mount a side cover between, to avoid the risk of trapping.
- Constantly be aware of objects under the shower bed.

The Shower & Changing Bed is maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition. It is recommended the table is serviced on an annual basis as recommended by PUWER regulations. This table does not come under LOLER requirements as it is purely for height adjustment.

Cleaning

To clean this product, do not use a high-pressure washer.

Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C.

Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

Accessories

Back support

The back support fits all shower bed lengths and uses two slats. No tools are required for mounting in the shower tray, as the back support has been designed to fit directly into the tray. The back support has three positions: 20°, 40° and 60°.

Item No. 40-25031



Recommendations for use

- Make sure that the unit has been placed correctly in the shower tray so that the two side rails of the bottom frame rest properly on the sides of the tray.
- See that the horizontal supporting rod has snapped into place before leaning on the back support.
- When folding down the back support after use, do not let go of the support, as it will fall down. Instead, carefully lower the back support to horizontal position.
- When folding down the back support, make sure that there are no fingers between the frame and the support in order to prevent fingers from getting caught.
- Note that the slats may become slippery when in contact with water and soap.

EU – Declaration of Conformity



I hereby declare under sole responsibility that the following products:

40-25023 and 40-25073	146 cm	Ropox Shower & Changing Bed,
40-25026 and 40-25076	178 cm	Ropox Shower & Changing Bed,
40-25029 and 40-25079	210 cm	Ropox Shower & Changing Bed,

are all under Classification Class I and in conformity with the following directives and standards:

DIRECTIVES

- The Council Directive No.2007/47/EEC, concerning Medical Devices
amended through Directive No.98/79/EC
- 2006/95/EEC Directive for Low-voltage, amended through Directive No.93/68/EEC
- 2004/108/EEC, EMC-Directive, amended through Directives No.92/31/EEC and 93/68/EEC

STANDARDS

DS/EN14971:2012

DS/EN 60601-1+CORR:2006

DS/EN12182:2012

DS/EN 60601-1-2:2007

DS/EN ISO9999:2009



Tommy Jensen
Adm. direktør

Date: 05-09-2014

Kingkraft Ltd, 26D Orgreave Crescent, Dore House Industrial Estate, Sheffield S13 9NQ
phone 0114 2690697 fax 0114 2695145 email info@kingkraft.co.uk web www.kingkraft.co.uk

WE PLAN + WE MANUFACTURE + WE INSTALL + WE MAINTAIN
