

By	Atos	Medical	Inc.
----	------	---------	------

Hospital		
Department		
Clinician		
Contact information		

Leading the way in ENT

## Artificial Larynx (AL): What is an Artificial Larynx?

#### NOTE: Before using any AL device read the manufacturers instructions for use carefully.

#### What is an Artificial Larynx (AL)?

An artifical larynx or electrolarynx (EL), is an electronic device that makes sound for you to produce speech.

### Why Do I Need an Artificial Larynx?

You may choose to use your artificial larynx as your primary way of communicating or use it as a backup or temporary means of communication. Many begin using it right after surgery before they are healed enough to begin using tracheoesophageal (TE) and/or esophageal speech.

# Why do I still need an AL once I can use Esophageal or Tracheoesophageal (TE) Speech?

- To communicate immediately after your surgery while you are healing
- To communicate while you are learning to produce esophageal or tracheoesophageal speech
- As a backup in case your other form of communication fails
- The AL may be a way of being heard over the noise in a crowd
- The AL may be an easier way to communicate if you are sick and unable to use your esophageal or TE voice

#### How Does an AL Work?

When sound moves into your mouth, you can form sounds into words using your lips, teeth and tongue. There are two ways that the artificial larynx (AL) is used. One way is placing the AL on your neck and the sound moves through your neck tissues into your mouth. Sometimes it is hard for sound to move through the neck tissues due to swelling and/or the stiffness of the neck tissues after radiation therapy or surgery. In this case, you would place an Oral Adapter with an Oral Straw onto the end of your AL. You then place the Oral Straw into your mouth and the sound goes right into your mouth.

Refer to your Speech Language Pathologist for further instruction or CareTips Sheet #9: Artificial Larynx (AL): Basic Training.



This text does not replace nor does it set forth the complete contents of the User Manual and/or Prescriber Information for the products in this text, and is not a substitute for reviewing and understanding that important information. Therefore, before prescribing and/or using any of the products included in this text, please review the entire contents of the respective User Manual and/or Prescriber Information. For more information contact us at: info.us@atosmedical.com - www.atosmedical.us - 1.800.217.0025