

RosieScan™ Bladder Scanner

By

Nurse Rosie Products, distribute by Life
Systems Inc.



Forward

Nurse Rosie Products, distributed by Life Systems, Inc. would like to thank you for your purchase of the RosieScan™ Bladder Scanner. Please read this manual carefully before using the RosieScan. Familiarize yourself with the features, methods of use, cautions and warnings, as well as any limitations of the scanner. This manual should be kept in a safe, convenient place for future reference.

No part of this manual should be reprinted or reproduced without written consent from Life Systems Inc., distributed by Life Systems Inc., maintains the right to modify the contents of this manual without prior notice. If you have any questions about the information presented in this manual or about other Nurse Rosie Products, distributed by Life Systems Inc., please contact us at:

*Nurse Rosie Products / Life Systems Inc.
7320 Central Avenue
Savannah, Georgia 31406
www.nurserosie.com
800-841-1109*

Table of Contents

Table of Contents

1. Introduction	6
1.1 General Description	6
1.2 Primary Components	6
1.3 Intended Use.....	6
1.4 Contraindication	7
1.5 Symbol Directory.....	7
2. Product Specification	8
2.1 Main Technical Specifications.....	8
2.2 Accuracy Range	8
2.3 Instructions for Environment Test	9
2.4 Sound Output.....	9
3. System Overview.....	10
3.1 Primary Components	10
3.1.1 Hardware Components	10
3.1.2 Embedded Software System.....	10
3.2 Pictures and Descriptions	11
3.3 Circuit Diagram and Part List	13
4. Control Panel.....	14
5. Optimal Conditions	15
5.1 For Use	15
5.2 For Storage.....	15
5.3 For Transportation	15
6. Installation	16
6.1 Installation Instructions	16
7. Navigation	18
7.1 Before Powering Up.....	18

7.2	Powering On.....	18
7.3	Ready Screen.....	19
7.4	System Reset Screen.....	21
7.4.1	Time	21
7.4.2	Date.....	22
7.4.3	Print Images	22
7.4.4	Image Display:.....	22
7.4.5	Hospital	23
7.4.6	Alarm.....	23
7.4.7	Sound	23
7.4.8	Volume Calibration	24
7.5	The Scan Screen	26
7.6	The Scan Results Screen.....	28
7.7	Gray Scale Results Screen	29
7.8	The Save/Edit Screen	30
7.9	Browsing Historical Records.....	31
7.9.1	Deleting Stored Records	32
7.9.2	Location Presentation	32
7.10	Printing Results	34
7.11	Exporting Data to a PC	34
7.12	Recharging the Battery	35
8.	Clinical Application.....	37
8.1	Instructions for Use in a Clinical Setting	37
8.2	Images of Invalid Scan Results	38
9.	Transportation and Storage	40
9.1	Transportation	40
9.2	Storage	40
10.	Maintenance and Service.....	41
10.1	Maintenance for Probe	41
10.2	Main Unit Maintenance	41

10.3	Service Guide	41
11.	Troubleshooting.....	43
12.	Quick Reference Guide	44
	Appendix A.....	45
	Appendix B.....	48
B.1	Warnings	48
B.2	Recharging and Discharging.....	48
B.3	Storage	49
B.4	Transportation	49
B.5	Other Instructions.....	49
	Appendix C.....	50
C.1	Pre-Scan Image to the Left of Center.....	50
C.2	Pre-Scan Image to the Right of Center	51
	Appendix D.....	53
D.1	Electromagnetic Emissions	53
D.2	Electromagnetic Immunity.....	54

Introduction

1. Introduction

1.1 General Description

The RosieScan (“Device”) is a noninvasive ultrasound device used to measure urinary bladder volume. During each scan, the RosieScan creates a three-dimensional image of the bladder. This image is then used to automatically calculate and display the urinary bladder volume. The results of the scan may be stored internally, printed using the on-board thermal printer or transferred to an external computer using a USB connection.

The RosieScan can be programmed to make calculations based on a male, female or pediatric bladder before each scan. The RosieScan can store up to 100 patient I.D.s and measurements. When the memory is full the results may be transferred to an external computer and the on board memory can be cleared.

1.2 Primary Components

The RosieScan consists of a Main Unit, Probe (standard frequency 2.6MHz), two Batteries, a USB cable, Adapter and Charger.

1.3 Intended Use

The device is intended to provide a noninvasive means of measuring urinary bladder volume by a licensed medical professional. All users must read this entire manual prior to using the RosieScan. Failure to comply with these instructions may compromise the performance of the device and the accuracy of its measurements.

Immediately after unpacking your new RosieScan, refer to the parts list below to ensure that all parts are included in your package. Also verify that the user’s manual, maintenance document and all accessories are included.

RosieScan Packing List		
No.	Name of Parts	Quantity
1	Main Unit	1
2	Probe	1
3	Adapter	1
4	Charger	1
5	Battery	2
6	USB Cable	1
7	Cross Screwdriver	1
8	Certificate	1

If any parts are missing please contact Life Systems using the contact information listed in the Foreword.

Check to ensure that all components are connected properly and free of defect before turning on the RosieScan. Do not use if any component is missing or damaged. Cease use immediately and contact customer service if the RosieScan appears to be functioning abnormally in any way. Examples include, but are not limited to the screen becoming abnormally bright, strange or unusual sounds coming from the device or the sudden smell of fumes or smoke. The RosieScan must be calibrated before its initial use and should not be used in close proximity to other electrical devices or high-frequency operational equipment.

Make sure to turn the RosieScan off after every use. Do not leave the RosieScan unattended while it is turned on or in use. Remove the batteries and place them in an easily accessible location before the RosieScan is transported or stored for an extended period of time.

1.4 Contraindication

The device is not suitable for those who have an abdomen wound or skin disease.

1.5 Symbol Directory

Symbol	Indicates	Symbol	Indicates
	USB screen		Class II device
Probe	Probe		B-type application
	Indicate arrow of battery insert direction	IPX4	Be away from liquid intrusion
	Open the battery buckle		Check the documents enclosed in the device
	Lock the battery buckle		DC
	Switch indicator		AC

Product Specifications

2. Product Specification

2.1 Main Technical Specifications

- 1 *Transducer: 2.6MHz*
- 2 *Probe: Mechanical Sector Scanning*
- 3 *Detect depth : ≥ 140 mm;*
- 4 *Host power supply: internal battery*
- 5 *Battery: Voltage 7.4V (standard value), capacity: ≥ 4200 mAh Charge the batteries for 3.5-4 hours for full charge. Fully charged batteries will provide 4 hours of sustained use*
- 6 *Charger : Input voltage : AC 220V $\pm 10\%$, 50 ± 1 HZ ;*
- 7 *Output voltage for adapter: 9.0V ;*
Output voltage for charger: 8.5V ;
- 8 *Output screen: USB for data storage and software upgrade*
- 9 *Readings displayed: scan results, present location and ultrasound image*
- 10 *Measured result: scan results are automatically measured and displayed.*
- 11 *Case management options: Store, browse and export*
- 12 *Max. measurement: 999ml;*
- 13 *Measurement accuracy: $0\text{ml} \leq V \leq 999\text{ml}$, $\pm 15\%$;*
- 14 *Storage capacity: 100 cases*
- 15 *Device dimensions: 310x260x50mm ;*
- 16 *Net weight: about 2.1 KG. (4.62 LBS)*

2.2 Accuracy Range

Accurate results are contingent upon proper use of device.

Explanation	Narrative
Bladder volume accuracy	$\pm 25\%$ ($60\text{ml} \leq V \leq 150$)
	$\pm 15\%$ ($150\text{ml} \leq V \leq 999$)
Bladder volume range	0-999ml

2.3 Instructions for Environment Test

The environmental testing for the device should be in accordance with requirements of GB/T 14710-2009 (Medical electrical equipment environmental requirement and test method)

2.4 Sound Output

The device is in accordance with GB16846-2008 and the announcement exemption for the public in chapter 6 of Requirement of medical ultrasound diagnosis device for its sound output (see Figure 2-1).

Transducer frequency: 2.6MHz

Index Label	MI	TIS		TIB	TIC	
		Scan	Non-Scan	Non-Scan		
			A aprt <= 1 cm	A aprt > 1 cm		
Maximum index value	0.89			0.96		
Associated Acoustic Parameters	p_{ra}	1.44				
	p					
	Min. of $[P_{a}(Z_s), I_{a, a}(Z_s)]$			81		
	zs			1.95		
	zbp			1.95		
	zb					
	z at max. / pi.e	1.95				
	d _{eq} (zb)					
	fawf	2.63			2.63	
	Dim of A aprt	X			1.3	
Y				1.3		
Other Information	td	1.04				
	pr	430				
	pr at max. / pi	3.5				
	d _{eq} at max. / pi					
	I _{pa, a} at max. / MI	212				
	Focal Length			5		
Operating Control Conditions	Control 1	---		---		
	Control 2	---		---		
	Control 3	---		---		
	...	---		---		

Fig 2-1 Technical Datasheet for Level of Sound Output

Device is in accordance with requirements of GB9706.9-2008 -< Part 2-37 in Medical electrical equipment : 51.2aa,dd) in requirements for particular safety of ultrasound diagnostic and monitoring equipment> . Exempted from showing MI, TI. MI <1 TI <1

System Overview

3. System Overview

3.1 Primary Components

The system consists of two parts --hardware circuit and embedded software.

3.1.1 Hardware Components

The hardware circuit consists of an embedded micro-controller system, B-mode transducer, B-mode signal processing module, B-mode signal A/D conversion module, Step-motor driving module, LED display system, print system, keyboard input, high speed USB screen and the power/battery management system. Its composition is shown in Figure. 3-1:

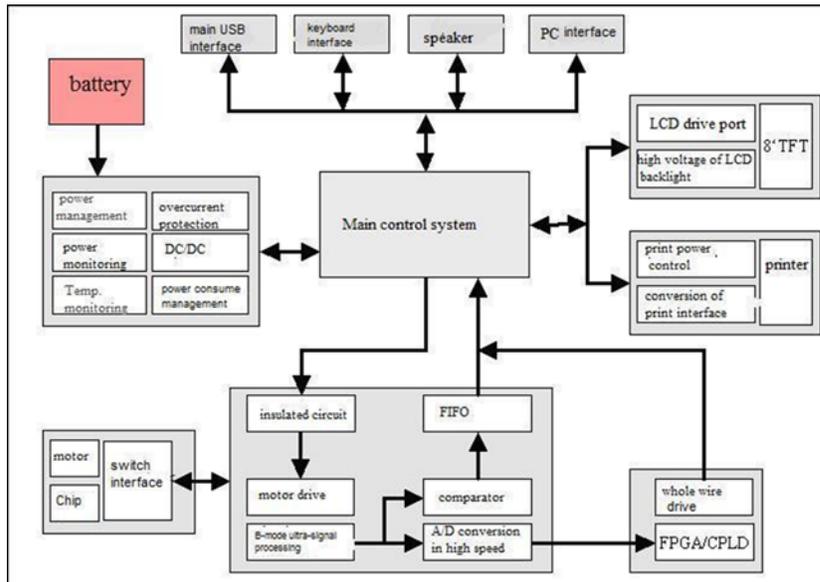


Figure 3-1
Hardware Configuration

3.1.2 Embedded Software System

The embedded software system is fully designed according to the hardware structure to allow for better stability, ease of use and a friendly user screen. The software modules include B-mode ultrasonic-image data acquisition, image edge identification, step-motor drive control, LED drive, graphical menu display, printing, power management and USB data communication. Its components are shown in Figure 3-2.

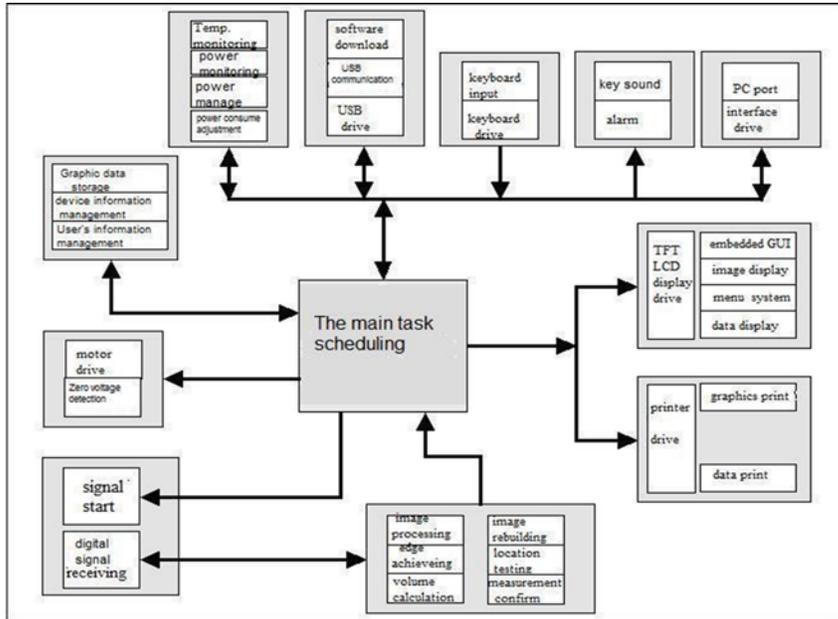


Figure3-2
Software Module Block
Diagram

3.2 Pictures and Descriptions

The RosieScan consists of a main unit, ultrasonic probe and external thermal printer.

1. RosieScan see Figure 3-3

1



Figure 3-3
Device

1. LED Display
2. Control Panel
3. Printer
4. Power Light
5. Probe
6. Screen Buckle
7. Printer Cover

2. Probe and USB Port see Figure 3-4



Figure 3-4
Ports
1. Probe Port
2. USB Port

3. 3D Probe see Figure 3-5

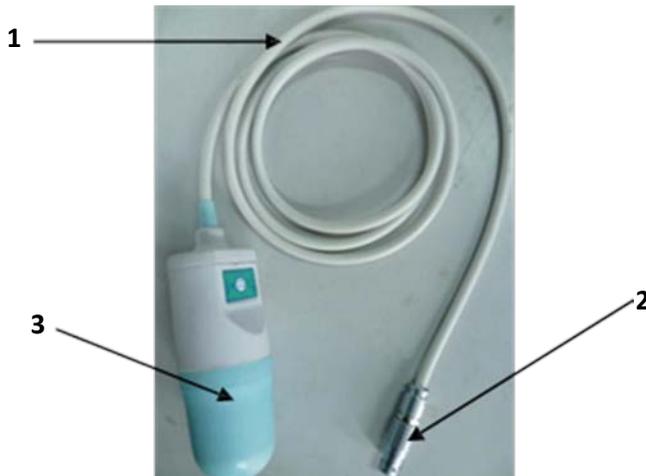


Figure 3-5
Ports
1. Probe Cable
2. USB Plug
3. Probe Perspective Window

4. Battery Box see Figure 3-6



Figure 3-6
Battery Box
1. Battery

3.3 Circuit Diagram and Part List

Please contact Life Systems Inc. for circuit diagram, parts list and any other information pertaining to maintenance or repairs. THE USER SHOULD NOT DISASSEMBLE OR TRY TO REPAIR THIS DEVICE.

Control Panel

4. Control Panel



Figure 4-1
Control Panel



Optimal Conditions

5. Optimal Conditions

5.1 For Use

1. Temperature range: +41~+105 F;
2. Relative humidity: ≤70%;
3. Atmospheric pressure: 700~1060 hPa;
4. Power supply of charger: AC 220V±10%, 50Hz±1Hz;
5. Keep device away from strong electrical fields, strong magnetic fields and high-voltage equipment.
6. Use the socket from an independent power supply.
7. Images on the display may appear faded if viewed in bright rooms. The device should be used in rooms that are clean, and pollution free. Loud noises may result in inaccurate readings.

5.2 For Storage

1. Temperature range: 23F~104F,
2. Relative humidity ≤ 80%
3. Scope of atmospheric pressure: 700 hPa~1060 hPa

5.3 For Transportation

1. Temperature range: 14 F~131 F
2. Relative humidity ≤ 80%
3. Scope of atmosphere pressure: 700 hPa~1060 hPa

Note: The battery should be removed before the device is transported.

Installation

6. Installation

6.1 Installation Instructions

1. Put the device on a soft, flat surface.
2. Turn the main unit over so the bottom is facing up
3. To ensure that the battery is being installed correctly, confirm that the arrow  on the unit corresponds with the arrow  on the battery.
4. Slide the battery lock into the unlocked position  and hold. See Figure 6-1



Figure 6-1

1. Batter Lock

5. Insert the battery into the battery slot while holding the battery lock in the unlocked position.
6. Then slide the battery lock into the locked position. 
7. When the battery is successfully installed it will resemble Figure 6-2. The arrow on the battery is pointing directly at the arrow on the main unit.



Figure 6-2
Back view of main unit
with battery in place

Note: The device must be switched off before removing the battery.

8. To remove the battery, put the device on a soft, flat surface. Move the battery lock to the unlocked  position, and then slowly pull the battery out of the unit.
9. To install the probe, find the red dots on the main unit port and on the plug at the end of the probe cord. See Figure 6-3. Align the two red dots and then plug the probe into the main unit. See Figure 6-4.



Figure 6-3
Probe Installation



Figure 6-4
Probe Installed

Note: Do not scratch the surface of the probe. This will result in inaccurate readings. Only the probe supplied by the original manufacturer may be used with this device. Please contact Life Systems, Inc. in the event a replacement probe is needed.

7. Navigation

7.1 Before Powering Up

1. Confirm that that battery is properly installed and locked
2. Confirm that the probe is properly installed
3. Confirm that the device is tightly secured on the stand (if applicable).

7.2 Powering On

To turn on the device, press the “POWER” key. Once turned on, the device will automatically check the battery level, printer, probe connection, memory capacity and screen. The screen show in Figure 7-1 will display after the device is turn on



Figure 7-1
Power On Screen

Note: Once the device self-checks successfully, the system will enter its main system screen. If any of the items fails during self-checking, an alarm will alert you. If the alarm sounds, press direction key **▶** before entering the main system. If the printer fails, it will not influence your measurements

Self-check displays:

Battery Checking.....OK	This means your battery is good and connected properly.
Probe Checking.....OK	This means the probe is connected correctly and ready to use.
Printer Checking.....OK	This means the printer is ready to print.
Storage.....100%	Percent represents the remaining storage space.

Note: If “Disconnect” is displayed instead of “OK” next to the probe-checking, remove and reconnect the probe. If “Disconnect” is still displayed, check the probe cord and contact Life Systems, Inc.

At the bottom right corner of the power on screen there will be two functional keys:  and . The  functional key displays when self-checking fails, so touch the  key or press the directional key  on the control panel to enter “Ready” screen of system. You may touch  with stylist pen or press Power button  on the control panel to turn off the device.

Note: The system will automatically power down if you do not press  key on the control panel within 20 seconds of a self-check failure.

7.3 Ready Screen

Once RosieScan is finished self- checking the Ready screen will display on the monitor, see Figure 7-2.

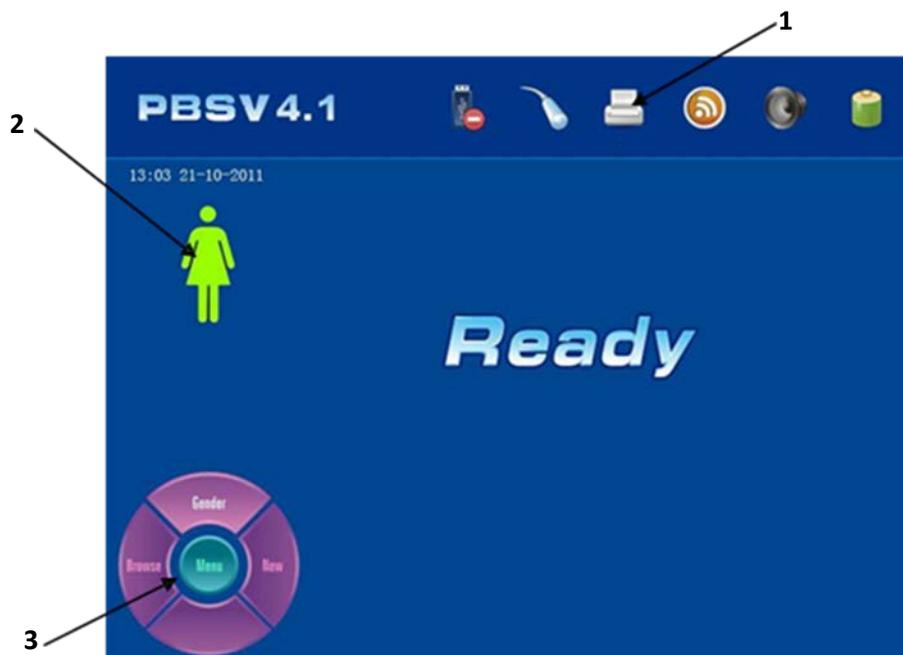


Figure 7-2
Ready Screen
1. Status Bar
2. Scan Mode
3. Touch Screen
Navigation Keys

On the top of the Ready screen you will see a Status bar. The icons below may change depending on the device's self-check results. Please consult the Figure 7-3 for the meaning of each icon. Please contact Life Systems Inc. if you have any questions.



Icon	USB	Probe	Printer	Prompt	Alarm	Battery
Normal or successful connection						
Problem or no connection						

Figure 7-3 Status Bar and Instructions

You will find two sets of navigation keys – one set on the Ready screen and one set on the control panel. See Figure 7-4 and 7-5.

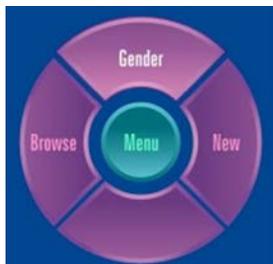


Figure 7-4 Ready Screen



Figure 7-5 Control Panel

Figure 7-4 and 7-5 Navigation Keys

- Press  key on the right or touch  on the Ready screen to delete the max tested volume without switching off the device;
- Press  key on the left or touch  to browse previously saved readings
- Press  key or touch to  to select gender. Select the  icon for male, the  for female and the  icon for Pediatric. Pediatric mode is only effective to measure volumes that are less than 100 ml.

- Press  key or touch  to enter the system reset screen. You can touch the screen to select features you need to change or use the directional keys on the control panel. For example touch "Scan Images" to choose between scanning 12 or 24 images. You must click on Save/Exit in order to save your changes and go to the pre-scan screen.
- Press  key before entering the **pre-scan** screen. If the probe is not connected, the device will not enter the pre-scan screen.

Note: Check to see if any icons on the status bar have changed. If they have, refer to Figure 7-3 on page 20. If you have any questions, please contact Life Systems Inc.

7.4 System Reset Screen

Press the  key or touch  to enter the system reset screen (Figure 7-6) from the Ready screen. There the user can set the time, date, print mode, and display mode, name of facility, key prompt sounds and alarm on/off in the system.

Press alarm on the screen for turning on/off alarm and sounds.

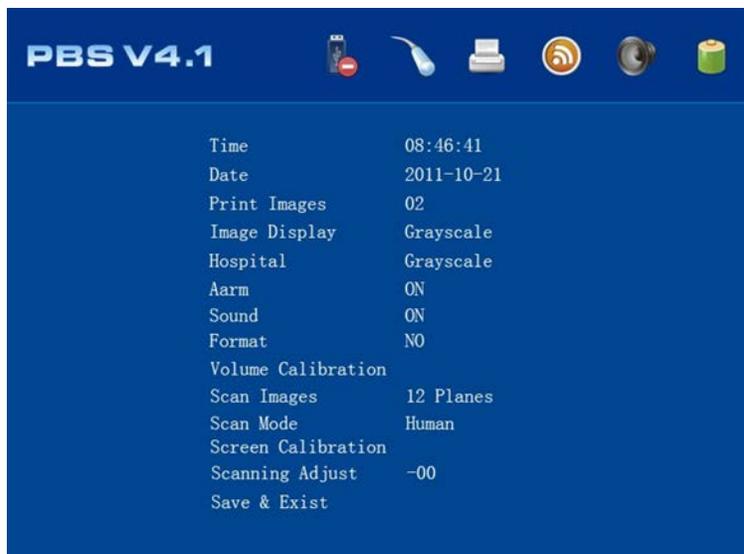


Figure 7-6
System Reset Screen

7.4.1 Time

Note: Time should be entered in the following format: Hour\Minute\Seconds

1. Press the ▲ / ▼ keys to move and select "Time" or use your stylus to select it on the screen (its background will turn purple).

2. Press the  key to underline HH and then press the ◀/▶ keys to move the underscore between HH/MM/SS (Hour/Minute/Seconds). Press the ▲/▼ keys to adjust the time where the underscore is located. You can also use your stylist to touch HH, MM, or SS to change the time. An underscore will appear under the number you wish to change. Tap the desired number (0-9) with the stylist pen. The underscore will then automatically move to the next option until you complete your setting.
3. Press the OK key to confirm the setting or touch “Time” with stylist pen again to save the setting.

7.4.2 Date

Note: *Date should be entered in the following format: YY/MM/DD (Year/Month/Day)*

1. Press the ▲/▼ keys to move the selected setting to “Date” or use your stylist to select “Date” on your screen (its background will turn purple).
2. Press the  key to underline YY, MM or DD, use the ◀/▶ keys to move the underscore among YY/MM/DD. Use ▲/▼ to adjust the dates. You can also use your stylist to select the date you want to change. Touch the desired figure (0-9) with the stylist pen. The underscore will automatically move to the next option until you have finished your setting.
3. Press the  key to complete its setting

7.4.3 Print Images

The device can print two types of images (gray scale + red line or gray scale)

1. Press the ▲/▼ keys or touch the “Print Images” option with a stylist. (Its background will turn purple.)
2. Press the  key to underline the number of images then press the ▲/▼ keys to set the number to be printed or use the stylist to select number of images from the screen.
3. Images can be selected in groups of 2's (2,4,6,8,10,and 12)
4. Press the  key to confirm the setting.

7.4.4 Image Display:

Images can be displayed in two color or gray scale.

1. Press the ▲/▼ keys or touch “Image Display” with the stylist pen (this background will turn purple.)

2. Press the  key to underscore the type of display image. Then press the ▲ / ▼ keys to select “Gray Scale + Red Line” or “Gray Scale.”
3. Press the  key to confirm the setting

7.4.5 Hospital

1. Select the “Hospital” option and press the ▲ / ▼ keys or select “Hospital” with the stylist pen to type in your name. (see Figure 7-7)
2. Press the print  key or touch  after editing the facility name to save the name and return to the system reset screen. Then press the  key or touch Save & Exit on the screen to save and exit this screen.



Figure 7-7
Facility Naming Screen

7.4.6 Alarm

1. Press the ▲ / ▼ keys to select “Alarm” and press the ▲ / ▼ keys to turn alarm on or off. You can also use your stylist to touch “Alarm” to turn the alarm on or off
2. Press  key or touch “Alarm” with your stylist to save the setting

7.4.7 Sound

1. Press the ▲ / ▼ keys to select “Sound” or touch “Sound” with your stylist (its background will turn purple.)
2. Press the  key to underscore the setting and press the ▲ / ▼ keys to change sound from on or off or touch ON/OFF with your stylist to select the ON/OFF option.

3. Press the  key or touch "Sound" again to save the setting.

7.4.8 Volume Calibration

A "phantom bladder" will be needed for calibration. The phantom mimics a human bladder with a known volume. The phantom should be scanned and the measured volume should be the same as the phantom's standard volume. If the measured volume is different from phantom's standard volume, use the correcting parameter to calibrate the device. The range of its correction is ± 40 ml.

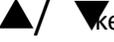
1. Press the  keys or touch Volume Calibration. Press  or touch Volume Calibration to enter password (see Figure 7-8)
2. Once you have entered the password, press  or touch  to move to the calibration screen (see Figure 7-9)
3. Press the  keys again to set '+' '-', then press  to set the calibration value
4. Press  key or touch  to save the calibration value. Press the  key or touch  to exit the calibration screen



Figure 7-8
Password Input Screen for Calibration



Figure 7-9
Calibration Screen

5. Scan Images: select the number of images to scan
 - a. Press the  keys or touch **Scan Images**
 - b. Press the  key to underline the numbers of images you need to scan.
 - c. Press the  keys or touch the number of images (12 or 24) to scan
 - d. Press the  key or touch **Scan Images** to save the setting

6. Scan Mode:
 - a. Press the ▲/ ▼ keys or touch Scan Mode with the stylist pen
 - b. Press the OK key or touch Scan
 - c. Press the ▲/ ▼ keys or touch Human OR Phantom before selecting Scan Mode
 - d. Press the OK key or touch **Scan Mode** to save the setting.
7. Screen Calibration:
 - a. Press the ▲/ ▼ keys or touch Screen Calibration
 - b. Press the  key or touch "Screen Calibration" (See figure 7-10).
 - c. Touch cursor with stylist pen to complete the calibration of the touch screen.

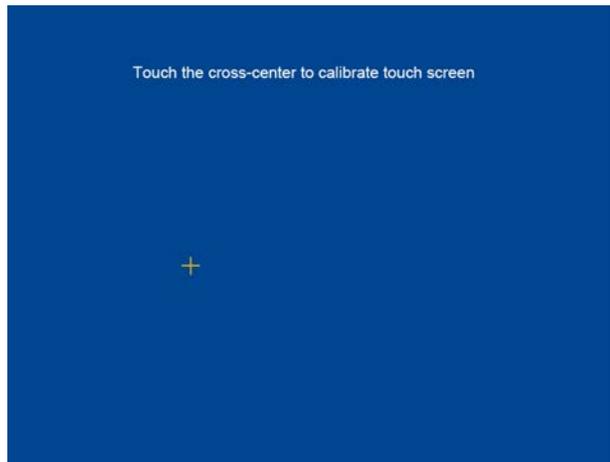


Figure 7-10
Touch Calibration
Screen

8. Scanning Adjust: start point for setting the probe location before scan.
 - a. Press the ▲/ ▼ key or touch Scanning adjust
 - b. Press the  key or touch "-/+", then press the ▲/ ▼ key or touch "-/+" again before selection
 - c. Press the ▲/ ▼ key or touch number character, selecting number setting. The number icon will appear once you touch the number, than touch the number icon for number setting.
9. Save and Exit: Exit from the setting before returning to Ready screen
 - a. Press ▲/ ▼ key or touch Save and Exit
 - b. Press the OK key or touch Save and Exit to save the setting

Note:

- *Facilities name may be up to 25 characters*
- *All saved records may be deleted using the "Format" option.*
- *When selecting Human mode, there is icon displayed on the top left of the Ready*

- *Screen.*
- *When selecting Phantom mode, Phantom will be displayed on the top left of the Ready screen.*
- *Instructions for calibrating and correcting the scanner can be found in Appendix A*

7.5 The Scan Screen

Figure 7-11 displays a real time image in the pre-scan screen. The straight red line indicates the center of the area being scanned. The dark area represents urine and must cross over this central line in every scan. Do not press any keys during pre-scan.



Figure 7-11
Pre-Scan Screen

Once in the pre-scan screen, press the  key on the control panel or the smiling face button  on the probe to scan the bladder, see Figure 7-12.

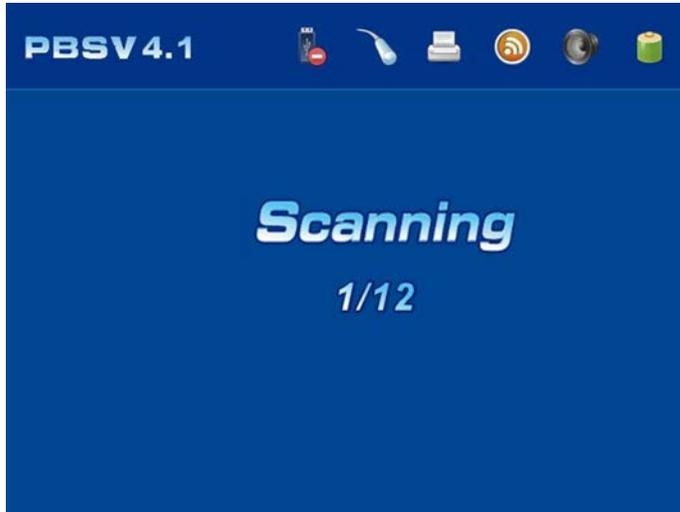


Figure 7-12
Scanning Screen

Do not disturb the device during scanning. The scan time is about 10 seconds for 12 images and 20 seconds for 24 images. The scanning progress will be displayed during the course of a scan (see Figure 7-12). “1/12” means it is scanning the first of 12 images. It will then enter “Calculating” screen after the scan is complete (see figure7-13). “Calculating... 12” mean it is scanning the twelfth image.

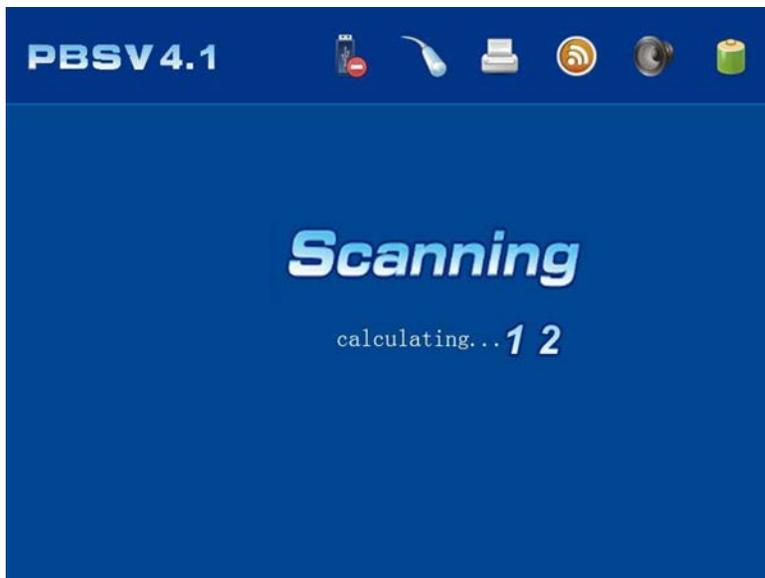


Figure 7-13
Calculating Screen

After calculation, the system will enter the “Scan Results.”

7.6 The Scan Results Screen

The scan results screen displays the results of the scan as shown in Figure 7-14.

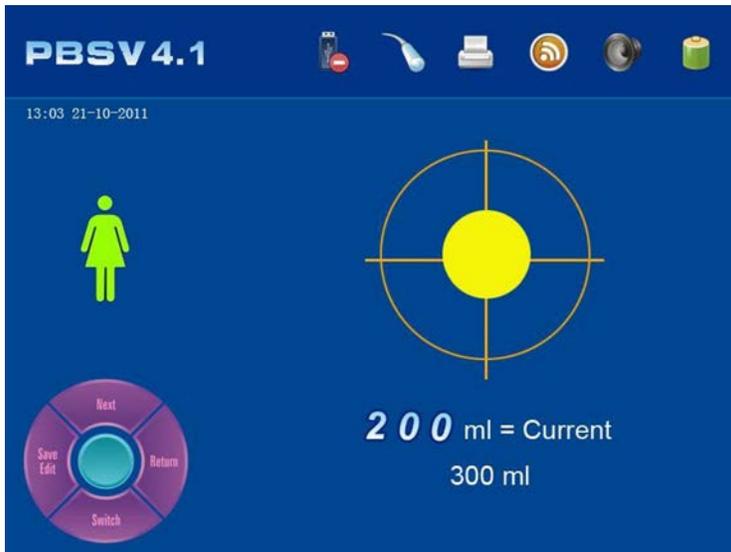


Figure 7-14
Scan Results Screen

1. Press  key or touch  to return to the ready screen
2. Press  key or touch  to enter the save/edit screen
3. Press  key or touch  to toggle between the before and after scanned results of 6 images
4. Press  key or touch  to switch between gray scale and 2 color images
5. Press  key to print the current image in 2 colors
6. Press  key to return to the pre-scan screen. Then scan by pressing the scan key again.

Note: When in pediatric mode  mode, if the current measured results are >100ml, invalid measurement will display on the screen. If this occurs, you need to change the mode to  (male) or  (female) and scan again. Please select correct mode to measure before starting pre-scan.

7.7 Gray Scale Results Screen

Press ▲ key to enter the gray scale results screen. There are two settings in 'Scan Page' of the system. If 12 images were chosen, they will be display on two pages with 6 images on each page. If 24 images were chosen, they will be displayed on 4 pages with 6 images on each page. Figure 7-15 displays what the gray scale results screen looks like.

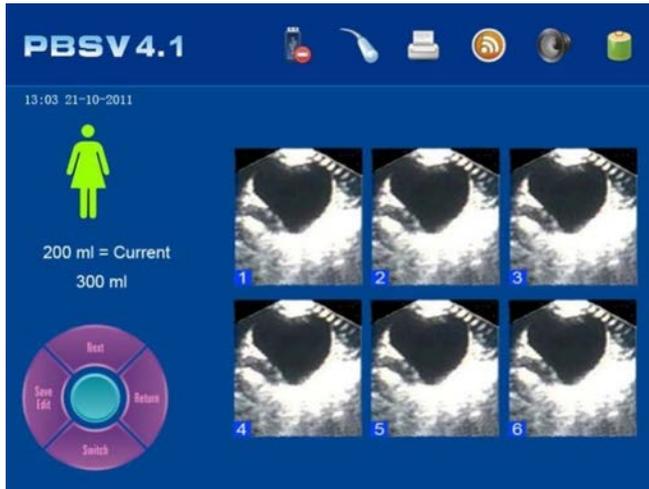


Figure 7-15
Gray Scale Results
Screen

How to operate in the Gray Scale Results Screen:

1. Press ► key or touch  to return to the Ready screen
2. Press ◀ key or touch  to enter the Save/Edit screen
3. Press ▲ key or touch  to switch to the next page of 6 screens
4. Press ▼ key or touch  to switch between gray scale and 2-color images
5. Press print  key to print the current image
6. Press scan  key to enter pre-scan mode. Then press the scan  or the  on the probe to perform the next scan

Note: It is suggested to press ▼ or touch  to confirm that the red line and the edge of the dark area in fluid are consistent (see Figure 7-16). If the deviation is too large, press the scan  or the  on the probe to rescan.



Figure 7-16
Scanned 2-Color Images
Screen

7.8 The Save/Edit Screen

Figure 7-17 show the save/edit screen view.



Figure 7-17
Save/Edit Screen

Press ◀ key or touch  to enter Save/Edit screen from the Scanned Results screen.

How to operate in the Saved/Edit Screen with functional keys:

1. Press the ▶, ◀, ▲, ▼ keys to choose the character needed. The selected character key will turn white on the screen

2. Press the OK  key to confirm the selected character
3. Press the ← key before pressing  to delete an unwanted character
4. Press the print  key to save and return to Ready screen
5. Press the scan  key to delete the saved information and return to the Ready screen

How to operate in the Saved/Edit Screen with touch screen keys:

1. Use a stylist pen to select the desired character
2. Press Print to save the record, it will take about 4 seconds. Do not press any keys while the record is being saved
3. Memory icon displays the remaining number of readings that may be stored.

Note: Patient ID may be up to 12 characters. Gender does not need to be input, gender is set in the Ready screen. Each patient ID is unique and specific to one individual. The same patient ID cannot be used for multiple patients.

7.9 Browsing Historical Records

Users can browse through stored records in the Browse Historical Records screen (see Figure 7-18). Each record includes the measurement result, the patient's information and their scanned image in gray scale. User can select current record to print or delete. After deleting the most current record, the next one in order will be the current record.

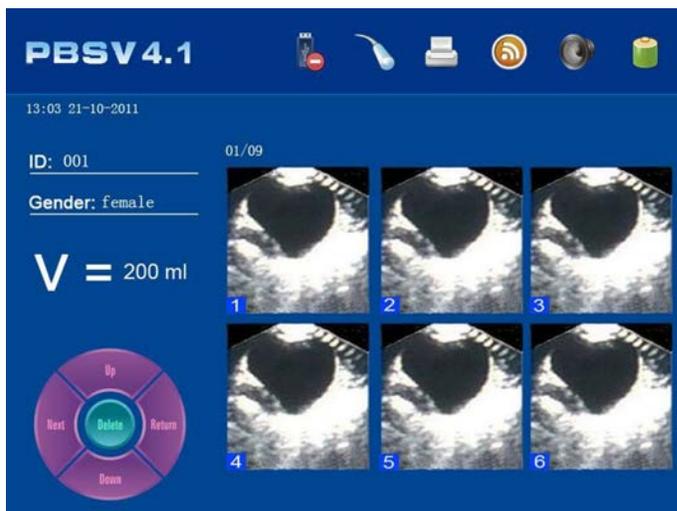


Figure 7-18
Browse Historical
Records Screen

1. Press ◀ key or touch  to enter the browse historical records screen.

- a. The screen will display the patient's ID, sex and measured volume record. The images from past scans will be displayed on the right under the patient ID. There will be 6 images displayed per screen.
 - b. For example, if "This is 01/09" is displayed, "01" means this is one of nine total records that have been stored
2. Press the  key to print the current record being displayed
 3. Press the ▲ key or touch  to display the previous record
 4. Press the ▼ key or touch  to display next record
 5. Press the ◀ key or touch  to display the next 6 images
 6. Press the ▶ key or touch  to return the **Ready** screen
 7. Press the  key or touch  to delete the current record

7.9.1 Deleting Stored Records

1. Press the ▲/ ▼ keys or touch Format
2. Press the  key or touch NO
3. Press the ▲/ ▼ keys or touch YES/NO again and toggle between YES/NO.
4. When display reads "YES", press the  key or touch Format again to delete all measured records

7.9.2 Location Presentation

When taking a measurement, the nurse should put the **scan key of the probe face to the head of patient**. After scan and measurement, if the picture of the bladder is located in the center of the coordinate, the scanned result is ok; if the picture of bladder is not in the center of the coordinate (see Figure7-19 through 7-26), you will need to rescan and adjust the measurement according to the operating instructions

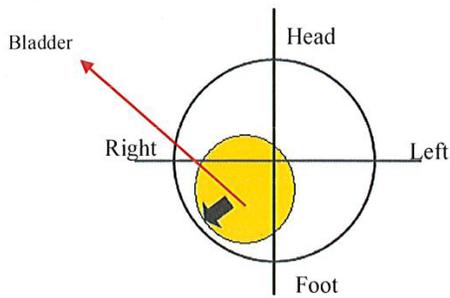


Figure 7-19

Move probe to lower right

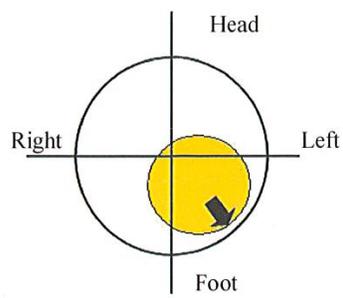


Figure 7-20

Move the probe to lower left

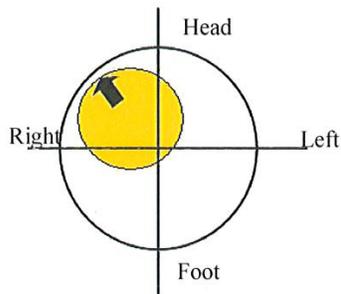


Figure 7-21

Move the probe to up right

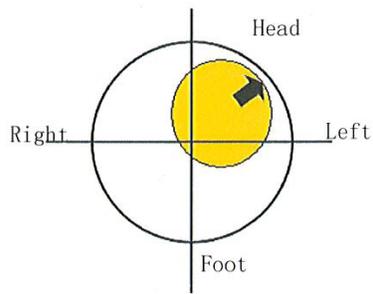


Figure 7-22

Move the probe to up left

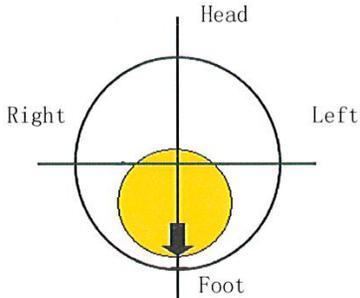


Figure 7-23

Move the probe towards patient's feet.

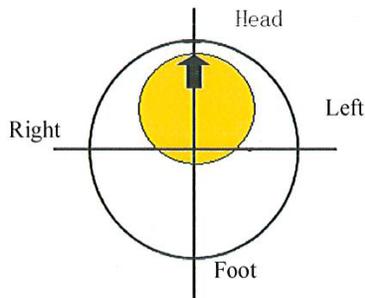


Figure 7-24

Move the probe towards patient's head

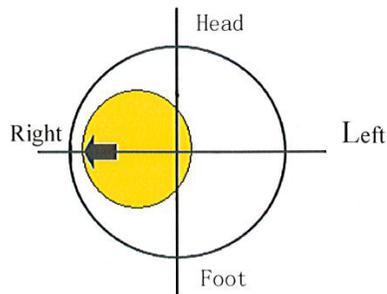


Figure 7-25

Move the probe to right

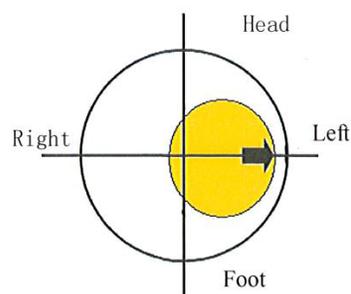


Figure 7-26

Move the probe to left

7.10 Printing Results

Scanned result can be printed immediately after performing a scan or recalled from the devices memory (if they were saved). User may select how many images to print on the **Print Page** (see Figure 7-27).

RESULT OF PBS
NAME: anonymous
GENDER: phantom
HOSPITAL: 00
DATE: 03/11/2010 11:22

Volume: 126ml



Figure 7-27
Two Images Printed

7.11 Exporting Data to a PC

You may export the historical data saved in the system to your PC using a USB connection. The system will also build a folder for each user. The same of the folder should correspond with the patient's ID. Only gray scale images and a copy of saved readings will be saved in each folder.

Instructions for establishing a USB connection:

1. Turn on the device and connect it to a PC using a USB cable
2. Check the status bar and confirm that  is displayed. If this symbol is displayed, it indicates that the connection is Ok. If the  icon is displayed, it indicates that the connection is not Ok. After connecting the device to a PC, the readings may be saved onto a CD or stored internally on the computer.

After the data has been successfully exported to a PC, turn off the device before disconnecting the USB cable.

Note: Measurements should not be taken while the device is connected to a PC. The copied file only can be viewed using picViewer.exe (See Figure7-28). Click on the folder icon in the toolbar and open the images to be viewed. Use left and right direction keys to toggle between images. Please contact Life Systems Inc. if you have and problems exporting, saving or viewing historical records.

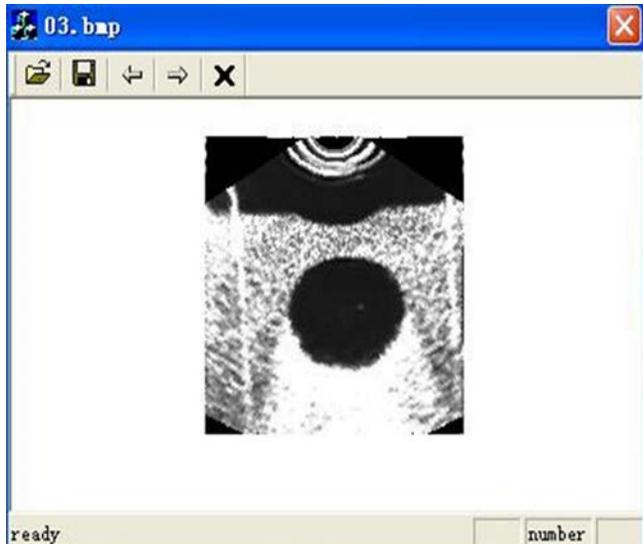


Figure 7-28
PC View

Thermal Printer

Instructions for changing the printer paper

1. Open the lid of printer as shown in Figure 7-29
2. Put the paper into paper box as shown in Figure 7-30
3. Pull out the paper about 5cm as shown in Figure 7-31
4. Close the cover of printer as shown in Figure 7-32

Note: *The printer will not work if paper is inserted in the wrong direction. Paper dimension is 30x57mm.*



Figure 7-29

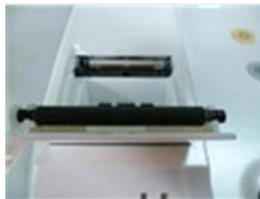


Figure 7-30



Figure 7-31



Figure 7-32

7.12 Recharging the Battery

The battery charger consists of power adaptor and charger.

1. Put the battery into the charger and ensure it is placed properly. The connection between battery plug and the socket of the charger must be correct.
2. Connect the charger with power supply before charging the battery. The red light will turn on when the battery has begins charging.

3. The green light will turn on after the battery is fully charged (It is suggested to charge the battery for at least 2 hours).
4. Disconnect the power between power supply and charger before removing the battery.

Note: Do not leave the charger unsupervised while the battery is charging. Do not place the battery, adapter or charger near fire, heat, flammable and explosive materials. All components meet the requirements of GB9706.1-2007.

Clinical Application

8. Clinical Application

8.1 Instructions for Use in a Clinical Setting

1. Patient should be in supine posture when a scan is performed.
2. Ultrasonic gel should be evenly applied to the acoustic window of the probe.
3. The probe should be placed on the human abdomen at about 2 cm above the pelvic bone. The probe should be slightly sloped towards to the head of patient, facing the direction of the bladder.
4. Press the SCAN key to enter pre-scan mode. Move the probe to find the largest concentration of dark (fluid) area of the bladder.
5. Press SCAN again to take measurement. Press the ▼ key to see if both edges of the red line and the dark fluid area in the gray scale image are overlapped. If the deviation is too large, rescan the patient's bladder by pressing the SCAN key. Please refer to Figures 8-1 to 8-10 for examples of invalid scan results (deviation between red line and dark fluid too large)
6. During scan, make sure to apply enough pressure with the probe to ensure that it is in full contact with the patient's abdomen (especially if ultrasonic gel is used).
7. Make sure the patient's posture remains stable throughout the scan. The patient should not speak or flex their abdominal muscles.

8.2 Images of Invalid Scan Results

The following images (see Figure 8-1 to 8-10) are examples where the red edge and dark fluid area do not coincide. These are invalid scans and require the patient to be rescanned.

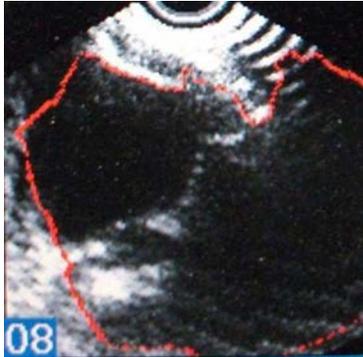


Figure 8-1. Too large red edge line



Figure 8-2 Too large red edge line

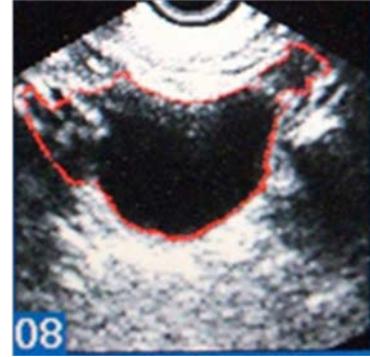


Figure 8-3 Too large red edge line



Figure 8-4 Too small red line

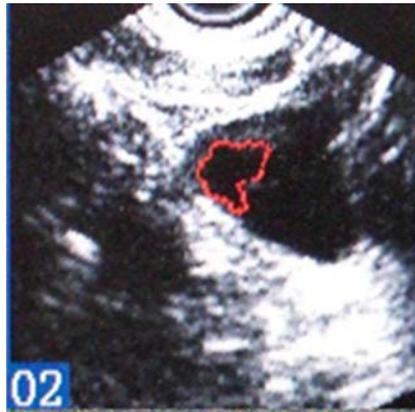


Figure 8-5 Too small red line



Figure 8-6 Red line is not in dark fluid area



Figure 8-7 Non-fluid area



Figure 8-8 Non-fluid area

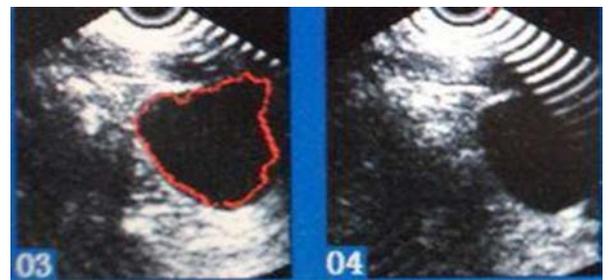


Figure8-9 Normal red line Figure 8-10 Non-red line

Note: If red edge is too large, then its volume will be too high, if red edge is too small, then its volume will be small, if red edge is not in the dark fluid area, its measured result will be small, and if the red edge is not in the dark fluid area, its scanned results will be ineffective.

Transportation and Storage

9. Transportation and Storage

9.1 Transportation

The marks on the device packaging case conform to the requirement of GB/T191-2008 <Packaging and Storage Illustrated Marks>. Avoid rain, snow, inversion and collision during transportation.

9.2 Storage

Do not store the device for more than six months without removing it from its case and turning it on for at least 4 hours. Do not place heavy objects on top of the device. The device should be stored in a dry and ventilated environment, away from strong sunlight and corrosion from causticity gas.

Note: See 5.2 for details about storage condition requirements.

Maintenance and Service

10. Maintenance and Service

10.1 Maintenance for Probe

1. The probe is very fragile. It should not be dropped or struck against any other objects. Please turn off the scanning key and put it into the case for probe, during paused of scanning. The acoustic window of the probe is especially sensitive. It is imperative that it not be damaged. Even a small scratch will impair scan results and reduce its life.
2. The probe is not made of a water-tight material. Its degree of protection into liquid is : IP×4;
3. In order to avoid probe or its protective material be from being corroded, do not expose it to any conductive or corrosive material.
4. Please check for any crack on the lens material of acoustic window of the probe and its plastic part before use. If there is, contact your local distributor immediately. Before use, also check the probe cord for any cracks or breakage. If either is discovered, discontinue use and contact Life Systems Inc. immediately.
5. Please use an ultrasonic gel with a neutral PH balance.
6. Softly scrub and sterilize the probe after use before it is put back in its case.
7. Do not take the probe off freely once it has been connected to the device. This will interfere with the connection.

10.2 Main Unit Maintenance

Clean the shell with soft and wet cotton cloth. Never use any chemical cleaners to clean (such as hydrochloric acid or bleach). Regularly check the visible part of device for any damages or defects. If any are discovered, please contact Life Systems, Inc. immediately.

1. Clean the probe using a soft cloth and alcohol (or an approved medical cleanser)
2. Dry the probe with a soft cloth before returning it to its case.
3. If the device needs to be disinfected, use a soft cloth in conjunction with a medical cleanser such as Cidex®, Cidex 7®, Sporidicin®.
4. d. The probe should also be cleansed thoroughly using a wet cloth and an approved medical cleanser before every use

10.3 Service Guide

If there is ever a problem with the device, first check all of the connections and confirm that the battery is adequately charged. If the problem persists, please contact Life Systems, Inc. and provide the following information:

1. Model of the device and its configuration;

2. SN of the device and the date of delivery;
3. Detailed problem description displayed on the screen.
4. The devices maintenance history, including the results of past repairs

Service will be provided free of charge if the device is still under warranty. The device has an estimated life of 6 years (excluding the battery). When the device is ready to be disposed of, please contact Life Systems, Inc. for instructions.

Troubleshooting

11. Troubleshooting

1. Ensure that the battery is inserted properly and has an adequate charge
2. Ensure that the connection between the device and the probe is secure
3. Check the printer's connection and confirm that paper is inserted in the right direction.
4. If problems persist, please contact Life Systems Inc.

Quick Reference Guide

12. Quick Reference Guide

- 1. Power On**

- 2. Select Scanning Mode – Select - Male, Female or Pediatric <60 lbs**

- 3. With Resident Supine – Apply gel (with no air bubbles) to midline approximately 1 inch (2-3 cm) above pubic symphysis**

- 4. Orient the probe per the pictures on the probe head. (feet and head)**

- 5. Aim towards the Bladder – a gentle tilt towards the coccyx.**

- 6. Press the scan button once – ensure that the red line is center in bladder. If not reposition**

- 7. Press scan again and print if necessary.**

- 8. Results will be displayed in ml.**

Appendix A: RosieScan Calibration

Appendix A

A1. Fix the probe on a holder. Make sure that the probe is placed vertically on the center of acoustic window of the phantom (standard phantom 128ml) See Figure A-1.

**Note: If you are using a phantom that was not manufactured by the manufacturer of this device, please check the standard volume of your phantom before calibration*

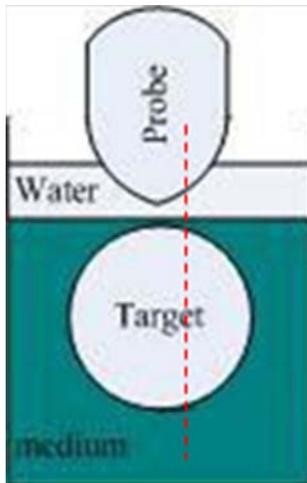


Figure A-1
The Place of Probe

A2. Please press "MENU/OK" to enter the system setting screen. (Figure A-2)



Figure A-2
System Setting Screen

After switching on the device, enter the system reset screen by selecting Phantom in the Scan Mode before returning to Ready screen.

A3. Enter pre-scan state by pressing SCAN key (FigureA-3).

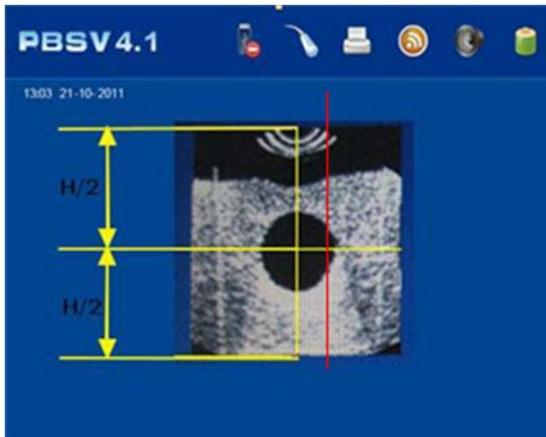


Figure A-3
Pre-Scan Image

A4. Adjust the distance between probe and phantom; let the round dark area (target) be in the middle of fan-image in pre-scan. The red line shown in Figure A-3 is the central line of the measurement. The object being measured must be across this center line

A5. Enter the Scan screen (see Figure A-4) by pressing the SCAN key



Figure A-4
Clear Two Color Images After Scan

A6. Take 5 separate measurements of the phantom to confirm the accuracy of the scanner. Calibration is not necessary if results are within 128 ± 10 ml. (or the standard volume for the phantom being used)

A7. If the measured result is not within $128 \pm 10 \text{ml}$, calibration is necessary. To begin, take 5 readings of the phantom and average the results. This will be the "mean value" that is to be entered in this formula: $128 \text{ (or standard value of phantom being used) } - \text{mean value} = \text{calibrating value}$

A8. Select Calibration in the system reset screen for the device calibration.

A9. Phantom Parameter:

Phantom Parameter:

- Sound speed of background material : $1540 \text{m/s} \pm 10 \text{m/s}, 23 \pm 3^\circ\text{C}$
- Attenuation of background material : $0.7 \text{dB}/(\text{cm} \cdot \text{MHz}) \pm 0.05 \text{dB}/(\text{cm} \cdot \text{MHz}), 23 \pm 3^\circ\text{C}$
- Protective plate : $\leq 0.4 \text{mm}$
- The internal target parameter of the Phantom
- Weight: $1322 \text{g} \pm 0.1 \text{g}$
- Density: $1021 \text{kg/m}^3 \pm 1 \text{kg/m}^3$
- Attenuation at 3MHz: $0.03 \text{dB}/(\text{cm} \cdot \text{MHz}) \pm 0.02 \text{dB}/(\text{cm} \cdot \text{MHz})$
- Sound speed at 23°C : $1547 \text{m/s} \pm 1 \text{m/s}$

Appendix B: Battery Instructions

Appendix B

B.1 Warnings

- Do not allow the battery to come into contact with sharp objects.
- The battery should be stored in a clean environment away from sharp objects
- The battery should not be perforated for any reason
- The battery should not be transported or stored with any other metallic objects
- The battery should not be dropped or have strong contact with any other objects
- Keep the battery away from all objects listed on the battery and charger
- Only use a battery manufactured for this device and supplied by your local distributor.
- Do not combine old and new batteries in the same device
- Only a charger produced for this device by the manufacturer and supplied by your local manufacturer may be used. **DO NOT LEAVE UNATTENDED WHILE CHARGING**
- Any electrical short circuit will result in serious damage to the battery. Under no circumstances should metal objects come in to contact with the positive and negative terminals on the battery!
- Do not use the battery if:
 - a. The battery(s) found to hot, emitting strange or unfamiliar odors, discolored, deformed or other anomalies should appear
 - b. Discontinue use immediately if battery is deformed due to a drop, bump or any other form of impact.

B.2 Recharging and Discharging

Recharging current and voltage is not allowed to exceed the standards listed below. Failure to comply may result in damage to the mechanical performance and/or security risks. Noncompliance to the below recharging standard may also result in heat and leakage.

- The charger must be used with constant current and voltage.
- The current of a single battery must be below 1C while recharging.
- The range of temperature must be 0~ 45°C (32F~113F) while recharging.
- The voltage must be less than 9.4V while recharging the battery

Note - Special caution: Immediately disconnect power supply if the battery is found to expand, smoke or break during charging. Do not touch the battery for at least four hours and NEVER use it again. Dispose of the battery immediately after it is safe to touch and removed from the charger.

Discharge current should never exceed the following standards:

- The discharge current must be less than 1C.

- The temperature range is -20~60°C (-4F~ 140F) during discharging.
- The terminated voltage must be more than 5.6V when the battery is fully discharged.

B.3 Storage

- The environment temperature for Long time storage: -5~40 °C (23F~ 104F)
- Relative humidity: <80%
- The battery needs to be recharged and discharged every 90 days if it is to be stored for an extended period of time. See 7.15 for recharging instructions.

Note: Battery should not be charged more than 50% if it will be stored for 1 year or more.

B.4 Transportation

Battery should not have more than a 50% charge during transportation.

B.5 Other Instructions

Please read the following precautions before using the battery.

- Do not disassemble the battery in any way
- Do not immerse the probe in water. It should be dried immediately after cleaning
- The battery should never be in close proximity to any heat source i.e. fire and heater
- Do not heat the battery or allow it to come in to contact with fire
- Do not perform any soldering directly on the battery
- Do not recharge next to fire or any other heat source
- Do not place the battery in a microwave or high pressure container
- Do not place the battery in any environment with a temperature above 60C (140F) or in contact with direct sunlight
- In theory, there is no flowing electrolyte in a polymer Lithium-ion battery. If any part of the battery leaks and comes into contact with your skin, wash the area thoroughly and contact your doctor immediately
- Do not use a battery that is damaged, deformed or suspect in ANY way
- Keep the battery away from fire. Especially if it leaks or smells of electrolyte
- Store in a cool, dry place away from children
- Keep the battery away from static fields at all times
- Failure to comply with any of the above mentioned precautions may lead to fire, explosion or a shortened battery life. The user will take the full responsibility for the consequences of misuse
- Do not casually dispose of batteries. Please consult and follow the proper procedure for disposing of Lithium-ion batteries. Recycle the battery by following the proper policy to avoid environment pollution.

Appendix C: Setting the Starting Location and Troubleshooting the Probe

Appendix C

There are 3 possible images on the screen in the pre-scanning.

C.1 Pre-Scan Image to the Left of Center

If the pre-scan image looks like Figure C-1, press the “-“button in the “Scanning Adjust” section in the system setting screen. Input the degree of $\angle a$, as Figure C-4 shows ($\angle a$ equals the center of the image between the red line’s degree).

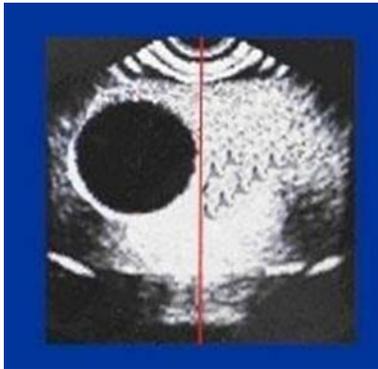


Figure C-1 Image center-left.

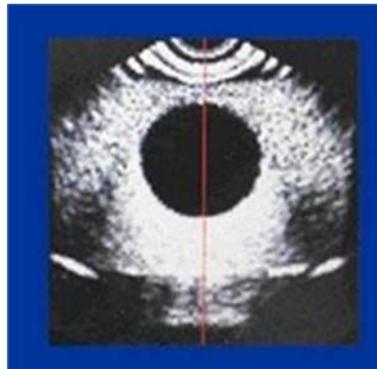


Figure C-2 Correct

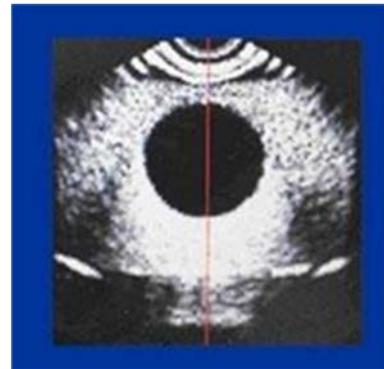


Figure C-3. Image center-right

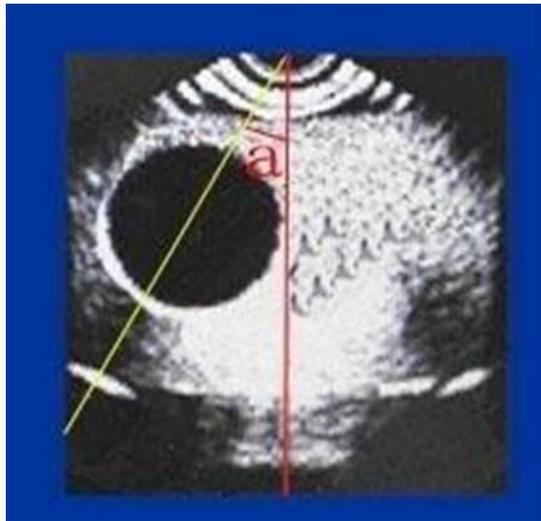


Figure C-4
 $\angle a$

Formula

- When $0^\circ < \angle a < 15^\circ$, then $0 < a < 10$
- When $15^\circ < \angle a < 30^\circ$, $10 < a < 20$;
- When $30^\circ < \angle a < 45^\circ$, $20 < a < 30$;
- When $45^\circ < \angle a < 60^\circ$, $30 < a < 40$;

Return to pre-scan mode to check the red line and the image's position. If it is still incorrect, please re-adjust until it is correct (as Figure C-2 shows).

C.2 Pre-Scan Image to the Right of Center

If the pre-scan image looks like Figure C-3, please press the "+" button in the "Scanning Adjust" section in the system setting screen. Input the degree of $\angle a$, as Figure C-5 shows ($\angle a$ equals the center of the image between the red line's degrees).

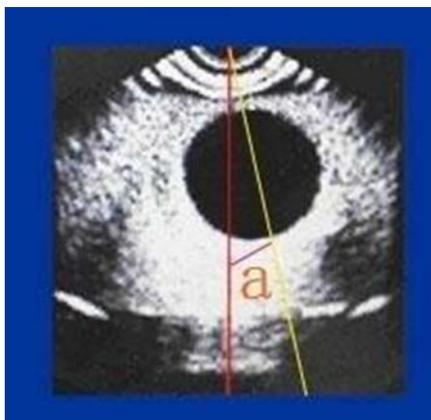


Figure C-5
 $\angle a$

Formula:

When $0^\circ < \angle a < 15^\circ$, then $0 < a < 10$

When $15^\circ < \angle a < 30^\circ$, $10 < a < 20$;

When $30^\circ < \angle a < 45^\circ$, $20 < a < 30$;

When $45^\circ < \angle a < 60^\circ$, $30 < a < 40$;

Return to pre-scan mode to check the red line and the image's position. If it is still incorrect, please re-adjust until it is correct (as Figure C-2 shows).

Appendix D: Guidance and Manufacturer's Declaration

Appendix D

D.1 Electromagnetic Emissions

Guidance and manufacturer's declaration: electromagnetic emissions		
The PBSV4.1 bladder scanner is intended for use in the electromagnetic environment specified below. The customer or the user of the SECP-II should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The PBSV4.1 bladder scanner uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The PBSV4.1 bladder scanner is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class B	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

D.2 Electromagnetic Immunity

Guidance and manufacturer's declaration: electromagnetic immunity			
<p>The PBSV4.1 bladder scanner is intended for use in the electromagnetic environment specified below.</p> <p>The customer or the user of the PBSV4.1 bladder scanner should assure that it is used in such an environment.</p>			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be made of wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT)	Main power quality should be that of a typical commercial or hospital environment. If the user of the PBSV4.1 bladder scanner requires continued operation during an
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency of magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<p>NOTE: UT is the a.c. main voltage prior to application of the test level.</p>			

Guidance and manufacturer's declaration: electromagnetic immunity

The PBSV4.1 bladder scanner is intended for use in the electromagnetic environment specified below. The customer or the user of the PBSV4.1 bladder scanner should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	Portable and mobile RF communications equipment should not be used close to any part of the PBSV4.1 bladder scanner, including cables (other than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.) Recommended separation distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz to 800MHz $d = 2.3 \sqrt{P}$ 800MHz to 2.5GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3V/m 80MHz to 2.5GHz	3V/m	

NOTE: At 80MHz and 800MHz, the higher frequency range applies.

NOTE: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the PBSV4.1 bladder scanner is used exceeds the applicable RF compliance level above, the PBSV4.1 bladder scanner should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the PBSV4.1 bladder scanner.

^b Over the frequency range 150kHz to 80MHz, field strengths should be less than 3V/m.

Recommended separation distances between portable and mobile RF communications equipment and the PBSV4.1 bladder scanner

The PBSV4.1 bladder scanner is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the PBSV4.1 bladder scanner can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the PBSV4.1 bladder scanner as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150kHz to 80MHz $d = 1.2 \sqrt{P}$	80MHz to 800MHz $d = 1.2 \sqrt{P}$	800MHz to 2.5GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE: At 80MHz and 800MHz, the separation distance for the higher frequency range applies.

NOTE: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Declaration:

All the pictures in this user manual are just for reference.