

**Advance
Mobility™**
EMPOWERING INDIVIDUALS

A Division of The Baby Jogger Company



Spirit Push Chair

This product is CE approved.

Assembly Instructions

Read these instructions carefully before use and keep them for future reference. Your passenger's safety may be affected if you do not follow these instructions.

For the sake of your precious cargo, please read these warnings before using your brand new Spirit Push Chair.



- **WARNING:** Prevent serious injury to your passenger by always using the safety harness.



- **WARNING:** Any load attached to the handle affects the stability of the push chair.



- **WARNING:** It may be dangerous to leave your passenger unattended.



- **WARNING:** Recommended weight for seat pocket is 2 pounds and basket is 10 pounds. Excessive weight may cause a hazardous unstable condition to exist.

- Accessories which are not approved by the manufacturer or distributor should not be used.
- Always use your wrist strap – this push chair can really roll, and the wrist strap gives you control if you should slip or fall down.
- The passenger should remain seated (*no standing in push chair*).
- Watch out for cars – never assume that a driver can see you.
- Intended for children aged 6 months and up.
- Total weight load recommended for this push chair is 75 lbs. (34 kg.) (combined passenger and baggage weight.)
- This push chair seats one passenger.
- It may be unsafe to use replacement parts other than those supplied or approved by the manufacturer.
- Do not run/jog with a baby under 6 months of age.

Frame



- 1** Unpack the push chair from the box and set wheels and footplate aside.



- 2** Unfold the push chair by lifting the front fork away from the handlebar until it clicks into place.




- 3** Turn stroller over to upright position.



Under Seat Basket

- 1** Using the two Velcro straps, secure the front end of the under seat basket to the frame just above the footplate holder slot.
- 2** Snap the rear basket straps around the frame.

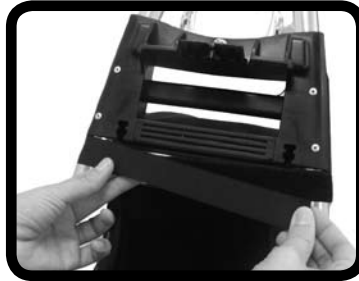

Footplate



- 1** Insert the footplate tab into the footplate holder slot.
- 2** Press down on footplate until it slides into place.

Front Wheel



1 To install, squeeze the two brake calipers together and push the black lever backwards to open brake pads. Release calipers so that they spring open.

2 Insert the front wheel into the fork tips by aligning the axle flats with the fork tip flats.



3 Tighten by turning the wheel lever clockwise while holding the opposite side.



4 Close the wheel lever by pushing it flush against the tubing. Be sure not to over tighten – light pressure is needed to close the wheel lever. You should not strain to close.




5 Squeeze the two brake calipers together against the metal rim of the front wheel and flip the black lever forwards to close brake pads.

Rear Wheels



1 To install, slide the axle peg of the rear wheel into the axle until you hear it click into place. Complete on both sides.

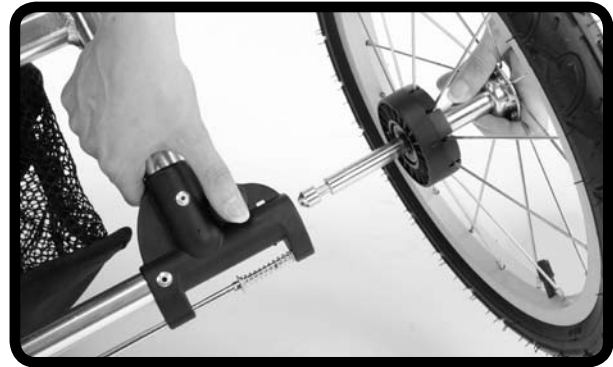

2 To remove, push the round button on the rear axle while pulling out the wheel.

Rear Brake



WARNING: Engage parking brake before loading and unloading passenger.



1 Press down on the brake pedal to engage rear wheel parking brake.



2 To release, flip the lever upwards.

Seat



- 1.** Start with snapping seat to upper frame and work your way down frame.



- 2** At knee area of the seat, secure the seat to the frame by threading one side of the Velcro straps through the D-ring on the other.



- 3** Tighten and secure the Velcro together.



- 4** At foot area of the seat, secure the seat to the frame by threading one side of the Velcro straps through the D-ring on the other.



NOTE: *The seat of your jogger can be removed for washing. Unsnap the fabric from the frame and undo any Velcro straps. See page 15 for washing instructions.*

5 Tighten and secure around the frame.



6 Snap the seat strap around the support bar on the frame.

Safety Harness



1 Un-clip top of shoulder strap from the seat and re-clip on the pair of seat loops closest to the child's shoulder height.



2 Adjust the strap length by moving the plastic guide up or down to lengthen or shorten.



3 Buckle harness by sliding harness pegs into the buckle on the crotch strap until it locks in place.



4 To unbuckle, squeeze the sides of the buckle while pulling the two harness pegs out of the buckle.

Reclining Seat



- 1** Simply adjust seat straps to desired degree of recline by releasing the length of the strap.



- 2** Pull on strap when you want to return the seat to its upright position.



- 2** Push chair is packaged with seat support board and can be easily removed or inserted into the seat back if additional support is desired

NOTE: *Be sure to remove the support panel from the back of the seat before washing.*

Accessory Bracket Adapter



Accessory bracket adapter allows for easy use of many Baby Jogger brand accessories

Canopy



- 1** Your canopy is equipped with side ventilation panels that Velcro closed or can be rolled up and secured with a loop and toggle.



- 2** Clear view windows are provided at two positions of the canopy. Roll up the window flap and secure with the Velcro tabs.



NOTE: *The canopy of your push chair can be removed for washing. Using a Phillips head screwdriver, unscrew the canopy brackets from the frame and remove the canopy fabric from the metal supports.*

Quick-Fold



WARNING: Always be sure the safety strap is securely snapped onto the frame on both sides before using the push chair.



- 1** Thread the fold strap through the opening in the seat.



- 2** Attach the folding strap to the trigger underneath the seat by sliding the open end of the triangle shaped fastener into the hole on the trigger.



- 3** Close the openings of the triangle shaped fastener by rotating the attached nut until it tightens in place. Repeat on both sides.




4 Snap the safety strap to the frame.

To fold:

- Unsnap the safety straps on the side of the frame. Lift up on the fold strap in the center of the seat until the push chair folds in half.



Hand Brake

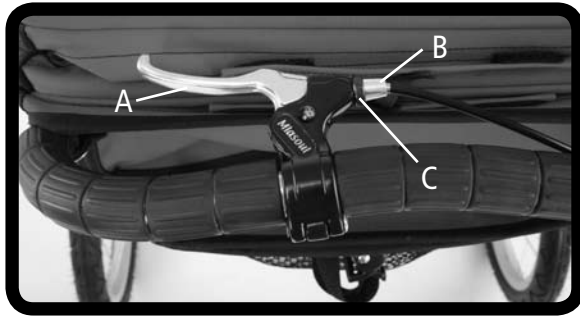

- Your stroller is equipped with a hand brake for your convenience.

Adjusting the hand brake:

- If you do not have good braking action, simply squeeze the brake handle (A) five or six times. If the problem persists, loosen the brake's cylinder ring (B) to adjust the small ring (C) and then move the small ring away from the handle to tighten the cable. (Please do not over tighten!) To lock the rings, tighten the cylinder ring (B) against the handle while holding the small ring (C) in place.



- 1** To use the hand brake make sure brake pads are properly installed (see page 3). Squeeze the hand brake to give you better control over your push chair.



- 2** To rotate the hand brake's position on the handle bar, loosen the bolt using the supplied Allen wrench. Rotate the hand brake assembly on the handlebar until it is in a comfortable position. Tighten the bolt to secure hand brake.

In Motion

Steering

- Steering your push chair is simple. It's balanced to turn with just a light touch on the handlebar.

Wrist Strap

- Make sure wrist strap is around your wrist when in use. The Spirit is ready to roll with or without you – and the wrist strap helps you stay in control of your push chair.

In the Dark

- Since we are concerned about safety, we recommend that you not use your push chair at night. However, if you choose to, please attach reflective strips to the wheel spokes and the seat. Bicycle lights can also be mounted on the handlebar.



Push Chair Care

Washing

- You can machine wash the Spirit's seat in cold water with a mild detergent. To minimize shrinkage, simply reattach the seat to the frame to dry. *(Please do not put the seat in the dryer!)*
- Be sure to remove the board from the back of the seat before washing.
- You may clean the frame using a soft, dry cloth, and an occasional spray of household wax will maintain its healthy glow.



Wheel Care

- Moisture – especially from the beach – can rust the wheel bearing case. Spraying regularly with a silicone lubricant will keep your wheels looking healthy.
- Also, if you use your push chair regularly, your push chair will benefit from a yearly visit to the bike shop for wheel alignment. Refer to the number on the tire wall for the optimum pressure for your tires.

Storing

- Sunshine can fade seat fabric and dry out tires, so storing your push chair indoors will prolong its good looks. However, if you do store it outside, using a tarp will protect against weather wear.

WARRANTY REGISTRATION ON-LINE AT www.advancemobility.org/register

Advance Mobility's Limited Lifetime Warranty:

Advance Mobility warrants that the frame is free of manufacturer defects for the lifetime of the product. Manufacturer's defects include but are not limited to the breaking of welds and frame tube damage. Textile or soft side materials and all other components shall be warranted for one year from the date of purchase (except for tubes and tires). Proof of purchase is required to make a warranty claim and the warranty is only extended to the original purchaser.

THIS WARRANTY DOES NOT COVER:

- Normal wear and tear
- Corrosion or rust
- Incorrect assembly or the installation of third party parts or accessories not compatible with the original design
- Accidental or intentional damage, abuse or neglect
- Commercial use
- Improper storage/care of the unit
- Accidents/injuries related to leaving the passenger unattended

This warranty specifically excludes claims for indirect, incidental, or consequential damages. The implied warranties of merchantability and fitness for a particular purpose is according to the specific warranty period for this model unit at time of purchase. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state. To know what your legal rights are in your state, consult your local or state consumer affairs office or your state's Attorney General.

E-MAIL: specialneeds@babyjogger.com

CALL: 1-800-241-1848 between 8am – 6pm EST, Mon. – Fri.

FAX: 1-804-262-6277

MAIL: ADVANCE MOBILITY
8575 Magellan Parkway
Suite 1000
Richmond, VA 23227

Thank You!

We wish you many happy journeys with your new push chair! Your push chair is a great product, and we are proud to manufacture it. If you have any questions or comments, please give us a call at:

1.800.241.1848

***Advance
Mobility***[™]
EMPOWERING INDIVIDUALS

Advance Mobility is a Division of Baby Jogger

8575 Magellan Parkway, Suite 1000 • Richmond, VA 23227
T: 800.241.1848 • F: 804.262.6277
www.advancemobility.org

