



270 Washington Street, Mount Vernon, New York 10553

Tel: 845-205-6004 Fax: 845-205-6005

www.proactivemedical.com

855-BE-PROACTIVE (855-237-7622)

Alternating Pressure Pump & Overlay Mattress User Manual

Standard Features

- ◆ LED on/off switch and pressure adjustable knob.
- ◆ Fixed book and air filter are provided on the bottom of pump.
- ◆ Waterproof, vapor permeable cover provides a low friction surface.
- ◆ A/B alternating provides pressure relief.


Product Functions

Pump

The functions of the Protekt Aire 2000 pump are described below. Please refer to the figures of the Protekt Aire 2000 pump.

- ✚ Power Switch(1)
 - The switch is at the right side of the pumps.
 - Turn ON/OFF the power, the pump will start/stop operation.
- ✚ Pressure adjust knob(2)
 - Turn the soft/firm knob to set a comfortable pressure level.
- ✚ Normal Pressure Indicator(3)
 - A visible indicator (green) tells the pressure has reached a preset or user-defined level.
- ✚ Low Pressure Indicator(4)
 - A visible indicator (yellow or red) warns the pressure is below a preset or user-defined level.
- ✚ Static Control(5)
 - Press to set the air mattress in static mode.

1. Power Switch
2. Pressure Adjust Knob
3. Normal Pressure Indicator
4. Low Pressure Indicator
5. Static Control

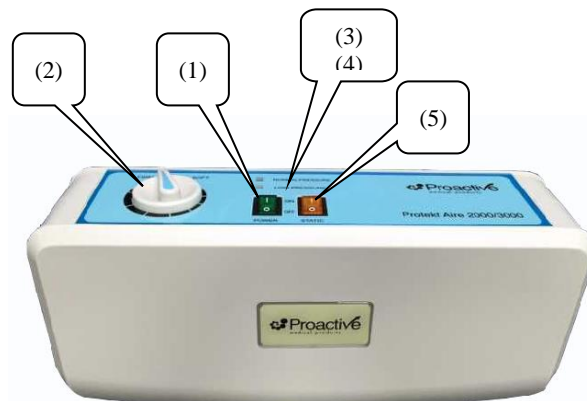



270 Washington Street, Mount Vernon, New York 10553
 Tel: 845-205-6004 Fax: 845-205-6005
www.proactivemedical.com
 855-BE-PROACTIVE (855-237-7622)

Operating Instructions

1. Position pump on horizontal surface or hang it by means of the hook.
2. Open mattress on the bed. Secure it with end flaps or straps. Air tubes must be at foot end of bed.
3. Connect air feed tubes to air outlet on the mattress and pump, ensuring that the tubes are not kinked or twisted. Allow space for bed rails to drop freely.
4. Plug the pump into a suitable electrical power source and switch on. The switch indicator should now illuminate.
5. Allow 20 minutes for mattress to inflate fully.
6. Cover with a sheet and tuck loosely.
7. Please use pressure adjustable knob to give maximum patient comfort.

Technical Specifications

Pump	Mattress
Item: Protekt Aire 2000	Item: Protekt Aire 2000
Power supply: 120V/60Hz	Size: 80" (L) x 35" (W) x 5" (H)
Air output: 8 liter/min.	Top cover: Nylon / PU with button
Pressure range: 40 mmHg – 100 mmHg	Air cells: Nylon / PVC with non skid bottom
Cycle time: 10 min.	Base: Nylon / PVC
Power consumption: 9W	18 air cells
With normal / low pressure light	With low air loss
Fuse: 1A	
Size: 12" (L) x 6" (W) x 4" (H)	
Weight: 3 Lbs	

Cleaning Guidelines

- ◆ Pad can be cleaned by disinfectant solution.
- ◆ Do not immerse the pump in water.
- ◆ Do not heat or steam in autoclave.

Warranty for Pump and Mattress: 18 Months