

CLINICIAN QUICK GUIDE FOR SETTINGS LOCK LEVELS

This reference will help you understand the lock settings or security levels of the Curlin Medical Infusion Pumps. For more detailed information, please refer to the User's Manual issued with the pump.

The security lock settings can be set from the Biomed Setup Menu or from each therapy Options Menu. The pump programmer will set the lock level according to the prescription access allowed for the intended patient/caregiver. A higher lock level means more restrictions (less access to functions of the pump).

1. Lock setting "OFF" allows the user access to all pump functions.
2. Lock level 1 (one) allows the user to make changes in the prescription fields and is often used in conjunction with the Titration feature.
3. Lock level 2 (two) allows the user to repeat the prescribed therapy and to reconfirm the prescription settings, but will not allow changes to be made to the prescription. This allows a user the ability to independently change IV bags and to repeat the same prescription for that IV bag, but prevents any changes in the prescription as a safety precaution for the patient.
4. Lock level 3 (three) requires that a trained clinician be present to change the IV bag. The user is allowed to pause the therapy and resume it again but not to use the Repeat Rx function to change an IV bag.

If the pump has a programmed security setting of 1, 2 or 3, an access code will be required to change the security setting or to access "locked" functions. These settings are designed to give added protection and security to the patient receiving a therapy while allowing as much independence as deemed safely possible.

Lock Setting Table					
Pump Operation	Active Therapy	Lock OFF	Lock 1	Lock 2	Lock 3
Change Lock Settings	All	All settings with correct access code			
Turn Pump Off/On	All	Yes	Yes	Yes	Yes
Pause/Run Therapy	All	Yes	Yes	Yes	Yes
Access Help/Options	All	Yes	Yes	Yes	Yes
Check Power Source	All	Yes	Yes	Yes	Yes
Resume Therapy	All	Yes	Yes	Yes	Yes
View Rx at "No to Review"	All	Yes	Yes	Yes	Yes
Repeat Therapy	All	Yes	Yes	Yes	No
Prime Pump	All	Yes	Yes	Yes	No
Change Audio Level	All	Yes	Yes	Yes	No
Change Rx screen information	All	Yes	Yes	No	No
Change Rx screen at "No to Review"	All	Yes	Yes	No	No
Access New Program	All	Yes	No	No	No
Change Air in Line Setting	All	Yes	No	No	No
Change Down Pressure	All	Yes	No	No	No
Change KVO Rate	Cont, TPN, INT, VAR	Yes	Yes	No	No
Delay Start	Cont, TPN, INT, VAR	Yes	No	No	No
Change Units (New Program)	Cont, PCA, INT, VAR	Yes	No	No	No
Change Concentration (New Program)	Cont, PCA, INT, VAR	Yes	No	No	No
Utilize Titration Option	Cont, PCA	Yes	Yes	No	No
Program Titration Limits	Cont, PCA	Yes	No	No	No
Change Administration Route (New Program)	PCA	Yes	No	No	No
Program Loading Dose	PCA	Yes	No	No	No
Program Clinician Dose	PCA	Yes	No	No	No
Access Early Down Ramp	TPN	Yes	Yes	Yes	Yes
Change Next Dose Time	INT	Yes	Yes	No	No