

# EZee Life™

## Raised Toilet Seat with Flip Back Arms & Lid

Model: CH2024 – 2”

Model: CH2027 - 4”



EZee Life™ Products  
34 Futurity Gate, Unit 15  
Concord, Ontario L4K 1S6



## Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

## Specifications:

Seat Height:	CH2024 – 2”	CH2027 - 4”
Seat width:	16”	
Seat depth:	16”	
Construction:	Moulded plastic	
Weight capacity:	220 lbs	
Product weight:	CH 2024 - 7 lbs	CH2027 - 8lbs

## Features:

- Padded flip back arms
- Molded plastic construction
- Gently contoured surface for comfort
- Stain and odour resistant
- Locks securely onto toilet
- Strong and durable design
- No tools required for installation

## Installation:

1. Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.
2. Lift toilet lid and seat assembly and place rear wings of toilet seat around rim of toilet bowl. (Some toilet bowls may require that seat assembly be removed to allow proper fit.) In addition the front adjustment bracket must clear the rim in order for the raised toilet seat to fit securely.
3. With raised toilet seat resting correctly on rim, straighten and center product over toilet bowl as necessary. Turn adjusting knob clockwise to tighten product securely in place. Hand tighten only - use no tools, and be careful not to overtighten, as threads on adjustment bracket might strip.
4. Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
5. If you have a style of commode that this product will not fit properly, or cannot be tightened securely as noted above, it is recommended that you discontinue use.
6. This product has a maximum user weight of 220 lbs.

## Removal Instructions

1. Rotate adjusting knob counter clockwise until seat is loose.
2. Lift front of raised toilet seat and pull forward to move.

## Flip Back Arms

This EZee Life raised toilet seat features flip back arms that allow a caregiver or the user to flip the arms back out of the way for transfers or hygiene purposes.

To flip the arms back ,simply lift the front of the arms up and direct back.

To replace the arms, fold the arms back down forward until they sit in the arm grooves moulded into the seat base.





## Care and Cleaning

- All parts can be cleaned with soap and hot water (up to 195°F / 90°C).
- Use a non-abrasive detergent or household cleaner with warm water.
- Rinse with warm water and wipe dry.
- **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may ensue.

## Cautions

- Do not hold onto the seat as a support when getting on and off the seat.
- Always be sure that the seat is correctly and securely locked in place before using.
- Be sure that the adjustment knob in front of the seat is tight at all times, thus securing the seat to the toilet bowl.
- This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
- On models with arm rests: The arms are for the purpose of providing assistance when getting on and off the seat. **DO NOT** try to use the arms to support full body weight during transfer, as this may result in failure of the unit.

### Imported / Distributed by:

Factory Direct Medical  
34 Futurity Gate, Unit # 15  
Concord Ontario L4K 1S6  
Local: 416-739-8393  
Toll Free: 855-235-2400