

Evolutionary solution for fat reduction

# Lipo Slim

**SE-C80**

**User Manual**



[Http://www.SPALife.com](http://www.SPALife.com)

# I . Structure of Liposlim

---

## I -1. General Structure



∅ 63 applicator connector

∅ 38 applicator connector

# I . Structure of Liposlim

---

## I -2. Description of Applicators



- $\phi$  63 Applicator
- Used to treat larger and wider area
- Red LED



- $\phi$  38 Applicator
- Used to treat smaller and targeted area
- Red LED

# I . Structure of Liposlim

---

## I -3. Operating Board



A :  $\phi$  63

B :  $\phi$  38


---




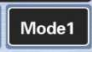
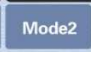
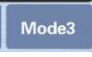




A. Big : Used to treat large area

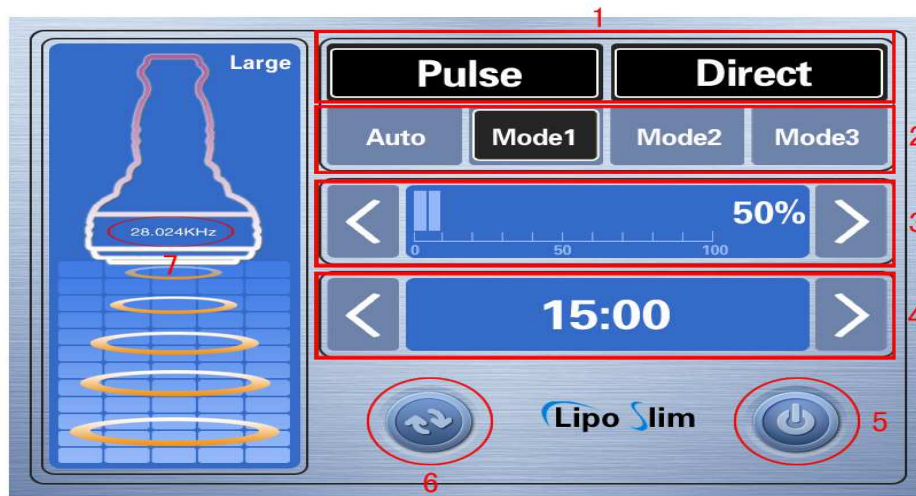B. Small : Used to treat narrow area

# I . Structure of Liposlim

## I -3. Operating Board



- ① Pulse/ Direct mode :  (shot) or  (continuous) mode
- ② Pulse button :      
Select the pulse (4 modes)
- ③ Power Level control button :   
Set the power level (Preset level :50%)
- ④ Timer control button :   
Set the treatment time (Preset time : 15 min.)
- ⑤ Start button 
- ⑥ Applicator conversion button :   
Change the applicator (Large to medium)
- ⑦ Frequency : Show the frequency

## II. Operation

---

### II-1 : Installation



1. Hold φ 63 applicator on the holder



2. Connect the applicator cable to the upper connector



3. Hold φ 38 applicator on the holder



4. Connect the applicator cable to the lower connector



5. Connect power cable to AC Inlet


## II. Operation

### II-2 : Operation

1. Select the hand-piece (  $\phi$  63 or  $\phi$  38)



2. Select the m

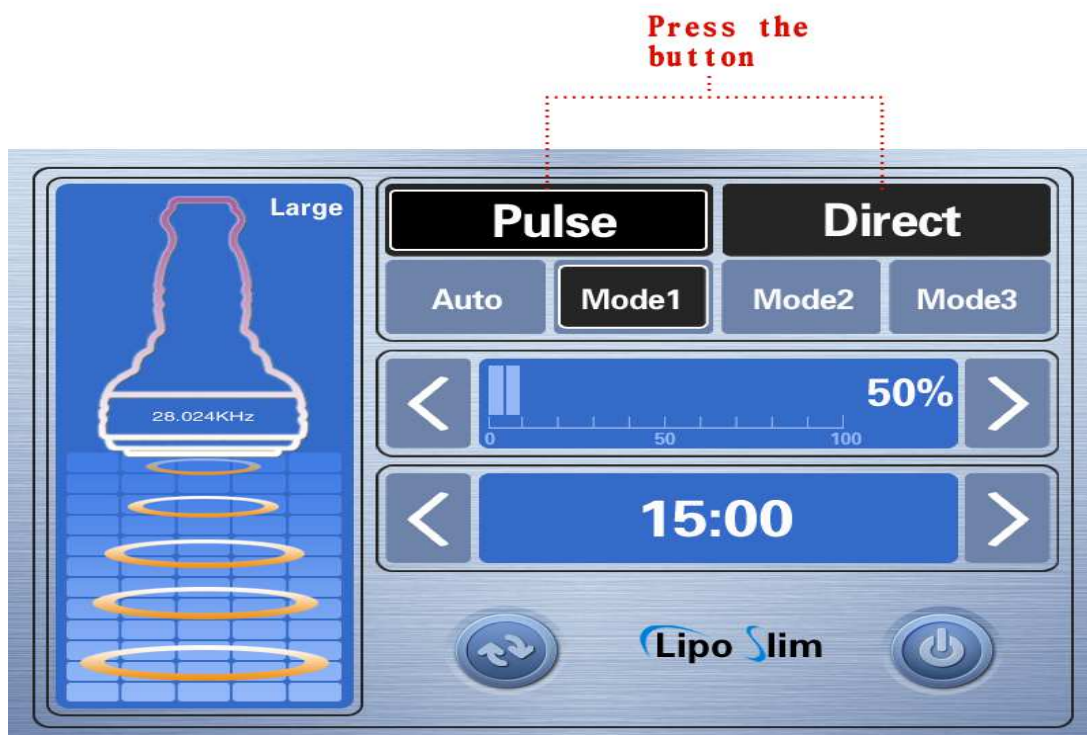

- Preset : Pulse Mode (ON for 5 sec. OFF for 3 sec.)


## II. Operation

### II -2 : Operation

3. Set the output level



4. Set the time

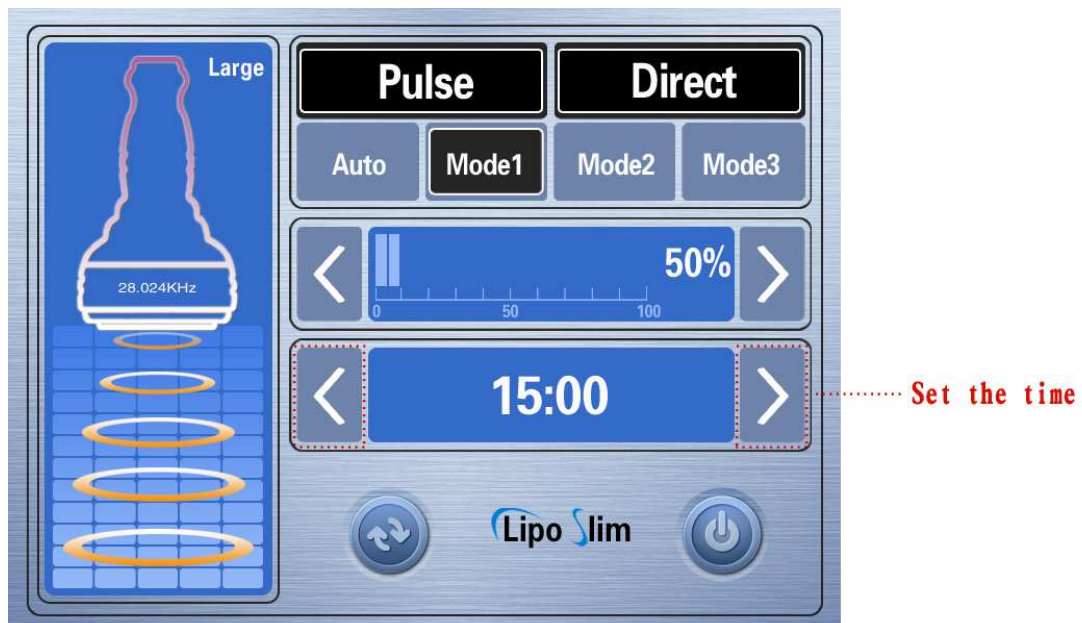


- Preset time : 15 Min. (Adjustable from 0 to 30 minutes)



## II. Operation

### II -2 : Operation

#### 5. Select the pulse

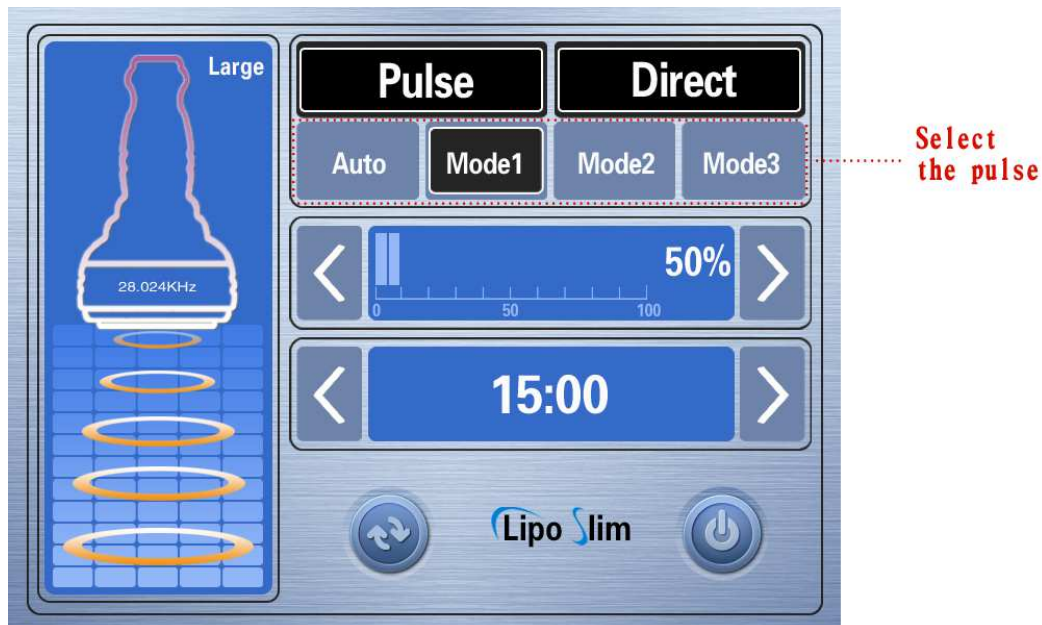

A u t o : Auto mode : (operated at "Direct" mode only)

[Continuous ]-[2/3]-[1/2] repeats automatically

Mode1 : Continuous pulsation

Mode2 : 2/3 pulsation

Mode3 : 1/2 pulsation


#### 6. Apply ultrasound gel on the treatment area

## II . Operation

---

### II -2 : Operation

7. Press “Start” button to begin operation

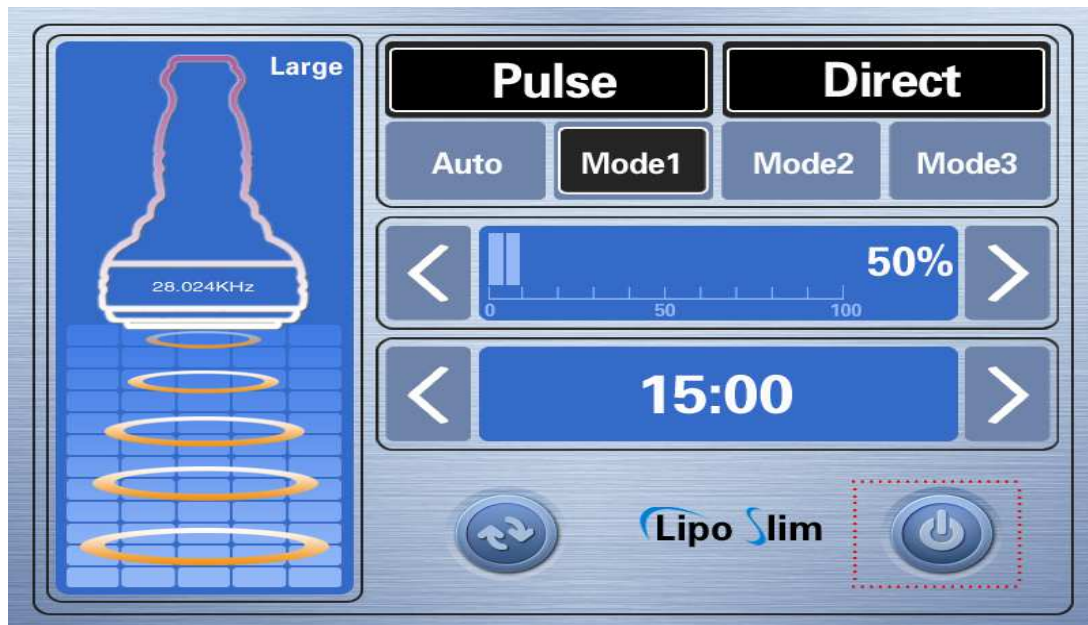

\* Once “Start” button is pressed, red LED on the applicator is automatically lit on.

\* If the chosen level or mode doesn't fit to the patients, adjust the level or mode.

### Cautions!!

No mode or button except power level control button can be changed when start button is on

### II -3 : Finish

---


1. When finish the treatment, press “Start” button off
2. Turn off the power switch on the back of the machine.
3. Clean the accessories after treatment.

### III. Practical Applications


#### III-1 : Abdomen / Back / Thigh

Applicator		Pulse	Direct	
φ 63	Mode	<b>mode2</b>	<b>mode1</b>	<b>Auto</b>
	Level	50%	50%	50 ~70%
	Time	5 min.	10 min.	5 min.


**Abdomen – 20 min.**

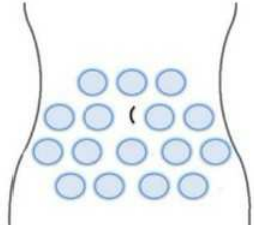

Pulse Mode  
(5 min) **mode2**

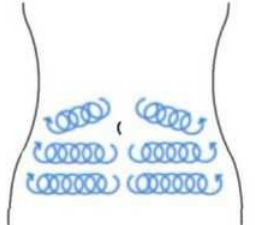


Direct Mode  
(10 min) **mode1**

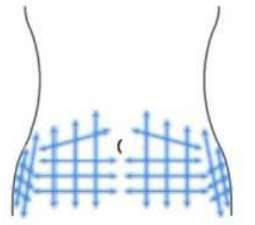


Direct Mode  
(5 min) **Auto**


**Back – 20 min.**

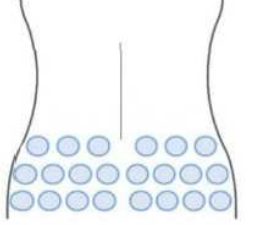

Pulse Mode  
(5 min) **mode2**

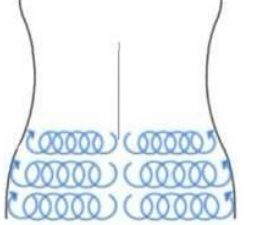

Direct Mode  
(10 min) **mode1**

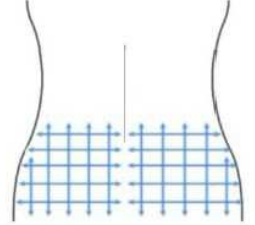


Direct Mode  
(5 min) **Auto**

√For abdomen circumference reduction, performing the treatment of Abdomen and back together get the best result.

### III. Practical Applications

**Thigh – 20 min.**

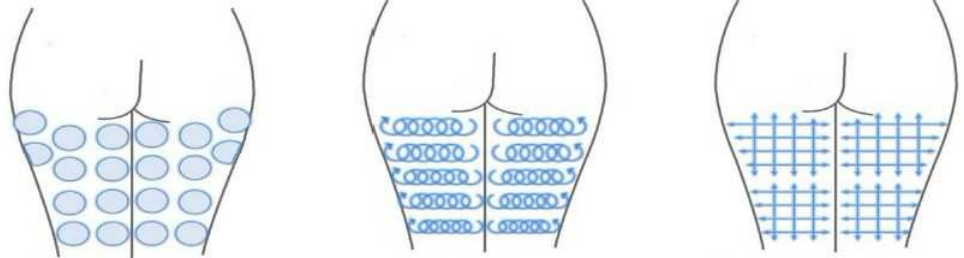

Pulse Mode (5 min) **mode2**


Direct Mode (10 min) **mode1**

Direct Mode (5 min) **Auto**

### III 2:Arm

Applicator		Pulse	Direct	
f 38	Mode	<b>mode2</b>	<b>mode1</b>	<b>Auto</b>
	Level	50%	50%	50%
	Time	2.5 min.	5 min.	2.5 min.

**Arm – 20 min. (10 min. per arm) Treat one arm, then treat the other.**  
 Avoid treating around eyes and thyroid, bones, lymph nodes

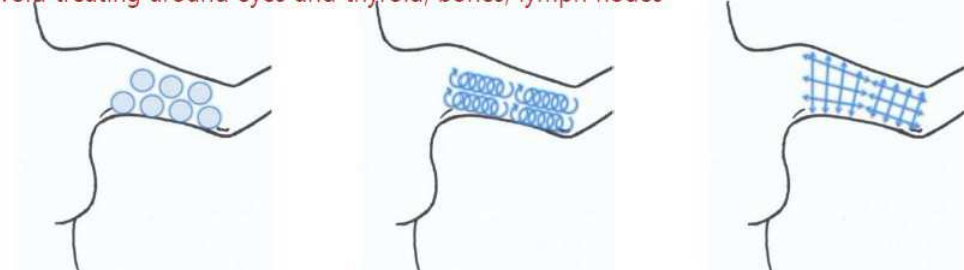

Pulse Mode (2.5 min) **mode2**

Direct Mode (5 min) **mode1**

Direct Mode (2.5 min) **Auto**

## IV. Caution

---

### IV-1 : Contraindication

Please avoid the treatment from these people or part as below,

- a. Pregnancy
- b. Scarring, infection of the treatment area
- c. History of keloid scarring
- d. Use of medication known to induce photosensitivity
- e. Fragile skin
- f. Patients undergoing the surgical procedure without consulting with a surgeon
- g. Patients with tumors
- i. Patients with serious disease
- j. Patients with low/high blood pressure

## IV-2 : Caution when use

---

1. This equipment should be used by the person who took a equipment training.
2. Avoid treating around eyes and thyroid, bones, lymph nodes.
3. Avoid treating on cuts, eyeballs, insensate parts, ears.
4. In case of resting during treatment, please “Start Off” to avoid over-output of the machine.
5. If the machine can't operate in normal condition, please turn off the power and contact the distributor or manufacturer immediately.

## IV. Caution

---

### IV-3 : Maintenance

1. After treatment, keep the power on about 3 minutes to make inner board cooling.
2. Please keep Liposlim in the shade, avoiding direct lay of light.
3. Don't drop the machine & keep it out of any shock.
4. Clean applicators after use.
5. Moving the machine, be careful not to drop applicators

### IV-4 : Solution

---

Light of power switch is not on although turn the power switch

- a. Please check whether power cord is put to plug on the machine
- b. Please check whether fuse has been broken down

## VI. Specification

---

Handpieces	63 $\emptyset$ , 38 $\emptyset$
Ultrasonic Frequency	43 KHz, 32KHz
Output Power	50 Watt
User Interface	Digital Touch Screen
Weight / Dimensions	7.9Kg / 50*42*49.5cm