



Candy

User Manual

The User Manual includes introductions on Candy, Candy's App and how to use Candy.



Catalogue

I Candy Overview	4
1、 Candy	4
2、 Accessories	5
3、 Buttons	5
II Getting Started	6
1、 Setting up candy’s App	6
2、 Registration	6
3、 Charging	6
4、 Cleaning	6
5、 Sterilising	7
6、 How to use Candy	7
III Other Functions	8
1、 App Functions	8
2、 The storage cage	9
IV The Warranty	10
1、 Warranty Instructions	10
2、 What is not covered by this warranty	10
3、 Other things need to know	11
Appendix: Information on Safety and Use	12




Candy Overview

1、Candy

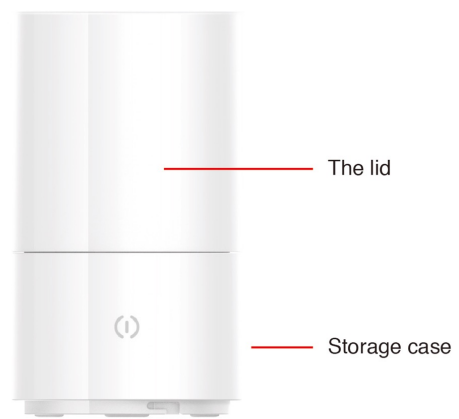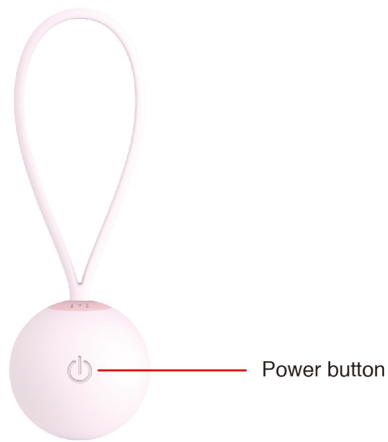


2、 Accessories

Box Contents:

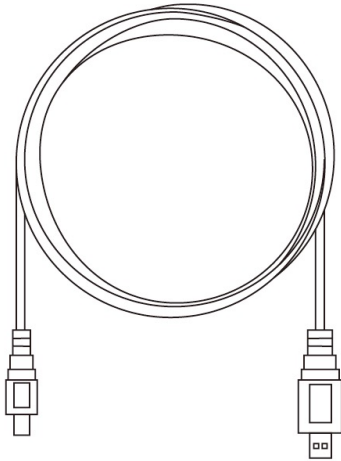

(1) 1xCandy Kegel ball, 1xstorage case(with build in wireless charging system and UV steriliser), 1 × USB power cable

Note: If the cable power becomes misplaced, please contact IMTOY customer service.

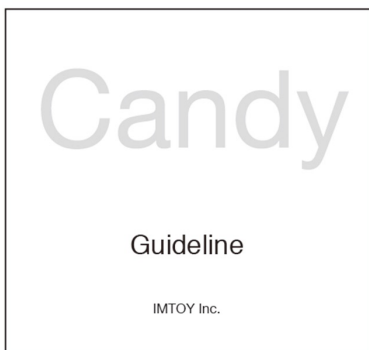

(2) User Manual: Includes instructions on charging, cleaning, operation, warnings, the warranty terms and the warranty card.

3、 Buttons Overview

Candy case's touch power button and indicator light: Controls both charging and sterilising.

Candy's power button: Charging Candy for 5 hours before make it start to work first time.



Getting Started

1、 Setting up Candy's App

Candy's exclusive App provides professional Kegel exercise guidance.

You may download Candy's App via the information below.

Note: To get a better experience, please visit us at www.IMTOY.com for compatibility downloading Candy's App.

(1) Scan the QR code on the package or on the User Manual, and follow the instructions onscreen to download.

(2) Please visit us at www.IMTOY.com and go to App downloads to scan the QR code.

2、 Registration

Create a new account when first using Candy.

3、 Charing

(1) Lower Candy into the recess of the base where it will be held by the magnet, then attach the lid by lining up the arrows. Now connect up the USB power cable.

(2) Press the power light on the case to charge Candy. The light will pulse whilst charging. When the power light is pulsing the light on Candy will also flash.

(3) A complete charge takes 5 hours.

4、 Cleaning

Rinse Candy with water and clean hands and dry using a clean flannelette (do not air dry in direct sunlight) .

5、 Sterilising

(1) Place Candy onto the glass platform of the base where it will be held by the magnet. (Simply flip over the base of the storage case from the charging side)

(2) We recommend 45 minutes for complete sterilisation.



Note①: Twist the lid into the locked position by lining up the arrows and then connect up the USB cable. Press the illuminated power light and UV sterilisation will begin, the light will now flash.

Note②: The glass platform is an important component of the UV sterilise. Do not place the UV side onto any objects that may break the glass.

6、 How to use Candy

Please find a comfortable training area:

(1) Press and hold Candy's power button for 2 seconds and the power light will flash.

Please make sure Candy has been fully charged for 4 hours before first time using it.

(2) Turn on bluetooth on your smart phone and follow the App's onscreen instructions to pair up and select the appropriate training course.

(3) Apply desired amount of lubricant on Candy with clean hands.

(4) Gently insert Candy deeply into your vagina. Please leave the remaining cord outside to easily remove Candy after use.

(5) Exercising according to the instructions on the App's training course.

(6) When exercise is finished, relax and remove Candy with the cord.

(7) When you are done, please press and hold the power button for three seconds to turn off Candy. And repeat the sterilising process according to step 4 and step 5.

(8) After sterilising, place Candy into the cave part of the storage case and made the lid into a locked position.



Other Functions

1、 App Functions

(1) Training course

Selecting courses: Go to courses>select courses to choose your training plan.

(2) Record

Record analysis: You may check the analysis of your training in the Record.(such as calorie consumption and butternut training progress)

(3) Settings

①Set a passcode: You may go to Settings to set a four-digit App unlock passcode to protect your privacy.

②Schedule your training plan: Go to Settings >“set the next course reminder”to schedule your training course.

③Bluetooth connection: Go to Settings >“Not connected”to build the connection between Candy and the App via bluetooth. Make sure you turn on bluetooth on your smart device.

④Clear all data: Go to Settings > “Delete all data” to clear all of your training data. The data clearance is irreversible.

⑤Reset password: Go to Settings > “Change account passcode”.

(4) Information

①General Guide: You may go to Information>”Instructions”to get a general guide about how to use Candy.

②FAQ: You may go to Information >“FAQ”.

③Check for Updates: You may go to Information>”Check for updates“to see whether you have the latest version of the Candy Player App.

2、 Storage Case Function

The storage case has two functions. The concave part of the base is for wireless charging of Candy. The reverse side of the base is the ultraviolet (UV) sterilising system.

(1) Charging: Lower Candy into the recess of the base where it will be held by the magnet and attach the lid. Connect up the USB power cable and press the power light on the case to charge Candy. The light will pulse whilst charging.

(2) Sterilising: Please make sure Candy attached to the glass platform of the base and the lid is in the locked position for the sterilising system of the storage case to work.

IV

The Warranty

1、 Warranty Instructions

(1) Please keep “The IMTOY Warranty Card”(referred to as “the warranty card”)safe.

(2) We guarantee the IMTOY branded hardware product and accessories contained in the original packaging against defects in materials when used in accordance with Candy’s user manual. With the warranty card, you will be able to receive any required services or replacements for free available under the One Year Limited Warranty for Candy.

(3) One-year warranty period is valid from the date of original purchase, according to your purchase receipt.

2、 What is Not Covered By This Warranty

(1) Not under one-year limited warranty.

(2) Unable to provide the original official warranty card or receipt.

(3) Damage caused by performed by anyone who is not a representative of IMTOY.

(4) Distributors and retailers are solely responsible for any free gifts or promotions offered.

(5) Normal aesthetic wear and tear (i.e. wear to the outer case and charging cable).

(6) Damage caused by accidents or improper use, such as improper operation, water/ liquid damage, scratches or cracks from dropping, damage caused by pets or animals.

(7) Any damage as a result of natural disaster.

3、 Other Things Need to Know

(1) Repairs or replacements may only apply when a valid warranty card and original invoice are provided.

(2) Repair or replacement service charges and shipping charges may apply if the product is not under the one-year limited warranty.

(3) The warranty service shall based on the warranty terms in User Manual.

Note: IMTOY Inc. All rights reserved.

Information on Safety and Use

1、 Important safety information

Warning: Failure to follow the following safety instructions could result in fire, electric shock, injury, damaged to Candy or other property. Please read carefully before using Candy.

(1) Handling

- ① Please handle Candy and its case with care. It is made of metal, high-density plastic, UV glass, medical silicone and has sensitive electronic components inside. Dismantling or tampering with it will void the warranty.
- ② Candy can be damaged if dropped, burned, punctured or crushed, or if it comes in contact with corrosive liquids. Do not use a damaged Candy as it may cause injury.
- ③ After cleaning Candy, please place it into the concave part of the case to protect it from scratching, dust and bacteria build up.

(2) Repairing

- ① Do not open Candy and do not try to repair Candy yourself as it may cause injury and will invalidate the warranty.
- ② Do not dismantle the base of the case as this will damage the unit, make it unsafe to use and invalidate the warranty.
- ③ Any repairing service performed by a non-authorized service provider will not be covered by the warranty. If Candy is damaged or malfunctions, please contact our customer service. You may visit us at www.IMTOY.com for more information.

(3) Battery

- ① Don't attempt to replace the Candy battery yourself as you may damage it, which could cause overheating and injury and will not be covered by the warranty.
- ② The lithium-ion battery in Candy should be replaced only by IMTOY or an authorized service provider and must be recycled or disposed of separately from household waste.

(4) Charging

- ① Please charge Candy with the original USB cable.

② Do not charge Candy in a high humidity environment.

(5) Prolonged heat exposure

① Candy and its power adapter comply with applicable surface temperature standards and limits. But, even within these limits, sustained contact with warm surface for long periods of time may cause damage. Avoid situations where your skin is in contact with Candy or its power adapter when it's operating or plugged into a power source for long periods of time.

② When Candy's plugged into a power source. It's important to keep Candy adapter in a well-ventilated in use or charging.

(6) Training Schedule

Please schedule your training appropriately according to the instructions.

(7) Note

Please do not share your Candy with other people to against infection.

2、 Important Handling Information

(1) Usage

When using Candy, please follow the onscreen instructions to get the best results.

(2) Hygiene

Please keep food and liquids away from the case to maintain hygiene.

① If Candy is exposed to certain liquids, such as cosmetics, cleansers and inks, please clean it with water as soon as possible.

② Although Candy is 100% waterproof, please do not place it in boiling water for sterilising. If you wish to sterilise Candy you may rinse it with water first, wipe it dry with a clean flannelette and then place it in the ultraviolet steriliser.

③ Please do not attempt to dry Candy or its case with any heat generating device such as a microwave oven or hair drier.

④ If you wish to clean the storage case, please do so with the power supply disconnected. Wipe it with a damp, soft tissue or flannelette and let it air dry (out of direct sunlight).

(3) Temperature

① Candy is designed to work in ambient temperatures between 32 and 98° F (0 and 37 °C) and stored in temperatures between -4° and 113°F (-20° and 45°C). Candy can be damaged and battery life shortened if stored or operated outside of these temperature ranges. Avoid exposing Candy to dramatic changes in temperature or humidity (for example in a hot car). When using or charging Candy, it is normal for it to get warm.

② If the interior temperature of Candy exceeds normal operating temperatures (for example, in a hot car or in direct sunlight for extended periods of time), you may experience the following as it attempts to regulate its temperature:

- Candy stops charging.
- Battery temporarily stops working.
- Candy will not turn on.

Important: You may not be able to use Candy if the interior temperature is too high. If Candy can't regulate its internal temperature, it will stop working. In this situation, move Candy to a cooler location out of direct sunlight and wait a few minutes before trying to use Candy. If Candy does not work again you may contact our customer service department.

Please visit us at www.IMTOY.com for more information or to contact us
support@imtoy.com