

INSTALLATION INSTRUCTIONS

This document provides guidance in the installation of the ENDODRY Storage and Drying System.



READ THE ENTIRE DOCUMENT BEFORE STARTING INSTALLATION

APPLICABLE DOCUMENTS

- ENDODRY System Site Requirements
- ENDODRY System Pre Installation Checklist
- ENDODRY System Installation Verification Checklist
- ENDODRY Storage and Drying System Users Manual

TOOLS REQUIRED

- 4mm hex wrench
- 8mm hex wrench

CUSTOMER REQUIREMENTS


- The customer is responsible for preparing the facility site prior to the ENDODRY Storage and Drying System arrival, including all required electrical and air supply needs.

INSTALLATION OF ENDOSTORE DRYING CABINET

IMPORTANT! Remove the drying cabinet and provided accessories from its packaging with care. Please inspect the equipment upon arrival, checking for any evidence of damage during shipping.

Inspect the equipment to ensure all supplied parts are complete

QTY	Description
1	ENDODRY Storage and Drying System
1	barcode reader with cable and holder (2x barcode readers for pass-thru models)
1	power cord
1	air lines with quick connector
1	Users manual
1	LAN cable

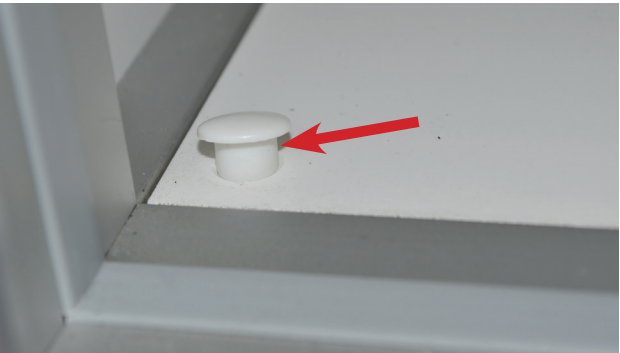

Move the ENDODRY Storage and Drying System to its final location.

Check the level of the cabinet front to back and left to right using a bubble level. If leveling is required, remove the four small white plugs in each corner of the base as shown. Use a 4mm hex wrench to adjust the level. Replace the plugs when complete.

IMPORTANT! For optimal user-safety and operation, it is recommended that the ENDODRY Storage and Drying System always be placed directly on the floor and not on an elevated platform type device.



- Install the supplied barcode scanner(s) and the holder(s).
- The barcode scanner holder(s) attach to the top of the cabinet as shown. Use the supplied screws to attach. Tighten the screw with a 8 mm hex wrench.

Note: Two barcode scanners and holders are supplied with the pass-thru cabinet. The barcode scanners on the pass-thru cabinet are side dependent. If they initially do not function properly, swap the barcode scanners.



- Connect the power supply, the air line and optional LAN connection to the panel located on the top of the system. The wall outlet should be within the operators reach for easy unplugging.

NOTE! Take care to clean the cabinet before operation. Disinfect all contact surfaces with an approved surface disinfectant.


Use only surface disinfectants appropriate for medical devices. Chemicals need to be compatible with polyethylene terephthalate, polyvinyl chloride (PVC), aluminum and stainless steel. For cleaning, only use a lint-free cloth. Avoid saturating solution into the control drawer, phematic system and connection block nozzles.

- Turn the system on using the ON/OFF switch. Confirm the system powers up to the ENDODRY™ Storage and Drying System home screen.

To verify the ENDODRY Storage and Drying System is operational, refer to the document 50098-679 - INSTALLATION CHECKLIST, to document and verify proper system installation.

Refer to the User Manual for the entry of operator and endoscope information.

MEDIVATORS Customer and Technical Support Toll Free: Phone: 1-800-444-4729 · FAX: 1-866-421-7696

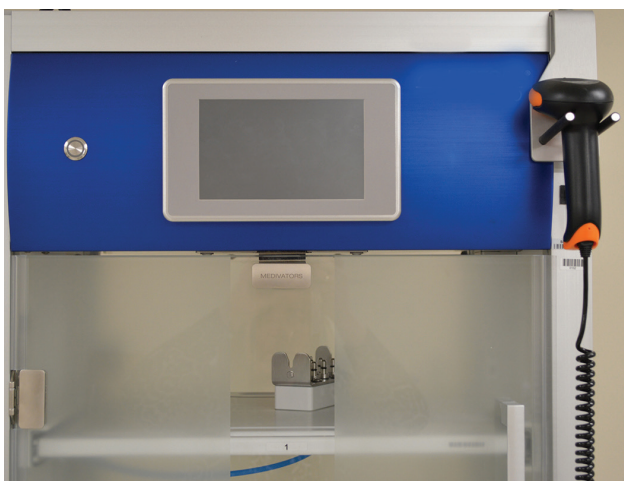

MEDIVATORS™ and ENDODRY™ are trademarks of Medivators Inc.



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.763.553.3300
Fax: +1.763.553.3387

Medivators BV **EC REP**
Sourthweg 11
6422 PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.595

www.medivators.com

Cantel Medical Asia/Pacific Pte. Ltd.
1A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

Medivators Inc. Beijing Representative Office
Room 1801, Floor 18th, Tower A,
Beijing Marriott Hotel, Office Building
No. 7, Jianguomen South Avenue,
Dongcheng District, Beijing 100005 China
Tel: +86 10 65204039