

TABS® mobility monitors are used to alert a caregiver that a patient needs assistance.

Mounting Options

TABS Professional monitors can be easily moved from a patient bed to a chair to a wheelchair. Choose the correct bracket and mount it securely before each use.



Wire Bracket



Side Chair Attachment



Wall Mount Bracket



Monitoring Options

Monitor patients with the recommended pressure pad method. Fall monitoring options include both audible alert from the monitor and/or through the nurse call system.



Bed and Chair Pads

- Record in-use date on Pad.
- Place Pad on bed or chair. For bed application, cover pad with a sheet.
- Insert phone-style plug into jack location labeled Pad on the TABS monitor.
- When the patient applies his/her weight, a beep will sound to indicate the device is alarmed.
- Alarm occurs when patient removes weight from pad.
- To silence or reset the alarm, press the black Reset button on the front on the monitor.
- Plug the nurse call cord into the jack labeled Nurse, and plug opposite end 1/4" phone jack into the nurse call outlet on the wall or bed.



Monitor Operations

- TABS monitors are powered by a 9-volt battery located under the back cover.
- A low battery alert (intermittent beep) reminds you to replace the battery.
- To record a message:
 - 1) Remove back cover.
 - 2) Push and hold down the black record button while you speak slowly into the microphone.
 - 3) Release button when finished.
 - 4) Replace back cover.
 - 5) Review your message by pressing the reset button on the front of the alarm.

- ▲ **1069 ETL Listed**
- ▲ **JCAHO compliant**
- ▲ **Always test the TABS device and pad before each usage**
- ▲ **For full instructions, refer to TABS Professional Monitor User Manual**

STANLEY

Healthcare Solutions

www.stanleyhealthcare.com