
















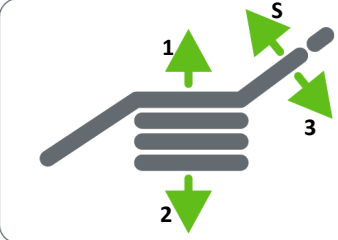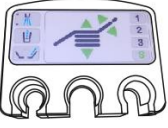



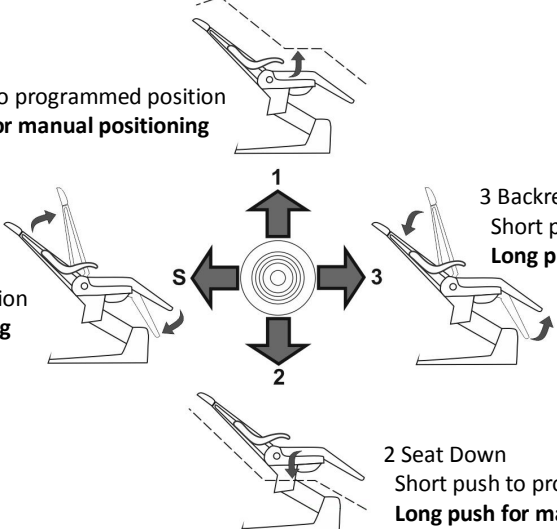


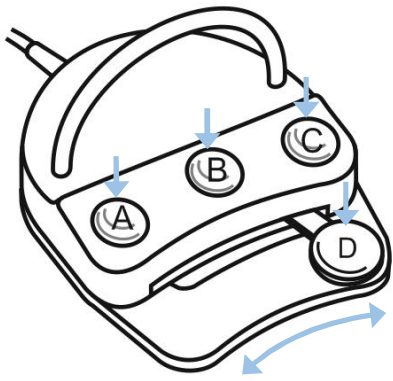
FINNIDENT 8000+ QUICK REFERENCE

PATIENT CHAIR MOTION

Main Control Panel			
S Backrest Up		To programmed position	Short press S
		Manual positioning	Long press S
3 Backrest Down		To programmed position	Short Press 3
		Manual positioning	Long Press 3
1 Seat Up		To programmed position	Short press 1
		Manual positioning	Long press 1
2 Seat Down		To programmed position	Short press 2
		Manual positioning	Long press 2
3+ Control Panel			
<p>Manual positioning Long press on arrow</p> 		<p>To programmed positions Short press on button</p> 	
	 1 Seat Up		
	 2 Seat Down		
	 3 Backrest Down		
	 S Backrest Up		
Chair Foot Control			
			
<p>1 Seat Up Short push to programmed position Long push for manual positioning</p>		<p>3 Backrest Down Short push to programmed position Long push for manual positioning</p>	
<p>S Backrest Down Short push to programmed position Long push for manual positioning</p>		<p>2 Seat Down Short push to programmed position Long push for manual positioning</p>	

FINNDENT 8000+ QUICK REFERENCE

DENTAL UNIT FUNCTIONS

Main Control Panel			
Door Open		Short press	Either door open or call assistant may be installed
Call Assistant		Short press	Either door open or call assistant may be installed
Clock		Press and hold to adjust time	
Timer		Short press repeatedly until the timer you want appears	
Bowl Rinse		Short press to start rinse	Press and hold to adjust rinse duration
Glass Fill		Short press to start glass fill	Press and hold to adjust fill duration
Air Spray		Short press to turn ON / OFF	Instrument must be active to use air spray
Water Spray		Short press to turn ON / OFF	Instrument must be active to use water spray
Chip Blow		Select instrument and shortly press to turn chip blow ON / OFF	For turbine and micromotor only
Increase / Decrease		Short press to increase or decrease function settings	
3+ Control Panel			
Suction Tube Rinse		Short press to activate rinse cycle	
Unit Foot Control			
<p>A Spray ON / OFF</p> <p>B Power range selection for the instrument</p> <p>C Micromotor reverse / forward</p> <p>C Additional functions in connection with ENDO use: Auto reverse Auto reverse forward Press C to switch between functions.</p> <p>D Press the pedal to start the selected instrument. To decrease the power level, slide the pedal to the left. To increase the power level, slide the pedal to the right.</p>			

For detailed instructions, please see the User's Manual | www.finndent.com
 Finndent Oy | Niittylänpolku 16 | Helsinki | 00620 | Finland