

Digital Multi-function Diamond Dermabrasion Machine User Manual




- Thank you for your purchase;
- Please read the instructions carefully before using the normal operation, after reading it you should keep for at any view;
- The use of this instrument is the beauty of healthy skin care as a precondition, for cosmetic purposes is prohibited to use for therapeutic purposes or sale.

Equipment surface layout



Accessories

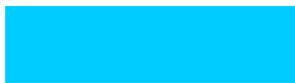

Miniature engraving

Diamond dermabrasion instrument

Diamond dermabrasion instrument is the mainstream beauty products, new technology is a new wave of beauty, it appears to radiate beauty, a new vitality, it has gradually become

a living quality and taste to enhance the performance ... it through a The product of the laser laser light tubes attract Diamond Head with deep, so that damage cellular automata from aging, damaged skin cells so that is no longer attached to the surface, and absorption through the skin to stimulate the growth of layers of oxygen and promote blood circulation, accelerate skin Tissue development, and promote collagen production and elastin production, combined with proper skin care program, the skin becomes soft and delicate, fresh and natural, flexible glossy, full of vitality, give you more beautiful ... beautiful ...

Diamond dermabrasion Beauty Instrument is now following the acid peels, laser skin after safest and most gentle and most popular skin surgery. The past, microdermabrasion technology, the role of the suburbs to the dermis layer of conflict, this is a medical field, along with the rapid development of science and technology, calls the skin are cosmetic surgery is the role of the epidermis, its technology has reached the safe, no side effects, Simple, easily controlled effect. At the same time, a fine, accurately eliminate the necrotic epidermis skin cells, accelerate the metabolism of the skin surface, so that the new skin speed up the metabolism, so its very broad application, the process calls the skin does not produce the traditional calls the skin red, swollen , thorns, heat, pain feeling.

At what level, regardless of age, the United States has been a dream of a woman, and no one wants to have a beautiful face, beautiful body, and no one wants to be able to arouse the envy of the attention, however, a number of troubles plaguing men and women each, Skin spots, fine lines show rough, acne, acne occurs, and even left pock uneven dents, these phenomena tend to affect the individual's appearance, we all hope to improve their appearance through a variety of ways to protect Beautiful face.

1. Diamond dermabrasion instrument comparison with other skin beauty equipment

In order to improve appearance, eliminate pock and restore youth, dermatology or plastic surgeons often use chemical abstract peels, dermabrasion or laser surgery surgery to achieve the goal, but the technique is the use of chemical peels chemical peels and break the skin, the depth Difficult to control, and sometimes a lack of experience because the physician side effects; and laser resurfacing surgery often produce easily understood because of the additional thermal effects of skin redness, darkening or recovery of the side effects of conventional mechanical dermabrasion surgery

rapid rotation of the metal or diamond dermabrasion Bit to rub off the skin,

patients must be anesthetized during surgery, and the situation is more serious pain and bleeding. As a longer recovery period.

Past the traditional microdermabrasion machine or cosmetic methods have many drawbacks, leading many patients in treatment less than ideal results, and now the company introduced the diamond miniature beauty instrument, and with the use of diamond miniature vacuum suction control suction strength

Will be worn shallow layer of the epidermis, is a kind of non-invasive, physical shallow dermabrasion. Diamond dermabrasion Beauty Instrument used to improve skin pigmentation disorder, shallow pock youth, age spots, rough skin aging after sun exposure and facial lines, can also promote skin renewal, presented immediately after surgery with smooth skin Touch.

After making the diamond dermabrasion, and then do import or use of ultrasound

whitening whitening agents, with the ice, the effect will be even better, it will reach your beautiful wishes!

2. Operation procedures

1. Diamond dermabrasion (dermabrasion scrub); time 5-10 minutes

Using miniature tube top inlaid with diamonds of different coarseness of the diamond particles to increase the friction by the way, look at the cumulative level of individual cuticle to do to adjust the pressure at different speeds.

Diamond dermabrasion instrument working principle

Diamond dermabrasion instrument is the use of patented diamond miniature tube, top inlaid with different degrees of thickness of fine diamond particles, by rubbing back and forth the way, and with vacuum suction control suction strength will wear off the shallow layer of the epidermis, is a kind of Non-invasive, physical, shallow dermabrasion. This kind of purely physical and mechanical principles, as opposed to acid skin by way of removing horny substances have the advantage of mild non-irritating to the skin, suitable for very sensitive skin. Through the suction pipe size from miniature to control the depth of granularity, less irritating, you do so is not easy to take care of the wound problems are not easy to have pigmentation or redness of skin problems, and the operation will not have crystal particles flying, residual Problem, the whole process is very clean, very safe.

Advantages of diamond miniature instrument

1. Conditioning skin and rebuild healthy skin;
2. to improve the accumulation of aging skin, rough and aging skin;
3. To improve the skin appearance of fine lines and relaxation;
4. Dilute the superficial stain;
5. To improve large pores, pock
6. To stimulate blood circulation and metabolism of the new array
7. no irritation, no irritation, do not need anesthesia;
8. Applicable to all kinds of skin, and also can do sensitive skin
9. the physical mechanical dermabrasion, we can control, no side effects, safe and reliable
10. You can see the effect immediately
11. Rapid recovery, a period of about 5-7 days
12. After nursing ,it won't effect the work andsocial work.

Operation method

1. The diamond miniature tube connection into the appropriate host port;
2. Connect to the power supply, press the control panel "POWER" button, the corresponding indicator light, and the system power, the instrument enters the work state.
3. Press the "Vacuum" function button, the indicator light will shinning on.
4. Adjust the intensity by turning the pressure adjustor, from left to right to enhance the pressure. And the pressure meter will show the outputting pressure.
5. To have "TIME" miniature diamond district ▲ and ▼ set the time required for operation, press according ▲ to growth in time,press ▼ to reduced time.
6. Diamond dermabrasion cosmetic operation is complete, press the "OFF" button to close the diamond dermabrasion function.

Note: The pump of the instrument set temperature is 85 degrees, if more than 85 degrees will automatically shut down, you can wait untill the body of the instrument cold down, and then boot to work.

Functional analysis of the diamond dermabrasion

● Role of diamond dermabrasion to activation in the dermis, repair, regeneration

1. removal of the aging dead skin cells, to avoid excessive accumulation of skin cells, keeps skin smooth and delicate;
2. Promote blood and lymph circulation, moisture and nutrients more rapidly transmitted to increase the bright and shiny;
3. Moisturizing repair function: increase in cell permeability, enhance product absorption, improve skin moisture and moisture;
4. Promote the regeneration of epidermal basal cells, accelerate skin renewal, the skin returned to normal, allowed to transfer water capacity balance.

● on the loose, wrinkled skin treatment occurred

1. Accelerate the clearing of the skin surface, easy to damage the cells of the dust pollutants, free radicals, and enhance the natural defense cells to avoid the degeneration of collagen fibers
2. Improve blood and lymphatic circulation, promote metabolism, so that the dermis layer of adequate nutrition, so fine lines gradually disappear.
3. increase skin permeability, make the product more effectively into the deep, and promote new collagen fibers, smooth wrinkles.
4. Promote the manufacture of collagen fibroblast collagen fibers and increase the defense capability of skin to prevent damage by free radicals and the environment

● on the dull, uneven skin color, the processing occurs

1. remove the old dead skin cells, to avoid the accumulation of dead cells to keep skin clear and detailed.
2. to improve the blood and lymph circulation, promote metabolism, increase skin oxygen content, making the skin red.
3. Increase the cell permeability and enhance absorption of the main products, thermal insulation and moisture to improve skin and leaves skin bright and shiny.
4. Promote the regeneration of epidermal basal cells, accelerate skin renewal, showing a uniform, the new soft white color.

● On the treatment of pigmented skin

1. Accelerate the removal of melanin in the epidermal keratinocytes and residue on the skin easily lead to cell damage and lipid peroxidation of environmental pollution substances such as embryo cream can be added quickly melanin spots.
2. Improve skin microcirculation, promote the metabolism of melanin, to strengthen the deep whitening, reduction of melanin, faster and more pronounced effect to fade away.

3. to increase skin permeability, so that cells absorb the depth of whitening

products, effectively inhibit the formation of melanin to prevent pigmentation on the face.

4. Promote epidermal basal cell regeneration, and increase the skin immune system, strengthen the fight against ultraviolet rays, keeping the skin clean, white and bright.

● Treatment effect

1. Cavity effect 5-10 times 80%-100%
2. Effect of pores 90%
3. Closed of acne 2 effects 100%
4. Seborrheic dermatitis effect of 100% 3-5 times
5. Rough skin effect 100% 1-2
6. Fiber lumps effect of 100% 2-3 times
7. Nasolabial folds, forehead wrinkles, as the case effect of 80%-100%
8. Raised age spots, as the case 80% -100% effectiveness
9. small-wrinkle effects of rosacea, as the case 80%-100%
10. Compaction effect of lift 1-10 times 80%-100%
11. Pock scars 3-5 times and 90% effect

Specifications and technical parameters

External voltage: 110/120-220/240V

Internal voltage: $\leq 36V$

Electric current: 3.3A

Power: $\leq 120W$

Single pump pressure flows: 14L/min

GW: 5KG

NW: 3.5KG

Packing specification: 350*350*280mm

Material: ABS plastic

Matters and attention after treatment

1. Often use ice or the use of living cells to do repair or stabilization of collagen products

2. be sure to take sunscreen.

3. For more information on the makeup, please use the pure mineral powder, in order to avoid crowding.

4. Please wash your face with cold water.

5. Do not wash the course of three days after the 10-5 contact with the heat source can be warm.

6. deep scar treatment will be itching 2-3 days after the peeling, do not scratch

or tear the skin.

7. for personal skin condition, vitality and depth of treatment site about interval of 1-3 weeks to different treatment.

Accessories list

Name	Unit	Quantity
host	set	1
Blackhead vacuum head (big, medium,small)	pcs	3
Spray bottle	pcs	1
Diamond tips	box	1
Thin straight pipe	bag	1
Red pipe	bag	1
Power line	pcs	1
User manual	copy	1

Product service and repair

1. This machine from the date of purchase (with a note) host one year warranty, one year free repair quality problems, such problems after one year will receive maintenance costs. Parts warranty for six months, the quality problem within six months, free repair, maintenance costs after six months to receive parts. Other warranty period has expired, he may be responsible for maintenance, spare parts and consumables costs charged fees.

2. causes the following use of personal failure, does not provide free warranty service.

- 1) The unauthorized entry, modification caused the failure of the product;
- 2) Use accidentally blows, falls caused by the fault;
- 3) Lack of reasonable maintenance caused the failure;
- 4) Do not at the correct guidance of instructions caused by the fault.

Maintenance Note

Maintenance Report

Product Name : _____ Date of Purchase : _____

Sub-office: _____ Add: _____ Tel: _____

Customer Name: _____ Add: _____ Tel: _____

Maintenance Date	Trouble Reason	Trouble Remark