

INSTRUCTIONS FOR USE:







Insert sample*

*also insert the little granule that is in the cap when using **EZ Test for THC tand EZ Test for LSD**. This is a catalyst that gets the reaction going.



Mix ingredients



Observe and compare

* Scrape a little bit of material off your pill or use a small amount of the powder, about the size of a pin head. * When finished, close ampoule with the cap and dispose of it responsibly. Please, do not litter.

WHEN THE COLOR IS SHOWN ON THE CHART:

This means the sample contains (some) of the substance (-family) indicated on the color chart. It does **NOT** mean the sample is "pure", there could be something else in it that wasn't detected by the test. It also does **NOT** mean the sample is "safe". No drug is completely safe, even when it is pure. It does NOT tell you how much of the substance is in the sample.

WHEN YOU SEE A DIFFERENT OR NO COLOR REACTION:

This means that you are definitely NOT dealing with any of the substances indicated on the color chart. In this case, you cannot say with certainty what kind of substance is present. This can be harmless, but might be **extremely dangerous!** If this happens to you, it is **safest to throw it away.**



















