

P137 & P138 Plesiocare Pager system

User Handbook



Models covered by this handbook

P 137AAA	Advanced 32 monitor Pager (legacy products): English
P 137AAB	Advanced 32 monitor Pager (legacy products): French
P 137BAA	Advanced 64 monitor Pager: English
P 137BAB	Advanced 64 monitor Pager: French
P 137BAC	Advanced 64 monitor Pager: German
P 137CAA	Advanced 8 monitor Pager: English
P 137CAB	Advanced 8 monitor Pager: French
P 137CAC	Advanced 8 monitor Pager: German
P 138AAA	Domestic 3 monitor Pager (legacy products): English
P 138AAB	Domestic 3 monitor Pager (legacy products): French
P 138BAA	Domestic 3 monitor Pager: English
P 138BAB	Domestic 3 monitor Pager: French
P 138BAC	Domestic 3 monitor Pager: German

One of a range of Alert-it Care Alarms available from:




UH1068B

Alert-it Care Alarms, Atherstone House, Merry Lees
Industrial Estate, LE9 9FE
0845 2179951, www.alert-it.co.uk, sales@alert-it.co.uk



After 30 sec inactivity the display returns to STANDBY with backlight off.

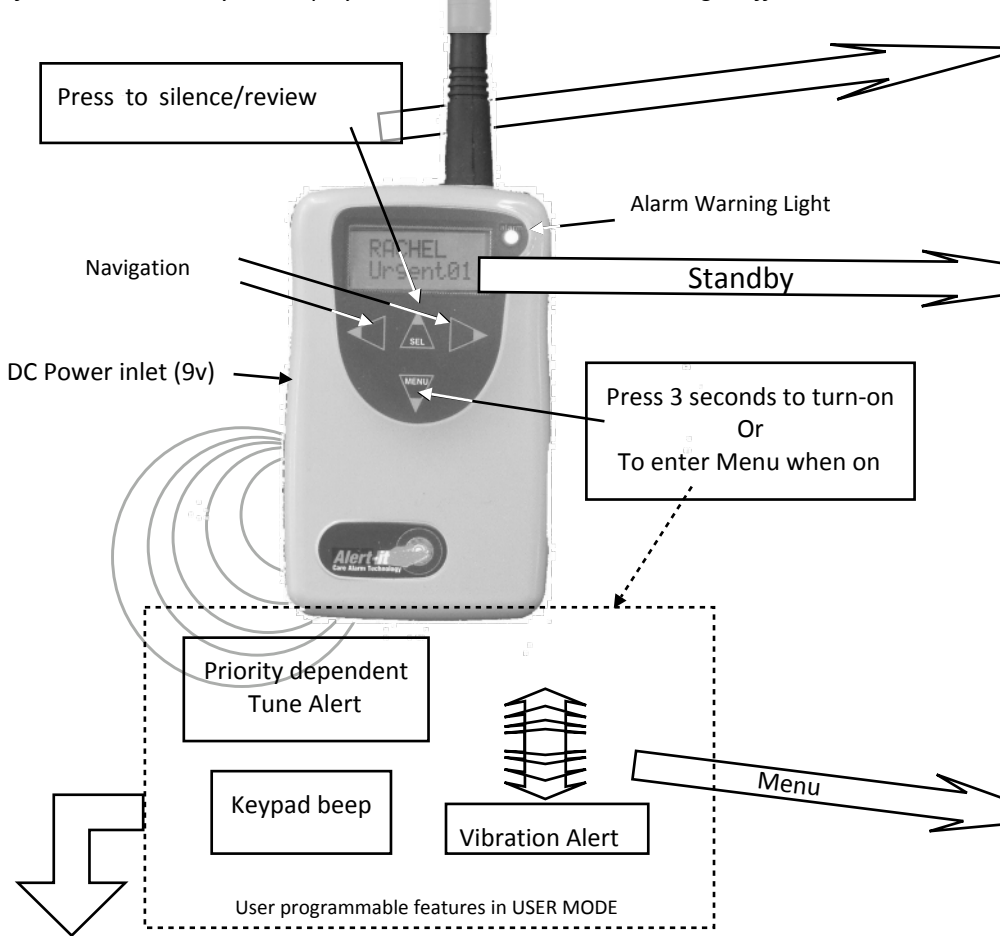


Silencing RF Fail Alarms.

The RF Fail alarm is used to warn of possible communication failure that would leave the client vulnerable. However it is also a natural consequence of turning off the client equipment, which may be required to prevent false alarms when the client is absent. In this case there is the option to silence the tune that accompanies this alarm. The display will still warn that the equipment is off, but the frequent audible alarm will no longer cause nuisance.

Any true distress alarm will over-ride the display and alert the carer as required.

New Monitors

If the name NEW! Appears on the screen, then contact the supervisor as this suggests there is an unpaired monitor in the area

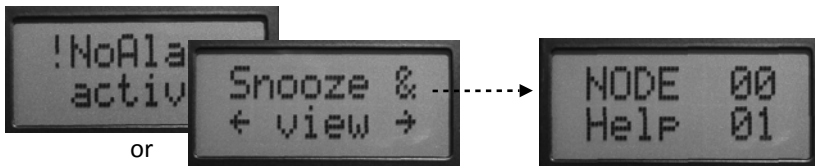

Navigate all active alarms



Also can indicate "charger"



Displays monitor names
as each transmits



Display changes when an alarm is active

Typical alarm display.
Backlight on 15 seconds

Alarm ACTIVE
monitor name

Indicates radio link has
failed and needs supervisor



Alarm Level Value



Navigate to
function with
left/right keys



Select function
with up key



Select option with left/right key



Save selection with up key



Abort with down key.
Eventually returns to
STANDBY

Extended User Functionality

The more advanced features of the Pager can be barred (by the Supervisor using a password) from the normal user by a 4 digit password. These include

- Turn the pager off
- Reset all alarms
- Modify the Name associated with a monitor

For details of these features please refer to the Installation Handbook UH1068A

Battery Maintenance

The Pager is powered by two rechargeable NiMH AA cells. These are charged in situ if:

- A 9v adapter (1.3mm pin) is connected to the DC inlet socket
- Or the unit is in the P146 USB Interface with a 12v adapter (2.1mm pin) connected.

With power applied the display will indicate “charger” in standby, instead of “battery” with two different icons for fast and standby charge.

On fast charge the pager may feel warm to touch, which is normal.

**DO NOT TURN OFF THE PAGER BY REMOVING THE BATTERIES AS THIS
CAN DAMAGE THE CONNECTIONS.**

**DO NOT USE NON RECHARGEABLE BATTERIES AS THESE CAN EXPLODE IF
THE CHARGER IS USED.**

Support

For technical support please fax or EMail:
HELP: 0845 2179951
FAX : 0845 2179953
Support@alert-it.co.uk

Designed by:
ITs Designs Ltd
Leicester
LE9 9FE UK

...using technology to care for carers

The Alert-it system has been designed with due regard to reliability and integrity. While it offers a highly vigilant monitoring method, it is always possible that a distress condition can go undetected for a variety of reasons (including malfunction) and in life threatening situations it is advisable to use the Alert-it system in conjunction with additional monitoring techniques (e.g. video). Neither the manufacturer nor its agent can accept legal responsibility to provide a system that is infallible. The carer is responsible for assessing the risks of using this equipment and any settings pertaining to it.