

# Liberty® Cyclor Training Test

1. When inserting the cassette, the plastic domes should be
  - a. Facing inward
  - b. Facing outward**
  - c. Up-side-down
2. The heater bag has the \_\_\_\_ color clamp
  - a. Blue
  - b. Yellow
  - c. Red**
3. It is necessary to mask and wash my hands before
  - a. Connecting the solution bags to the tubing**
  - b. Watching television
  - c. Turning on the cyclor
4. I should use \_\_\_\_\_ to clean the surfaces of the cyclor
  - a. A damp cloth with a dilute bleach solution**
  - b. Formula 409
  - c. Windex
5. When I travel, my Liberty cyclor can
  - a. Be checked luggage on a plane**
  - b. Be hand carried on the plane
  - c. Not go on a plane
6. When I experience a DRAIN ALARM, I will press STOP to mute the audible alarm and to display the HELP screen. After checking for possible causes of the alarm, I will
  - a. Roll over and go back to sleep
  - b. Press stop again to interrupt the treatment for 10 minutes
  - c. Press OK to continue with the treatment**
7. If the power fails during a treatment
  - a. All lines will automatically be closed, the pump will stop, and the screen will be blank**
  - b. An alarm will sound and a HELP screen will appear
  - c. Power failures cannot occur before starting your treatments
8. How do you bypass a phase of your treatment?
  - a. You may not bypass any part of your treatment
  - b. Hold the OK key for four seconds and press "Bypass" on the screen**
  - c. Bypassing only works when the machine is turned off



# Liberty® Cyclor Training Test

9. It is important to check the solution bags before use. Check for
- a. **Solution, clarity, amount, leaks, expiration**
  - b. Type of plastic, flexibility, amount
  - c. Clarity, color, cost
10. Under the 'hardware' tab of "My Settings" page, you can change
- a. Letters
  - b. **Alarm and key loudness, screen blanking and brightness**
  - c. Percentage of power used
11. Under "Patient Data", I can enter my
- a. Address
  - b. **Blood pressure**
  - c. Bank account number
12. If you are having problems with your Liberty cyclor, you should call
- a. The neighbors
  - b. **Fresenius Technical Support**
  - c. Your best friend
13. A bag will be attached to the line with the green clamp for the
- a. Medication option
  - b. **Last bag option**
  - c. Heater bag option
14. Always check your prescription for
- a. **Correct information**
  - b. Hand washing instructions
  - c. Number of unused lines
15. Your treatment will begin
- a. Once you are lying down
  - b. **Once you are connected and both catheter and blue patient line clamps are open and you press OK**
  - c. Once the machine alarms twice
16. The stay-safe® organizer
- a. **Assists you in making the connection between your catheter and the blue patient line**
  - b. Verifies that all of the clamps are open
  - c. Prevents solution from draining



# Liberty® Cyclor Training Test

17. **When moving the cyclor**  
a. You can drag it by the cord  
b. Push it with your foot  
**c. Use the handles on either side**
18. **One of the things that the OK button does on the machine is**  
**a. It saves your settings**  
b. It goes back one screen  
c. It puts the machine in "sleep mode"
19. **If you are using an IQdrive, check that it**  
a. Has been recently cleaned  
**b. Is inserted properly into the slot on the back of your machine**  
c. Is free from obstructions
20. **Be sure the power cord is**  
a. Connected to an extension cord  
**b. Plugged into a grounded, 3-prong electrical outlet**  
c. Plugged into the back of your television set
21. **When can the cassette be inserted into the Liberty cyclor?**  
**a. When the cyclor prompts you to "insert cassette"**  
b. Any time during a treatment  
c. When the machine is turned off
22. **For better visibility, when connecting the lines to the ports, the bags can be placed**  
a. On the ground  
b. On the sink  
**c. On the heater tray**
23. **Where can I find a complete listing of alarms?**  
a. In the dictionary  
**b. In the Liberty Cyclor User Manual**  
c. In the Handi-Guide
24. **Why do warnings usually occur during treatments?**  
**a. To alert you about the conditions that need to be taken care of**  
b. To disturb your sleep  
c. To initiate treatments



## Liberty® Cyclor Training Test

- 25. The range of height for the Liberty cyclor in your room is**
  - a. Just above the floor
  - b. On the windowsill to the dresser
  - c. One foot higher or one foot lower than the bed height**
  
- 26. For optimal performance, place the heater bag**
  - a. In the suitcase
  - b. Between the lips on either side of the heater tray**
  - c. Hanging over the edge of the heater tray
  
- 27. A BLUE status bar indicates**
  - a. The door sensor is broken
  - b. Patient interaction is required**
  - c. The next dwell is changing
  
- 28. If the temperature of the Delflex® bag exceeds 38°C, the Liberty cyclor will display**
  - a. Please wait for the solution to cool down**
  - b. A bypass has occurred
  - c. Press OK to start fill
  
- 29. "F" under My Records Details screen indicates**
  - a. Fun for all
  - b. Fill up the supply bags
  - c. Power failed**
  
- 30. The Liberty cyclor will not allow a large volume to be programmed**
  - a. In a short amount of treatment time**
  - b. As a time saver
  - c. If the cone is not broken



Fresenius Medical Care

Fresenius Medical Care North America  
920 Winter Street  
Waltham, MA 02451

PN# 101092-01 Rev. 00 06/2009

success@home™

