

V.A.C. Ultra™ Negative Pressure Wound Therapy Unit with V.A.C.® Therapy

V.A.C. Ultra™ Therapy Unit





Setting up the Therapy Unit

1. Attach the canister



V.A.C.® Therapy: 500 mL or 1000 mL canister (Acute Care only)

2. Connect the Dressing Tubing (after applying the dressing) Align, Push, and Twist

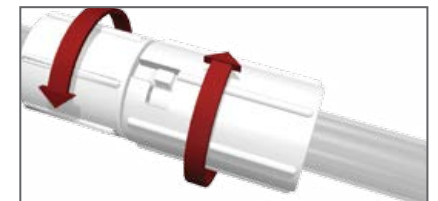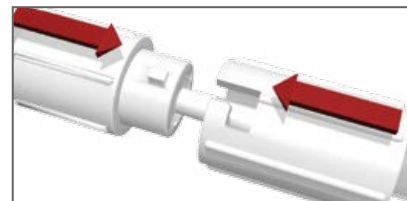


NOTE: Refer to the Instructions for Use for complete details regarding Dressing Application.

Initiating V.A.C.® Therapy

Press and hold power button until light comes on.



Choose Therapy screen

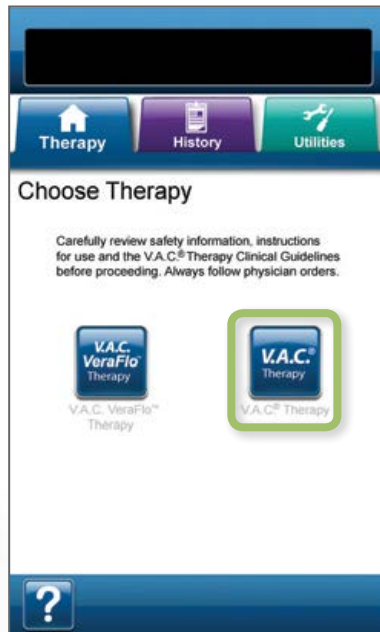


Select **V.A.C.® Therapy**

V.A.C.® Therapy Settings screen




Configure V.A.C.® Therapy as prescribed. Choose **Continuous** (default) setting and select **OK**. See below for DPC* option

Confirm Settings screen

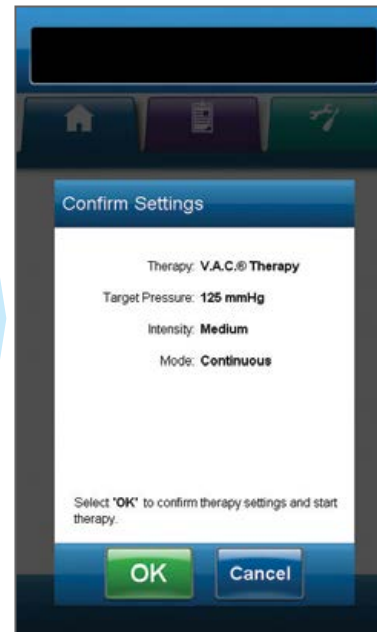

Select **OK** to accept settings

SealCheck™ Leak Detector screen

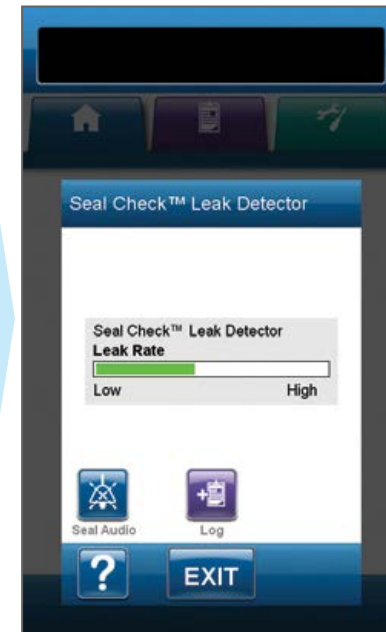


V.A.C.® Therapy Begins

Enhanced Features


Seal Check™ Leak Detector

V.A.C.® Therapy Home screen

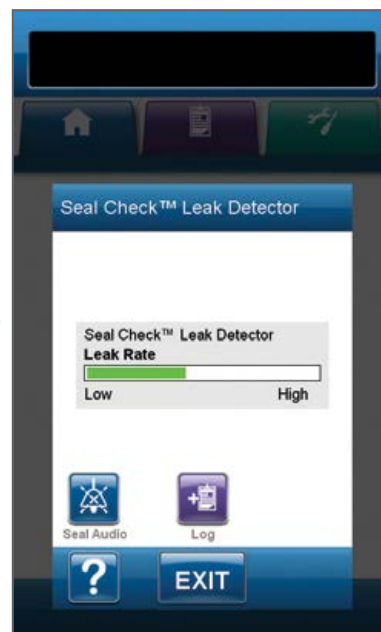

*Dynamic Pressure Control™

V.A.C.® Therapy Settings screen

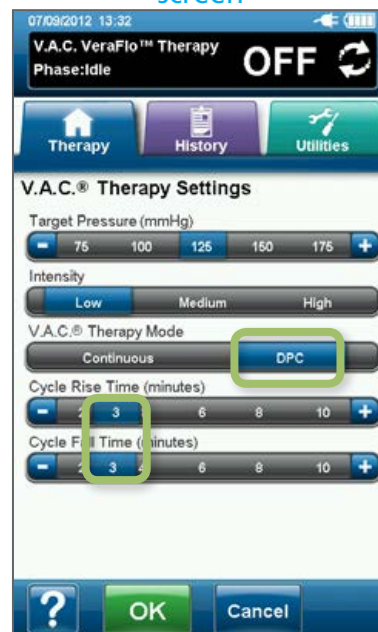

Configure V.A.C.® Therapy as prescribed and select **OK**.

Contraindications

Do not place foam dressings of the V.A.C. Ultra™ Therapy System (including both V.A.C.® Therapy and V.A.C. VeraFlo™ Therapy Dressings) directly in contact with exposed blood vessels, anastomotic sites, organs, or nerves.

V.A.C.® Therapy is contraindicated for patients with:

- Malignancy in the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistulas
- Necrotic tissue with eschar present (**NOTE:** After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used.)
- Sensitivity to silver (V.A.C. GranuFoam Silver® Dressing only)

NOTE: Refer to the Warnings and Precautions section of the Safety Information Sheet for additional information including details on potential risks associated with bleeding and infection. For complete information on indications, contraindications, and other safety information please reference the User Manual and Safety Information Sheet.

For more information, please contact your KCI Representative: _____ at _____


NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.

Important: This quick reference is provided as a basic overview only. Refer to the V.A.C. Ultra™ User Manual and Safety Information Sheet for detailed instructions, including important safety information. For medical emergencies, consult treating physician or call emergency services. Carefully read and follow the KCI V.A.C.® Therapy Clinical Guidelines and all literature and labeling accompanying product. Contact KCI at 1-800-275-4524 if you have any questions before initiating therapy. Additional information may be available at www.VACUltra.com. Rx only.


V.A.C.Ulta™ Negative Pressure Wound Therapy Unit with V.A.C. VeraFlo™ Therapy

Setting up the Therapy Unit

1. Attach the V.A.C. VeraLink™ Cassette

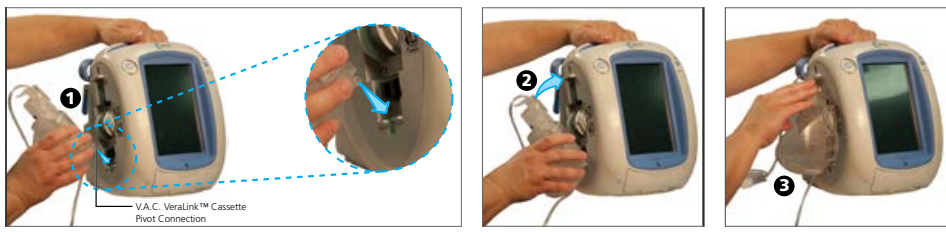


2. Extend the Solution Arm

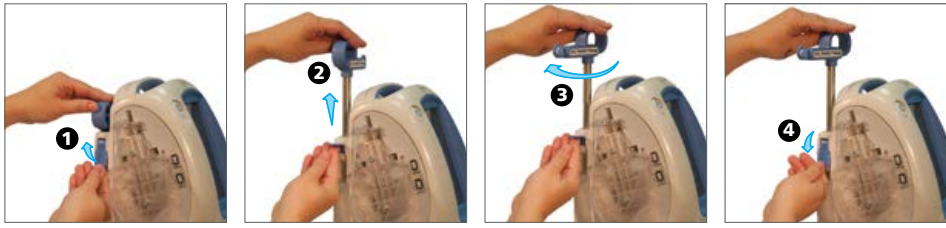


3. Attach the 500 mL or 1000 mL Canister



4. Spike and Hang the Solution Container Bag/Bottle

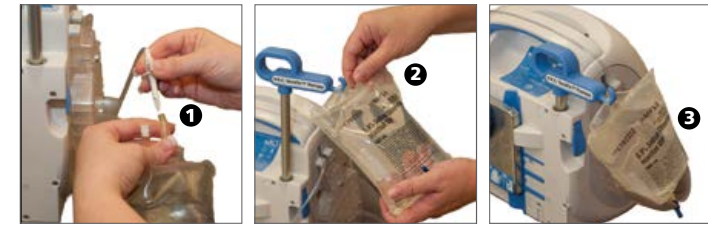

5. Connect the Dressing Tubing (after applying the dressing) Align, Push, and Twist

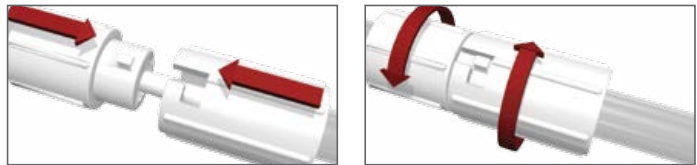

Connect the Instillation Line and then the V.A.C.® Therapy Line

NOTE: Refer to the Instructions for Use for complete details regarding Dressing Application.

V.A.C.Ulta™ Therapy Unit



Contraindications

Do not place foam dressings of the V.A.C.Ulta™ Therapy System (including both V.A.C.® Therapy and V.A.C. VeraFlo™ Therapy Dressings) directly in contact with exposed blood vessels, anastomotic sites, organs, or nerves.

V.A.C.® Therapy and V.A.C. VeraFlo™ Therapy are contraindicated for patients with:

- Malignancy in the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistulas
- Necrotic tissue with eschar present (**NOTE:** After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used.)
- Sensitivity to silver (V.A.C. GranuFoam Silver® Dressing only)

NOTE: Refer to the Warnings and Precautions section of the Safety Information Sheet for additional information including details on potential risks associated with bleeding and infection. For complete information on indications, contraindications, and other safety information please reference the User Manual and Safety Information Sheet.

Additional Contraindications Specific to V.A.C. VeraFlo™ Therapy:

- Do not use V.A.C.® Dressings with Octenisept®, hydrogen peroxide or solutions that are alcohol-based or contain alcohol.
- Do not deliver fluids to the thoracic or abdominal cavity due to the potential risk to alter core body temperature and the potential for fluid retention within the cavity.
- Do not use V.A.C. VeraFlo™ Therapy unless the wound has been thoroughly explored due to the potential for inadvertent instillation of topical wound solutions to adjacent body cavities.

*Not available in the United States. Brand name referenced is not a trademark of KCI, its affiliates, or licensors.

Initiating V.A.C. VeraFlo™ Therapy

Press and hold power button until light comes on.

Choose Therapy screen

Select **V.A.C. VeraFlo™ Therapy**

V.A.C. VeraFlo™ Settings screen

Configure V.A.C. VeraFlo™ Therapy as prescribed and select **OK**.

Confirm Settings screen

Select **OK** to accept settings

V.A.C. VeraFlo™ Therapy Begins

Enhanced Features

Fill Assist (with test cycle)

Seal Check™ Leak Detector

Dressing Soak

For more information, please contact your KCI Representative: _____ at _____

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.

Important: This quick reference is provided as a basic overview only. Refer to the V.A.C.Ulta™ User Manual and Safety Information Sheet for detailed instructions, including important safety information. For medical emergencies, consult treating physician or call emergency services. Carefully read and follow the KCI V.A.C.® Therapy Clinical Guidelines and all literature and labeling accompanying product. Contact KCI at 1-800-275-4524 if you have any questions before initiating therapy. Additional information may be available at www.VACUlta.com. Rx only.