



Mobile Shower Chair and Commode User Manual



Model no. ZT-0510B

Maximum User Weight: 125kg/19.6 stone

Overall Width: 56cm/22"

Overall Depth: 78cm/30.7"



**Please ensure that this manual is read and understood before using this Shower
Chair/Commode**

(Dec 09/1)

Introduction

This Z-Tec Shower Chair is manufactured from powder coated aluminium tubing with swing away detachable footrests and lift up arms. The back support incorporates a pushing handle for attendant use when required. A gap fronted seat allows the chair to be used as a commode or over a toilet pedestal.

Directions for Use

The product is shipped with the backrest and footrests detached. The backrest is easily fitted in to the base tube and is secured by tightening the two black hand knobs. (Photo 1)

Photo 1



Black hand knob

To fit the footrests locate the hanging posts in the base frame making sure that the left and right hangers are correctly located. Swing the footrests towards the centre of the chair until a click is heard. (Photo 2)

Photo 2



To adjust the height of the footplates slacken the height adjustment screw with a hexagon key. Adjust the footplate to the required position and tighten the screw to secure. (Photo 3)

Photo 3



Height Adjustment Screw

(Dec'09/1)

The armrests can be lifted to allow for side transfer.

The Mobile Shower Chair is fitted with four 125mm castors and is designed for the user to be positioned by an attendant. When in use the chair can be secured by locking the brakes on the castors with your foot by pressing down on the castor levers. To release move the castor lever to the up position. (Photo 4)

Photo 4



Castor lever in braked position

The toileting bowl can be removed from the rear of the chair. When refitting the bowl ensure the bowl is seated in both slides underneath the seat and is centrally positioned. (Photo 5)

Photo 5



(Dec'09/1)

Safety Precautions

Do not stand on the footrests when getting in or out of the chair as this will tip the chair forwards.

Always apply the brakes when using the chair for showering or toileting to avoid any unnecessary chair movement.

It is recommended that the user maintains an upright posture and leans against the backrest. Be aware that if the user leans too far forward in the chair and takes the weight off the rear wheels the chair could slide backwards even with the brakes supplied.

Do not sit on the front edge of the seat as this will also tend to tip the chair forward.

Cleaning

To clean, wipe over with a soft damp cloth. Use general purpose neutral detergent and warm soapy water. The shower chair may be placed under running water at low pressure or showered down to assist cleaning. Do not use abrasives or bleach based cleaners to avoid surface damage and discolouration.

Maintenance

It is important that the following parts are checked on a daily basis:

Brakes - check for effective use

Castors - check for effective use

Seat - seat is properly secured in place

Armrests - securely locked down

Footrests - secure in position

Footplates - secure in position

Warranty Information

This Shower Chair/Commode is guaranteed for 12 months from date of purchase, this warranty is not transferable. Should any part of the Shower Chair require repair, or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. The guarantee excludes all items that have been subjected to undue wear and misuse. Under normal circumstances, no responsibility will be accepted where the Shower Chair has not been maintained in accordance with the manufacturer's recommendations. This does not affect your statutory rights.

If any other issues arise that are not mentioned in this user manual please write to the address below.

Disposal – Contact your local authority to dispose of the shower chair when no longer required.

Z-Tec Mobility, Unit 1B Albion Works, Moor Street, Brierley Hill, West Midlands DY5 3SZ

(Dec 09/1)