# STORZ

**KARL STORZ — ENDOSKOPE**

**Anleitung Thorax 1/2004**

**Instructions Thorax 1/2004**

**Instrucción Tórax 1/2004**

Anleitung

Instruccions

Instrucción

Anleitung Thorax 1/2004

Instructions Thorax 1/2004

Instrucción Tórax 1/2004

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**Important information for  
users of KARL STORZ  
instruments**

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, these products are the result of years of experience and great care in manufacture. You and your organization have decided in favor of modern, high-quality items of equipment from KARL STORZ.

These instructions contain information for personnel authorized to prepare instruments for thoracic surgery. The instruments and accessories manufactured by KARL STORZ are, as a rule, sensitive precision instruments which require particular care during preparation. To prevent damage during preparation, support is provided here for disassembly, cleaning, sterilization and assembly of the KARL STORZ endoscopic instruments.

*KARL STORZ is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be derived from the information, figures and descriptions in these instructions.*

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**Important information for users of KARL STORZ instruments .....U2**  
**About these instructions ..... 4**

Using these instructions ..... 4  
Symbols and hazard information ..... 5  
Safety information ..... 6  
General instructions on cleaning and sterilization ..... 8

**Esophagoscopy –  
diverticuloscopy ..... 9**

Optical esophagoscope for adults and children ..... 9  
Optical instruments ..... 12  
Semirigid and rigid inseparable forceps .... 13  
Take-apart instruments with U-spring handle ..... 14  
Flexible forceps ..... 16  
Instruments with three-ring handle ..... 18  
Distending diverticulosopes ..... 19  
**CLICK<sup>line</sup>** Instruments with and without connector pin for unipolar coagulation ..... 20  
Coagulation and dissecting electrodes ..... 23  
General information ..... 23  
Cable connections ..... 24  
Disassembly ..... 25  
Adapter for coagulation and dissecting electrodes ..... 26  
Fiberoptic light carriers, prismatic light deflectors, clip-on light deflectors ..... 26

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Optical instruments  
see Chapter *Esophagoscopy – Diverticulo-  
scopy*  
Semirigid and rigid forceps  
*See section Esophagoscopy – Diverticuloscopy*  
Coagulation and dissecting electrodes  
*See section Esophagoscopy and  
diverticuloscopy* .....  
FREITAG stent forceps  
*See corresponding instruction manual*  
Fiberoptic light carriers, prismatic deflectors  
and clip-on deflectors  
*See section Esophagoscopy – Diverticuloscopy*

**Autofluorescence ..... 32**

Suction valve insert ..... 32  
Battery Light Source ..... 33  
Universal bronchoscope  
*See section Bronchoscopy – Tracheoscopy*  
Optical forceps  
*See section Esophagoscopy – Diverticulo-  
scopy*  
Steerable instrument guide  
*See section Bronchoscopy – Tracheoscopy*  
Fiber optic light carrier  
*See section Esophagoscopy – Diverticulo-  
scopy*

**Mediastinoscopy –  
Thoracoscopy ..... 36**

Mediastinoscope blade ..... 36  
Rigid forceps  
*See section Esophagoscopy – Diverticuloscopy*  
Rigid trocars with silicone lip valve,  
2.5 mm and 11 mm ..... 37  
**CLICK<sup>line</sup>** instruments  
*See section Esophagoscopy-diverticuloscopy*  
MATERN/WALLER Ergonomic  
Multifunction Handle ..... 38  
Surgical Sponge Holder ..... 41  
**CLICK<sup>line</sup>** Grasping Forceps, Scissors  
and Biopsy Forceps, angulating ..... 44

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RoBi® Instruments.....	48
Dissecting electrodes with exchangeable tips .....	48
Dismantling fan retractor .....	49
Handle with trumpet valve .....	50
Suction and irrigation tubes with two-way stopcock .....	51
Needle holder.....	51
Handle 30810 for irrigation, suction and coagulation.....	52
Handles for suction and irrigation Models 37112 T/37113 T .....	55
Metal suction and irrigation tubes .....	57
Needle holder and fascial closure instrument with ring handle .....	57
Needle holder with straight and curved handle, with and without ratchet .....	58

**Instruments for video-assisted thoracic surgery – lung resection 59**

Instruments for video-assisted thoracic surgery – lung resection.....	59
Coagulation electrodes <i>See section Esophagoscopy – Diverticuloscopy</i>	

**Holding systems .....60**

Articulated stand .....	60
-------------------------	----

**Maintenance .....63**

Operating/storage conditions .....	63
Disposal .....	63
Directive compliance .....	63
Limitation of liability .....	63
Warranty .....	64
Important information .....	64
Servicing and repair.....	65
Repair and replacement program .....	65

**Appendix .....66**

Cleaning Agents and Disinfectants .....	66
---	----

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**Using this manual**

This manual does not contain instructions on how to use the instruments, but instead provides general information regarding assembly/disassembly, cleaning, care and sterilization of thorax instrument sets from KARL STORZ.

This manual presumes knowledge of the 'Cleaning, sterilization and care of KARL STORZ instruments' manual, in which the various cleaning and sterilization methods are described in detail. If you do not yet have this manual, it can be requested without charge from KARL STORZ.

The section 'General information on cleaning and disinfection' contains fundamental instructions, knowledge of which is essential for correct cleaning, disinfection and sterilization.

Instruments whose cleaning, dismantling and reassembly as well as disinfection and sterilization do not require any special knowledge are not listed separately in these instructions.

Particularly sensitive product groups, such as telescopes or fiberscopes, have their own manuals. Separate instruction manuals and service manuals are available for units. Other general regulations (Normal use, User qualification, Safety rules when using instruments, etc.) must be heeded.

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**Symbols and hazard information**

Please read this manual and follow its instructions carefully. The words Warning, Caution and Note convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words Warning and Caution stand out more clearly, they are accompanied by a pictogram.

**Warning:** A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

**Caution:** A Caution indicates that particular procedures or precautions must be followed to avoid possible damage to the instrument.

**Note:** A Note indicates special information about operating the instrument or clarifies important information.

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**Safety instructions**

**Warning:** The listed cleaning, disinfection and sterilization methods alone do not guarantee disinfection or sterility. This can only be achieved if staff follows recognized and validated preparation methods.

**Warning:** New instruments are delivered unsterilized and must therefore be thoroughly cleaned, disinfected and, if necessary, sterilized prior to initial use and each subsequent reuse.

**Warning:** For all cleaning of contaminated endoscopes and instruments, the Accident Prevention Regulations (UVV) must be observed (e.g. wearing of protective gloves and safety goggles).

**Warning:** Remember that sterilization is only then successful if

- the instruments have been freed of all organic materials and cleaning residues, as only clean surfaces can be reached by the sterilization.
- work proceeds according to recognized and validated methods.
- the recommended sterilization parameters are maintained. These are only valid with sterilization equipment that is properly maintained and calibrated.

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**Caution:** Use of preparation methods other than those listed here may have a negative effect on the function and service life. Any damage resulting from this is not covered by the guarantee.

**Caution:** The following information on cleaning and sterilization only applies to the thermostable instrument set described in these instructions. For telescopes and fiberscopes, special rules apply (see instructions for telescopes and fiberscopes).

**Caution:** For the production and application of solutions, precisely observe the information from the manufacturer concerning dilution and immersion times. A maximum immersion time of 60 minutes (including in water) must not be exceeded. Prolonged immersion can lead to damage. After cleaning it is recommended to rinse the instruments with completely demineralized water.

**Caution:** Constant changing between various preparation methods is especially hard on the materials and must therefore be avoided. A one-off change of method, due to procurement of new cleaning and sterilization equipment for example, is however possible.

Observe the guidelines of the Employer's Liability Insurance Association and equivalent organizations.

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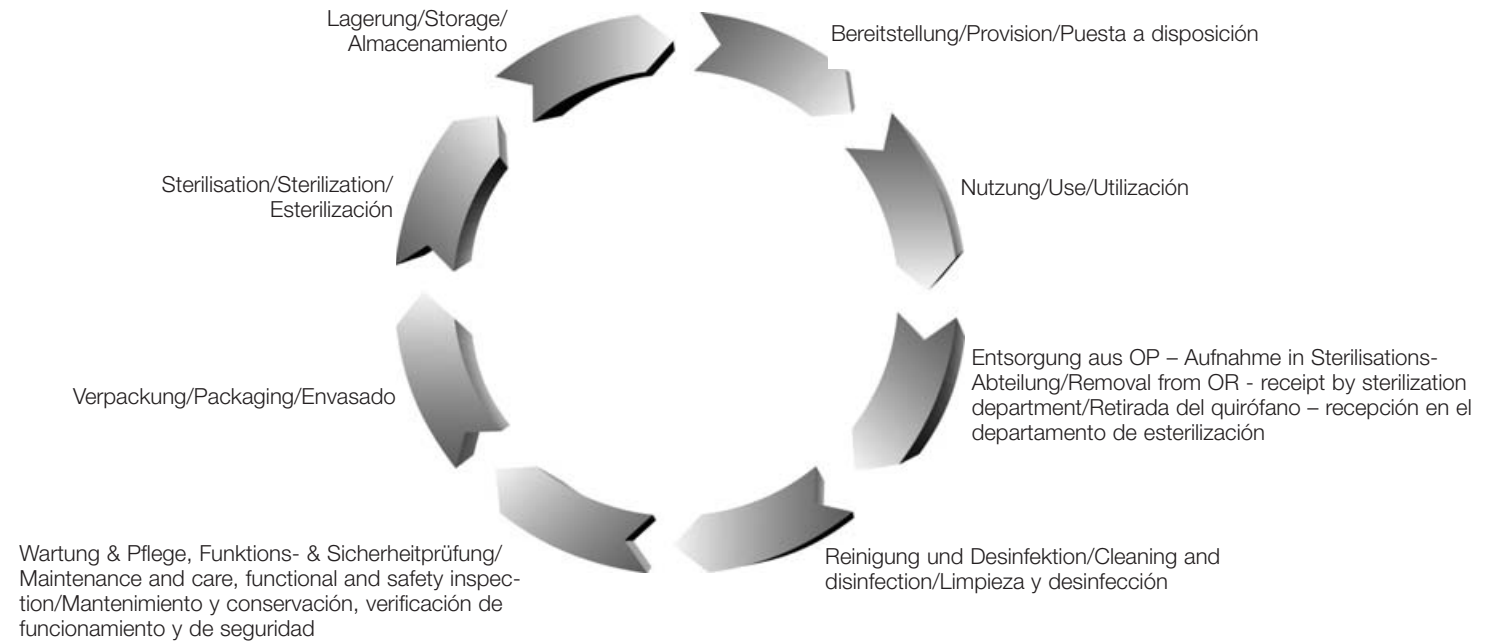
**General instructions on cleaning and sterilization**

The medical device cycle shown below illustrates the sequence of steps for advisable instrument preparation:

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According to the instrument cycle, preparation of the instruments described here generally comprises the following 6 steps:

**Caution:** The instructions for cleaning and sterilization in the manual 'Cleaning, sterilization and care of KARL STORZ instruments' must be observed. The methods for cleaning, disinfection and sterilization are explained there in detail.

**Note:** Recommended cleaning accessories and sterilization containers can be found in the catalog chapter 'Cleaning, sterilization and care'.

### 1. Manual preliminary cleaning

The instruments may require manual preliminary cleaning on account of their design (e.g. slits and long, narrow lumens) and depending on the type and degree of soiling.

In addition, a prolonged waiting time before preparation (which allows the contamination to dry on) may make preliminary cleaning necessary while still in the operating room. Organic residues and pharmaceutical products, for example, must be removed mechanically (by brushing, rinsing through with a water gun, etc.), and the instruments must subsequently be rinsed off with completely demineralized or distilled water.

Used and contaminated instruments should always be prepared with the minimum possible delay.

Separable instruments must be disassembled following the instructions. Articulating instruments and scissors must be treated whilst open.

Tissue remnants sticking or baked to the jaws must be removed with a cleaning brush, and the instrument must be precleaned with a 3% H<sub>2</sub>O<sub>2</sub> solution as soon as possible after use. Special care must be taken when cleaning the jaws, as they are precision-engineered parts.

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## 2. Ultrasound treatment

All non-separable scissors, coagulation instruments and microinstruments must be pre-cleaned with ultrasound (3 - 5 minutes exposure time). The cleaning effect of the ultrasound bath reaches even inaccessible hollow spaces. For this reason, all non-separable scissors and forceps, with or without irrigation channel, rail-sheath instruments, aspirators, circular punches, coagulation instruments and microinstruments must be preliminary cleaned with ultrasound. Ultrasound has no cleaning effect on soft plastic parts because the oscillations are attenuated.

Hollow instruments (aspirators etc.) should be held at an angle in the ultrasound bath so that the hollow spaces can fill. Position large instruments in the bath so that no acoustic shadows are formed. The following items, amongst others, are not suitable for cleaning in an ultrasound bath: all rigid and flexible telescopes, electrical devices, cables, prisms, rubber articles, magnetic discs and fiberoptic light cables.

After cleaning, rinse thoroughly with completely demineralized water and then dry immediately.

Place the instruments in a suitable instrument carrier (e.g. a tray holder).

*Note: If manual preliminary cleaning is mandatory, the necessary steps are explained in this manual for each instrument*

## 3. Manual/machine cleaning and disinfection

For preparation, multi-part instruments must be dismantled, and seals, cock plugs and protective caps removed.

### Manual cleaning/disinfecting cleaning

For manual cleaning or disinfecting cleaning, use active-cleaning, non-protein-fixing chemicals, where necessary with a verified range of action (DGHM (German Association of Hygiene and Microbiology) listing or comparable).

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**Caution:** When preparing and using the solutions follow the manufacturer's instructions paying close attention to proper dilution and immersion times.

**Note:** A list of products which have been tested and shown not to attack the surface and approved by KARL STORZ for manual cleaning and disinfection can be found in the appendix of this manual and on the KARL STORZ home page.

When immersing instruments in cleaning and disinfectant solutions, ensure that air can escape from hollow spaces so that all the internal surfaces are completely wetted. It is best to immerse the instruments at an angle until no more air bubbles can be seen. During cleaning, always ensure that all hollow spaces (sheath lumens, stopcocks, valves etc.) are also cleaned and that there are no obstructions (flow test, for example with a syringe).

Thoroughly clean surfaces, interiors and jaws with a suitable soft brush, sponge or cleaning gun, and rinse. Take care when cleaning small parts.

The cleaning gun with accessories (Art. no. 27660) is ideal for rinsing out hollow spaces with water and drying them with sterile compressed air.

After cleaning, rinse the instruments copiously with completely demineralized water and dry with sterile compressed air.

#### **Thermal cleaning and disinfection in a machine**

As far as possible, thermal cleaning/disinfection should be carried out in a machine, followed by steam sterilization at 134°C. The advantage of preparation by machine is that it reduces the risks for personnel and standardizes the cleaning procedure (validatable or validated processes are demanded).

As there is sometimes strong movement of the water, it is necessary to secure all parts in holding devices to avoid damage. To ensure that the instruments are adequately rinsed out, it is essential to connect up the hollow spaces appropriately to the washer/disinfectant.

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It is not advisable to use lubricants etc. in the machine because they may cause deposits to form on the instruments and may damage plastics. Selective manual care and lubrication of joints etc. is safer and more effective.

It is necessary to perform a final thorough rinse with completely demineralized water in order to remove all chemical residue, as such residue can lead to chemical cross reactions or the formation of deposits during final sterilization.

**Note:** A list of products tested and approved by KARL STORZ for machine cleaning/disinfection can be found in the appendix of this manual and on the KARL STORZ home page.

#### 4. Functional testing

All instruments should be checked for correct functioning before sterilization, at the latest. This functional check should include the following aspects in particular:

- Success of previous cleaning / signs of residual dirt (as a rule via visual checks of the critical areas)
- Smooth operation of the joints
- General signs of wear
- Exact closing of forceps, clamps and punches etc.
- Secure closing of valves and seals
- Sharpness of cutting instruments
- Condition of telescopes
- Condition of the insulating sheaths of cables, including cold light cables
- Broken fibres in cables

Damaged instruments must be separated out and sent for expert, qualified repair. For this purpose we recommend the KARL STORZ Repair Service, which ensures consistent quality and guaranteed safety of your instruments and units.

Prior to sterilization, instruments with a closure mechanism or joint must be treated with lubricants capable of withstanding steam sterilization (e.g. instrument oil Art. no. 27656 B).

**Note:** Apply a thin coat of special grease to sealing and running surfaces, such as stop-cocks and plungers (Art. no. 27657). This thin layer of grease not only serves to lubricate the metal surfaces which are in contact with one

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another, but also makes the stopcocks leakproof. Oil is not suitable as a sealant.

**Warning:** Only use special white oil-based lubricants as the sterilant is not able to penetrate all types of oil in order to sterilize the underlying surfaces.

**Note:** Leave the stopcocks dismantled for sterilization and only insert them into the housing once sterilization is complete.

## 5. Sterilization

For almost all instruments, steam sterilization by the fractionated vacuum method at 134°C (3.04 bar.), gas sterilization (formaldehyde, ethylene oxide), plasma sterilization and chemical sterilization (STERIS) are possible. Steam sterilization at 134°C is usually the gentlest and most economical method of sterilization. This also applies to rigid telescopes because the longer exposure time for steam sterilization at 121°C can have a negative effect.

Sterilization is only ever possible on surfaces that have been cleaned.

Almost all instruments can be sterilized without being dismantled. Only tighten screws and closures gently, so that they are easy to undo later.

**Note:** If the design of the instrument necessitates sterilization in the dismantled state to ensure effectivity, this is expressly stated for the article in question in this manual.

For steam sterilization (with saturated steam at 134°C + 3°C), the process defined in EN 285 or DIN 58946 Part 1, Number 3.25.2 should be used. Use prevacuum methods for instruments with a lumen. The steam must fulfill the requirements of EN 285, Appendix B, in order to avoid discoloration and stains.

These procedures are also permissible for telescopes labeled 'AUTOCLAVE'.

**Caution:** When sterilizing telescopes, please follow the instructions in the Telescopes manual.

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**Warning:** Long lumens, slits and channels, and assembled instruments can only be sterilized without restrictions with the fractionated prevacuum method.

**Note:** If it is necessary to deactivate the pathogens responsible for Creutzfeldt-Jakob disease, the steam sterilization method recommended by the WHO can be used (18 min at 134°C), the sterilization time may be extended to up to an hour at 134°C in certain cases.

As there are various types of cleaning, disinfection and sterilization, it is essential to ensure that recognized and validated (tested) preparation methods are used.

Refer to the manual 'Cleaning, sterilization and care of KARL STORZ instruments' for more detailed information.

## 6. Storage/Transportation

To avoid damage during transportation and storage, place the instruments carefully in suitable holding devices.

Suitable accessories, storage systems, and sterilization trays for special instrument sets can be found in the relevant chapter of every KARL STORZ catalog.

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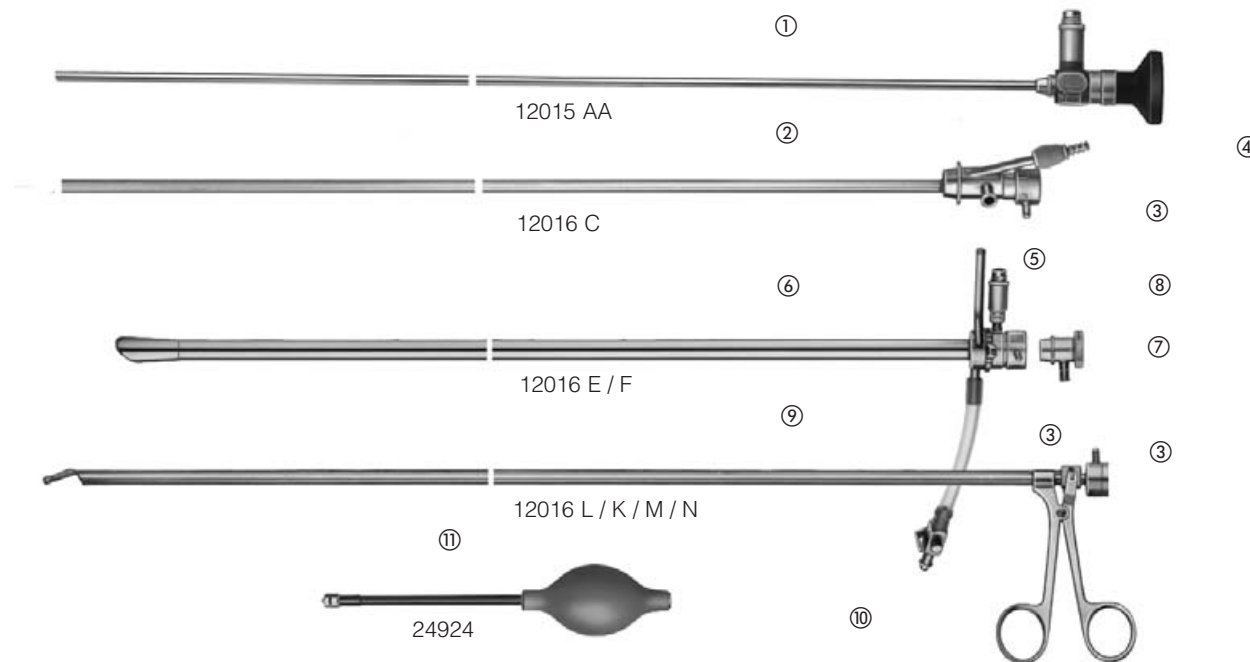
**Optical esophagoscope for  
adults and children**

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- ① Telescope
- ② Tube insert
- ③ Unlocking lever
- ④ Instrument channel
- ⑤ Anti-fog channel
- ⑥ Esophagoscope tube
- ⑦ Cap with observation window
- ⑧ Connection point for fiberoptic light carrier, including 2 unscrewable adapters
- ⑨ Optical forceps
- ⑩ Air inflation facility
- ⑪ Rubber bulb for air inflation

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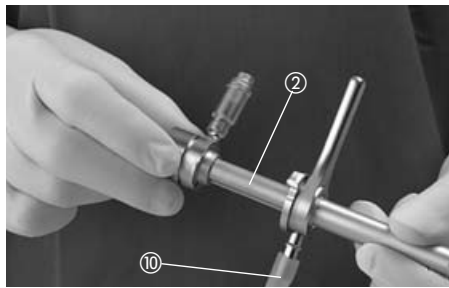
**Disassembly:**

Open the telescope lock (3) by giving it a ¼ turn counterclockwise. Remove telescope (1) or cap (7).

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
If necessary, undo the lock (3) of tube insert (2) or the optical forceps (9) by giving it a ¼ turn counterclockwise. Remove the tube insert/instrument. Remove silicone hose of the air inflation device (10) from the suction outlet.

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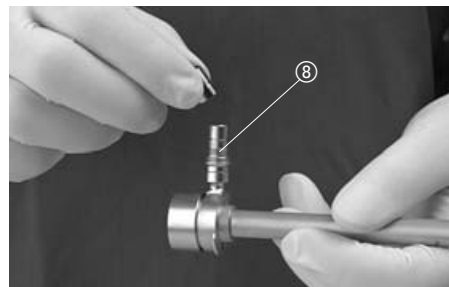Remove the two adapters from the fiberoptic light cable connection point (8). Lubricate the contact surface of the locks using KARL STORZ special grease. Oil moving parts with KARL STORZ instrument oil.

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**Cleaning:**

To clean the instrument, completely dismantle the esophagoscope. Clean the outer surfaces with a soft sponge. Clean out the lumen thoroughly with a brush. Rinse with water and dry with sterile compressed air. Then clean the front and rear surfaces of the prism with a soft cloth, sponge, or cotton swab impregnated with 70% alcohol.

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**Assembly:**  
Screw the fiberoptic adapter onto fiberoptic cable connection point (8).  
Slide tube insert (2) at the 12 o'clock position into the esophagoscope tube (6). Close the lock of the tube insert by giving it a ¼ turn clockwise.

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Slide telescope (1) into tube insert or fit cap (7) and engage the lock.  
Slide on the air inflation silicone hose (10). If necessary, connect up inflation bulb (11) to the air inflation device (10).

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**Pediatric esophagoscope**  
Remove rubber cap, check for damage and if necessary replace with a new one.  
Clean the exterior with a soft sponge; thoroughly clean the lumen with a brush ...

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... rinse out with water and dry with sterile compressed air.

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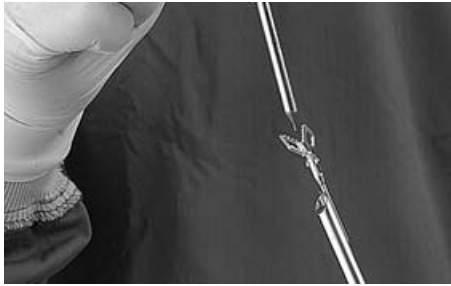
**Optical instruments**

Clean the exterior of the forceps with a sponge and the jaws with a brush. Brush out the telescope channel. Then clean in an ultrasound bath. Rinse out with water and blow dry with sterile compressed air.

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Oil the joint parts of the forceps jaws. Hold the instrument so that oil can flow into the joint from the distal end and the wire pull is also wetted.

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The joints on the handle also have to be supplied with oil.

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**Semirigid and rigid inseparable  
forceps**

Clean the sheath of the semirigid or rigid instruments with a sponge.

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Clean the parts of the jaws carefully with a brush. Then clean in an ultrasound bath. Rinse with water and blow dry with sterile compressed air.

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Finally oil the joint of the jaws of the instrument, holding the instrument so that the oil can seep into the jacket and thus also wet the wire pull.

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**Take-apart instruments with  
U-spring handle**

**Disassembly**

Press the guide wire ① downward and then detach from the grip.

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Pull out the insert ② toward the front.

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Remove the seal, check for damage and replace if necessary.

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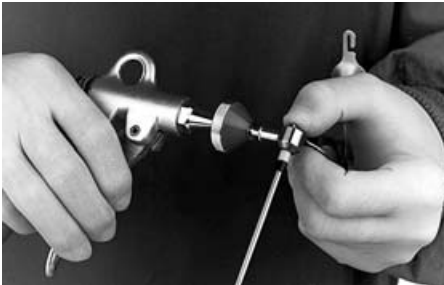
Connect a syringe or cleaning machine to the LUER-Lock fitting and rinse out with water. Dry with sterile compressed air.

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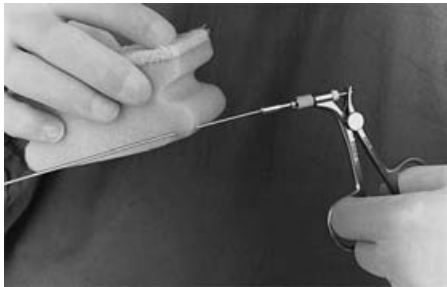
**Assembly**

Reassemble the instrument in reverse order.

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**Flexible forceps**

Clean the sheath of semirigid instruments with a sponge.

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A brush is more suitable for cleaning the outer spiral of flexible instruments.

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Carefully clean the jaws with a brush.

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**Disassembly**

Disassemble the handle piece. Undo the knurled screw ...

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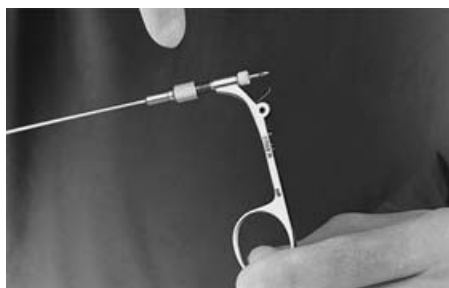
... and remove the rear handle part from the groove of the forceps insert.

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Pull off the front handle part toward the rear.  
Then clean in an ultrasound bath.  
Flush through with water and dry with sterile compressed air.

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Then oil the joint of the forceps jaws, holding the instrument so that the oil can flow into the jacket and thus also wet the wire pull.

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**Assembly**

Oil the wire pull from the proximal end.  
Afterwards reassemble the instrument in reverse sequence.

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**Instruments with three-ring handle**

To disassemble first undo the rear knurled screw.

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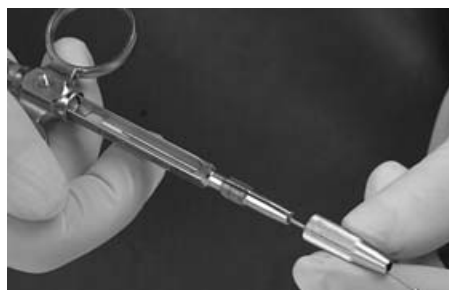
Undo the knurled screw on the lower part of the handle (turn counterclockwise).

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Remove insert.

Clean as described for „Forceps with U-spring handle“ (section on Esophagoscopy - diverticuloscopy).

To reassemble, place the insert in the guide groove on the handle.

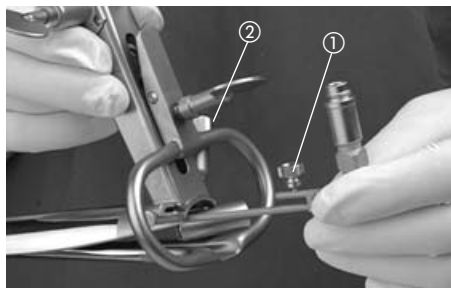
First tighten up the rear knurled screw, and only then lock the lower screw.

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**Models 12067 A / B / C / V**

**Disassembly:**

Undo knurled screw ① and remove light carrier.

On instruments with telescope holder: undo knurled nut ② and remove telescope holder.

**! Caution:** Hold with the handle pointing upward because the knurled nut contains individual parts.

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**Cleaning:**

Clean the external surface with a sponge or brush. Rinse with water.

Dry with sterile compressed air.

**Assembly:**

Attach light carrier. Oil moving parts.

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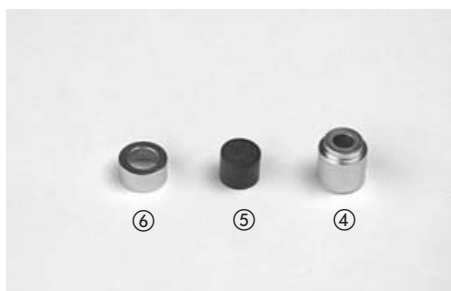
If telescope holder is fitted:  
Position ball of telescope holder ③.

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The individual parts of the telescope holder are assembled in the following order (from bottom up):

- Ball ③
- Brake cylinder, comprising
  - sleeve ④
  - rubber bolt ⑤
- Sleeve ⑥
- Screw cap ⑦

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**CLICKline® instruments with and without connector pin for unipolar coagulation**

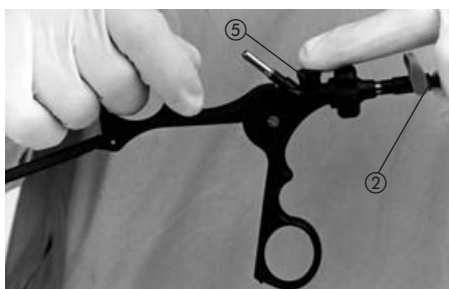
**Disassembly**

Open handle completely. If necessary, unlock the locking mechanism on the handle.

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
Press and hold the locking button ⑤. The handle is now opened completely in order to push the sheath tube ② out toward the front.

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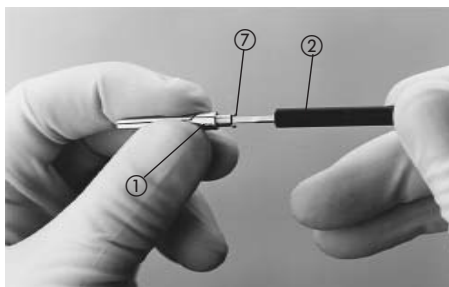To remove the internals, hold the jaws ① tightly and open the bayonet mount ⑦ by rotating the sheath tube ¼ turn. Pull out the internals.

**⚠️ Warning:** Grasp pointed or sharp jaws with a cloth.

**Caution:** Please note that with instruments in the 2 and 3 mm sizes and curved instruments the internals cannot be removed from the sheath tube.

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**Manual cleaning**

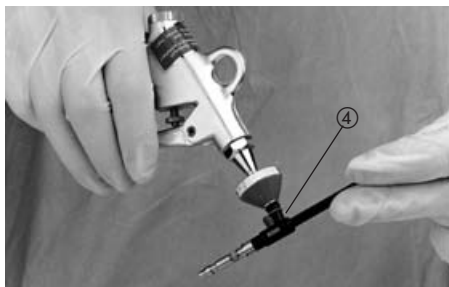
Remove tissue residue with a cleaning brush. Brush and rinse out sheath tubes on the inside. Blow dry with sterile compressed air.

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**Machine cleaning**

Make certain all parts are positioned securely so that damage during cleaning is prevented. The sheath tube must be rinsed out on the inside. By means of the LUER-lock irrigation connection ④, easy attachment to a syringe or cleaning machine is also possible.

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**Assembly**

Check all parts for damage. Replace if necessary. Check jaws and joints especially carefully, and additionally the insulation on instruments for HF use. Insert the internals from the front into the sheath tube.

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Lock the internals by rotating ¼ turn in the bayonet mount on the sheath tube.

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Insert the sheath with the jaws closed as far as it will go into the completely opened handle.

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Then lock the sheath tube by closing the handle completely.

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Place a drop of instrument oil (art. no. 27656) in the joints.

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**Coagulating and Dissecting  
Electrodes: General information****Cleaning of coagulation instruments during  
surgery**

The coagulation surfaces on coagulation instruments should be wiped off with a moist compress between uses during surgery. Incrustations may be loosened by briefly immersing in a 3% H<sub>2</sub>O<sub>2</sub> solution. Before reusing, rinse with sterile water and wipe off again.

**Manual cleaning**

Remove sticky tissue remnants on the tip carefully with a cleaning brush.

Sheath tubes with lumen must be rinsed out well on the inside.

Dry with sterile compressed air, particularly in the area of the plug.

**Note:** The HF cable and electrode connectors must be clean and dry. Moisture and dirt or residue from disinfectants may lead to malfunctions during operation.

**Assembly and care**

Incidents during high-frequency surgery which have resulted in damage to the instrument set or even injuries to the user or patient, may be traced back, under closer examination, to incorrect use or defective equipment. A frequent source of errors here proves to be damaged cable and/or electrodes. A further cause may lie in the use of unsuitable accessories from other manufacturers which are not compatible with KARL STORZ units. Safe and proper use of KARL STORZ instruments is only possible using KARL STORZ high-frequency cables and electrodes.

When testing for proper operation, check the insulation material especially for damages such as cracks and chipped material.

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### Cable connections

**Warning:** The use of damaged HF cables may lead to very serious dangers. Check the cable before every use for visible damage. Damaged cable must never be reused.

HF cables are subject to a certain degree of wear, particularly with improper handling and maintenance. For example, damage to the cable terminal or cable insulation may lead to HF voltage flashovers on the working element and therefore to the metallicly conductive parts of the instrument. During patient application the parts of the instrument at HF potential then become dangerous to the patient and surgeon.

Careful inspection of the HF cable used is therefore absolutely necessary before every application.

Particular attention must be paid to

- burn marks on the cable terminal
- defects on the connectors
- breaks in the cable or cable insulation
- missing pin for closing the second hole on the cable terminal.

If damage is found or suspected on HF cable, it must be discarded.

### Cleaning high frequency cables

Clean with a brush or a sponge.

All KARL STORZ high frequency cables may be autoclaved up to 134°C or gas sterilized.

Disinfection by immersing in solution is only possible under certain circumstances: a) the electrical contacts must be dried afterwards with compressed air, b) steam sterilization afterwards in a fractionated prevacuum.

Moisture or residue from disinfectants on the plug-in connections could lead to malfunctions.

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**Coagulation and Dissecting  
Electrodes**

**Disassembly**

Separate the electrode ② from the handle ① by undoing the knurled nuts ④.

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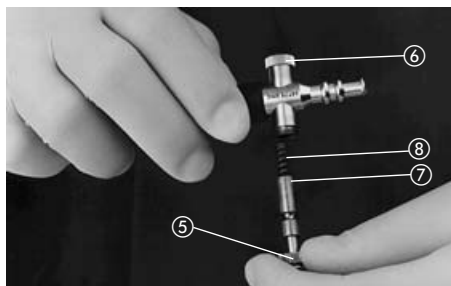
Remove plunger ⑦.  
Take spring ⑧ out of the plunger.

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**Assembly and functional test**

Lightly grease plunger ⑦.  
Insert spring ⑧ into plunger ⑦.  
Screw knurled nut ⑤ tightly.

Insert plunger with spring into valve.  
Screw knurled nut ⑥ tight with plunger ③ slightly depressed.

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**Assembly**

Screw the electrode into the handle ①. Fix in place with the knurled nut ④.  
The knurled nut ⑨ is used to fix the rotatable sheath in position.

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**Adapter for coagulation and dissecting electrodes**

To use 5 mm sized coagulation or dissecting electrodes on handles intended for 10 mm suction/irrigation tubes, an adapter is required.

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**Fiberoptic light carrier, prismatic light deflector, clip-on light deflector**

**Cleaning:**

Remove the light carrier connecting adapter. Clean with a sponge.  
Dry with sterile compressed air. Clean the light inlet and outlet surfaces with a soft cloth, sponge, or cotton swabs soaked in 70% alcohol.

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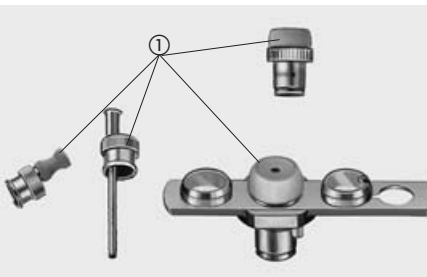
**Bronchoscopes**

Remove all accessories.  
Clean the external surface with a sponge.  
Brush out the lumen. Rinse with water and dry  
with sterile compressed air.

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**Accessories for bronchoscopes**

Clean accessories with a brush. If there are  
sealing caps ①, remove and replace if neces-  
sary. Brush out channels, rinse with water, and  
dry with sterile compressed air.

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**Steerable instrument guide and control instrument**

Remove sealing cap from connection point,  
check for damage, and replace if necessary.  
Clean external surface with a sponge. Clean  
the distal end carefully with a brush.

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Brush out the lumen of the sheath. Then clean  
in an ultrasound bath. Rinse with water and  
blow dry with sterile compressed air. Oil the  
joints.

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**TONN Stent Applicator**

Remove contaminants with a sponge or a soft brush. Following this, rinse out thoroughly with completely demineralized water and dry with sterile compressed air.

The instruments can be steam, gas or plasma sterilized, or immersed in solution at temperatures of up to 134°C.

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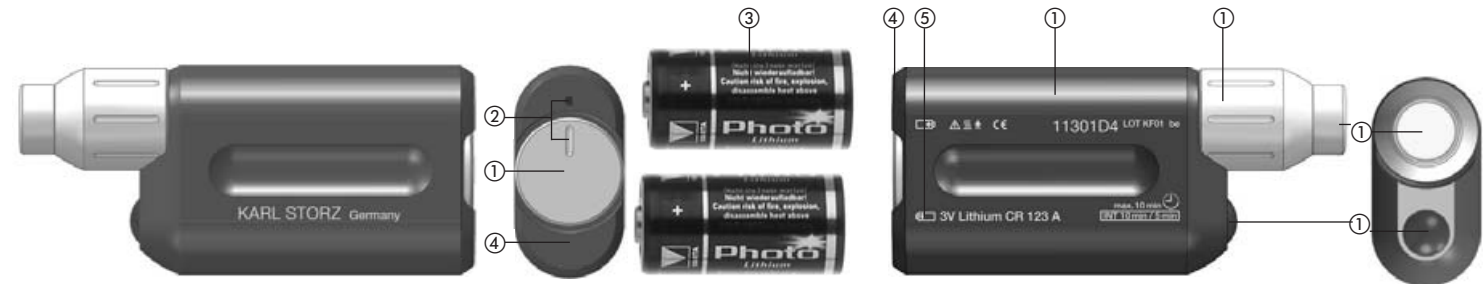
- ① Locking mechanism for the cover
- ② Mark: locked
- ③ Batteries 2 x 3 V/CR 123 A (Art. no. 121306 P)
- ④ Cover
- ⑤ Mark: battery symbol with polarity
- ⑥ Housing
- ⑦ Screw connection on endoscope/fiberscope light inlet
- ⑧ Light outlet
- ⑨ Button (on/off)

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**Cleaning and disinfection**

**Caution:** *It is essential to prevent any penetration of liquid into the housing.*

Wipe down the battery light source with a wet cloth soaked in standard surface disinfectant.

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**Changing batteries**

**Caution:** *The batteries ③ are loose in the housing and could fall out!*

Open the cover ④: Put your thumbs on the cover's ① locking mechanism, turn counterclockwise and remove the cover from the housing ⑥.

Always change both batteries at the same time; take out both batteries ③ and insert two new batteries, making sure to observe the polarity (see battery symbol ⑤).

Put the cover ④ back on. Turn the cover's locking mechanism a clockwise as far as it will go. The marks ② are now aligned and the cover is locked.

Dispose of old batteries in an environmentally friendly way.

Check that the battery light source is functioning.

**Warning:** *Ensure that the battery light source is switched off after every functional test.*

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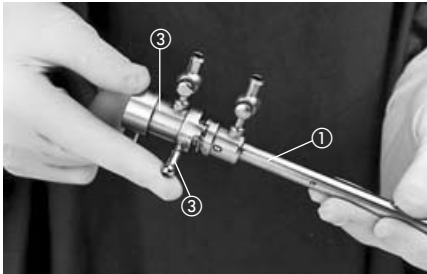
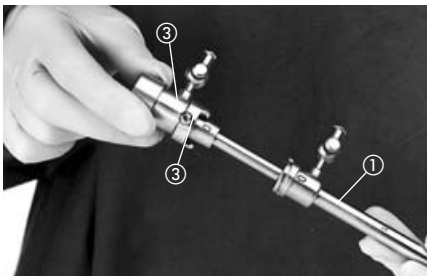
After loosening the locking ring ③, remove the outer sheath ① from the inner sheath ②.

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**Cleaning**

Remove sticky or caked tissue remnants with a cleaning brush. Clean small parts carefully. Clean the instrument in an ultrasonic bath. Rinse out sheath tubes. Dry with sterile compressed air.

**Assembly**

Slide the outer sheath ① onto the inner sheath ②, fix in place with the locking ring ③. Disassemble again for sterilization.

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**CO2 laser coupler**

**Disassembly**

No disassembly is necessary. The brake a for the mirror lever b must not be completely unscrewed.

**Cleaning**

**Caution:** Do not allow any moisture to enter the coupler to prevent deposits on the optical components.

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**Manual cleaning**

The laser coupler may be wiped off on the outside with a cloth soaked in a cleaning and disinfectant solution. No liquid must enter the unit. Clean the exterior glass surfaces with alcohol.

**Machine cleaning**

Machine cleaning is not permitted.

**Adapter for CO2 laser coupler**

The adapters may be cleaned and disinfected like a normal instrument set.

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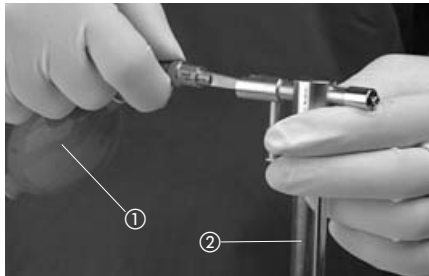
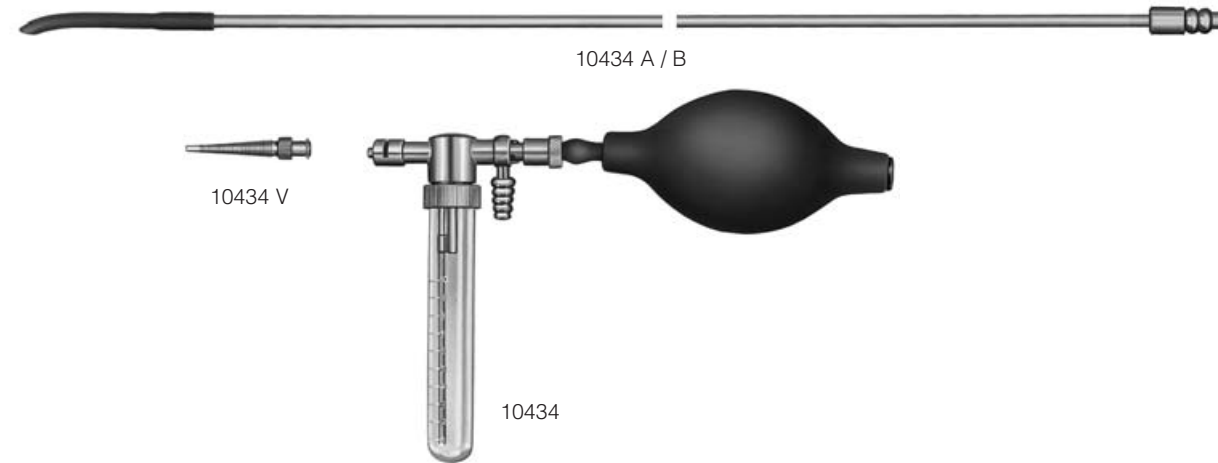
**Aspirator und Bronchuspüler**

**Aspirator and bronchial irrigator**

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**Disassembly**

Detach rubber bulb ①. Undo the collecting bottle ②.

**Cleaning**

Clean the external surfaces with a sponge or brush. Rinse off with water. Dry with sterile compressed air.

**Note:** In the case of the rubber bulb only disinfect the surface; it must not be autoclaved.

The plastic collecting bottle can be autoclaved a number of times at 121 °C.

**Assembly:**

Attach the rubber bulb and collecting vessel.

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**HASLINGER/LEMOINE Head Rest**  
Models 10900 A / B

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- ① Handwheel with crank (self-locking)
- ② Fixing screw for swivel joint
- ③ 2 fixing screws for head tray
- ④ Mount for table top
- ⑤ Swivel joint
- ⑥ Toothed quadrant rod

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**Handling**

**Warning:** Familiarize yourself with the use and operation of the head rest before initial application on the patient.

Check the head rest for functional capability before every application.

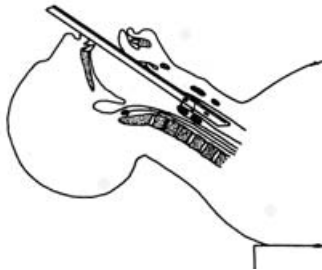
**Warning:** Make certain the patient's head is securely positioned. Take appropriate measures, if necessary, to secure the patient's head from sliding off the head rest.

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By lifting the head above the level of the positioning surface (approx. 5-15 cm, depending on the patient), while simultaneously tilting the occipital joint, the cervical spine is stretched. At the same time, the cervical muscles relax and the bronchoscope experiences much less resistance at the base of the tongue and the upper row of teeth during introduction. The mobility of the head rest is quite versatile. The movements of the head necessary to adjust individual segment bronchia can be performed more exactly by the examiner himself/herself than would be possible with an assistant holding the head.

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**Note:** There are several versions of the HASLINGER/LEMOINE head rest available for attachment to different makes of operating table. You will find a current list in the "THORAX" catalog.

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**Lerut DCI® Video Mediastinoscope**

Clean the exterior of the blade with a sponge.

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Thoroughly clean the lumen with a brush and then place in the ultrasonic bath.  
Rinse with water and clean with sterile compressed air.

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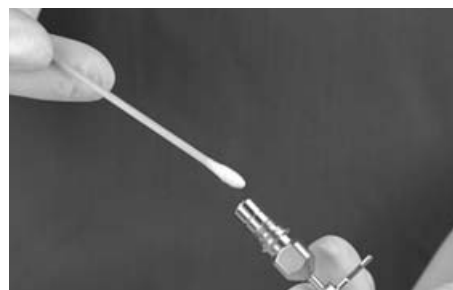
**Mediastinoscope blade**

Detach light carrier if fitted, remove light carrier adapter, clean the external surface of the blade with a sponge and then in an ultrasound bath.  
Rinse with water and dry with sterile compressed air.

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Clean the front and rear faces of the prism with a soft cloth, sponge, or cotton swab soaked in 70% alcohol.  
Attach the light carrier.

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**VERESS pneumoperitoneum cannulas**

**Note:** Disassembly, cleaning and assembly of LUER-lock stopcocks see "Instructions for LUER-lock stopcocks" (page 22)

**Disassembly**

Unscrew the outer tube ① by rotating the knurling ② on the stopcock housing ③ and pulling off towards the front.

For manual cleaning, the stopcock is only disassembled after the inside tube has been rinsed out.

**Manual cleaning**

Connect a hose to the LUER connection and rinse out the inside of the sheath tube well. Keep rinsing until clear water escapes from the distal end.

Dry with sterile compressed air.

**Machine cleaning**

The tubes must be rinsed out on the inside. To do this, an irrigation hose from the machine is connected at the LUER connection. Slide a suitable hose onto the knurling over the outer tube.

**Assembly**

Check the needle tip for damage. Blunt and broken off needles must be replaced.

Lightly grease the cock plug and assemble. Check the movement of the spring mechanism ④. Slide the outer tube ① from the front onto the inner tube ⑤ and screw together. Again, check the movement of the spring mechanism.

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**Trocars with multifunction valve**

**Note:** Disassembly, cleaning and assembly of LUER-lock stopcocks see “Instructions for LUER-lock stopcocks” (page 22)

**Disassembly**

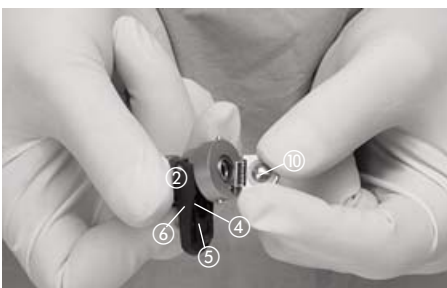
Remove trocar tongue.  
Take off seal bonnet, check for damage; replace if necessary.  
Loosen bayonet mount ① on the trocar cannula. Take off the top of the valve ②.

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
The top of the valve ② may be further dismantled in order to replace the small sealing cap, the tappet d or the inside seal ⑩

To do this, pull out the pusher ⑤ on the lever ⑥ on the top of the valve ②.

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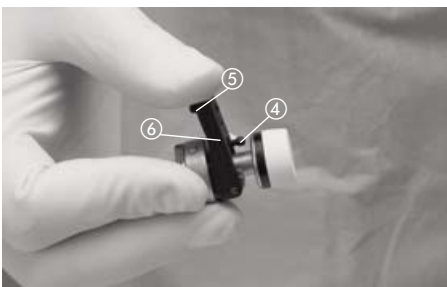Keep the lever ⑥ pressed down and pull out the tappet ④ towards the back.  
For trocars size 6 mm (color code: black) and size 7 mm (color code: yellow), first unscrew the rear top part.

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**Note:** It is easier to unscrew the rear top part if the sealing cap is in place.

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Check the inside seal ⑩ for damage, replace if necessary. To do this, turn the screw ⑪ over the seal clockwise (left-handed thread).

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**Manual cleaning**

Hold the valve flap open when cleaning the top of the valve.

**Caution:** Clean the valve flap carefully, since misalignment of the valve flap prevents smooth functioning.

**Machine cleaning**

Fix the flap in an open position using the opening rod (art. no. 26 G) or a suitable holding device.

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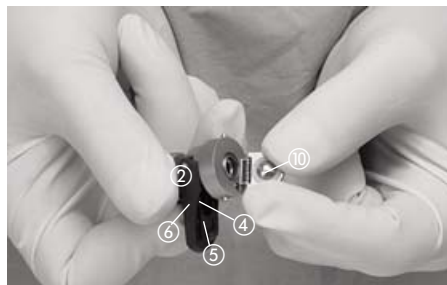
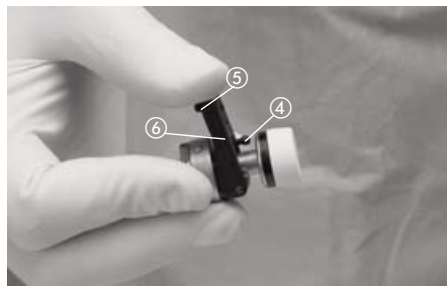
**Assembly**

Oil the joint on the flap valve ⑧ and lateral suspension of the thumb plate ⑨ precisely. If the top of the valve has been further disassembled: Put on the small sealing cap. Insert inside seal ⑩ in the recess in the flap and fix in place with the screw ⑪ by turning counterclockwise. Tighten the screw only slightly. Check the flap valve for correct seating in its closed condition.

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With the thumb plate ⑥ depressed, insert the tappet ④ from behind. Release the thumb plate ⑥ until the pusher ⑤ can be slid over the ball-shaped holder. For 6 and 7 mm size trocars, screw on the rear top part.

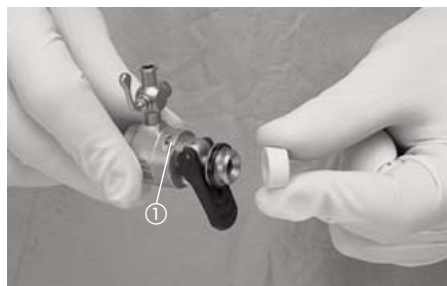
**Note:** It is easier to screw on the rear top part if the sealing cap is in place.

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Insert the top of the valve ② into the trocar cannula and close the bayonet mount ①. Put on the sealing cap. Insert the trocar tongue while operating the thumb plate.

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**Note:** To ensure the correct positioning of the trocar tongue tip of the REMORGIDA trocar tongues, the positioning aid ⑫ must lock into place on the insufflation stopcock.

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**Rigid trocars with silicone leaflet valve, 2.5 mm and 11 mm**

**Disassembly**

Remove trocar tongue.

To remove the silicone leaflet valve ①, unscrew the knurled ring ②.

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Take out the silicone lip valve, check for damage and replace if necessary.

**Note:** We recommend replacing the silicone lip valve after every use.

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Trokardorn einführen.

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**Assembly**

Insert the silicone leaflet valve ① into the knurled ring and screw on.

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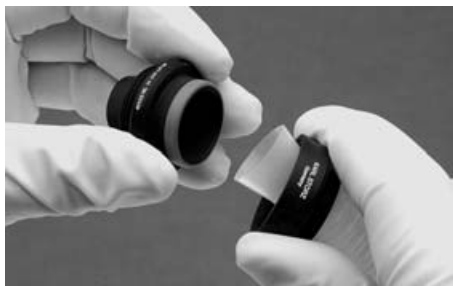
**Flexible trocars**

**Disassembly**

Remove trocar tip.  
Turn knurled ring clockwise (left-handed thread)  
and remove from flexible trocar sleeve together  
with the top of the valve.

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**Assembly**

Place knurled ring over the flexible trocar slee-  
ve and screw to the mounted valve top.  
Put on trocar tip.  
Put on protective cap

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**Flexible trocars with silicone lip  
valve, 9 mm**

**Disassembly**

Remove trocar tip.  
Take off the sealing cap, check for damage and  
replace with a new one if necessary.

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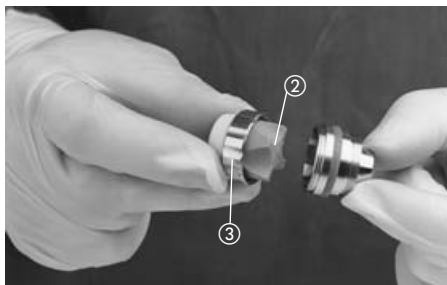
Unscrew the knurled ring ① clockwise (left-  
handed thread) and remove with the top of the  
valve ⑤ from the flexible trocar cannula

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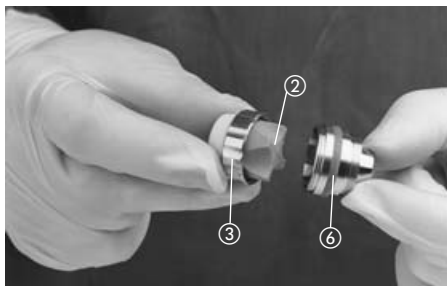
To remove the silicone lip valve ②, screw on the knurled ring ③. Take the silicone lip valve ② out of the knurled ring ③, check for damage, replace if necessary.

**Note:** We recommend replacing the silicone lip valve after every use.

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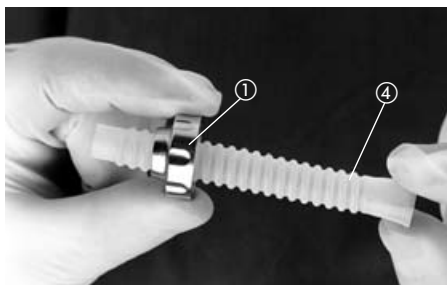
**Assembly**

Insert the silicone lip valve ② into the knurled ring ③ and screw onto the top part ⑥.

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Place the knurled ring ① over the flexible trocar cannula ④ and screw together with the top of the valve ⑤ in place.  
Put on the sealing cap.  
Insert the trocar tongue.  
Put on the trocar protective cap.

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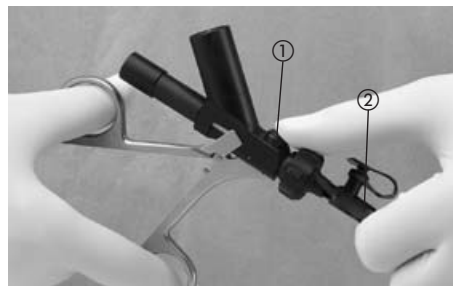
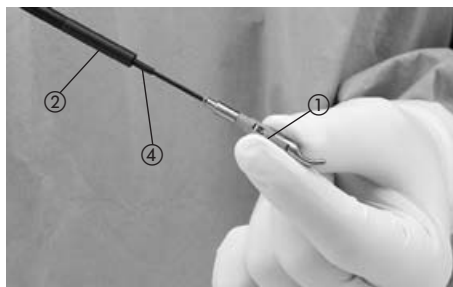
**Caution:** Do not turn locking button ①, as this may cause it to snap off.

Press and hold the locking button ①. The handle is now opened completely in order to pull the outer tube ② out towards the front.

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**Warning:** Grasp pointed or sharp jaws with a cloth.

To remove the internals, hold the jaws ① tightly and open the bayonet mount ④ by rotating the sheath tube ② ¼ turn. Pull out the internals.

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**Cleaning**

**Note:** For heavy soiling, we recommend using an ultrasonic bath of five minutes maximum duration.

**Manual cleaning**

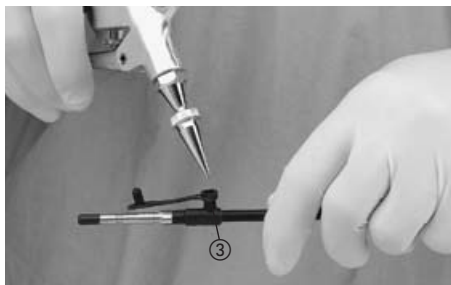
Remove tissue residue with a cleaning brush. Brush and rinse out sheath tubes on the inside with a cleaning brush (cat. no.27650 G). Blow dry with filtered compressed air.

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By means of the LUER-lock irrigation connection ③, easy attachment to a syringe is also possible.

**Machine cleaning**

Make certain all parts are positioned securely so that damage during cleaning is prevented. By means of the LUER-lock irrigation connection ③, easy attachment to a cleaning machine is also possible.

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**Assembly**

Check all parts for damage. Replace if necessary.

Check jaws and joints and the insulation on instruments for HF use especially carefully.

Insert the internals from the front into the sheath tube ②. Lock the internals by rotating ¼ turn in the bayonet mount ④ on the sheath tube ②.

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Insert the outer tube, with the jaws closed, as far as it will go into the completely opened handle. Then lock the outer tube by closing the handle completely.

Place a drop of instrument oil (Art. no. 27656) in the joints.

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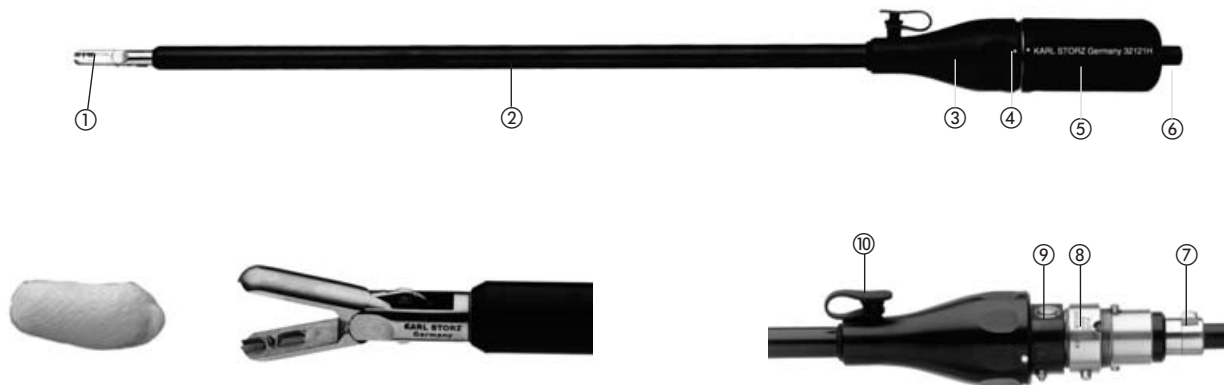
- ① Jaws (sponge holder)
- ② Outer sheath,
- ③ Handle, fixed
- ④ ○ locked  
● replace sponge  
▶ disassembly
- ⑤ Handle,
- ⑥ Button
- ⑦ Unlock jaws (rectangular)
- ⑧ Threaded ring
- ⑨ Unlock outer sheath (round)
- ⑩ Irrigation connection

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
Recommended spherical sponges  
In accordance with DIN 61630/VM24, or spher-  
ical sponges with a similar size and weight.

Surgical sponge holder	spherical sponges
Size 5 mm	4 cm x 4 cm
Size 10 mm	8 cm x 8 cm

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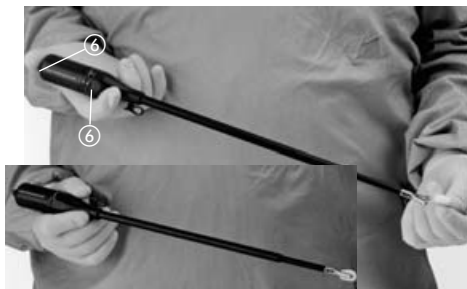**Application**

**Picking up or replacing spherical sponges**

Place handle ⑤ in the “○ replace sponge”  
position.  
Press the button ⑥: The jaws open.  
Insert the spherical sponge.  
Release the button ⑥: The jaws close  
and hold the spherical sponge securely.  
Place the handle ⑤ in the “○ locked” position.  
The sponge holder is ready to use.

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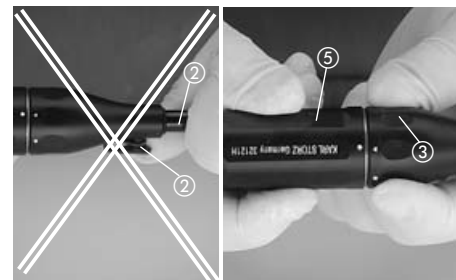
**Disassembly**

 **Caution:** When disassembling the  
instrument, do not hold it by the outer  
sheath ② or by the irrigation port ⑩.

Grip the instrument by the recesses on the  
fixed handle part ③, then turn the rotatable  
handle part ⑤ into the “▶ Disassembly”  
position and pull off.

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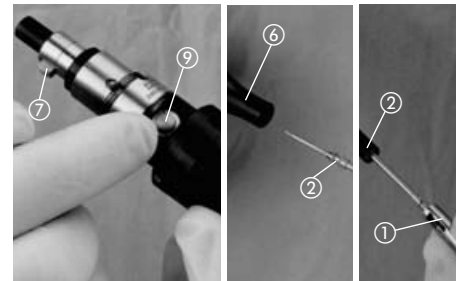
First press the unlock jaws button ⑦ then the  
unlock outer sheath button ⑨.

Remove the handle ⑤ from the outer sheath  
②.

To remove the jaws ①, hold them firmly and  
open the bayonet mount by rotating the outer  
sheath ② ¼ turn.

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A cleaning machine or syringe can be connec-  
ted to the LUER-Lock irrigation connection ⑩.  
Remove tissue residue with a cleaning brush.

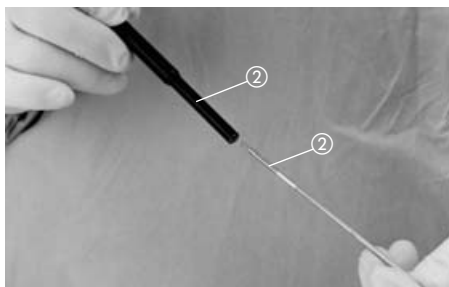
Brush and rinse out the outer sheath ② on the  
inside.  
Dry with sterile compressed air.

Lubricate movable parts precisely with instru-  
ment oil (Art. no. 27656) at the points of move-  
ment.

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**Assembly**

Check all parts for damage.  
Replace if necessary.

**Outer sheath with jaws preassembly**

Insert the jaws ① from the distal end into the outer sheath ②.  
Lock the jaws ① by rotating ¼ turn in the bayonet mount of the outer sheath ②.

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**Handle preassembly**

**Caution:** When assembling the instrument, do not hold it by the outer sheath ② or by the irrigation port ⑩.

If necessary, turn the threaded ring ⑧ as far as it will go.  
Grip the instrument by the recesses on the fixed handle part ③, then attach the rotatable handle part ⑤ and turn to the “● Replace sponge” position.

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**Final assembly**

Slide the assembled handle over the premounted outer sheath with jaws until the outer sheath lock clicks into place.

Press the button ⑥ so that the jaws lock clicks into place.

Test the instrument for proper operation.

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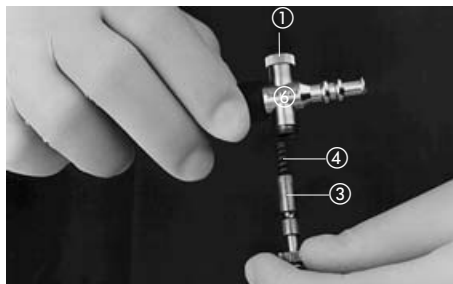
**Assembly and functional test**

Lightly grease piston ③.  
Insert spring ④ into piston ③.  
Screw knurled nut ① tightly.

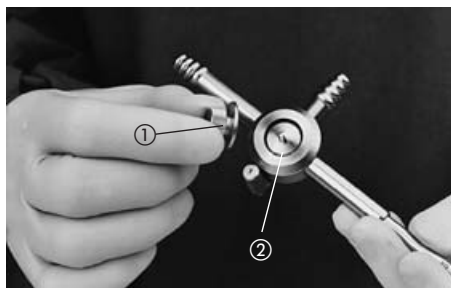
Insert piston with spring into valve ⑥.  
Screw knurled nut ② tightly with piston ③ slightly depressed.

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**Suction and irrigation tubes with two-way stopcock**

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**Assembly**

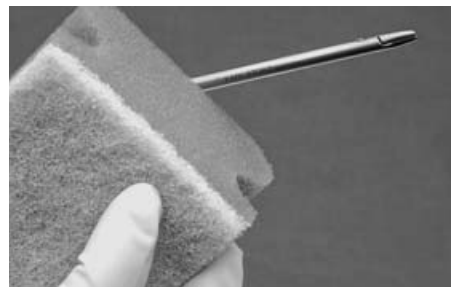
Lightly grease the contact surfaces on the cock plug ②.

Insert the cock plug ② and fix with the spring cap ①. Tighten slightly. Test for proper operation.

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**Needle holder, with straight and curved handle**

**Note:** Straight handles cannot be dismantled.

**Cleaning**

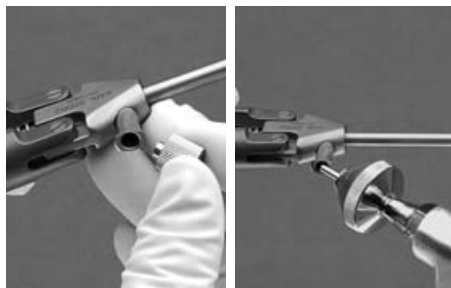
Clean the exterior of the needle holder with a soft sponge.

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Remove cover of the irrigation port. Clean instrument using irrigation port and then clean again in the ultrasonic bath.

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- ① Adapter for 5mm sheath
- ② Locking ring
- ③ Head piece
- ④ Handle
- ⑤ Removable plug
- ⑥ Suction port
- ⑦ Irrigation port
- ⑧ Suction valve
- ⑨ Irrigation valve
- ⑩ Sealing ring
- ⑪ Spherical seal

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**Disassembly**

Pull tube out of suction port ⑥. Loosen and  
remove tube on irrigation port ⑦ by turning  
counterclockwise.  
Preparation of the silicone tubing set is  
described in the suction/irrigation pump  
instruction manual.  
Unscrew suction/irrigation sheath.  
Unscrew adaptor ① for 5mm sheath, if fitted.

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**Cleaning**

If necessary, clean the dismantled parts in an ultrasound bath. Thoroughly clean the lumen and suction/irrigation channels with a brush. Carefully clean the surfaces with a sponge or soft brush. Clean the valves ⑧+⑨ with button pressed down. Then, rinse thoroughly with completely demineralized water and dry with sterile compressed air. Thermal machine disinfection at 93°C is possible, taking into consideration the chemicals approved by KARLSTORZ.

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**Assembly**

Insert head piece ③ correctly in the handle ④ (observe white markings).

Screw on locking ring ②.

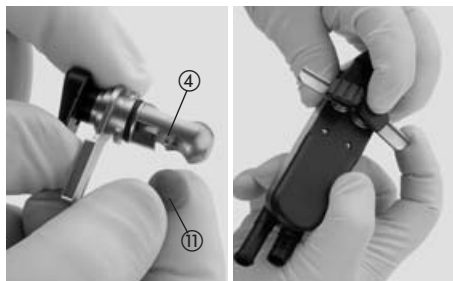
Additionally for pistol-shaped handle:  
Insert removable plug ⑤ correctly and fix by turning clockwise.

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Place the spherical seal ⑪ in the suction valve ⑧ and in the irrigation valve ⑨; make sure to observe the various sizes.

Place the suction valve ⑧ and irrigation valve ⑨ in handpiece ④ and then lock into position by holding down the button: rotate the metal plate inwards.

Screw in suction/irrigation sheath (if necessary, use the adapter ① for the 5mm sheath).

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**Note:** Adapter Art. no. 37112 TA (option for 37112 T/37113 T) facilitates the use of standard irrigation tubing with LUER lock.

Connect the suction and irrigation tubing.

**Warning:** Do not confuse the suction and irrigation ports!

**Warning:** Follow the instructions for the suction and irrigation unit!

Test the functioning of the instrument.

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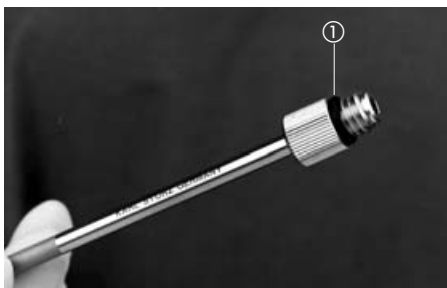
**Metal suction and irrigation tubes**

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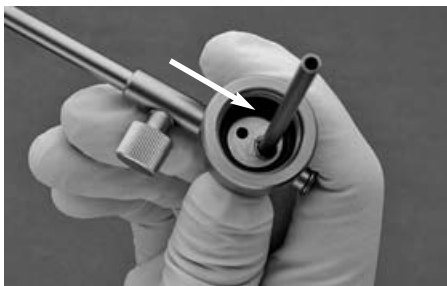
**Powder blower**

Remove rubber bulb from rubber tube.  
Remove rubber tube from spray tube.

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Unscrew the screw-neck bottle from the spray tube.  
Take out seals, check them and if necessary replace with new ones.

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**Cleaning**

**Caution:** Liquid must not be allowed to enter the rubber bulb.

**Manual cleaning**

Clean the screw-neck bottle and spray tube with a brush.  
Rinse with water and dry with sterile compressed air.

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### **Machine cleaning:**

The spray tube can be connected to a washer/disinfector via the LUER lock connector.

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### **Sterilization**

All parts, including the rubber bulb, can be steam sterilized at up to 134°C

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### Instruments for video assisted thoracic surgery – lung resection

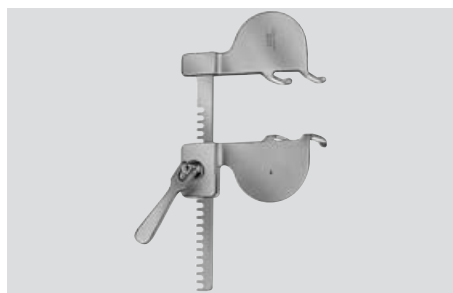
Clean the exterior with a sponge. Clean the parts of the jaws carefully using a brush. Then clean in the ultrasound bath.

Rinse with water and dry with sterile compressed air. Finally oil the joint.

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### CLICK'liné instruments

See **CLICK'liné instruments** in section *Esophagoscopy – diverticuloscopy*

### Coagulation electrodes

See section *Esophagoscopy – diverticuloscopy*.

### Forceps

### Retractors

### Instruments for suture and ligature

See section *Esophagoscopy – diverticuloscopy*.

### Holding System

see section *Holding Systems*

### Light Cables

see corresponding *instruction manual*

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

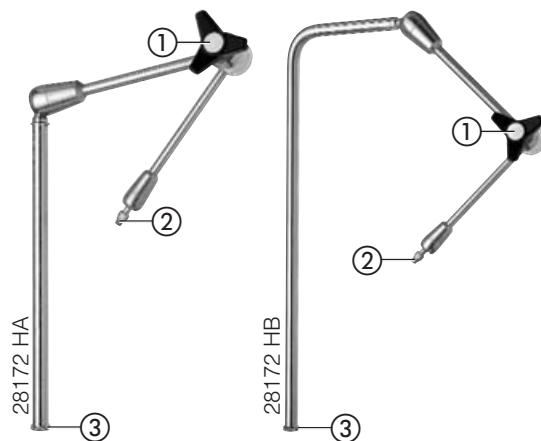
**Articulated Stand**

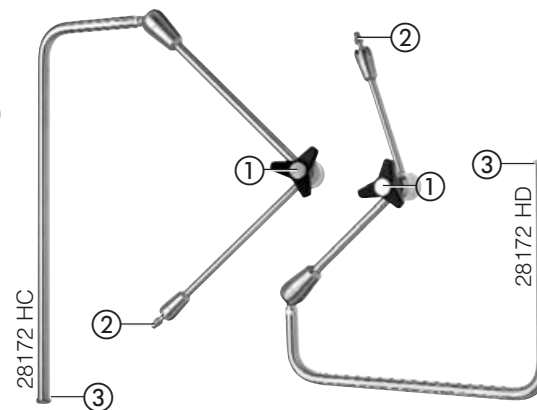- ① Star knob
- ② Clamping jaw mount with locknut
- ③ Thumb screw
- ④ Rail clamp
- ⑤ Stand clamp

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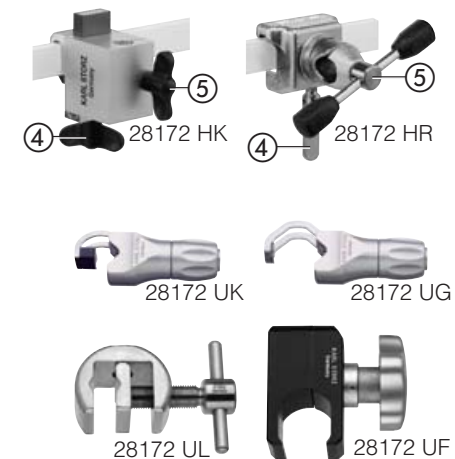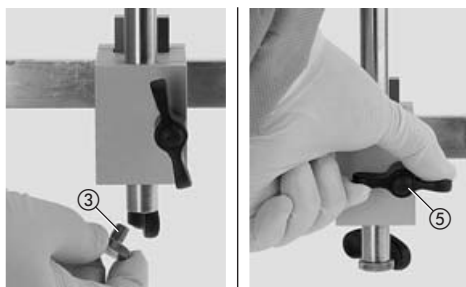
**Assembly**

Place the base on the operating table rail and fix in place with the rail clamp ④.

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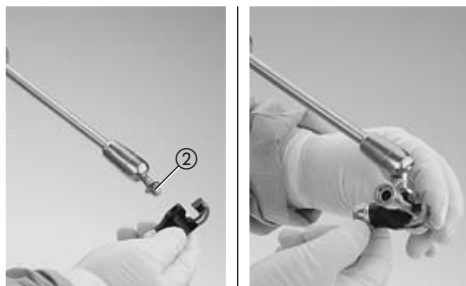
If necessary, remove the thumb screw ③ from the articulated stand. Insert the stand into the opening in the base. Screw the thumb screw ③ into the articulated stand and tighten.

Set the stand to the desired height and direction (for base with Art. no.: 28172 HR) and fix in place using the stand clamp ⑤.

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Select the correct clamping jaw for the instrument. Screw the clamping jaw onto the clamping jaw mount ② and secure using the locknut.

Insert the instrument into the clamping jaw and secure.

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Firmly hold the free end of the articulated stand whilst releasing the central clamp: turn the star knob ① in a counterclockwise direction.

Position the instrument as required.

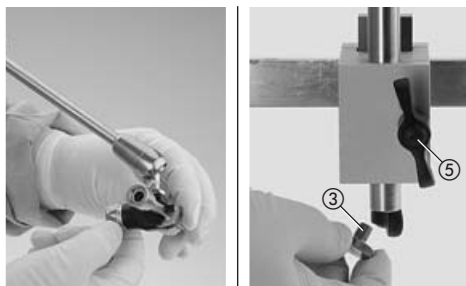
Turn the star knob ① clockwise until the central clamp is fixed in place and, consequently, the articulated stand is stable.

Ensure that the stand functions correctly.

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**Disassembly**

Release the clamping jaw and remove the instrument.

Remove the thumb screw ③ from the articulated stand, release the stand clamp ⑤ and remove the articulated stand from its base.

Release the rail clamp ④ and remove the base from the operating table rail.

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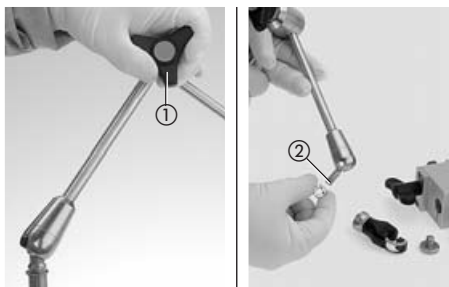
Release the central clamp: turn the star knob ① in a counterclockwise direction.

Unscrew the clamping jaw and locknut from the clamping jaw mount ②.

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### Cleaning

If necessary, disassemble the articulated stand (see Disassembly). Prepare the disassembled parts separately.

Clean the surface of the articulated stand with a commercial surface disinfectant, making sure that liquid does not enter the interior.

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### Operating/storage conditions

	Temperature	Rel. humidity
Storage	-20°C ... +60°C	10% ... 90%
Operation	+10°C ... +40°C	30% ... 70%

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### Disposal

Standard batteries and rechargeable batteries may contain toxic substances, and must be disposed of in an environmentally friendly way.

To dispose of the instrument set, no special measures are necessary.

National laws and regulations must be observed.

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### Directive compliance

The medical devices mentioned bear the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

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### Limitation of liability

As the supplier of these medical devices we only regard ourselves as responsible for the safety, reliability and performance of the products if assembly, system expansions, readjustments, modifications, or repairs have been performed by persons authorized by KARL STORZ, and the product has been used in accordance with its operating instructions at all times.

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**Warranty**

For **two** years as of delivery to the end customer, we agree to replace the goods free of charge if proof can be provided of faulty materials or workmanship. In doing so we cannot accept the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG  
Postfach 230  
D-78503 Tuttlingen/Germany

Any repairs or modifications to medical devices by unauthorized persons shall release us from any liability for their safety. Any such repair or modification performed during the warranty period shall void all warranty. KARL STORZ gives no warranty on the correct functioning of units or instruments which have been repaired by unauthorized third parties.

**Important information**

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and units must be cleaned and sterilized/desinfected before they are sent for repair. We reserve the right to return contaminated instruments/units to the sender.

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**Servicing and repair**

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts. Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights.

**Repair and replacement program**

KARL STORZ maintains a repair and replacement warehouse which is normally adequate to ensure prompt replacement of damaged telescopes and instruments. Under the repair and replacement plan, you receive an identical as-new instrument and are only charged the repair costs for the defective instrument.

For replacement telescopes, you receive a warranty of 1 year, and for replacement instruments 6 months.

For fiberscopes and units, individual repair is necessary. To bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to:

KARL STORZ GmbH & Co. KG  
Repair Service Department  
Dr. Karl-Storz-Strasse 34  
D-78532 Tuttlingen

In other countries please contact your local KARL STORZ branch or authorized dealer.

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**Chemicals for the preparation of telescopes and endoscopic instrument sets**

**Manual cleaning and disinfection/ultrasound**

The following instruments and telescopes produced by KARL STORZ must not be fully immersed in a liquid: HAMOU telescopes with ocular focusing, magnifiers with focusing, motors and their handles, IMPERATOR drill handles, prismatic light deflectors. The following instruments produced by KARL STORZ must not be placed in an ultrasound cleaning bath: rigid telescopes, flexible telescopes, other instruments with optical components made of glass.

When preparing and using the solutions, follow the solution manufacturer's instructions, paying close attention to proper dilution and recommended immersion times. Regarding the microbiological effectiveness please ask the manufacturer. The following cleaning and disinfectant solutions have been approved regarding material compatibility:

Ecolab GmbH & Co. OHG	Sekusept forte	2, 3
	Sekusept Plus	1, 2, 3
	Sekusept Pulver Classic	1, 2, 3
	Sekusept Pulver Classic + Aktivator	1, 2, 3
Esteer GmbH	Ultrasdes mit AF	2, 3
	Ultrasseptin Aktiv	1, 2, 3
	Ultrasseptin Classic	1, 2, 3
Fresenius AG	Ahd	1, 2
	Ahd plus (neu)	1, 2
Indeba Industria E Comercio LTDA	Sporcid FF	1, 2
	Letahdeido	2
Jose Collado S.A.	Darodor 4000 Liquido	1
	Darodor 9000	2
	Darodor Sinaldehyd 2000	2
	Aldasan 2000	1, 2
Lysoform Dr. Rosemann GmbH	Almyrol	1, 2
	Desoform	1, 2
	Lysoformin 3000	1, 2
	Edisonite Super	3
Merz+Co.	Mucadont-IS	2, 3
	Mucadont-Zymaktiv	1, 3
	Mucocit-T	1, 2, 3
	Orochemie	A 10
Promagent AB	ID 210	2
	Wavacide	2
Schülke & Mayr GmbH	Gigasept	2, 3
	Gigasept FF	2, 3
	Gigasept Instru AF (former Lysetol AF)	1, 2, 3
	Lysetol V	2
	Descoton forte	2
Schuhmacher, Dr. GmbH	Perfektan Endo	2
	EnzyCare 2	1, 3
Steris	neodisher LM 2	1
	neodisher mediclean	1
	neodisher mediclean forte	1, 3
	neodisher medizym	1, 3
Weigert, Dr. GmbH & Co.	Aidal Plus	2
Whiteley Industries PTY Ltd	Aidal Plus	2

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**\* Caution:** The use of CAUSTIC SODA can cause changes to the surface of aluminium parts (including coated parts), plastics and soldered connections, and may reduce the service life of the instrument set.

For flexible endoscopes, only immerse the examination sheath, not the housing/handle part.

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**Typ**

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths

**Note:** You will also find the current list of approved products on our Web site

([www.karlstorz.de](http://www.karlstorz.de) or [www.karlstorz.com](http://www.karlstorz.com)).

**Note:** When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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**Machine cleaning and disinfection**

The following instruments and telescopes produced by KARL STORZ must not be prepared in a cleaning and disinfecting machine: HAMOU telescopes with ocular focusing, magnifiers with focusing, motors and their handles, IMPERATOR drill handles, standard, tungsten carbide and diamond-tipped drills, prismatic light deflectors, and magnetic instrument rack. Selection of a method for cleaning and disinfection must be discussed with the machine manufacturer and the manufacturer of the chemical reagents. Only special methods must be used which have been verified for this purpose. Regarding the microbiological effectiveness please ask the manufacturer. Thermal disinfection is preferable. The following agents for machine cleaning and disinfection have been approved regarding material compatibility:

Schuhmacher, Dr. GmbH	Thermoton Endo	2
Steris	Instru-Klenz	1, 3
	neodisher FA	1
Weigert, Dr. GmbH & Co.	neodisher FA forte	1, 3
	neodisher mediclean	1
	neodisher mediclean forte	1
	neodisher mediklar	5
	neodisher medizym	1
	neodisher Septo DA	2
	neodisher Septo DN	2

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**Type**

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths

**Note:** You will also find the current list of approved products on our Web site ([www.karlstorz.de](http://www.karlstorz.de) or [www.karlstorz.com](http://www.karlstorz.com)).

**Note:** When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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Firmenstempel/Unterschrift:

# WARRANT

For a period of **two** years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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Tipo del aparato: Número de serie:

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Firma/Fecha:

To be filled out by instrument owner:  
Return address/company stamp:

Field of application:

Type of instrument: Serial no.:

Purchase date:

Signature/Date:

Sello o  
estampilla

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KARL STORZ GmbH & Co. KG  
Postfach 230  
78503 Tuttlingen/Alemania

Please attach  
sufficient  
postage

REPLY CARD

KARL STORZ GmbH & Co. KG  
Postfach 230  
78503 Tuttlingen/Germany

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Absender/Firmenstempel:

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Geräte-Typ: Serien-Nr.:

Kauf-Dat.:

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ANTWORTKARTE

KARL STORZ GmbH & Co. KG  
Postfach 230  
78503 Tuttlingen

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KARL STORZ GmbH & Co. KG  
Mittelstraße 8, 78532 Tuttlingen, Germany  
Postfach 230, 78503 Tuttlingen, Germany  
Phone: +49 (0)7461 708-0  
Fax: +49 (0)7461 708-105  
E-Mail: [info@karlstorz.de](mailto:info@karlstorz.de)  
Web: [www.karlstorz.com](http://www.karlstorz.com)

KARL STORZ Endoskope Berlin GmbH  
Ohlauer Straße 43  
10999 Berlin, Germany  
Phone: +49 (0)30 30 69090  
Fax: +49 (0)30 30 19452

KARL STORZ Endoscopy Canada Ltd.  
2345 Argentia Road, Suite 100  
Mississauga, ON, L5N 8K4  
Phone: +1 905 816-8100  
Fax: +1 905 858-0933  
E-Mail: [info@karlstorz.ca](mailto:info@karlstorz.ca)

KARL STORZ Endoscopy-America, Inc.  
600 Corporate Pointe  
Culver City, CA 90230-7600, USA  
Phone: +1 310 338-8100, +1 800 421-0837  
Fax: +1 310 410-5527  
E-Mail: [info@ksea.com](mailto:info@ksea.com)

KARL STORZ Veterinary  
Endoscopy America, Inc.  
175 Cremona Drive  
Goleta, CA 93117, USA  
Phone: +1 805 968-7776  
Fax: +1 805 685-2588  
E-Mail: [info@karlstorzvet.com](mailto:info@karlstorzvet.com)

KARL STORZ Endoscopy Latino-America, Inc.  
815 N. W. 57th Avenue, Suite 480  
Miami, FL 33126-2042, USA  
Phone: +1 305 262-8980  
Fax: +1 305 262-8986  
E-Mail: [info@ksela.com](mailto:info@ksela.com)

KARL STORZ Endoscopia  
Miramar Trade Center  
Edificio Jerusalem, Oficina 308 A  
La Habana, Cuba  
Phone: +53 72041097  
Fax: +53 72041098

KARL STORZ Endoscopia México S.A. de C.V.  
Lago Constanza No 326  
Col. Chapultepec Morales  
D.F.C.P. 11520, México, Mexico  
Phone: +52 55 52 50 56 07  
Fax: +52 55 55 45 01 74

KARL STORZ Endoscopia Argentina S.A.  
Cerviño 4449 Piso 10°  
1425 Buenos Aires C. F., Argentina  
Phone: +54 11 47 72 4545  
Fax: +54 11 47 72 4433  
E-Mail: [marketing@karlstorz.com.ar](mailto:marketing@karlstorz.com.ar)

KARL STORZ Endoskop Sverige AB  
Storsätragränd 14  
12739 Skärholmen, Sweden  
Phone: +46 8 50 56 4800  
Fax: +46 8 50 56 4848  
E-Mail: [kundservice@karl-storz.se](mailto:kundservice@karl-storz.se)

KARL STORZ Endoskopi Danmark A/S  
Skovlytoften 33  
2840 Holte, Danmark  
Phone: +45 45162600  
Fax: +45 45162609  
E-Mail: [marketing@karlstorz.dk](mailto:marketing@karlstorz.dk)

KARL STORZ Endoscopy (UK) Ltd.  
392 Edinburgh Avenue, Slough  
Berkshire, SL1 4UF, Great Britain  
Phone: +44 1753 503500  
Fax: +44 1753 578124  
E-Mail: [customerservice@karlstorz-uk.com](mailto:customerservice@karlstorz-uk.com)

KARL STORZ Endoscopie Nederland B. V.  
Phone: +31 651 938 738  
Fax: +31 135 302 231

KARL STORZ Endoscopy Belgium N. V.  
Phone: +32 22 67 3810

KARL STORZ Endoscopie France S. A.  
12, rue Georges Guynemer  
Quartier de l'Europe  
78280 Guyancourt, France  
Phone: +33 1 30484200  
Fax: +33 1 30484201  
E-Mail: [marketing@karlstorz.fr](mailto:marketing@karlstorz.fr)

KARL STORZ Endoskop Austria GmbH  
Landstraßer Hauptstr. 146/11/18  
1030 Wien, Austria  
Phone: +43 1 71 56 0470  
Fax: +43 1 71 56 0479  
E-Mail: [storz-austria@karlstorz.at](mailto:storz-austria@karlstorz.at)

KARL STORZ Endoscopia Italia S. r. l.  
Via dell'Artigianato, 3  
37135 Verona, Italy  
Phone: +39 045 8222000  
Fax: +39 045 8222001  
E-Mail: [info@karlstorz.it](mailto:info@karlstorz.it)

KARL STORZ Endoscopia Ibérica S. A.  
Parque Empresarial San Fernando  
Edificio Francia – Planta Baja  
28830 Madrid, Spain  
Phone: +34 91 6771051  
Fax: +34 91 6772981  
E-Mail: [marketing@karlstorz.es](mailto:marketing@karlstorz.es)

KARL STORZ Industrial\*  
Gedik Is Merkezi B Blok  
Kat 5, D 38-39, Bagdat Cad. No: 162  
Maltepe Istanbul, Turkey  
Phone: +90 216 442 95 00  
Fax: +90 216 442 90 30

OOO KARL STORZ Endoscopy – WOSTOK  
Gogolevski Blvd. 3/1/2  
119019 Moskau, Russia  
Phone: +7 095 73 78113  
Fax: +7 095 73 78114  
E-Mail: [kste-wostok@karlstorz.ru](mailto:kste-wostok@karlstorz.ru)

KARL STORZ Endoskope  
Regional Center for Endoscopy S.A.L.  
St. Charles City Center, 5th Floor  
Phoenicia Street, Mina Elhosn  
2020 0908 Beirut - Lebanon  
Phone: +961 1 368181  
Fax: +961 1 365151

KARL STORZ South Africa, (Pty) Ltd.  
P. O. Box 3039,  
Cape Town 8000, South Africa  
Phone: +27 117815781  
Fax: +27 117872660  
E-Mail: [info@karlstorz.co.za](mailto:info@karlstorz.co.za)

KARL STORZ Endoscopy India Private Ltd.  
C-126, Okhla Industrial Area, Phase-1  
New Delhi 110020, India  
Phone: +91 11 26 81 54 45-51  
Fax: +91 11 26 8129 86  
E-Mail: [karlstorz@vsnl.com](mailto:karlstorz@vsnl.com)

KARL STORZ GmbH & Co. KG  
Resident Representative Office  
80/33 (44/19) Dang Van Ngu, F.10-Q. Phu Nhuan  
Ho Chi Minh City, Vietnam  
Phone: +848 991 8442  
Fax: +848 844 0320

KARL STORZ Endoscopy (S) Pte. Ltd.  
#05 – 08 San Centre, 171 Chin Swee Road  
Singapore 0316, Singapore  
Phone: +65 65 32 5548  
Fax: +65 65 32 3832  
E-Mail: [karlstorz@pacific.net.sg](mailto:karlstorz@pacific.net.sg)

KARL STORZ Endoscopy China Ltd.  
Suite 1601-2, 16/F  
Chinachem Exchange Square  
1-7 Hoi Wan Street, Quarry Bay  
Hong Kong, China  
Phone: +8 52 28 65 2411  
Fax: +8 52 28 65 4114  
E-Mail: [kstorzhk@netvigator.com](mailto:kstorzhk@netvigator.com)

KARL STORZ Endoscopy China Ltd.  
Beijing Representative Office  
Unit 1806, China Life Tower  
No. 16, Chaowai Street  
Beijing, 100020 China  
Phone: +86 10 8525 3725  
+86 10 8525 3726  
Fax: +86 10 8525 3728

KARL STORZ Endoscopy China Ltd.  
Shanghai Representative Office  
Rm 3901-3904, 1 Grand Gateway  
1 Hongqiao Road  
Shanghai, PRC., 200030 China  
Phone: +86 21 6113-1188  
Fax: +86 21 6113-1199

KARL STORZ Endoscopy China Ltd.  
Chengdu Representative Office  
F-5, 24/F., Chuanxing Mansion  
18 Renming Road (South)  
Chengdu, Sichuan, PRC.,  
610016 China  
Phone: +86 28 8620-0175  
+86 28 8620-0176  
Fax: +86 28 8620-0177

KARL STORZ Endoscopy China Ltd.  
Guangzhou Representative Office  
Rm 1119-20, Dongshan Plaza  
69 Xianlie Road (Middle),  
Dongshan District,  
Guangzhou, PRC., 510095 China  
Phone: +86 10 8732-1281  
+86 10 8732-1282  
Fax: +86 10 8732-1286

KARL STORZ Endoscopy Japan K. K.  
Bois Hongo Building 6FI, 3-42-5 Hongo  
Bunkyo-ku, Tokyo 113-0033, Japan  
Phone: +81 3 5802-3966  
Fax: +81 3 5802-3988  
E-Mail: [info@karlstorz.co.jp](mailto:info@karlstorz.co.jp)

\*Sales for Industrial Endoscopy

**STORZ**  
KARL STORZ – ENDOSKOPE

**KARL STORZ GmbH & Co. KG**

Mittelstraße 8  
78532 Tuttlingen

Postfach 230  
78503 Tuttlingen  
Germany

Telefon: +49 (0)7461 708-0  
Telefax: +49 (0)7461 708-105  
E-Mail: [info@karlstorz.de](mailto:info@karlstorz.de)  
Web: [www.karlstorz.com](http://www.karlstorz.com)