Flo-Tech Plus kørestolspude XXL-Rehab Plus cushion

DK

UK

152-195 KG

DK

Kan med fordel anvendes til:

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Instructions for XXL-Rehab Plus cushion

⚠ CAUTION ⚠ CAUTION ⚠

- This product should not be used without proper instruction from a healthcare professional.
- Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.
- XXL-Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- If components are damaged or missing, contact XXL-Rehab immediately. Do NOT use substitute parts.
- XXL-Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from XXL-Rehab.

⚠ To ensure proper use of the XXL-Rehab Plus cushion, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with ⚠. Also remember that in order to prevent the risk of injury to the user, the cushion may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

USE

XXL-Rehab Plus cushion is designed exclusively for bariatric users at risk of developing pressure sores. The cushions weight capacity is maximum 152–195 kg, depending on the width of the cushion.

⚠ If users are weighing more than 152/175/195 kg there is a risk that the cushion will break or take damage.

DESCRIPTION

XXL-Rehab Plus cushion is developed to It is made of a molded base and has a sealed silicone bag in the middle. The molded base provides stability and seat support and the silicone bag provides unique pressure relief on the Tuber Ischii.

DESCRIPTION OF COVER

The cover is made of polyamide and polyurethane and is an incontinence cover which is also washable and flame retardant.

BEFORE USE

Place the cushion on a chair/in a wheelchair with the zipper placed at the bottom. The shaped surface should face upwards.

⚠ The XXL-Rehab Comfort cushion must always be used with the cover zipped correctly.

⚠ Do not place any further layers or padding between the cushion and the user as this may reduce the effect of the cushion.

DURING USE

⚠ Inspect the cushion on a regular basis to ensure optimal effect.

⚠ Do not attempt to move the cushion by its handle when the user is seated on it.

MAINTENANCE AND CLEANING

Of cushion

⚠ The cushion cannot be cleaned.

Of cover

The cover may be cleaned using a solution of neutral detergent and hot water, applied with a single-use wipe or alternatively, laundered at 71° C with a dilute detergent solution. The cover must be thoroughly drip dried before returning to the cushion.

⚠ Do not tumble dry.

⚠ Do not use cleaning agents containing phenols or alcohols.

⚠ If the cover is damaged do not use the cushion before the cover is replaced. When fitting a cover, care must be taken to ensure that the corners of the foam core are positioned correctly into the corners of the cover.

WARRANTY

The XXL-Rehab Plus cushion is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended uses; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by XXL-Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

07/2014/JP

MANUFACTURER

Invacare Ltd.
South Road
Bridgend Industrial Estate
Bridgend CF31 3PY
England

DEALER

XXL-Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
Fax: +45 70 25 25 44
www.xxl-rehab.com

Maximum weight capacity is	152 kg (seat width 50 cm) 175 kg (seat width 56 cm) 195 kg (seat width 61 cm)
Weight	Approx. 2.5 kg
Base cushion	Molded foam
Gel	100% silicone
Cover	Polyamide / polyurethane
Incontinence cover	Yes
Washable cover	Yes
Flame retardant cover	Yes

Strongly engineered and functionally designed exclusively for bariatric users!

XXL-Rehab Minimaxx

XXL-Rehab Minimaxx
Push Motor

XXL-Rehab Manual
Wheelchair - The Eclipse

XXL-Rehab Manual
Wheelchair - The Eclipse Tilt

XXL-Rehab
HD Electric Wheelchair

XXL-Rehab Wheelchair Cushions

XXL-Rehab Shower Bench

XXL-Rehab
Bedside commode

XXL-Rehab
Shower Commode

XXL-Rehab
Shower Commode Wheeler

XXL-Rehab
Shower Commode Rise N Recline

XXL-Rehab
Shower Commode Tilt

XXL-Rehab Barton Transferstol

XXL-Rehab Patient/Dinner chair

XXL-Rehab Rollator King

XXL-Rehab Rollator Support

XXL-Rehab Stand Tall

XXL-Rehab Bariatric walking frame

XXL-Rehab Canes

XXL-Rehab Multifunktional Scale

XXL-Rehab Lift and Slings

XXL-Rehab 400 bariatric hospital bed

XXL-Rehab Home Care Bed

XXL-Rehab Floor raiser

XXL-Rehab
Comfort Foam Mattress

XXL-Rehab
Low Air Loss mattress

XXL-Rehab
Alternating mattress

XXL-Rehab Bariatric suit

BRUGSANVISNING · USER MANUAL

mauryathome.dk

07-2014 Flo-Tech Plus kørestolspude / XXL-Rehab Plus cushion

XXL REHAB®

XXL-Rehab · Fuglebækvej 1D · DK-2770 Kastrup · Denmark
Telephone: +45 7025 2522 · Fax: +45 7025 2544
info@xxl-rehab.com · www.xxl-rehab.com