

User Manual



1. Micro--current head

By orientation, layer-oriented, ration, directly inject many kinds of nutrition into deep-seated cells, assist to melody the muscale, lymph drainage, promote metabolism, lymph and blood circulation, comparing it with the single skin care products, effectively absorb products higher 2000 times, so get the best beauty effect at one-step.



2. Ultrasonic head

After the nutrition being injected into the skin, to make it better absorption, then use the ultrasonic heads to activate it, prevent the circumstance pollution, renew the skin pathological changes, so get the better treatment.

Protect and repair the skin with cell membrane protection and skin joyful genes, special anti-aging, defend the skin damage from circumstance pollution, repair pathological changes of the skin. increase the muscle, stimulate lymph circulation, increase the absorption of nutrients.



3. Nutrition locked "Cold head"

Appease the skin, clear the sensitivity, exhaust out of skin waste and toxins, strengthen collagen fibres, enhance skin elasticity, close the pores. Frozen therapy and keep nutrition. **Close pores hammer to freeze the nutrition in the skin.**

After the skin care products being injected into the skin, to avoid the nutrition releasing, can use the hammer to freeze the skin, consolidate the nutrition in the skin well.



4. BIO head for eyes

For eyes beauty and massager, get rid of dirty things on the eyes. And put the cream on the eyes, then move softly and slowly to relax the eyes skin.


4. C: OFF, V.D ----- the fourth line is "cold head and diamond dermabrasion head"

<6>. The "V.D---diamond dermabrasion, it's independent, it doesn't connect with those four heads mentioned before. You can press the "V.D" button to operate it. On the right of the machine, there has a intensity watch, this's special for adjusting the strength of the dermabrasion.

<7>. For The pad , it has to be used together with BIO head, remember. You can put the pad on the body anywhere, then collect the power and hold the bio head, you will feel the electricity current.

<8>. For the dermabrasion connector, if you want put out the pipe, just press the metal slice down the connector, then it will be out. Don't take out by strong strength, it will break the pipe.

You have to choose the right model, then it can work well, if not. It doesn't work.

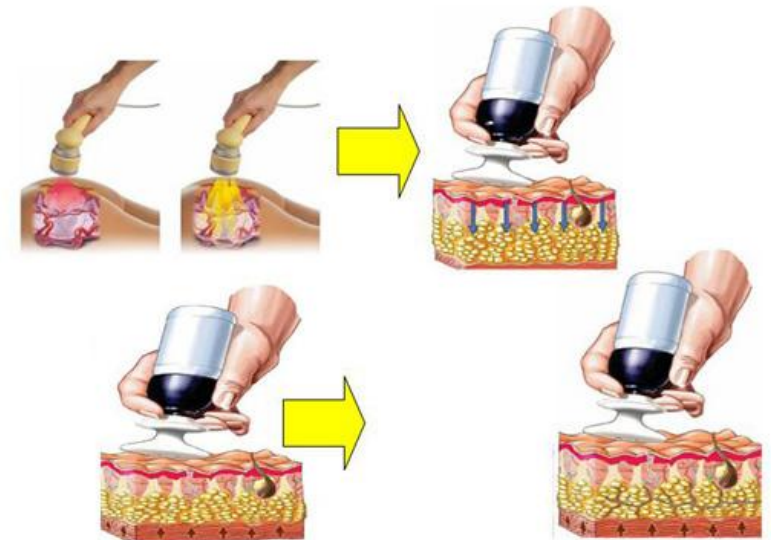

Suitable people

1. Skin reconstruction of the face

(1) Oily skin - big pores, thick cutin,
Purify the skin, remove unwanted thick cutin.
Purify the pores, dissolve the grease, dredge and tighten the pores,
cleanse and smoothen the skin.
Skin color dark, tiredness, paleness, dropsy
Rejuvenate the skin, speed up oxygen scent, improve circulation and
skin color dark.

(2) Mature type, the skin slack, short of water, wrinkle, dry,
Lift and tighten the skin, tighten the slack skin, make up water to
deep-seated skin.
Renew the skin, reconstruct and rebirth the cells, recover the youth
skin state.

<3> Eyes skin treatment

Remove the black orbits, eye bags, eye wrinkles, relax the dropsy of
the eyes.

5. Dimand dermabrasion:



Safety notice

Please note that do not stay in one treatment place over time when
applying the treatment to avoid any hurt caused by cool head.

1. The machine is professional instrument for beauty salon, in order to

"select" button is only controlled by this "U-ultrasonic"head model
then other heads model can't control the "select" button. So the
"select" button only belongs to the corresponding mother menu
heads.

<4> then Press the "start/stop" button, you will see on the screen the
time line is shining and the corresponding alphabet is shining. Now
it's working. Remember: only these this two lines are shining together,
then the corresponding head can work. That is the time line and the
alphabet line.

<5>.for "INT:.", it's frequency, when you operate the machine, it has to
be up 0. For the strength of the intensity, you can press "up" and
"down: button to exchange.

Attention: for the screen,

1. "TIME: 29:30" -----the first is show the time.

2. "U. M.01 INT.01"-----The second line is the "ultrasonic
head" fuction, "INT.01" is the frequency, it has to bigger than 0

3. "M.A M.P INT:00"---- the third line is "Micro-current", it
contains two models, the "M .A" is the bio head, the "M.P" is the big



micro-current head, this is "M.P".

"INT.01" is the frequency, it has to bigger than 0

<1>.connect all the heads correctly. Then connect the power and open the machine. make sure all the lights are lighten, and the screen will be blue color.

<2>. Then Press the"out" button, it means output/input., it's the main controlling button.it's the mother menu. This mother menu controls 4 heads as follows:

- the "U---ultrasonic head"
- , the "C----cold head"

✍ the "M-A-----the smaller micro-current BIO, for eyes beauty.

④the "M-P-----the bigger micro-current head.

When you press the "out" button, these four models can exchange one another. You can choose the head you need. For example, if you choose the "U", then the machine will be fixed on this model, other models head are not working. When you choose"C", also other models are not working. So when you fix on some model, only the selected model works, other models are not working.

For M-A , M-P , has two ways to exchange. The first way: you can press the "up" and "down" button to exchange each other. The second way: you can choose the "out" button to select, just need press several times to exchange to the model you need.

Attention: these four models are circulatory.

<3>. After press the "out" button, then press the "select" button, choose the model you need. in this situation ,the"select" button only can control the submenu of the "out" button menu. For example, if at first you choose the model of "U-ultrasonic" head, so now the

avoid any unnecessary hurt, need professional beautician to make operation.

1 Do not disassemble the machine or operate the machine that not explained in the instruction book. All the maintenance services shall be carried out the professional maintenance technician approved by our company.

2 Do not install or operate the machine with wet hands or near the water sources so as to avoid the liquid spraying/swashing over the machine.

5. Pull out the plug and connect with the maintenance service center in the following cases:

A. liquid spray/swash over the machine or the machine gets caught in rain or water.

B. the machine has peculiar smell, fume/smog or abnormal noise

C. power lines damaged or scratched.

D. instrument fell down or the shell damaged.

6. Do not put any item on the power wires, or install the instrument in the place where its power wires will be trampled.

7. Pull out the power wires at the end of use for the sake of safety.

8. Do not insert any type of item into the instrument opening; otherwise there will be a fire or electric shock. In case of something fell into the

🔊 : continue wave



🔊 : Pulse wave

Remark to ultrasonic:

<1>. Big probe for face, continuous wave, for 15 minutes.

2. Small probe for eyes, pulse wave, 5 minutes for each.

Pulse wave: for agnail, purulent skin tissue, expanded for capillary vessel skin treatment.

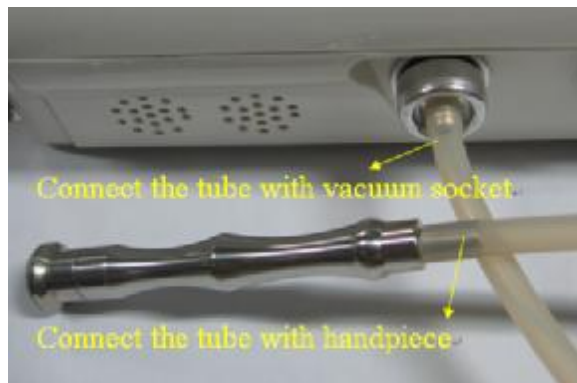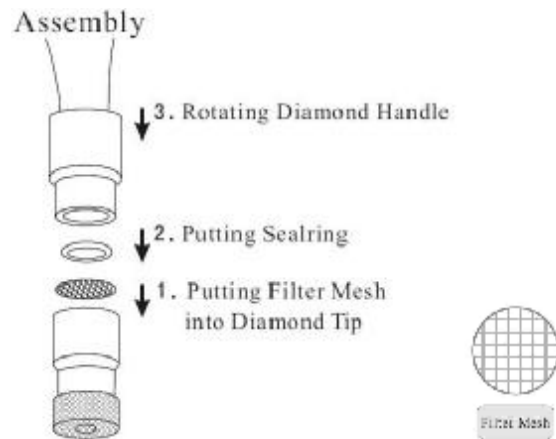

9diamond peel tips size as below:

reliable way to transport the nutrition, direct into fundus cells. Skin activation and reconstruction finish in one time.

Q. Any difference with the traditional ultrasonic?

A. It overpasses the traditional ultrasonic and galvanic ion therapy, effectively passes through the protection, penetrates skin care products into dermis layer. The equipment can let products into the skin deep-seated 10mm, so as to absorb nutrition better.

6. How to Operate the machine:



U---ultrasonic, M-P-----micro-current head,

M-A-----bio eyes head

C---cold head, V.D---diamond dermabrasion

pull out the 6x4 size vacuum hose. Then push in the black plastic ring on the vacuum hose.

2. Put a drop of cream or milk lotion on the vacuum hose nozzle, push in the movable ring and insert in the vacuum hose to the limit.

3. Insert a cotton swab into the Diamond Head, then tight up the Diamond Head on the hand piece. Remove the cotton swab after every treatment.

Attentions:

1. Don't treat on whom with epilepsy.
2. Don't treat on whom with heart diseases, such as arrhythmia, especial whom with heart pacemaker.
3. Don't treat on whom with cancer.
4. Don't treat on who with infection diseases.
5. Don't treat on the pregnant women.
6. Don't treat on the diabetic, or who had ever received big operations before.
7. Don't treat on whom with the metals in the body.

Normal questions and answers

Q. Applicable range?

A. It is based on the skin care products, to do skin whitening, wrinkle free, skin cleansing, acne removal etc.

Q. Any damage to the skin?

A. Of course not. Cause the equipment uses electromagnetic field of electrophoresis, make pores of the skin open within two seconds, through electric transportation channel and the safest and most



- #100 -----for leg
- #150-----for body
- #180-----for neck
- #200-----for face
- #250-----for eye

Diamond Dermabrasion


The Micro-crystal Dermabrasion was designed by Italian Florence's Mattioli at first, until now had over 20 years' history. This kind of technology only supplies to the dermatology expert and doctor at first uses, but more and more SPA House & Beauty Saloon also gradually uses, it was called the best method of medical beauty. This method successfully has developed in Europe for several years, cures patient thousands of examples, and has obtained the extremely satisfactory effect.

Now we want to introduce the Diamond Dermabrasion, it is a creative improvement of the Micro-crystal Dermabrasion. The Diamond Dermabrasion provided a non-surgical skin refinish procedure, by using sterile diamond heads to abrade or rub off the top skin layer, then Vacuuming out the particles along with any dirt and dead skin back up.

1. Do not use the machine in wound and eye area
2. Unsuitable for children, the pregnant and the insane.
3. This machine will be stopped automatically after 15 –minutes of continuous operation. To continue operation, please restart the machine.
4. During operation, the frequency generated is close to acoustic frequency, so it generates peep sounds, which is normal. Please take your ease


3. Removes aged cuticles

With an operating angle of 45°, the machine removes aged cuticles ultimately right away.

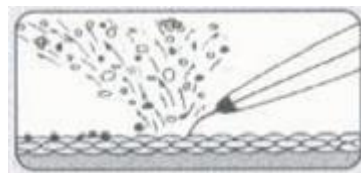


4. Removes wrinkles and regulates sebum secretion

By means of ultrasonic vibration and warm heat action, the machine helps to soften fat, activate cells, and remove excessive sebum, while eliminating wrinkles and muscular looseness.

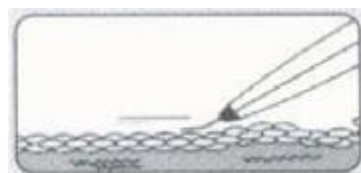

5. Reduces or removes melanin

By means of ultrasonic vibration, the machine works to inhibit tyrosinase, reduce and break up melanin.

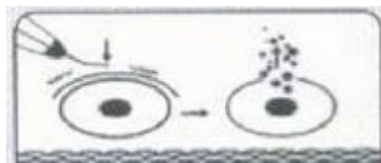

Precaution:

1. While using this ultrasonic machine, please work with cleansing lotion, exfoliating gel or tonic lotion.
2. To cleanse skin and remove aged cuticle, please select continuous wave. To massage skin and remove wrinkles, please select intermittent or continuous wave.
3. Before and after operation, please cleanse and disinfect the

This procedure removes skin debris, imperfections, blemishes, wrinkles and unwanted pigmentation on the skin. Moderate usage of Diamond Dermabrasion, accompany with skin products that penetrate through top skin layer that reach the dermis level, helps to replenish natural nutrients, restore cell activity, and promote skin health. The results after Micro Dermabrasion treatment is usually a healthy, glowing, beautiful skin.

*** Preparation before treatment

1. A consultation included the examination of the skin, a plan for treatment.
2. Operator should wear vinyl gloves and a facemask.
3. On oily or acne skin, a steamer may be used prior to the treatment, or a degreasing lotion may be applied prior to the cleanser.
4. Cleansing the skin with Cleansing gel or water, then wipe off with cotton pad. Allow the skin to dry.

Operating Instruction for Diamond Dermabrasion

1. Push in the movable ring of copper nozzle at the hand piece, and