

EZee Life™

Shower Stool

Model: CH2040



EZee Life™ Products

34 Futurity Gate, Unit 15
Concord, Ontario L4K 1S6



Limited Warranty:

EZee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. EZee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

Specifications:

Seat Height:	14" to 20"
Seat width:	12.5"
Seat depth:	12.5"
Width of legs:	13.5"
Construction:	Aluminum frame/composite seat
Weight capacity:	300lbs
Product weight:	2.64 lbs

Features:

- Round Seat
- Easy to clean
- Sturdy extruded aluminum construction
- Lightweight
- Durable composite seat
- Won't rust or scratch
- Attractive design

Assembly Instructions

1. Bath Stool comes fully assembled.
2. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per (figure 1).
3. Ensure legs are locked into place by pushing down on or by the pulling legs.

Safety Information:

DO NOT stand on the bath stool.

DO NOT lean too far in any direction while seated on the stool.

DO NOT use this stool if the user exceeds the weight capacity (300 lb).

Ensure all legs are set at the same height and securely locked.

DO NOT use if any damage is apparent.

Cleaning Information:

EZee Life products can be cleaned with most commercial cleaning products. Do not use abrasives.

We recommend cleaning the surfaces of the product and attachments with a solution made from a mild detergent (dish soap, etc.) to remove organic materials.

Once cleaned, it can be disinfected by wiping the product down with a 5% (1 cup bleach to 1 gallon water) solution of bleach and water. Allow the bleach solution to remain on the surfaces for 3 to 5 minutes, then rinse thoroughly with clear water.

Note: Bleach reacts with ammonia and organic matter (feces) causing gases that can irritate the eyes and lungs. Use in a ventilated area and avoid contact with other chemicals and organic matter.





Information

Imported / Distributed by:

Factory Direct Medical
34 Futurity Gate, Unit # 15
Concord Ontario L4K 1S6
Local: 416-739-8393
Toll Free: 855-235-2400