

Contents

Welcome using the semi-automatic biochemical analyzer	IV
What you should know	VI
Chapter 1 Brief Description	1
1. Appearance	1
2. Functions	2
3. Range applicable	3
4. Technical specifications	3
Chapter 2 Installation	4
1. Before installing	4
2. Installation	5
2.1 Installation	5
2.2 Adjustment of display	5
2.3 Install the print paper as figure below:	6
2.4 Check flow channel	6
Chapter 3 Basic operations	8
1. Operations of soft keyboard	8
1.1 soft keyboard window	8
1.2 Move the soft keyboard window	8
1.3 Instruction of soft keyboard	8
2. Windows Routine Operation & Adnex	8
2.1 Mouse	9
2.2 Windows	9
2.3 Blinking cursor	10
2.4 Control	11
2.5 Focus	11
2.6 Radio Button	11
2.7 Edition Bar	12
2.8 Check Box	12
2.9 Option Button	13
2.10 Combo Box only can be chosen	13
2.11 Combo Box can write in	14
2.12 List Box	15
2.13 Date Time Picker	15
3. Interface of the keyboard	16
3.1 【Key】	17
3.2 【Paper】	17
3.3 【Rinse】	17
3.4 【ASP】	17
3.5 【←,→,↑,↓,Enter】	17
4. Introduction of operation interface window	17
Chapter 4 Boot-strap	19
1. Process of boot-strap	19

2. Interface of boot-strap	20
3. Main interface window.....	23
Chapter 5 System setting.....	24
1. System Setting	24
2. Printer set	26
3. Instrument Maintenance.....	27
3.1 Touch Screen Calibration.....	28
3.2 IP Setting.....	29
4. Data Maintenance	30
4.1 Data Backup	30
4.2 Data Import	30
4.3 Delete Data	31
4.4 Data Transmission	32
Chapter 6 Animal Setting	34
1. Select Animal.....	34
2. Add Animal.....	35
Chapter 7 Test Setting.....	37
1. Select test.....	37
2. Add New Test.....	38
3. Test Parameters Edition	41
4. ABS Parameters.....	42
5. Calibration Parameters.....	43
6. QC Parameter.....	44
7. Select template.....	45
8. New template.....	46
9. Edit template.....	47
Chapter 8 Sample Test.....	48
1. Sample Test Interface.....	48
2. Sample Test	49
2.1 Test with non-template.....	49
2.2 Test with template	54
Chapter 9 Calibration Test.....	55
1. Calibration Test Procedures	55
Chapter 10 Quality Control Test	58
1. QC Test Procedures	58
Chapter 11 Sample Records	61
1. Summary	61
2. Browse sample records	61
3. Query sample records	63
4. Print sample records.....	64
5. Add, edit, or delete sample records	64
6. Add, edit, or delete test records	65
Chapter 12 Calibration Records.....	67
1. Summary	67
2. Query and print the calibration records	67
Chapter 13 Quality Control.....	69

1. Summary	69
2. Query and print QC records.....	69
Chapter 14 Turn Off	70
Chapter 15 Software Maintenance	71
1. Start the process of system maintenance	71
2. Update	72
3. Restore	72
4. Restore factory setting.....	73
Chapter 16 Routine Maintenance	74
1. Clean the instrument	74
2. Clean the flow cell	74
3. Instrument maintenances.....	75
4. Trouble-shooting	77
Chapter 17 Packing, Storage and Transportation	78

Welcome using the semi-automatic biochemical analyzer BS600V

Dear Users,

Welcome using the semi-automatic biochemical analyzer BS600V. It is our honor for the semi-automatic biochemical analyzer . In order to know our product well, we offer you this operational manual which includes the features, appearance and dimension, using methods, usage instructions, maintenance, packing, storage and transportation, etc. We try our best to edit the operational manual to be easily understood so as to make sure you can know the instrument better, Please read the manual carefully before operation and it is great helpful for you to operate the instrument correctly.


We try our best to offer correct, all-sided and reliable information in the manual and avoid the man-made mistakes. Please forgive us for some little mistakes unchecked out or some carelessness uncontrollable. We will update hardware and software at any time so as to promote the capability and reliability. Please forgive us it may be result in the manual cannot be match to changes in time.


Please contact us at any time if you find any mistake or carelessness in the manual during your operation.

Before operation, please do as follows at first

1. Make sure the actual fittings of the instrument consistent to the packing list. Any question, please contact the sales personnel or distributors.
2. Make sure write the service guaranty earnestly and mail the product guarantee card to manufacturer. Otherwise, it may restrict your service offered by our company.
3. Please read all the packed materials and store them properly.

Notice and the hint mark referring to safety

 **Warning:** The information (you should know) about how to avoid being hurt during the operation and consult the marks in the operational manual.

 **Notice:** The information (you should know) about how to avoid any damage to the instrument.

 **Hint:** The important hint information during the operation.

What you should know



Warning

- This instrument should avoid working in humidity condition or with corrosive gas.
- If peculiar smell or smoke generated during the operation, please cut off electricity in time and notify the manufacturer or agent for maintaining.
- The operational personnel should avoid touching the inner circuit and only the certificated personnel can repair the instrument.

The operational and maintain personnel should wear rubber glove when operating or repairing. Please treat the trash as medical garbage.




Notice

- Keep the working environment aeration; avoid strong electromagnetic interference, dust, sunshine and other strong direct light. Make sure the instrument is placed on the worktable stably.
- Do not damage the electrical wire, hold plug tightly when pull the plug and do not draw the wire.
- Do not put container with water or small metal object on top of the instrument, to avoid water or metal object falling in the inner, which may result in short-circuit and damage of the instrument.
- The instrument should use power wire with three cores and make sure grounding well.
- Clean the flow cell to make it clear after using it and turn off the power to prolong the usage life of the lamp.

Chapter 1 Brief Description


1. Appearance

1) Front view

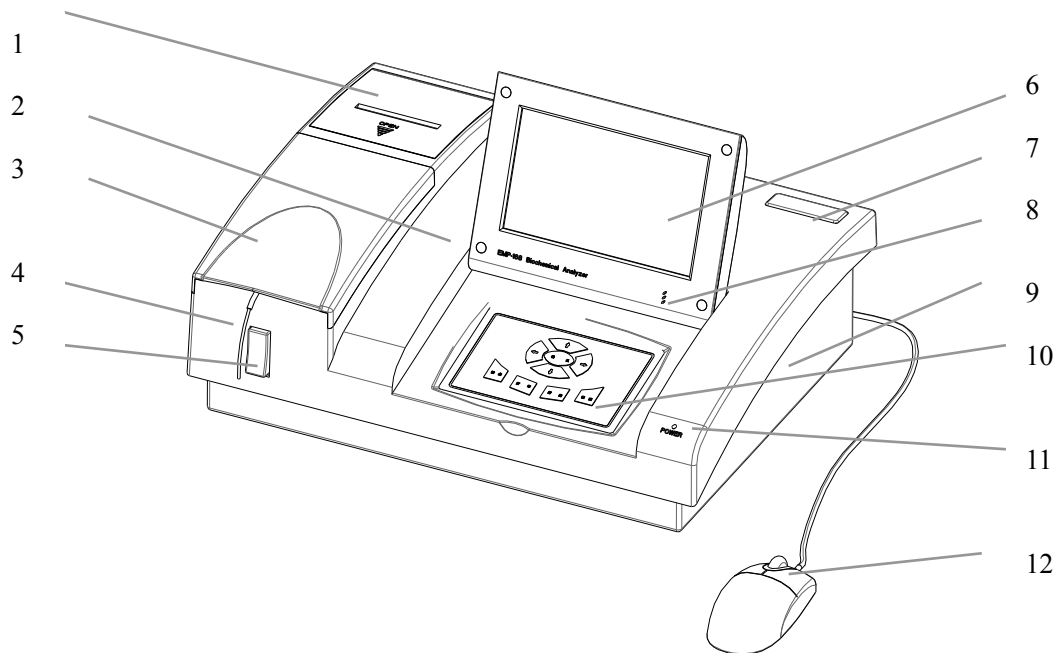

- | | | | |
|---------------------|-------------------|---------------------|------------|
| 1. Cover of printer | 2.Face shell | 3. Removable cover | 4.Pipette |
| 5. Aspirating key | 6. Display screen | 7. Logo | 8. Speaker |
| 9. Bottom shell | 10. Keyboard | 11. Power indicator | 12. Mouse |

2)Back view

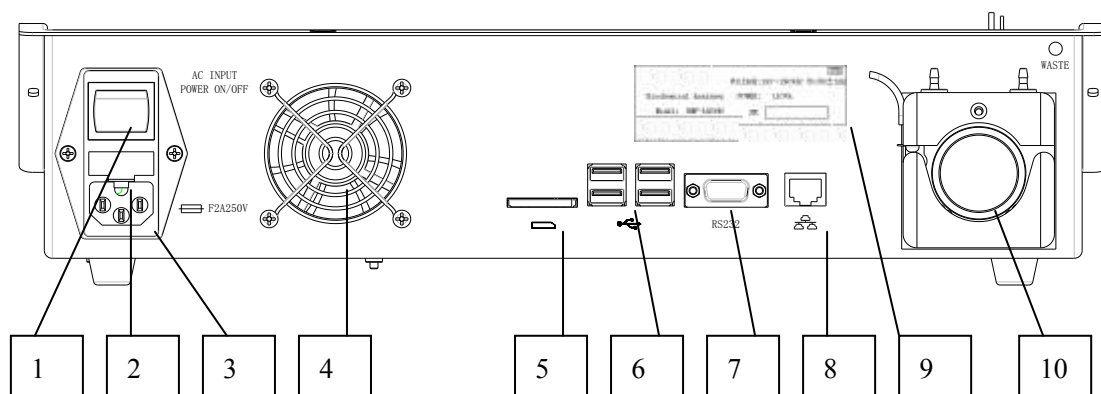

- | | | | |
|-------------------|----------------|-----------------|--------------------|
| 1. Power switch | 2. Fuse socket | 3. Power socket | 4. Fan |
| 5. SD card socket | 6. USB socket | 7. RS232 socket | 8. Ethernet socket |
| 9. Nameplate | 10. Pump | | |

2. Functions

- Colorized windows interface.
- Double-channels test, you can also use single-use cuvette available to eliminate crossed pollution.
- Advanced optics test system, built in 8 filters (340, 405, 505, 546, 578, 620, 670 and one optional filter available)
- Methods available as follows:
 - End-point
 - Fixed-time
 - Kinetics
 - Two-wavelength
 - Multi-standard
 - Sample-blank
- Urine, blood serum and body fluid can be tested.
- Reagent opened, both of oversea and home reagents available.
- Pipeline drains out automatically after testing, protecting the tubes and flow cell.
- 7"LCD display, reaction curve shown on the test interface, abnormal phenomena can be found in time.
- Great capacity for storage of user data and abundant programs.
- Perfect functions for quality control (quality control range: 31 days).
- Port for Ethernet available, data transmissible.
- External mouse, keyboard and printer available (based on USB)
- Adopt flow cell with volume 32 μ l.
- Built in Thermal Printer.
- The format of printing report supports the print function according to the project, animal, and case-history.
- Automatic sensing of flow cell/cuvette insertion.
- The intrinsic species of animals are 8 types, and user can customize 8 others.

3. Range applicable

This analyzer is applicable to veterinary clinics, pet hospitals or animal research institutions etc.

4. Technical specifications

Resolution	0.001Abs
Photometric range	0.000-3.000 Abs
Lamp	Halogen 6V/10W
Optics system	340-670nm, 7 filters built in: 340,405,505,546,578,620,670 nm, precision for 340 nm ± 2 nm, width of half-wave 10nm;precision for other wavelength ± 1 nm, width of half-wave 8nm; User can add a wavelength if necessary.
Stability of absorbency	Changes less than 0.005 Abs within 1 hour
Temperature control	Room temperature, 25℃, 30℃, 37℃ ± 0.1 ℃
Cuvette	Crystal flow cell with volume 32 μ L
Repeatability	CV $\leq 0.5\%$
Cross pollution	$\leq 1\%$
Linear of absorbency	r ≥ 0.999
Storage	Port for SD card, great capacity available through SD card
Ports	Port for RS232, Ethernet and SD card, 4 USB ports
Display screen	7" TFT LCD display
Printer	Built in Thermal printer, and supports laser printer and ink-paint printer available based on PCL print language and USB port.
CPU	High speed embedded processor
Weight	7 kg
Power supply	AC100 - 250V,50/60Hz
Dimension	420(L) \times 310(W) \times 152(H)
Power input	110VA
Working environments	Temperature: 10℃-30℃; relative humidity: $\leq 70\%$; atmosphere pressure: 86KPa-106KPa
Temperature for storage and transportation	-20℃-55℃

Chapter 2 Installation

1. Before installing

Unpack the package and check the packed parts and documents, prepare to install if all is right. If any question, please contact with the service department of our company. At the meantime, please write the guarantee for service and mail it back to our company, which is convenient to track and offer you available service.



Notice:

Working temperature :10℃-30℃;

Relative humidity:≤70%;

To make sure the instrument work normally, please avoid placing the instrument in following environments

Humidity, corrosive gas, dusty, strong electromagnetic interference;

Crowd and non-aeration;

Non-steady worktable;

Do not place the instrument near the centrifugal.

Requirement for power supply

- a) AC100 - 240V,50/60Hz;
- b) The instrument cannot use the same power supply socket with some high-power equipments including centrifugal, refrigerator, and oven, etc.
- c) The power supply should earth well and connect the power supply with three-cores wire; the voltage between the neutral line and earth line <5V.

2. Installation

2.1 Installation

- 1) Take the instrument out from the package and place it on a stable worktable.
- 2) Check its appearance and start to install.

Check the power supply at first and insert one terminal of itself three-core wire to the socket of the instrument and plug the other terminal to the electrical outlet. Put out the mouse and connect it to the USB socket. Prepare a container and put the liquid waste pipe in the container. Keep the package case and packed parts well.

2.2 Adjustment of display

A 7" TFT LCD display offered to this instrument; In order to obtain a best visual angle, the user can adjust the active display freely. Operations about adjusting the display as following figure

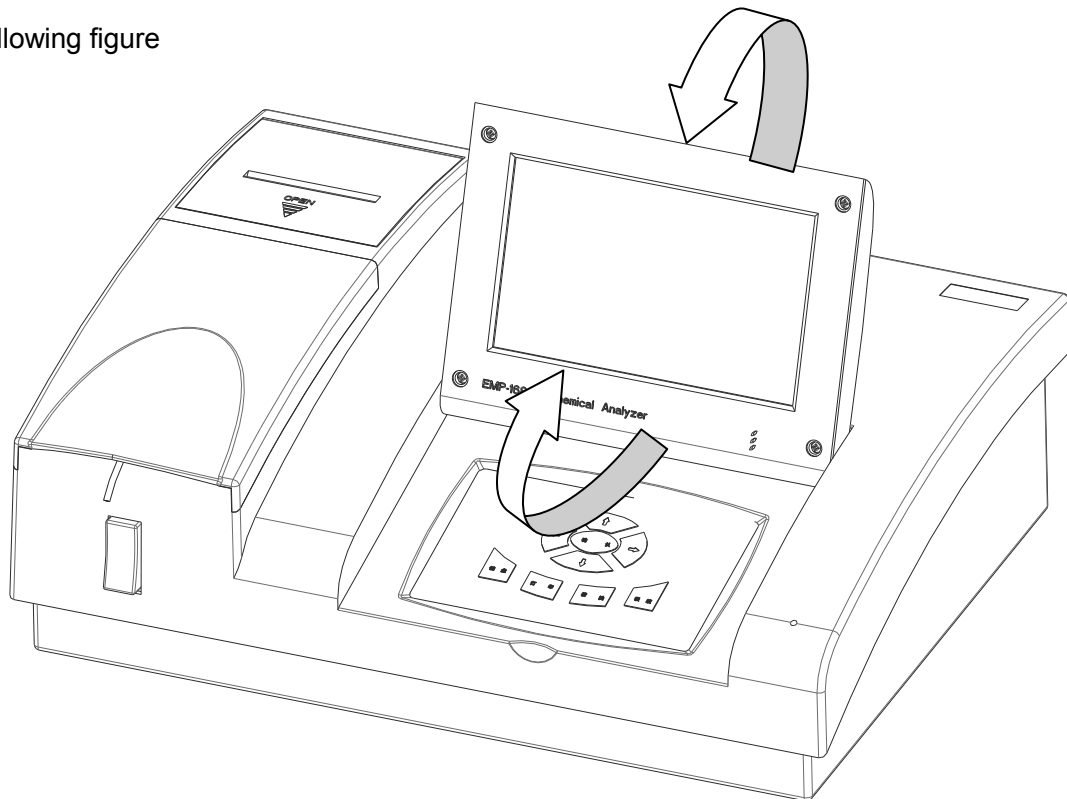


Figure 2-1

2.3 Install the print paper as figure below:



1, lever 2, the cover of printer 3, paper 4, platen knob

Figure 2-2

- Press down [OPEN] and open the cover of printer.
- Get rid of the package of the print paper and place it in the flute. Pay attention to the positive and negative direction of the print paper for thermal printer.
- Insert the print paper to the inlet of the printer, put up the lever, roll the platen knob, till the paper walk out 10 cm, and then push down the lever.
- Lead the paper through the cover of printer, close the cover and finish the installation.

2.4 Check flow channel

Lift the removable cover, check the position and direction of the flow cell. Figure 2-3

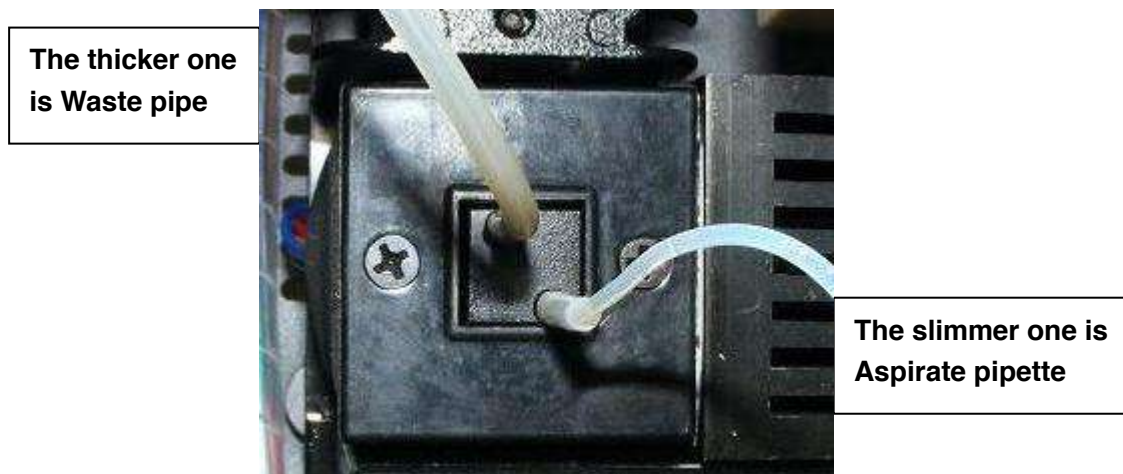


Figure 2-3

Push the aspirate pipette out as Figure 2-4. Then you can put back the cover.

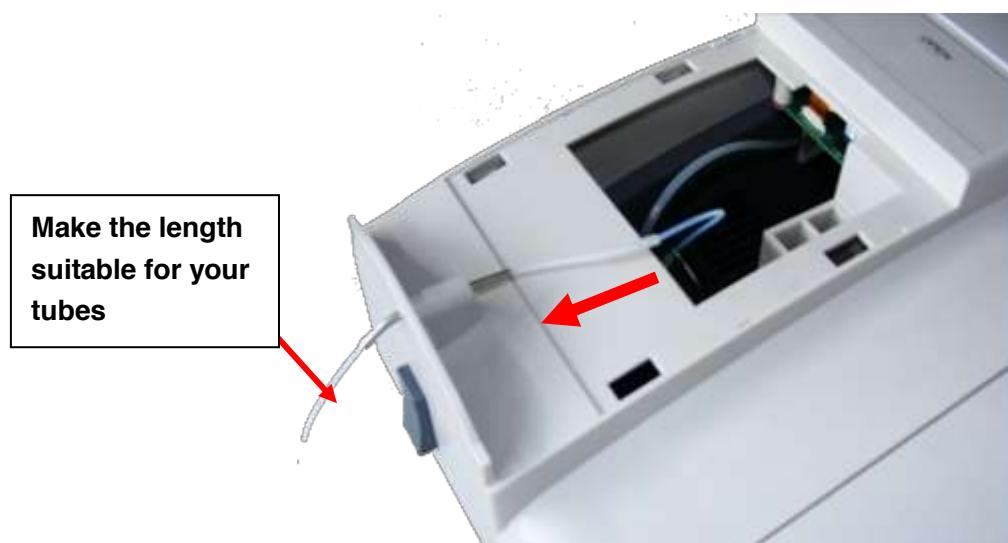


Figure 2-4

The waste pipe may be conglutinated in storage and transport. For your first time of using, it may not aspirate normally, and cannot pass the self-test. Please loosen the pump lock (press down the handle, as Figure 2-5); cancel the retrying of self test (Figure 2-6); rinse for several seconds (at the main interface), till the pump can roll normally.

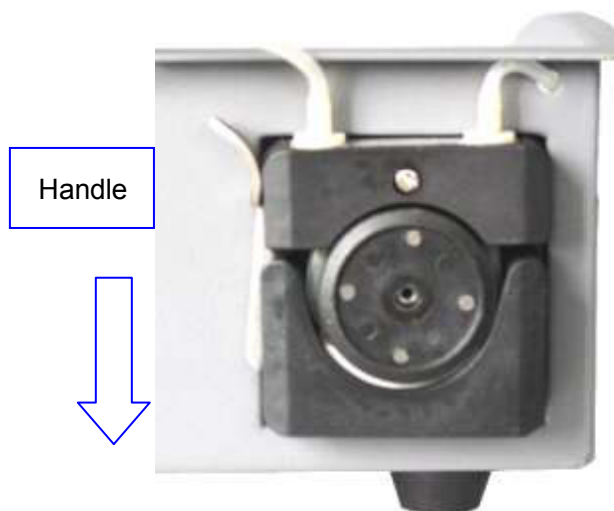


Figure 2-5

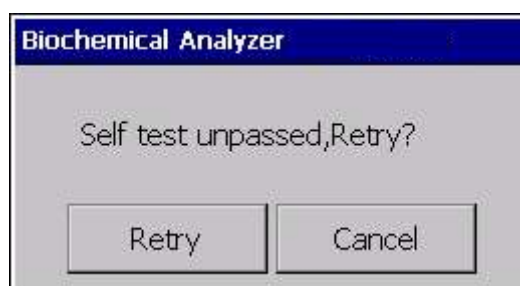


Figure 2-6

Chapter 3 Basic operations

1. Operations of soft keyboard

1.1 soft keyboard window


A soft keyboard set in this instrument. Users can press **【key】** in the keyboard interface or click the icon  in the window. The soft keyboard can be changed among closure, English input. The icon of soft keyboard shown as figure below:



Figure 3-1 Soft keyboard

1.2 Move the soft keyboard window

In the **【Input interface】** of soft keyboard window, hold the left key and move the mouse to a needed location, then loosen left key.

1.3 Instruction of soft keyboard

【←】 key: Delete the character before the cursor.

【Tab】 key: Change the focus.

【CAP】 key: Switch English letter between capitalization and lowercase.

2. Windows Routine Operation & Adnex

Hint:


You can skip if you are familiar with the operation of Microsoft Windows system.


2.1 Mouse

Mouse is available by connecting USB port of the device, such as Figure 3-2 shows:




Figure 3-2 Mouse

Put mouse on the stable worktable, hold the mouse and move it. Then you can see a cursor like  moving on the screen.

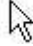Cursor of mouse: there is a cursor like  moving on the screen when we move the mouse, we call it mouse cursor.

The mouse cursor may change to different shapes:

Cursor  means some program is being processed and the user need wait.

Cursor  means that information can be input after the cursor.

Click, single click the left key of the mouse: means press down the left key of the mouse and then loosen it, but not move the mouse during the course.

Right click: press down the right key of the mouse and then loosen it, but not move the mouse during the course.

Double-click: click the left key of mouse two times swiftly.

2.2 Windows

Window is an interface for information display and operation, so window is also called interface. General speaking, put one or several control, window will have title bar and control button etc. Such as Figure3-3&3-4 shows:



Figure 3-3 Full Screen Window



Figure 3-4. Window with title bar and close button

2.3 Blinking cursor

We call the flickering vertical line “|” as blinking cursor, where you can input words.

If you click the character key on the keyboard; the character will be inserted to the location where the cursor is.

2.4 Control

Control means the displayed information or received element of the window such as: Radio Button, Edit box, Check Box, Optional Button, Combo Box, List, time & date picker (refer to the following section 2.6 to 2.13 is the instruction about control)

2.5 Focus

Focus designates the object input from keyboard. When a control or window hold focus, the control or window can be input from keyboard. Freely move the focus in the control by the keys [Tab], [Shift +Tab]. Judge holding focus or not, user can detail read below instructions about the controls.

2.6 Radio Button



Button without focus。



At the state of forbidden, the words become to gray and it cannot be clicked.



Button with focus, the dash rim stand for this button has focus

When control holds focus, the function of direction key and enter key as follows:

- 【↑】 : Move focus to previous control which can receive it, function is equal to (Shift +Tab) key on the keyboard.
- 【↓】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.
- 【←】 : Move focus to previous control which can receive it, function is equal to (Shift +Tab) key on the keyboard.
- 【→】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.
- 【Enter】 : Equal to pressing left key of the mouse or pressing Space key

2.7 Edition Bar



When hold focus, there will be a flickering vertical line or the letter is selected; the selected letter has blue background. Otherwise, it means the edition bar have no focus



Edit dialog box at the forbidden state, it become gray and its words cannot be input in the dialog box.

When control holds focus, the function of direction key and enter key as follows:

【↑】 : Move focus to previous control which can receive it, function is equal to (Shift + Tab) key on the keyboard


【↓】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.

【←】 : Move cursor to previous letter, function is equal to {←} key on the keyboard.

【→】 : Move cursor to next letter, function is equal to {→} key on the keyboard.

【Enter】 : Move focus to next control which can receive focus, the function is equal to (Tab) key on the keyboard.


2.8 Check Box

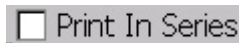

Unselected status, without focus currently.



Forbidden status.

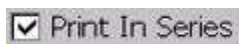

Selected status, no focus currently.



Unselected status, hold focus: words with broken line around them mean holding focus.

When control holds focus, the function of direction key and enter key as follows:

【↑】 : Move focus to previous control which can receive it, function is equal to (Shift + Tab) key on the keyboard.

【↓】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.

【←】 : Move focus to previous control which can receive it, function is equal to (Shift + Tab) key on the keyboard.

【→】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.

【Enter】 : Equal to pressing the mouse or Space key

2.9 Option Button



Selected status for flow cell without focus; unselected status for cuvette without focus.



Forbidden status for both flow cell and Cuvette.



Selected status for flow cell with focus; unselected status for cuvette without focus.

When control holds focus, the function of direction key and enter key as follows:

- 【↑】 : Move focus to previous control which can receive it, function is equal to (Shift +Tab) key on the keyboard.
- 【↓】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.
- 【←】 : Function is equal to the (→) on the keyboard
- 【→】 : Function is equal to the (→) on the keyboard
- 【Enter】 : Function is equal to pressing mouse or (Space) key on the keyboard.

2.10 Combo Box only can be chosen



Without focus currently, value of the selection is "No blank"



Forbidden status



Status with focus, value of the current selection is "No blank"



Hide or get down the dialog box by pressing the little triangle

hold focus and get down dialog box.

When control holds focus, the function of direction key and enter key as follows:

- 【↑】 : Choose previous entry of the current one, function is equal to (↑) key on the keyboard
- 【↓】 : Choose next entry of the current one, function is equal to (↓) key on the keyboard

【←】 : Move focus to previous control which can receive it, function is equal to (Shift +Tab) key on the keyboard.

【→】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.。

【Enter】 : Move the focus to the next control which can receive it, the effect as same as (Tab) key.

2.11 Combo Box can write in



The combo will display a flickering vertical line “|” or letter is selected when hold focus, the selected letter have blue background. Otherwise, it means this combo box have no focus.



Forbidden status.



Hold focus, letter is selected, and the background is blue.



Hold focus, letter is selected, and the background is blue, and the combo box is shifted down.。

When control has focus, the function of direction key and enter key on the device keyboard are as follows:

【↑】 : Select previous item of the current one, function is equal to (↑) key on the keyboard

【↓】 : Select next item of the current one, function is equal to (↓) key on the keyboard

【←】 : Select left item, function is equal to {←} key on the keyboard

【→】 : Select right item, function is equal to {→} key on the keyboard.。

【Enter】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.。

2.12 List Box

ab.	Name
1	TestDr1
2	TestDr2
3	TestDr3

There is no focus currently. No item selected.

ab.	Name
1	TestDr1
2	TestDr2
3	TestDr3

There is no focus currently. Select third item.

ab.	Name
1	TestDr1
2	TestDr2
3	TestDr3

There is focus currently. No item selected.

ab.	Name
1	TestDr1
2	TestDr2
3	TestDr3

There is focus currently. Select second item.

When control has focus, the function of direction key and enter key on the device keyboard are as below:

【↑】 : Select previous item of the current one, function is equal to (↑) key on the keyboard

【↓】 : Select next item of the current one, function is equal to (↓) key on the keyboard

【←】 : Select left item, function is equal to {←} key on the keyboard

【→】 : Select right item, function is equal to {→} key on the keyboard..

【Enter】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard..

2.13 Date Time Picker

2007-05-08 11:41:12 ▼

There is no focus in current

2007-05-08 12:49:41 ▼

Forbidden status

2007-05-08 11:41:12 ▼

There is focus in current



There is focus, and shifted down

When control has focus, the function of direction key and enter key on the device keyboard are as follows:

【↑】 : Select previous item of the current one, function is equal to (↑) key on the keyboard

【↓】 : Select next item of the current one, function is equal to (↓) key on the keyboard

【←】 : Select left item, function is equal to {←} key on the keyboard

【→】 : Select right item, function is equal to {→} key on the keyboard.。

【Enter】 : Move focus to next control which can receive it, function is equal to (Tab) key on the keyboard.。

3. Interface of the keyboard

There are 9 keys in the keyboard. The function of the key is as follows. (Figure 3-5)

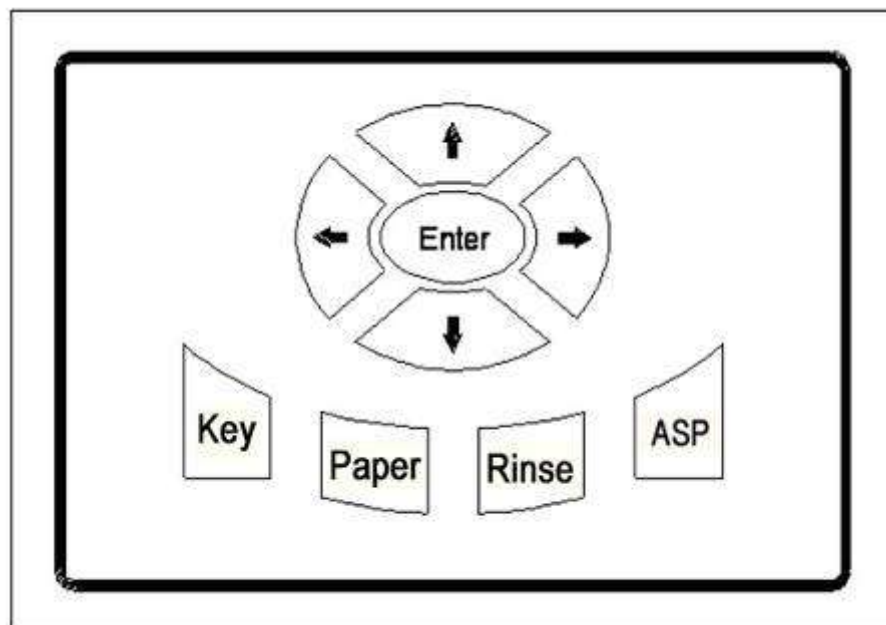


Figure 3-5 interface of the keyboard

3.1 【Key】

Control the soft keyboard among the off status, English language input status, Chinese language input status. See the section *1.1 soft keyboard window* for more information.

3.2 【Paper】

Control the printer paper, press the key to move paper, and loosen to stop.

3.3 【Rinse】

Press【Rinse】 to control the machine rinsing. When you press it once, the device would continuously aspirate the cleaning agent through the flow channel. Press it again to stop rinsing. Please notice that it is not effective to press 【Rinse】 when the machine is in testing. The function of this key is the same as the 【Rinse】 button in test interface.

3.4 【ASP】

Press 【ASP】 to aspirate testing liquid, which is as the same function as the 【ASP】 key under the pipette. ASP only works with the hint for aspirating by system.

3.5 【←,→,↑,↓,Enter】

The five keys are corresponding to those on the soft keyboard. Normally, user could use mouse and soft keyboard to finish all operation. User could use them instead of mouse and soft keyboard when they can't work.

4. Introduction of operation interface window

The signs below only appear on the main operation interface window.



Display the temperature of the testing room. Display as “room temperature” if no operation for temperature control.



Display the real time. (See the section “System setting” for adjusting the time)



Click it to control the soft keyboard of the off status, English language input status. It is the same as the 【key】 on the keyboard.



The status of the lamp is ON.



The status of the lamp is OFF.



Return. Press it to turn off the current window and return the previous window.



Return is in useless status. Please find [cancel] button for return.

Chapter 4 Boot-strap

1. Process of boot-strap

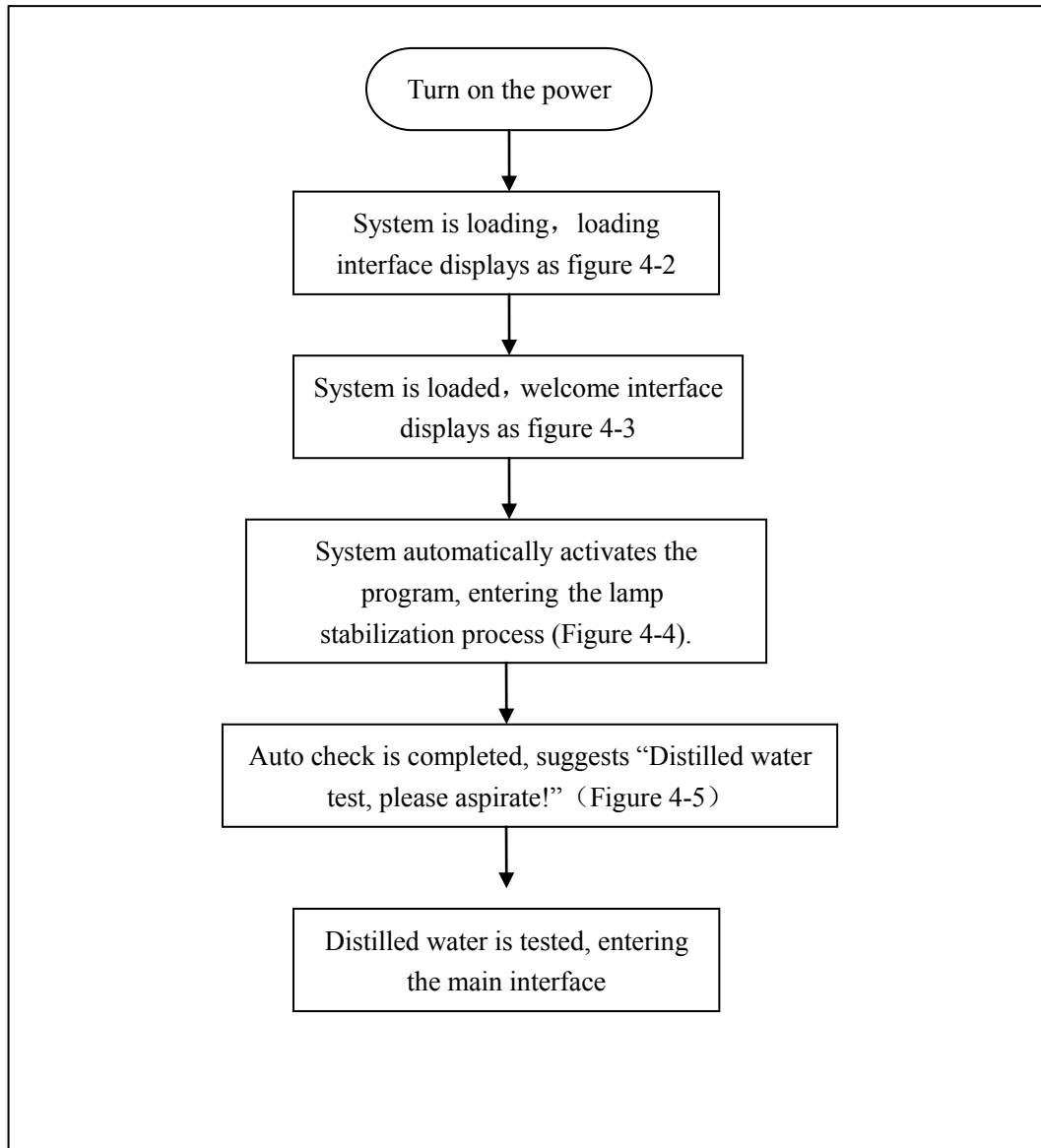


Figure 4-1 boot-strap

Hint:

Press **【Rinse】** once to aspirate cleanout liquid to rinse the flow system in no distilled water testing. Press **【Rinse】** again to finish rinsing. Do not test distilled water while it is in rinsing. It only could be operated after the rinse is finished.

2. Interface of boot-strap

Turn on the power, the system loading interface is as follow while the power light is on.

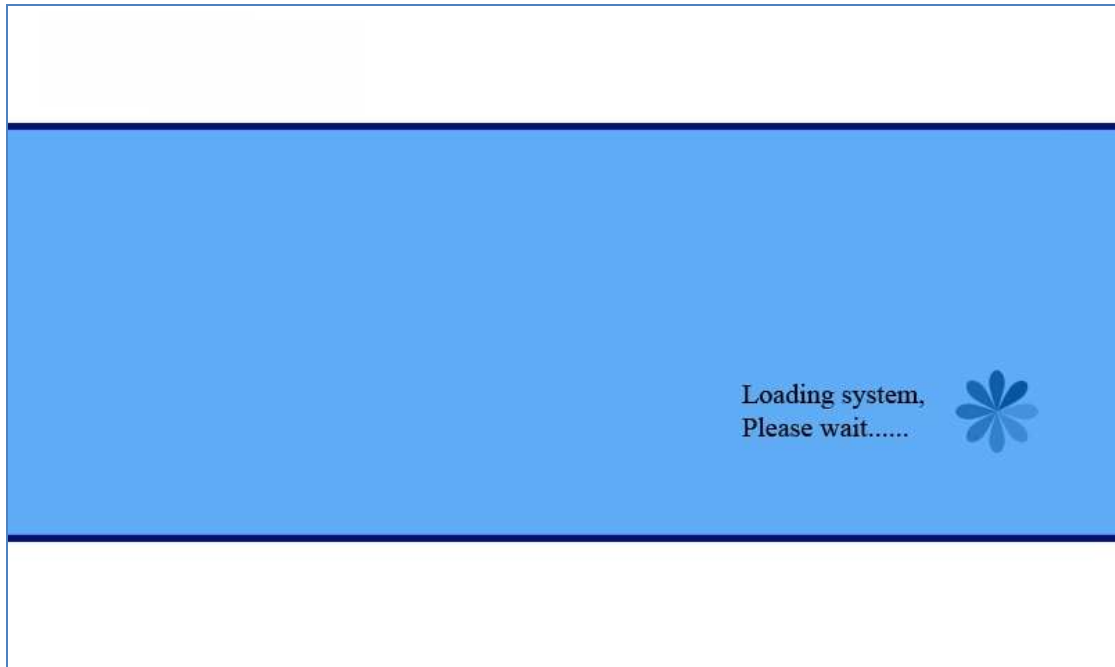


Figure 4-2

System is loaded and turns to system welcome interface Figure 4-3



Figure 4-3

Wait for auto check finishing. Figure 4-4

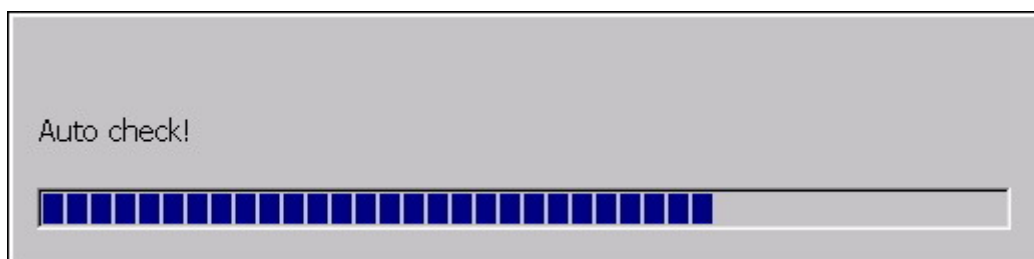


Figure 4-4

Wait for distilled water test .Figure 4-5

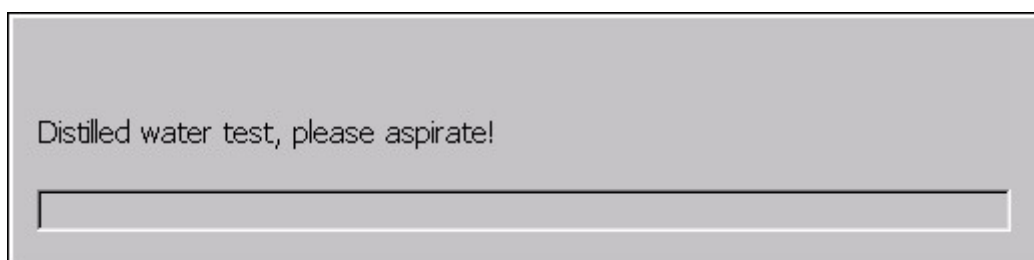


Figure 4-5

Put the pipette into distilled water, then press **【ASP】** to aspirate the distilled water.

Device start distilled water testing with each wavelength after aspirating. Figure 4-6



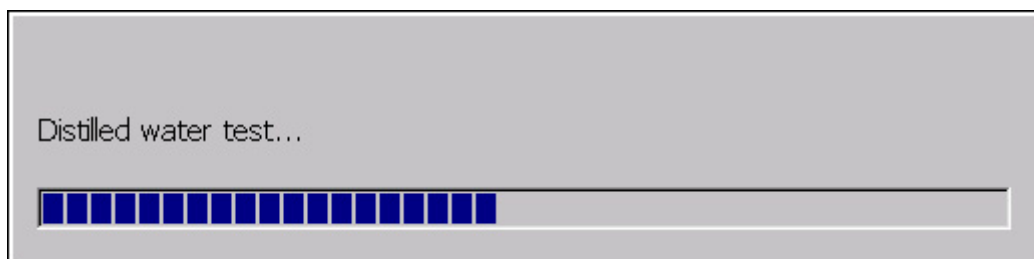


Figure 4-6

When self-test and distilled water test is qualified, device automatically enters the main interface (shown in Figure 4-9). Otherwise, system suggests that in figure 4-7 (1) and 4-7 (2), if user chooses to retry, device will begin self-test and distilled water test again.

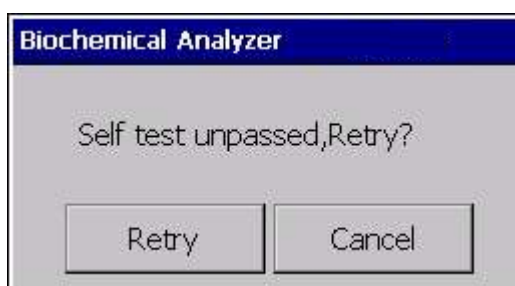

(1)

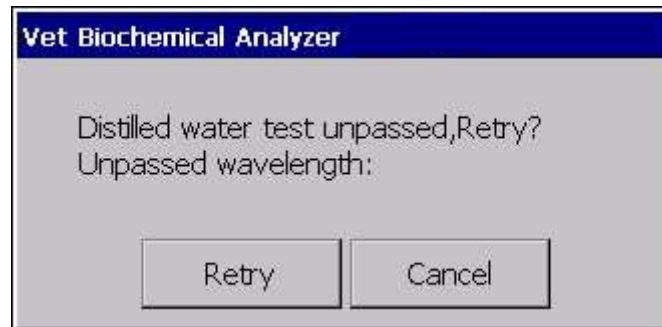

(2)

Figure 4-7

【Retry】: Press it to begin self-test and distilled water test again.

【Cancel】: Press it to enter main interface window as figure 4-9.

Hint:

- If self-test is failed, user can retest distilled water several times. Self-test failed may be attributed to the failure of optical or flow system. Users can retest after rinsing the pipe. If the problem cannot be resolved, you can contact the designated maintenance points.
- User could do repetitious test when distilled water test is unqualified. It could be caused by pollution of distilled water reagent, or even the flow channel. User could aspirate ethanol to rinse. If it is still unqualified after repetitious rinse, the device maybe in malfunction. Please contact with the maintain personnel.
- Press 【Rinse】 for repetitious rinse to renormalize when it appears abnormal after device be in long time storage or transportation.

System would display serial communication failure if system inner communication is in malfunction (Figure 4-8). Please contact with distributor or our company if user meets this problem.

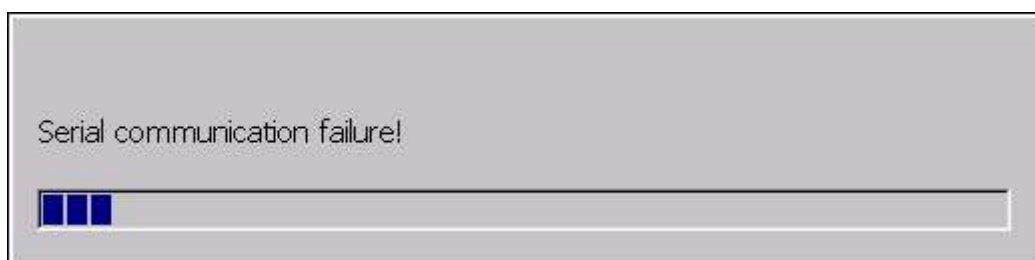


Figure 4-8

3. Main interface window

System enter main interface after distilled water testing. Figure 4-9



Figure 4-9 main interface

【TEST】: Press it and enter **【Select Test】** interface.

【RECORDS】: Press it and enter **【Sample Records】** interface.

【SYSTEM】: Press it to show a pop-up dialog box. User could select the QC Records、
Calibration Records、System setting interfaces

【EXIT】: Press it to turn off the device.

Chapter 5 System setting

1. System Setting

Press **【System】** in main interface, then select **【System Setting】** to enter the setting interface. User could set Hospital Name、System Date or any other parameters. Except **【Hospital Name】**, all other items were set already before left factory.

Setting

Hospital Name:

Instrument SN:

Lamp Idle Time(s):

Printer: ☐ Print In Series

Colorimetry Method

☒ Flowcell ☐ Cuvette

☐ Check Colorimetry Cell or Cuvette Position.

Data Transmission

☐ ASTM Mode ☒ RS232

Set Printer

Instrument Maintenance

Data Maintenance

Edit Save Cancel

System Time:

Edit Save Cancel

Figure 5-1

① General parameters setting area

【Hospital Name】: Input hospital name here.

【Instrument SN】: It is as the same as the nameplate of the instrument, which can't be amended for after-service.

【Lamp Idle Time】: In order to protect lamp, set a period time after testing to turn off the lamp.

【Printer】: There is a built-in thermal printer in instrument. Also it supports external printer based on PCL language such as HP LaserJet 1150 and HP LaserJet 1160.

【Print In Series】: Only available with external printer, please refer to the print sample record in the part of sample record.

【Colorimetry Method】: Flow cell or cuvette both available in the instrument. It is easy operation with small dosage reagent to select flow cell. It could reduce crossed pollution if user selects cuvette.

【Check Colorimetry Cell or Cuvette Position】: Automatic sensing of cuvette/flow cell insertion

【Data Transmission Method】: A method of data uploaded from device to computer by RS232 and use ASTM protocol or not.

【Edit】: Edit parameters above.

【Save】: Save the parameters user edited.

【Cancel】: Cancel to edit.

【Set Printer】: Click to enter the **【Printer Set】** interface.

【Instrument Maintenance】: Click to enter the **【Instrument Maintenance】** interface.

【Data Maintenance】: Click to enter the **【Data Maintenance】** interface.

② **System time setting area**

【System Time】 Display and set system time.

【Edit】 Click to edit date.

【Save】: Save the date user amended

【Cancel】 Cancel user's edit

2. Printer set

If you selected external printer, the button **【Set Printer】** becomes available. Click it to open the print option interface as figure 5-2 shows.

Figure 5-2

【Printer】 : Set printer model. If you use HP laser jet, please select “PCL Laser”.

It is enough if you set the printer, the options below are all unnecessary.

【Port】 : Default LTP1; please keep it.

【Path】 : Network shared printer. It is not available now.

【Paper size】 : Default A4, all the usable printers use A4 paper.

【Draft mode】 : Print lightly to save ink or toner.

【Print area】 : Default full, please keep it.

【Orientation】 : Default Landscape, please keep it.

【Margins】 : Set the margins of each side, Default 25.4mm (1 inch).

Press **【OK】** button to save the settings.

Available printers

	Brand	Model	Type
1	HP	2014	PCL laser
2		P1505N	PCL laser
3		ProK5300	Ink-jet
4		ProK5400d	Ink-jet
5		ProL7380	Ink-jet
6		ProL7580	Ink-jet
7		460cb	Ink-jet
8		ProK850	Ink-jet

	Brand	Model	Type
9	Epson	ME1	Ink-jet
10		ME1+	Ink-jet
11		ME30	Ink-jet
12		ME Office 70	Ink-jet
13		ME Office 80W	Ink-jet

3. Instrument Maintenance

Press **【Instrument Maintenance】** in System Setting interface, then enter the Instrument Maintenance interface as figure 5-2.

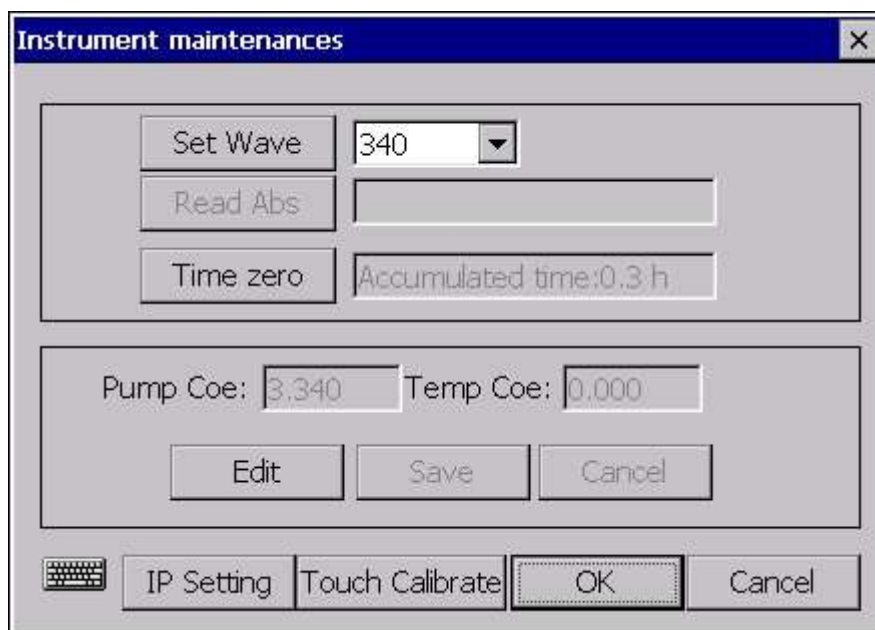


Figure 5-2

【Set Wave】: select wavelength in the combo box, then click this button to set wave.

【Read ABS】: click it to read AD and ABS.

【Time Zero】: click it to clear the accumulated time after you replace the lamp.

【Pump Coe】: correct it to adjust the aspirate volume error.

【Temp Coe】: correct it to adjust the temperature error.

【Edit】: Edit parameters above.

【Save】: Save the parameters user edited.

【Cancel】: Cancel to edit.

【IP setting】: Click it to enter the IP setting procedures (if no network is selected, this button will disappear in the interface)

【Touch Calibrate】: Click it to pop-up the "sure to touch calibrate?" dialog box. System enters the touch screen calibration procedure when the "Yes" button is clicked. (If touch screen isn't selected, this button will disappear in the interface).

3.1 Touch Screen Calibration

Click the **【Touch Calibrate】** in system setting interface, system pops up the dialog box as figure 5-3.

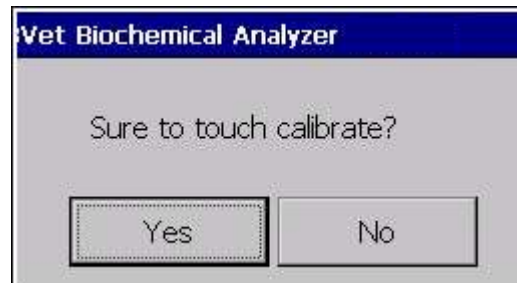


Figure 5-3

Click **【Yes】** to enter the interface in figure 5-4.

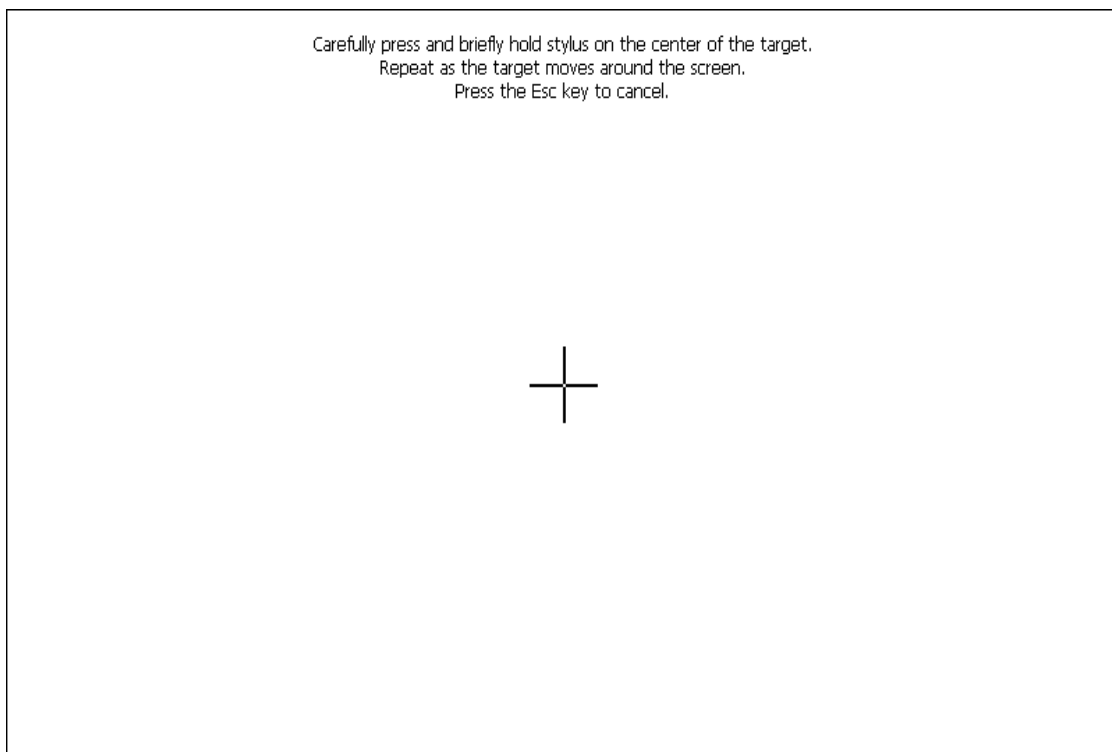


Figure 5-4

According to the tips on the screen, user can operate it with a touch pen. When calibration is accurate after clicked five times, you can click once on the screen to store the information and exit; if not, system will automatically re-enter the calibration status.

Hint

User can only use this function when the touch screen is purchased.

3.2 IP Setting

Click **【IP Setting】** in the system setting interface, enter the interface in figure 5-5.



Figure 5-5

Double-click network connection icon in the interface, enter the IP setting interface, shows in figure 5-6

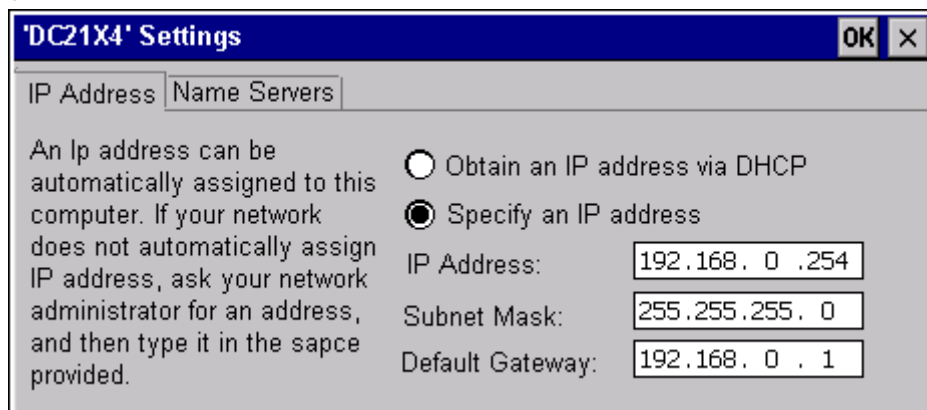


Figure 5-6

Because of the differences of network environment, this manual can not specifically introduce the IP setting process. If user can not understand how to operator, you can ask the local network administrator. If the IP Setting is finished, data can be uploaded from device to personal computer through data transmission.

Hint

User can only use the function when the network is selected.

4. Data Maintenance

Click **【Data Maintenance】**, enter the interface as figure 5-7, user can select the operation from “Data Backup, Data Import, Delete Data, or Data transmission”.

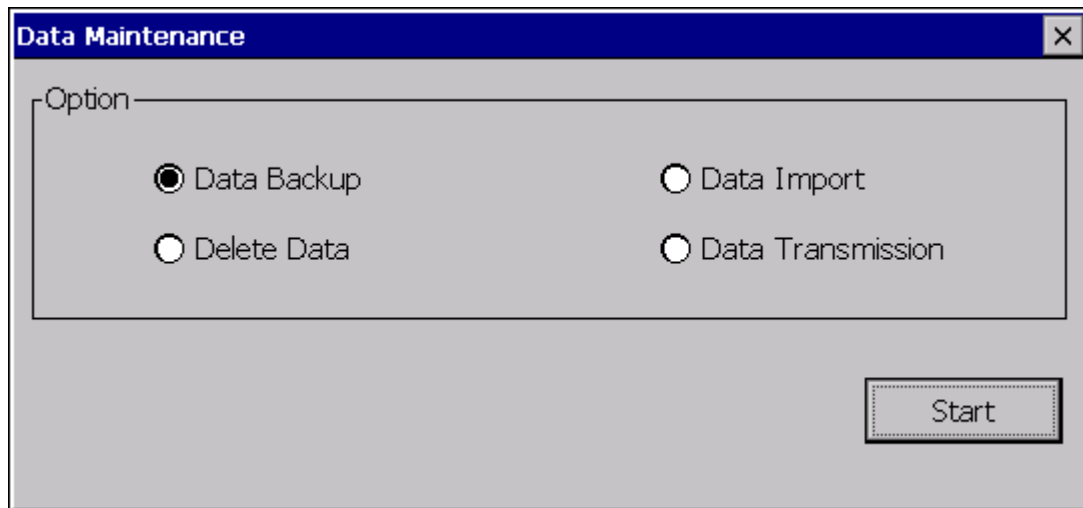


Figure 5-7

【Data Backup】: Select **【Data Backup】** and click **【Start】** to backup the data.

【Data Import】: Select **【Data Import】** and click **【Start】** to import the data.

【Delete Data】: Select **【Delete Data】** and click **【Start】** to delete the data.

【Data Transmission】: Select **【Data Transmission】** and click **【Start】** to transmission the data.

4.1 Data Backup

Insert SD or Flash Card, then select **【Data Backup】** in figure 5-7 and click **【Start】**, system will backup the data to SD or Flash Card.

4.2 Data Import

Insert SD or Flash Card, then select **【Data Import】** in figure 5-7 and click **【Start】**, show the dialog box as figure 5-8.

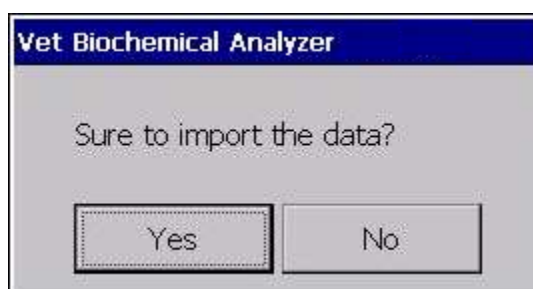


Figure 5-8

【Yes】: Click to import the data. System search the data file and import the data automatically.

【No】: Click to quit importing data , return to the **【Data Maintenance】** interface.

4.3 Delete Data

Select **【Delete Data】** in figure 5-6, the interface update as figure 5-9.

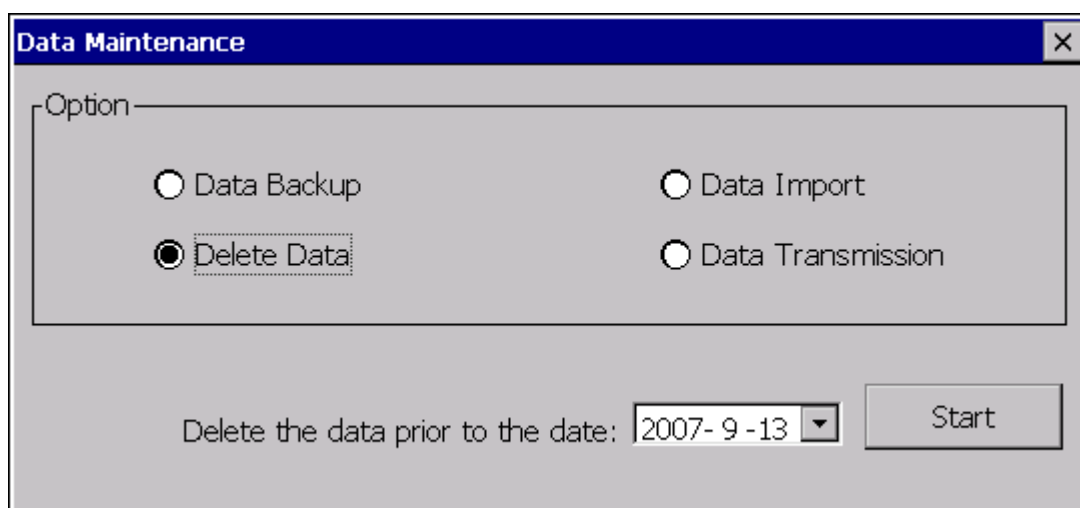


Figure 5-9

Select the date in figure 5-9 and click **【Yes】**, show the dialog box as figure5-10.

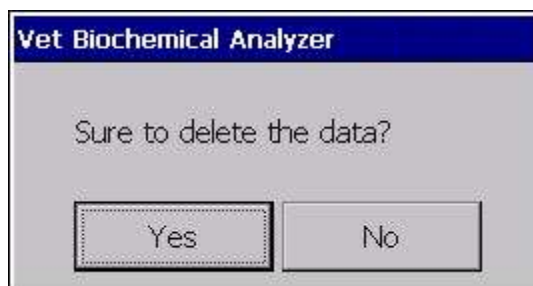


Figure 5-10

【Yes】: Click to delete the data. System deletes the control and sample records prior to the selected date as figure 5-9.

【No】: Click to suspend the implementation of deleting data, return to the 【Data Maintenance】 interface.

4.4 Data Transmission

Select 【Data Transmission】 in figure 5-7, the 【Data Maintenance】 interface update as figure 5-11:

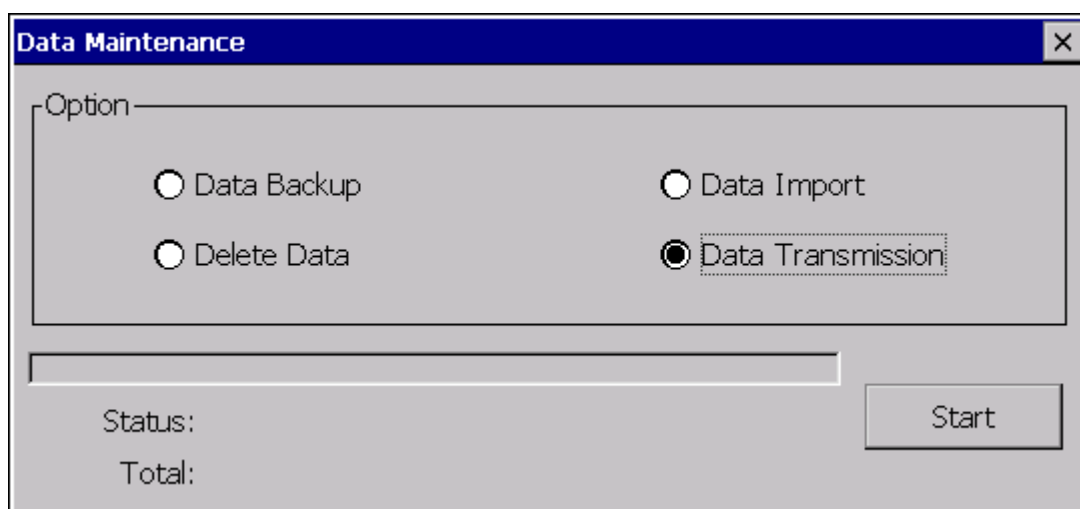


Figure 5-11

Click 【Start】, the 【Data Maintenance】 interface enters the data transmission status as figure 5-12.

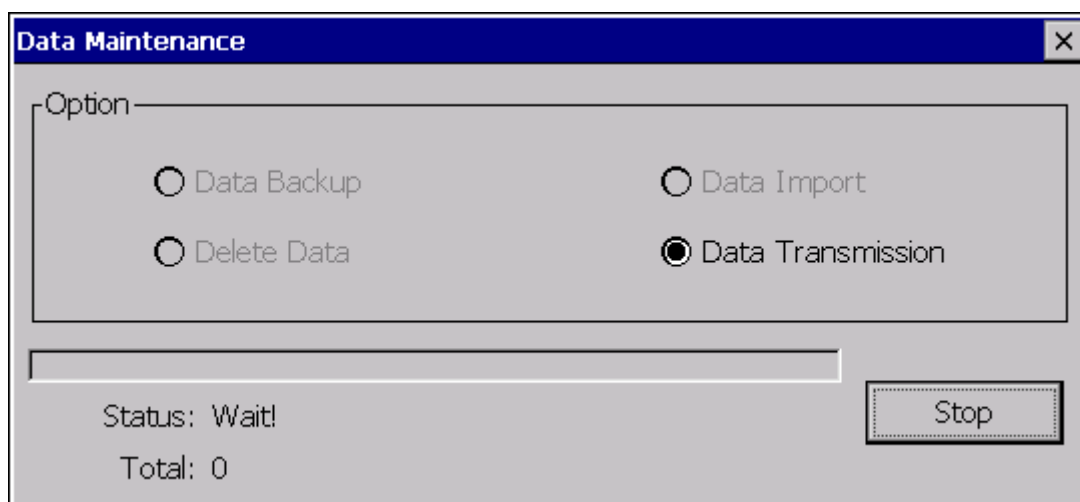


Figure 5-12

【Stop】: Click it to pop up the dialog box in figure 5-13.

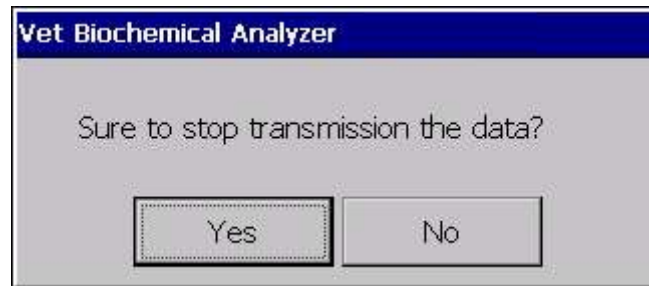


Figure 5-13

【Yes】: Click it to stop data transmission.

【No】: Click it to return data transmission status.

Hint

Steps of data transmission from device to computer are as follows

- According to the device setting and actual usage situation. User can link up the device and computer with serial line or net line.
- Set "data transmission mode" to be a corresponding mode in the 【System Setting】 interface.
- Operate it reference to the section "data transmission", and then enter the data transmission status.
- Run "BCA Data Center" software in personal computer, user can operate the software reference to help document and obtain data from the device.

Chapter 6 Animal Setting

User must select the type of test object before testing. User can select it in the **【Select Animal】** interface. If you do not see the choice as to the types, you can increase the animals by importing new animal names and adding new animals Photo.

1. Select Animal

As shown in Figure 4-10 in the main interface, click **【Test】** button, enter the **【Select Animal】** interface first as shown in Figure 6-1.

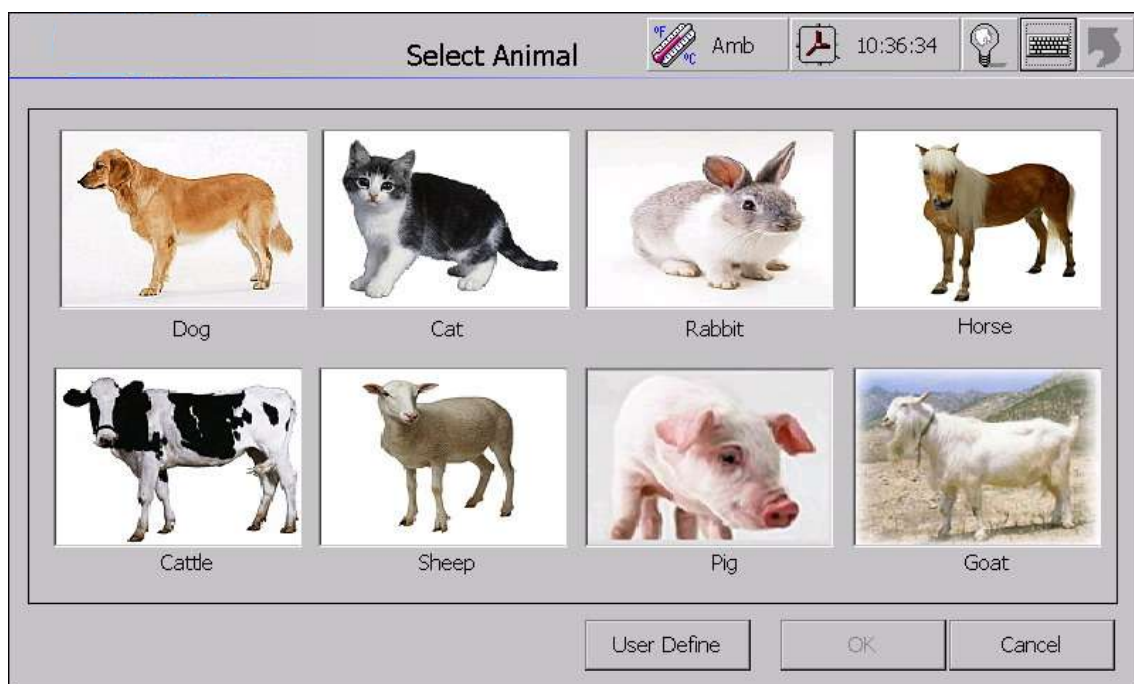


Figure 6-1 Select Animal

【User define】 : If the intrinsic species of animals doesn't include testing animal, click this button to enter the **【Add Animals】** interface for adding new animal species.

【OK】 : Click the button to select after clicked the picture of testing animals.

【Cancel】 : Click the button to suspend the operation and return to the former interface.

Hint: Double-click the pictures of some animals is equivalent to click the pictures and **【OK】** button.

2. Add Animal

Click **【Customize animal】** button in the **【Select Animal】** interface, then enter the **【Customize animal】** interface, show as follow.

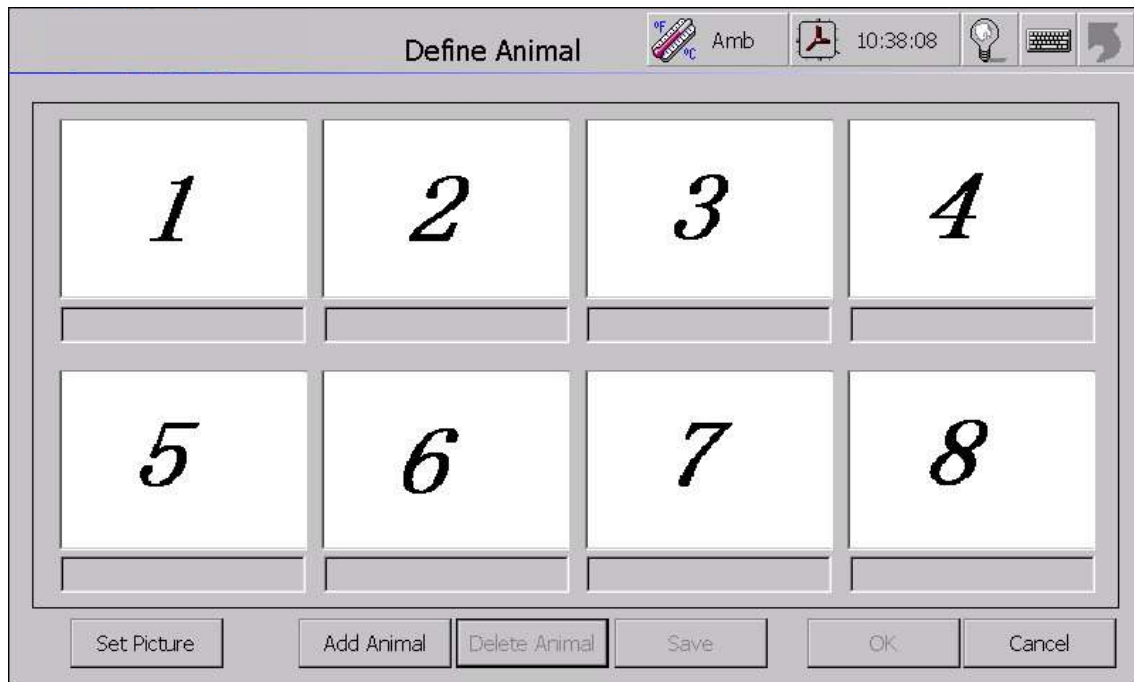


Figure 6-2 Select Animal

The edit box Below each picture frame display animal names, if the edit box isn't blank , show that there has been set up animal species, on the contrary, show that there is not yet provided animals.

【Set Picture】 : Click this button after Click a picture to pop up figure 6-3 choice dialog, select the picture files, Click **【OK】** to add a picture.

【Add Animal】 : Click to turn the edit box to be inputted, then user can input new animal species name

【Delete Animal】 : Click the button after click the type of animal pictures which has been set up, and then delete the type of animal. Purchase edit box empty, and restore pictures to the factory initial state.

【Save】 : Click to save new animal names and the edit box can't be inputted.

【OK】 : Click the button after clicked the animal to select it.

【Cancel】 : Abandon the current operations, return to the former interface.

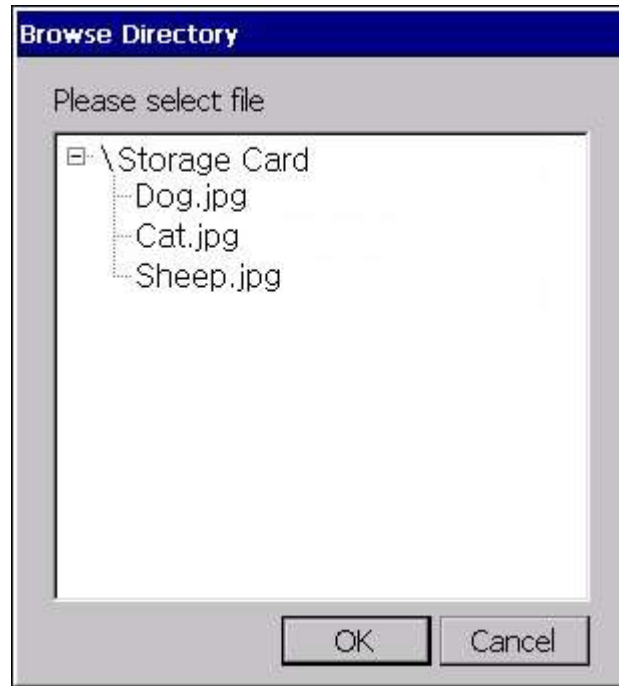


Figure 6-3 Select file

Hint:

- Double-click the picture of some animal is equivalent to click the picture and **【OK】**.
- The format of the picture must be "Jpg" format, and size is less than 50KB.
- Delete animal species will remove all relevant records.
- When user need to add new types of animals after 8-defined animal species have been set up, user could add new types of animals by deleting some of the definition of animals.

Chapter 7 Test Setting

User could select test in 【select test】 interface. If there is no test user required, user could add test. Input test name and correct parameters when add test, then save it.

Ensure test parameters is as the same as the reagent manual. The parameters maybe different by different reagent manufacturer, please correct parameters when change the reagent manufacture.

1. Select test

In the main interface as figure 4-10, click 【Test】, and enter 【Select animal】, select some animal and click 【OK】, then enter 【Select Test】 interface, click 【Non-Template】, display as the figure 7-1。

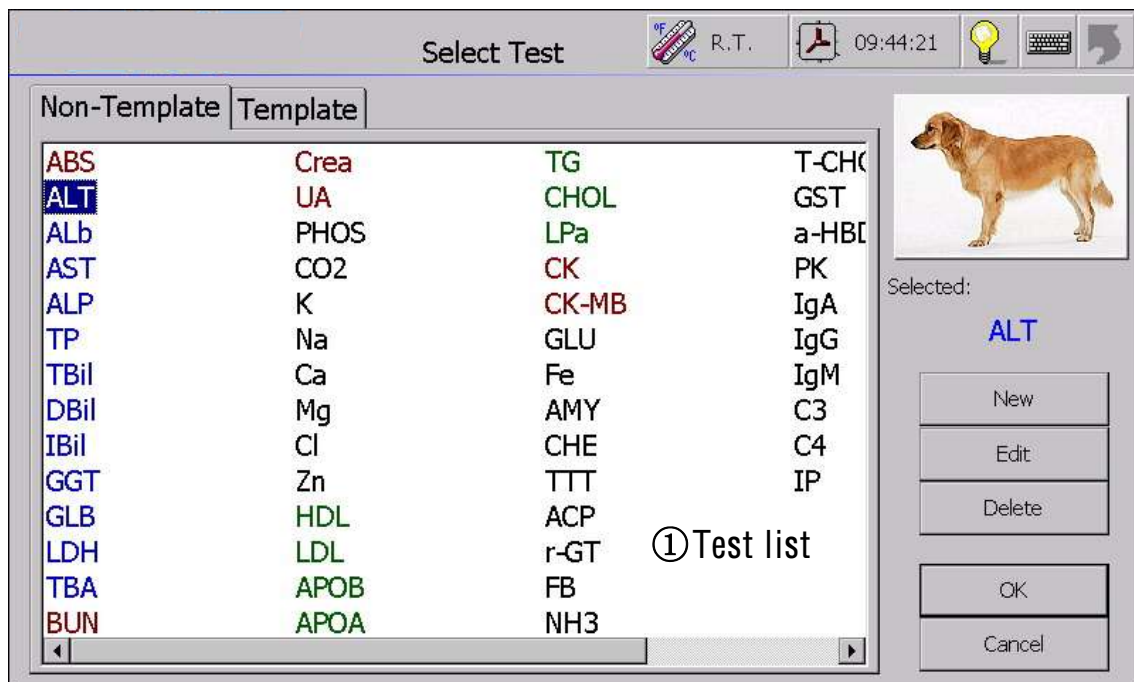


Figure 7-1 Select Test Interface (Non-Template)

【Non-Template】: Test with non-template. The test list will display all tests. Select one and click 【OK】 to begin testing with non-template, See the Chapter8<Test with non-template>。

【Template】: Test with template. Select all or part tests of one template, and then click【OK】 to begin testing with template. You can amend the tests of template or increase the custom template. See the section 7-9 of present chapter.

- ① Test list displays all current tests. The background color of test would turn blue when click the test (Figure 7-1). For example, the selected test ALT appear blue background color, and ATL is displayed in down side of the window in blue color. Then press **【OK】** to enter **【test】** interface .

【New】: Click to enter **【Edit Test Parameter】** interface for adding a new test.

【Edit】: Click to enter **【Edit Test Parameter】** interface for amending parameters for selected test.

【Delete】: Delete the selected test.

【OK】: Click to enter test interface for testing after selecting the test in **【test list】** .

【Cancel】: Cancel this operation, return to previous interface.

 **Hint:**

- The operation 'select test and press **【OK】** 'is equal to double click test in the test list.
- New test is custom test. Existent tests are fixed before the instrument leave factory. User only delete custom test.
- Deleted test would delete relevant control, calibration, sample records, and so confirm the operation is correct before delete. System would offer operation notice.

2. Add New Test

In the **【Select Test】** interface, in the non-Template mode, Select “New Test” display as figure 7-2 and set every parameter of the new test. Select **【QC】** and set the QC parameters. Select **【Calib. Para】** and set the calibration parameters. Click **【Save】** to save and exit.

Test Parameter

R.T. 09:49:57

Name: Full Name:

Reagent Batch No.: Rgt. Manf.:

Primary Wavelength: Units:

Secondary Wavelength: Test Method:

Temp.: Factor:

Sample Vol(ul): Delay Time(s):

Reagent Vol(ul): Measuring Time(s):

Aspirate Vol(ul): Blank Method:

Ref Upper Limit:

Ref Lower Limit:

Linearity:

Lower Limit:

Linear Error: %

7-2 Parameter Setting

【Name】: A brief name of the test, e.g. "ALT". A new name within 15 letters must be given before any other operation of the new test.

【Full Name】: The full name (within 50 letters) of the new test, e.g. "Alanine Aminotransferase "

【Reagent Batch No.】: Reagent batch number within 15 letters.

【Rgt. Manf.】: Reagent manufacture name within 50 letters.

【Primary Wavelength】: There are 7 levels of wavelength selectable (340, 405, 505, 546, 578, 620, and 670) and one user define wavelength available.

【Secondary Wavelength】: Please set the secondary wavelength when in double wavelength method.

【Temp.】: Testing temperature with R.T.(Room Temperature), 25℃, 30℃, 37℃ selectable. There is no temperature control in atmosphere mode.

【Sample Vol(ul)】: Sample Volume. Please pay attention to the proportion between Sample and Reagent in the operation manual of Reagent.

【Reagent Vol(ul)】: See **【Sample Vol(ul)】** .

【Aspirate Vol(ul)】: It means the aspirating volume of tested liquid each time. We recommend it no less than 400μl each time to make sure the accurate result. (Standard volume=500ul).

【Units】: Test unit within 15 letters (See the reagent manual).

【Test Method】: There are three methods selectable. (End-point, Fixed-time, Kinetics)

【Factor】: User can input the factor with the keyboard according to the operation manual

of the reagent. Or the instrument can test standard material to calibrate the factor after inputting the standard parameters. If the calibration method is non-linear, user can calibrate only.

【Delay Time(s)】: It means the time before the tested liquid ready. Normally, it is 5 seconds in End-point testing. For Fixed-time and Kinetics testing please see the operation manual of the Reagent.

【Measuring Time(s)】: It means the reaction time for the calculation. The time should be “0” for the reaction is finished; it could not be input by the user. And the time should be more than 0 when the test method is Fixed-time or Kinetics.

【Blank Method】: It is including Reagent Blank Test, Sample Blank Test and No blank Test. Control Test and Standard Test need control and standard blank.

【Ref Upper Limit】: The reference upper limit(maximum) of normal testing data (>0).

【Ref Lower Limit】: The reference lower limit(minimum) of normal testing data (≥ 0). Upper Limit $>$ Lower Limit.

【Linearity】: Please set the Linear Upper Limit (≥ 0) and Lower Limit (≥ 0) according to the user manual of reagent to control the testing data in the normal range. Please regulate the proportion of the tested liquid when over range data occurred. Linear Upper Limit \geq Linear Lower Limit.

【Lower Limit】: See **【Linearity】**.

【Linearity Error】: It is used to control the Linear of testing data in the normal range. The instrument will alarm when over range data occurred.

【Preview】: It is used to preview the data and parameters to be printed.

【Print】: It is used to print the data and parameters.

【QC Para】: Click to set the control parameters.

【Calib. Para】: Click to set the calibration parameters.

【Save】: Click to save the settings and return.

【Cancel】: Click to return without saving.

Hint: About the test method


End-point: It means to test the absorbency of the tested liquid after the reaction between Reagent and Sample finished. Then the concentration will be calculated according to the Lambert-Beer equation.

Kinetics: It means to test the absorbency of the tested liquid continuously during the reaction. And test the slope during the linear reaction. Then the concentration will be calculated according to the Lambert-Beer equation.

Fixed-time: It is a especial example in Kinetics. It means to test the absorbency of the tested liquid continuously during the reaction. And measure the absorbency of two different points during the linear reaction. Then the concentration will be calculated according to the Lambert-Beer equation.

3. Test Parameters Edition

Enter **【Select Test】**, testing with non-template, Select a test in the interface as figure 7-1, e.g. 'ALT' . Click **【Edit】** or click **【Para...】** in **【Test】** interface to display the interface as figure 7-3. Click **【Save】** to save after finished parameter setting. (See part 1 of this chapter about Parameter Setting.)

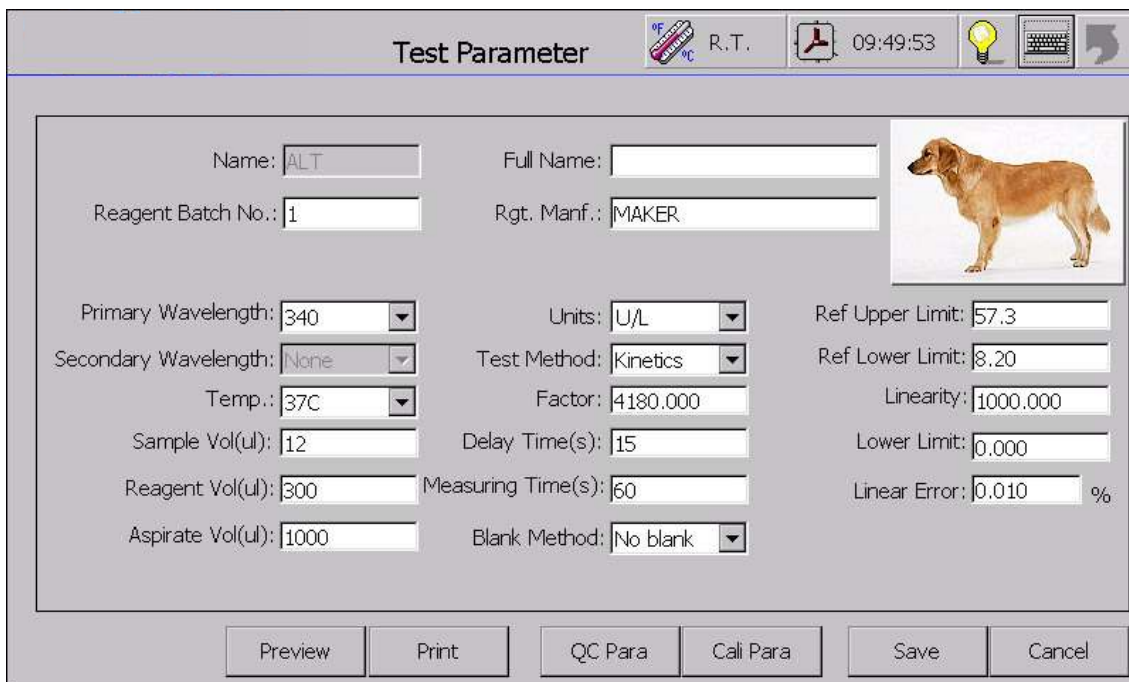

The 'Test Parameter' dialog box contains the following fields and controls:

- Title Bar:** Test Parameter, with icons for temperature (°F/°C), R.T., a printer, a clock (09:49:53), a lightbulb, a keyboard, and a refresh button.
- Main Area:**
 - Name:** ALT (text field)
 - Full Name:** (empty text field)
 - Reagent Batch No.:** 1 (text field)
 - Rgt. Manf.:** MAKER (text field)
 - Primary Wavelength:** 340 (dropdown menu)
 - Units:** U/L (dropdown menu)
 - Secondary Wavelength:** None (dropdown menu)
 - Test Method:** Kinetics (dropdown menu)
 - Temp.:** 37C (dropdown menu)
 - Factor:** 4180.000 (text field)
 - Sample Vol(ul):** 12 (text field)
 - Delay Time(s):** 15 (text field)
 - Reagent Vol(ul):** 300 (text field)
 - Measuring Time(s):** 60 (text field)
 - Aspirate Vol(ul):** 1000 (text field)
 - Blank Method:** No blank (dropdown menu)
 - Ref Upper Limit:** 57.3 (text field)
 - Ref Lower Limit:** 8.20 (text field)
 - Linearity:** 1000.000 (text field)
 - Lower Limit:** 0.000 (text field)
 - Linear Error:** 0.010 % (text field)
- Buttons:** Preview, Print, QC Para, Cali Para, Save, Cancel.

Figure 7-3 Testing Parameter Setting



Notice:

1. The name of test could not be changed by editing in an existed test.
2. The system will alarm "Factor or calibration information may be invalid" if no calibration information after changing the parameter of Reagent Batch No., Primary Wavelength, Secondary Wavelength, Temperature, Test Method, Delay Time, Measuring Time and Blank Method. Please input the correct factor or to calibrate in this occasion.

4. ABS Parameters

ABS is used to inspect the performance of the instrument according to ABS graph during the test. There is no calculation result in ABS Test.

Select ABS in the interface as figure 7-1 to display following interface as figure 7-4.

The screenshot shows a 'Test Parameter' window. The title bar includes a thermometer icon, '37.0C', a clock icon, '10:53:46', a lightbulb icon, a keyboard icon, and a help icon. The main area contains the following parameters:

Parameter	Value
Name	ABS
Primary Wavelength	340
Blank Method	No blank
Temp.	R.T.
Num Of Read Points	256
Read Time(s)	256
Aspirate Vol(ul)	500

Buttons: Save, Cancel

Figure 7-4 Parameter Setting of ABS

【Num of Read Points】: The total points of reading the ABS.

【Read Time(s)】: The total time(seconds) of reading the ABS.

See part 2 of present chapter for others.

5. Calibration Parameters

Click **【Calib. Para】** in the **【Parameter Setting】** interface, then display as figure 7-5:

Calibration Parameter

Test Name: ALT

Calibration Method: Linear

Repeat Times: 1

No.	Conc.
-----	-------

Add

Delete

OK Cancel

7-5 Calibration Parameters

【Calibration Method】: There are linear and non-linear selectable. There is only one standard concentration point in linear method. There are not less than 2 and not more than 10 standard concentration points in non-linear method.

【Repeat Times】: Repeat the test of standard concentration points and calculate calibration factor according to the average to improve the precision. The repeat times should not less than 1 and not more than 3.

【Standard Concentration】: It is to list the concentration of existed standard concentration points.

【Add】: Click to add a standard concentration point.


【Delete】: Click to delete a standard concentration point from the list.

【OK】: Click to save and return to interface 7-2.

【Cancel】: Click to return to interface 7-2 without saving.

6. QC Parameter

Click **QC Parameters** in the **Parameter Setting** interface, enter the **QC Parameters** interface ,show as figure 7-6:

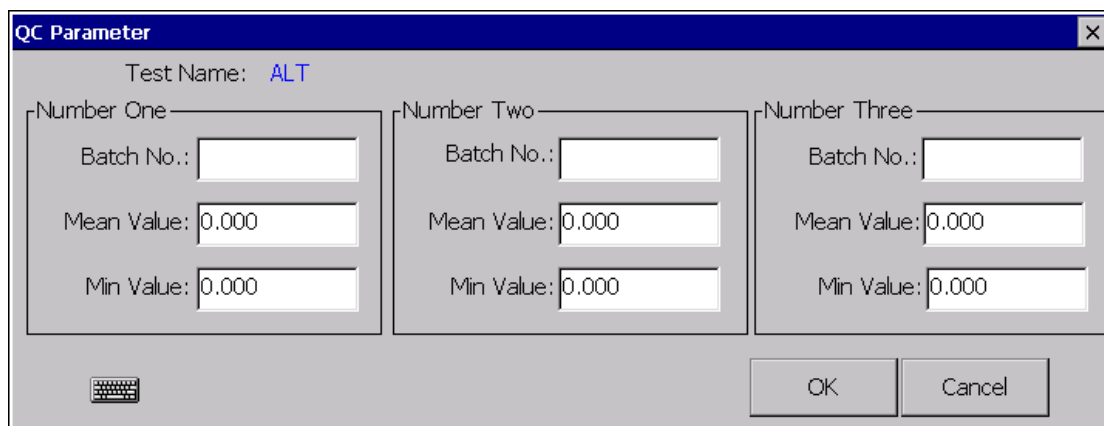

The screenshot shows a 'QC Parameter' dialog box. At the top, the title bar reads 'QC Parameter' with a close button. Below the title bar, the text 'Test Name: ALT' is displayed. The main area is divided into three columns: 'Number One', 'Number Two', and 'Number Three'. Each column contains three input fields: 'Batch No.', 'Mean Value: 0.000', and 'Min Value: 0.000'. At the bottom left, there is a keyboard icon. At the bottom right, there are 'OK' and 'Cancel' buttons.

Figure 7-6 QC Parameter Setting

Please input the parameters according to the operation manual of Control material. This instrument allowed a maximum of 3 Control test.

Batch No.: It means the batch number of the control material.

Mean Value: The mean value of the control material.

Min Value: It means the minimum value of the control material.

OK: Click to save the QC parameter setting and return to **Test Setting Parameter** interface.

Cancel: Click to return to the **Test Setting Parameter** interface without saving.

7. Select template

In the main interface as figure 4-10, click **【Test】**, and enter the **【Select Test】** interface after select one animal, then click **【Template】** to display as figure 7-7.

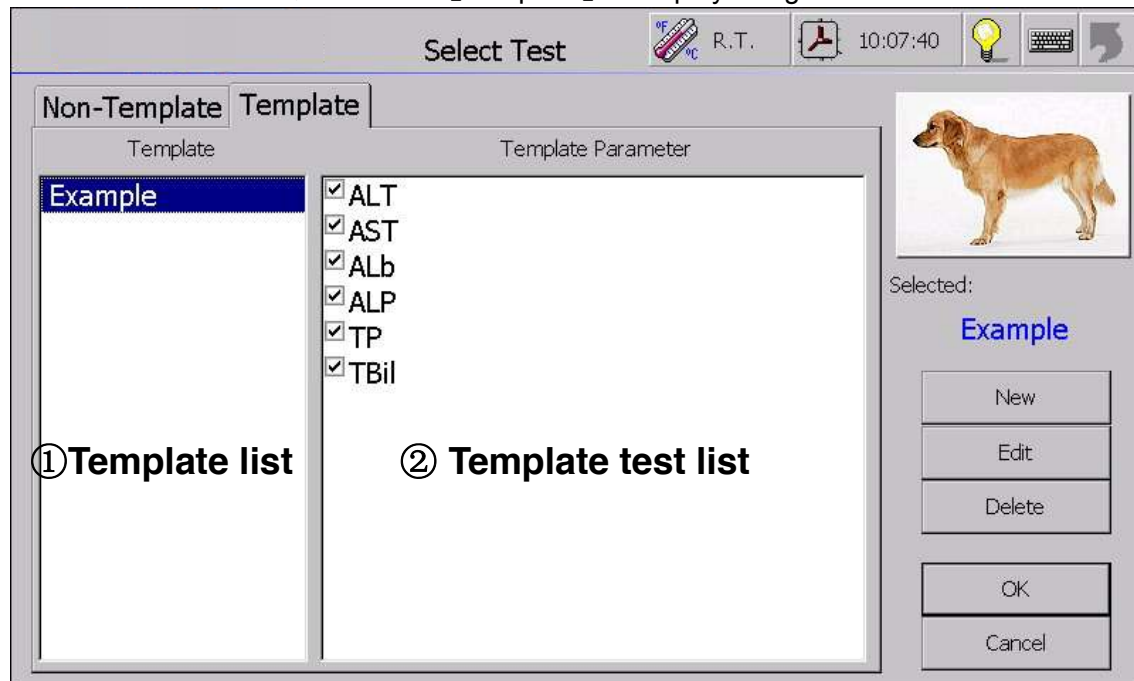


Figure 7-7 the Select Test interface (In the template mode)

【Non-template】: See the section 1-6 of present chapter for the non-template mode.

【Template】: Test with template. In the template mode, select all or part tests of the one template, and click **【OK】** to begin testing with template. You can amend the test of template or increase the custom template. See the section 1-6 of present chapter for more information.

- ① The template list show all the template, click one and the background will become blue. The selected template “Example” background is blue as figure 7-7, at the same time, system will prompt the selected “Example” template with blue words at the bottom of the window.
- ② The test list of template shows all the tests of this template. There is a check-box in front of the test respectively. If the check-box is selected, the test is selected. For example as figure 7-7, all the projects of the template “Example” are selected. Then click the **【OK】** to begin testing with template, and test the tests in turn. Please see the 《2.2Templet Test Process》 of chapter 7.

【New】: Click it to enter the **【Template Parameter】** interface, and add new a template.

【Edit】: Click it to enter the **【Template Parameter】** interface, and amend the project of present template.

【Delete】: Click it to delete present template.

【OK】: Select template on the 【Template List】 , and then select the test on 【Test list of template】 , click it to enter the test interface to begin testing the selected test.

【Cancel】: Quit operating the selected test, back to previous windows.

Hint:

- The new template is user-defined. Users can only edit or delete the user-defined template.

8. New template

In【Select test】interface, testing with template, Click 【New】, display as figure 7-8. Input name of new template, and select the test of this template, then click 【OK】 to add a new template.

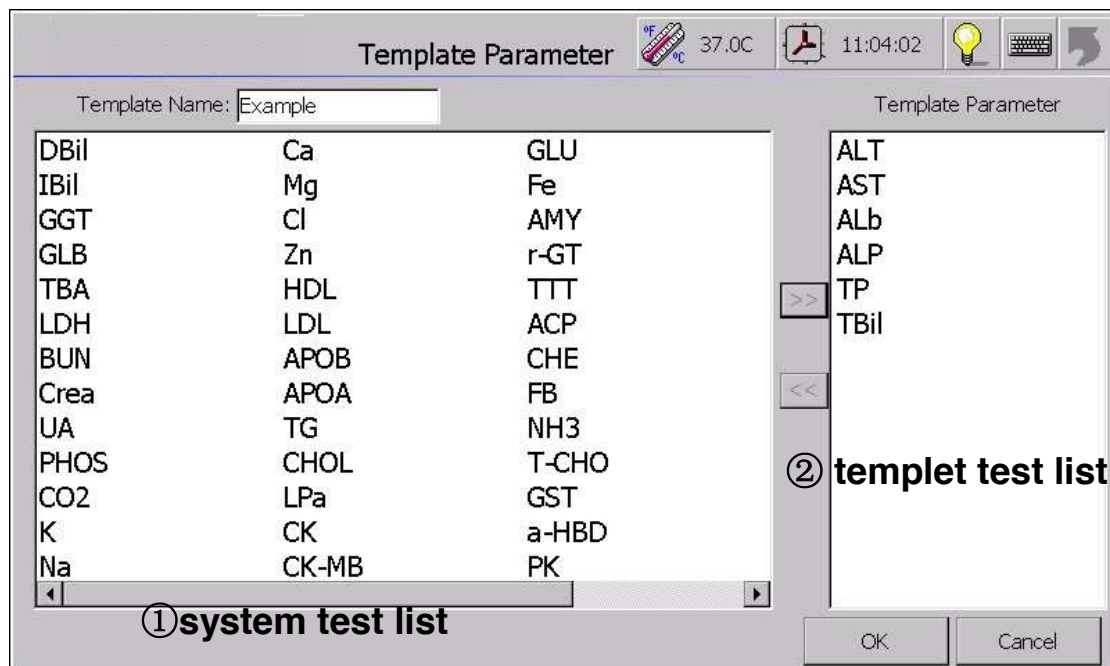


Figure 7-8 Template Parameter interface

① The system project list shows all the tests except tests of the selected template.

② The template test list shows all the tests of present template.

【>>】 Select one or more tests on system test list, and click the button to add the selected test to present template.

【<<】 Select one or more projects on template test list, and click the button to delete the selected test from present template.

【OK】 Finish editing present template and save.

【Cancel】 Quit editing present template.


9. Edit template

In **【Select Test】** interface, testing with template, click **【Edit】** and show the interface as figure 7-8, then edit the test of the template, and click **【OK】** to finish editing, See the section 8. New Template' of chapter 8 for others.

Chapter 8 Sample Test

1. Sample Test Interface

Select the non-template mode in【Select Test】interface, then select one and click 【OK】 to enter 【Test】 interface, display as figure 8-1.If select the template mode, the interface nearly accord with figure 8-1 except the button 【Test...】changed to【Next Test】.Select the test name and click 【Select Test】 to enter the 【Test】 interface as figure 8-1.

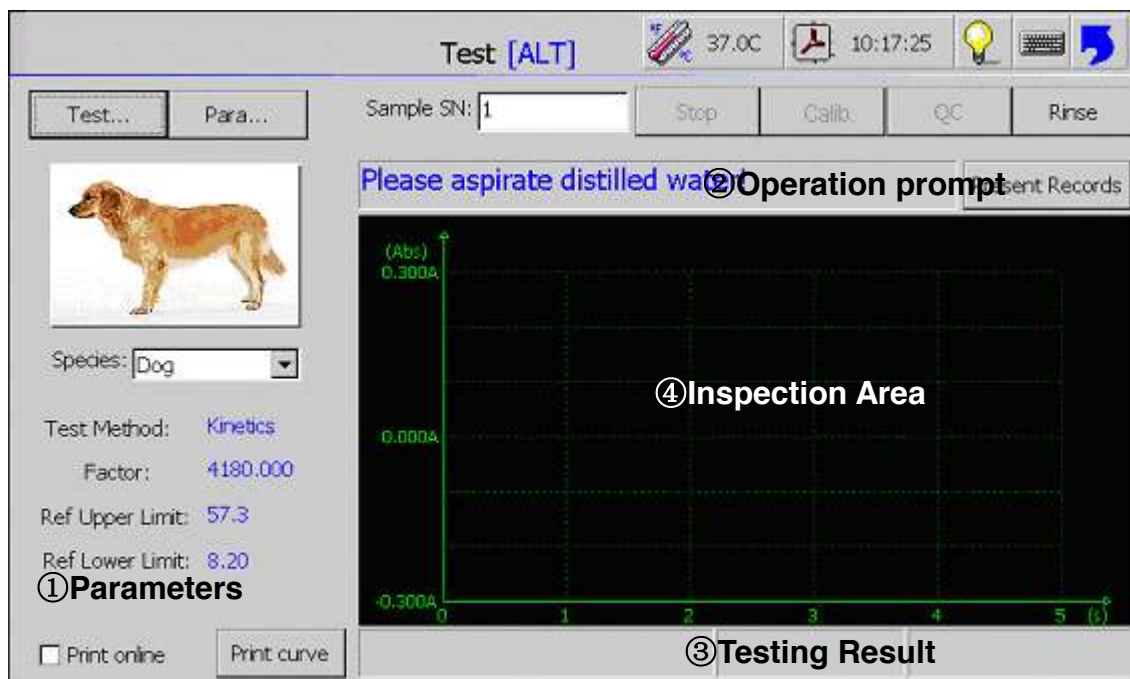

8-1 Testing Interface

- ① Parameters: To display the parameters of the test selected.
- ② Operation Notice: To display the operation being carried on and notice the next step.
- ③ Testing Result: To display the result calculated by the device.
- ④ Inspection Area: To display real time ABS graph during the test.

【Test...】: Click to enter the 【Select Test】 interface.

【Para...】: Click to review and edit the parameters selected.

【Species】: select animal.

【Stop】: Click to stop the test being carried on.

【Calib.】: Click to calibration.

【QC】: Click to start QC Test.

【Rinse】: The function is the same as “Rinse” on the Keyboard.


【Present Record】: Click to enter the 【Sample Records】interface to review the test record today.

【Print curve】: Click to print the current reaction curve.

【Print Online】: Click to print the testing result online.

2. Sample Test

2.1 Test with non-template

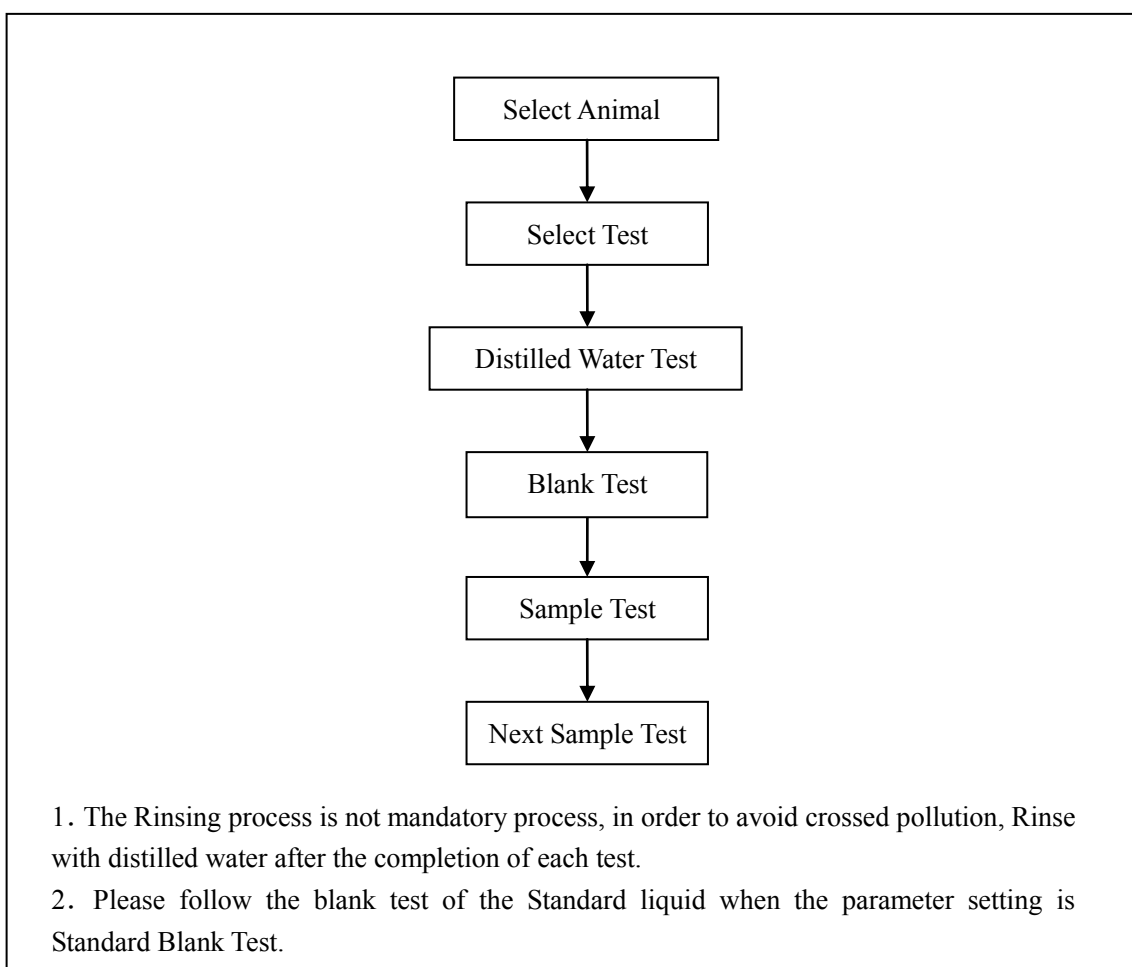

8-2 Sample Testing Procedures

E.g. Test ALT with Sample Blank:

- 1) Turn on the instrument to the main menu. Click 【Test】 to 【Select Animal】 interface, select a animal and click 【OK】 ,enter 【Select Test】 interface. Select ALT and click Enter (or double click ALT) to the 【Test List】 interface. Click 【Edit Test】 to enter the

【Test Parameter Setting】 interface. Please set the parameters if the parameters existed could not comply with the operation manual of the Reagent.

- 2) The system will adjust the Lamp and Temperature automatically. The real time temperature will be displayed on the top right corner of the interface. The operation notice “Please aspirate distilled water!” will display after the adjustment.
- 3) Press 【ASP】 under the pipette or on the keyboard to aspirate distilled water. The operation notice “Please aspirate sample blank!” will display after distilled water test.
- 4) Press 【ASP】 to aspirate sample blank (See the operation manual of the reagent.) and wait for the sample blank test.

The system will alarm “Factor or calibration information may be invalid!” if there is no valid factor after the blank test or the parameters has been changed after done calibration last time (See figure 8-3) This notice means the user must input the correct factor or calibrate before sample test. The system will notice “Please aspirate Sample!” after valid factor occurred.



Figure 8-3

- 5) Please confirm the Sample SN before test then press 【ASP】 to aspirate the sample (See the operation manual of reagent). Click 【Rinse】or press【Rinse】 on the keyboard to aspirate distilled water to rinse after every test to avoid crossed pollution and keep the instrument clean. Click 【Rinse】 or Press 【Rinse】 again to stop rinsing.

The system will save the result data automatically. And the instrument will print the result automatically if ‘Print Online’ has been settled. The system will be ready for next sample test. The Sample SN will go on automatically if the number is figures. Or it will be cleaned. Start next test when the system notice “Please aspirate sample blank!”.

Hint:

The system will check the Sample number before aspirating. Interface as figure8-4 will display if the Sample SN has not been inputted.

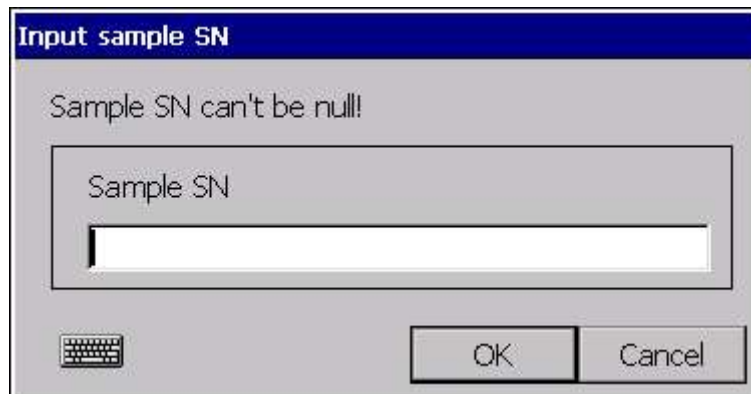


Figure 8-4

【Sample SN】: Please input the Sample SN of current test.

【OK】: Click to save the sample SN.

【Cancel】: Click to cancel and return to the testing interface.

Hint:

The system will check the sample records before aspirating. Interface as figure8-5 will display if the record has existed in the records.

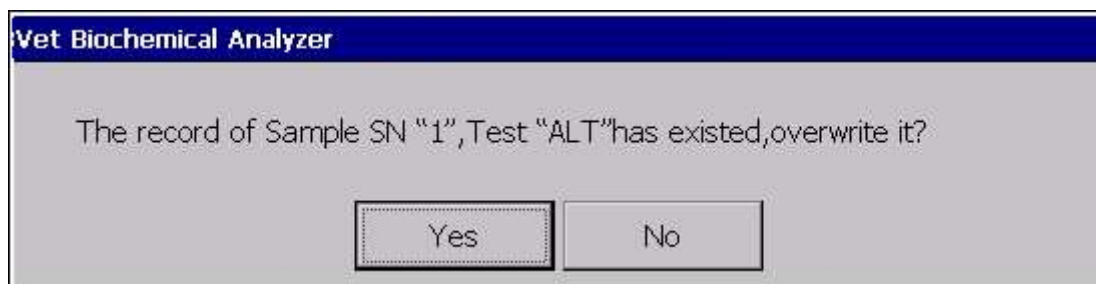


Figure 8-5

【Yes】: Click to make the new test instead of the record.

【No】: Click to display figure 8-4 to input another sample SN.

Hint:

Before user click **【Aspirate】** to aspirate liquid, interface as figure 8-6 will display, then user can input sample info.



The dialog box titled "Input Sample Info" contains the following fields and controls:

- Animal Name:** A text input field.
- Master Name:** A text input field.
- Animal Sex:** A dropdown menu with "Male" selected.
- Phone:** A text input field.
- Animal Age:** A text input field followed by a dropdown menu with "Y" and "M" options, and a numeric input field with "0" selected.
- Buttons:** "OK" and "Cancel" buttons at the bottom right.

Figure 8-6

【Animal Name】: Input the animal name here.

【Animal Sex】: Input the animal sex here.

【Animal Age】: Input the animal age here.

【Master Name】: Input the master name here.


【Phone】: Input the master's phone number here.

【OK】: Click it to save the info.

【Cancel】: Click it cancel and return to the testing interface. Also user can edit the sample information in the sample records interface.

Hint:

If anything is abnormal in distilled water test, system will suggest as shown in figure 8-7. Then system return the distilled water test interface after clicked 【OK】, and user can not continue testing process. At this time, user can retest water blank after rinsed the pipe to eliminate abnormal gaps. If user can not resolve the problem through retried several times, you can contact the designated maintenance points.

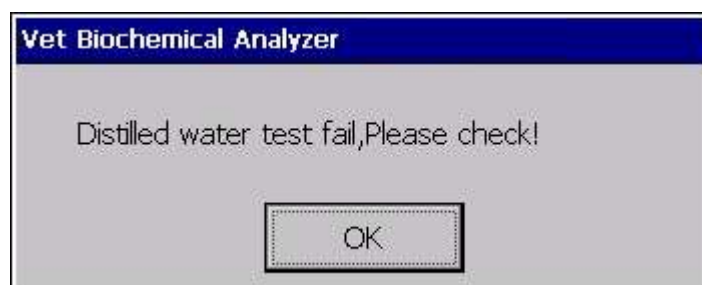

The dialog box titled "Vet Biochemical Analyzer" displays the following message and control:

- Message:** "Distilled water test fail, Please check!"
- Button:** "OK" button at the bottom center.

Figure 8-7

Hint:

If Abs value is transnormal in distilled water test, system will show the dialog box in figure

8-8.

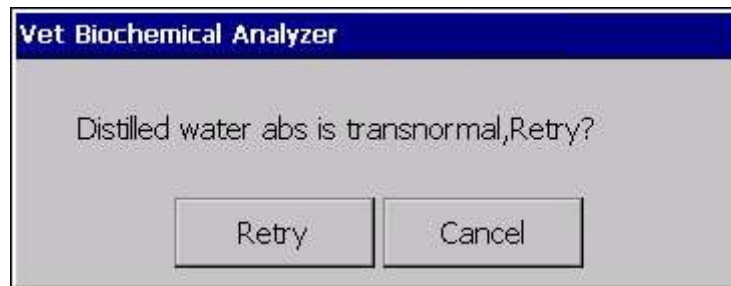


Figure 8-8

【Retry】: Click it to retest distilled water.

【Cancel】: Click it to continue testing behind.

Hint:

When a reagent blank is selected in the project parameters, system will store the reagent blank value after tested if the value is greater than 0. Then while device tests the same project next time, system will show the dialog box in figure 8-9.

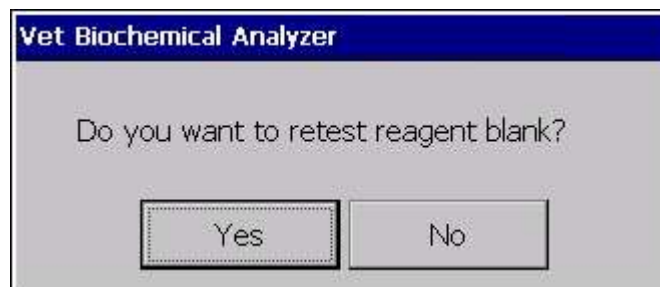


Figure 8-9

【Yes】: Click it to retest the reagent blank.

【No】: Click it to cancel testing the reagent blank, system will calculate the result with the stored reagent blank value.

2.2 Test with template

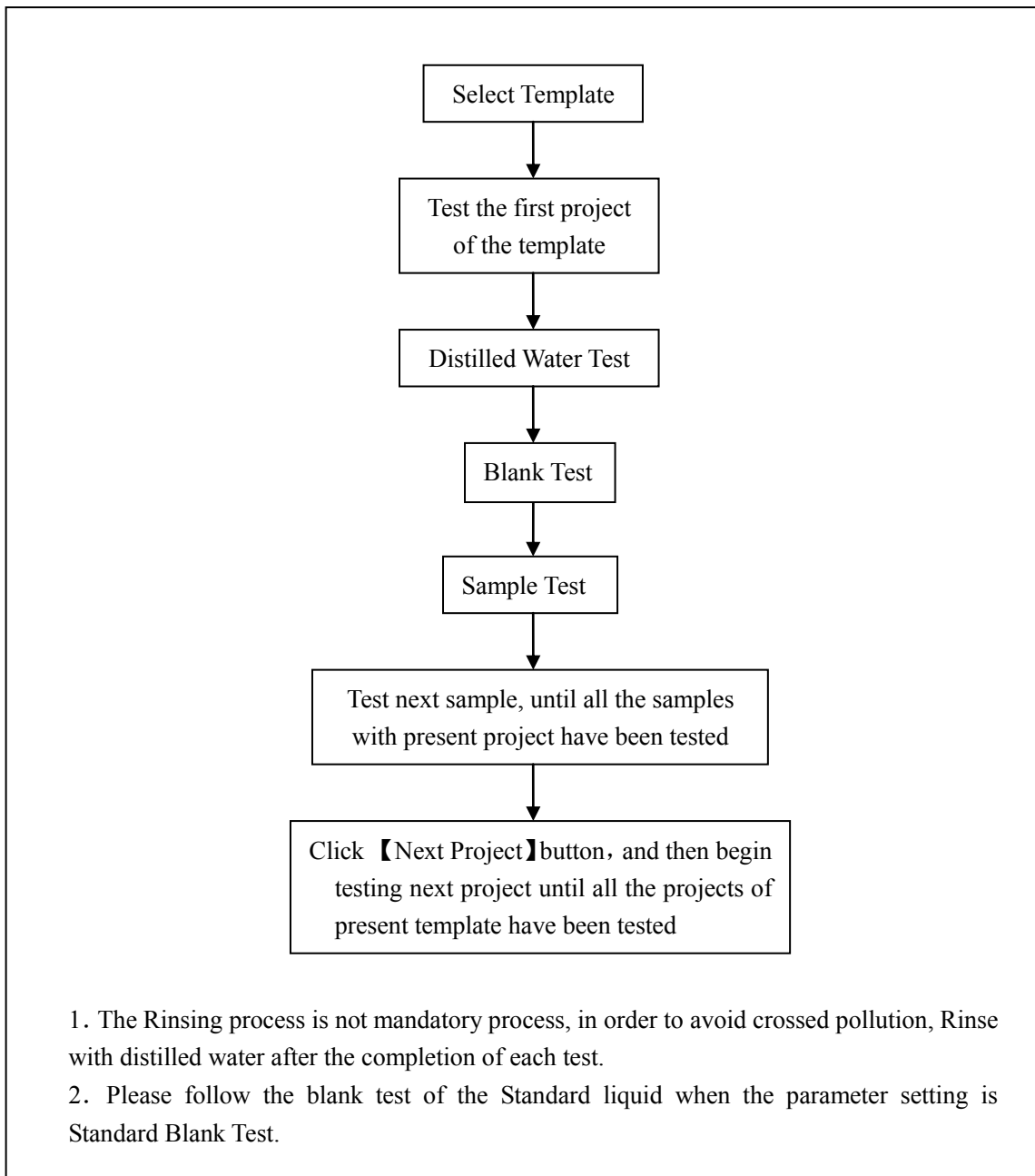


Figure 8-10 Sample Test Procedures

The single test procedure with template is as the same as the test with non-template mode.

Chapter 9 Calibration Test

1. Calibration Test Procedures

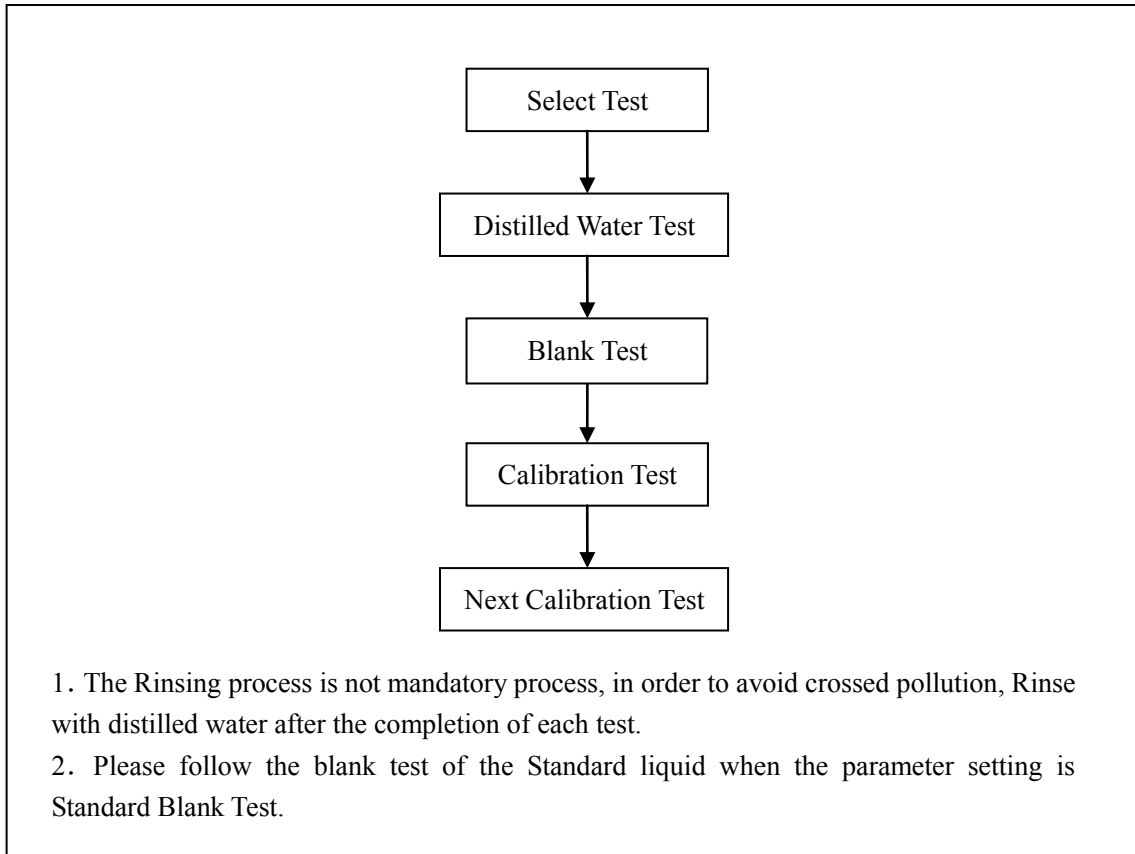


Figure 9-1

This instrument provides two calibration tests: linear and non-linear.

E.g. ALT calibration Test Procedures:

- 1) Turn on the instrument to enter the main menu. Click **【Test】** to the **【Select Animal】** interface ,select a animal and click **【OK】**,then enter the **【Select Test】**. Select ALT and click Enter (or double click ALT) to **【Test】** interface. Click **【Edit】** to the **【Test Parameter Setting】** interface. Please set the parameters if the parameters existed could not comply with the operation manual of the Reagent.
- 2) The system will adjust the Lamp and Temperature automatically. The real time temperature will be displayed on the top right corner of the interface. The operation notice “Please aspirate distilled water!” will display after the adjustment.

- 3) Press **【ASP】** under the pipette or on the keyboard to aspirate distilled water. The operation notice “Please aspirate sample blank!” will display after distilled water test.
- 4) Press **【ASP】** to aspirate sample blank (See the operation manual of the reagent.) and wait for the Sample blank test.

The system will alarm “Factor or calibration information is invalid!” if there is no valid factor after the blank test or the parameters has been changed after done calibration last time (See figure 8-2) .This notice means the user must input the correct factor or calibrate before sample test. The system will notice “Please aspirate Sample!” after valid factor occurred.



Figure 9-2

- 5) Click **【Calib.】** and the system notice “Please aspirate Standard Blank!”. Press **【ASP】** to aspirate the Standard Blank liquid according to the first concentration point. (See the operation manual of reagent). Click **【Rinse】** or press **【Rinse】** on the keyboard to aspirate distilled water to rinse after every test to avoid crossed pollution and keep the instrument clean. Click **【Rinse】** or Press **【Rinse】** again to stop rinsing.

The system will notice “Please aspirate ***% standard liquid! Test 1” after test. Press **【ASP】** to aspirate certain concentration standard liquid. The instrument will repeat the test on this standard point. (Repeat time≥1) The system will display following interface (Figure 9-3) after all the standard test is finished.

Calibration Result

Calibration details

Index	Conc.	ABS
1	1.000	0.002

New factor:500.000

Save Cancel

9-3

【Save】: Click to make the new factor to instead of the existed one.

【Cancel】: Click to cancel this test.

Chapter 10 Quality Control Test

1. QC Test Procedures

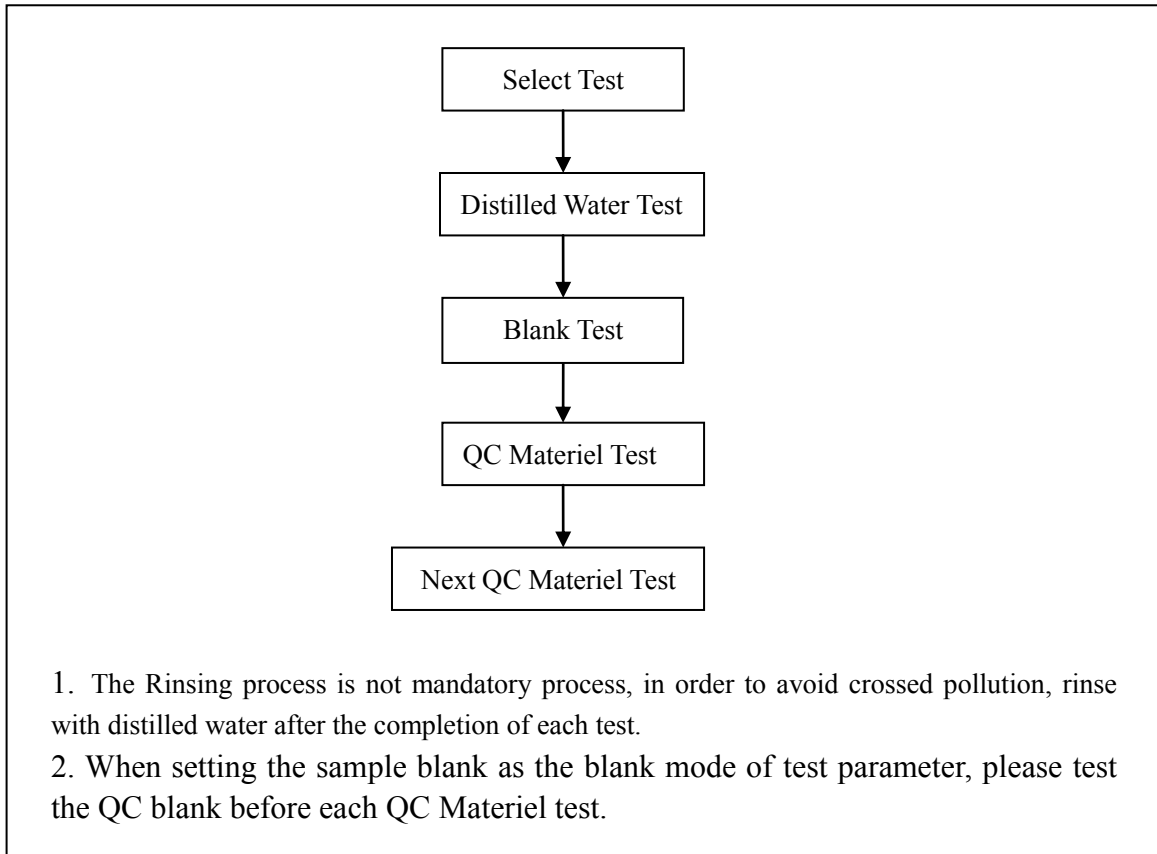


Figure 10-1

E.g. test “ALT” with sample blank as follows:

- Step 1: Turn on the instrument, enter the main interface , Click **【Test】** to the **【Select Animal】** interface ,select a animal and click **【OK】** ,then enter the **【Select Test】** . In **【Select Test】** interface, click “ALT” and **【OK】** (or double click “ALT”) to enter **【Test】** interface. In order to confirm test parameter of “ALT” is correct, click **【Edit】** to enter **【Test Parameter Setting】** interface, then click **【Calibration Parameter】** to examine or change.
- Step 2: Selected “ALT”, system will adjust the Lamp and Temperature automatically. While the light source is steady and the temperature is correct, **【Operation Notice】** shows “Please aspirate distilled water ”

Step 3.Press **【ASP】** key under the pipette or on the keyboard to aspirate distilled water, wait for testing, **【Operation Notice】** shows “Please aspirate the sample blank”

Step 4.Press **【ASP】** key to aspirate the prepared sample blank(See the reagent manual for preparing method),wait for testing.

The system will alarm “Factor or calibration information is invalid!” if there is no valid factor after the blank test or the parameters has been changed after done calibration last time (See figure 9-2) .This notice means the user must input the correct factor or calibrate before sample test. The system will notice “Please aspirate Sample!” after valid factor occurred.

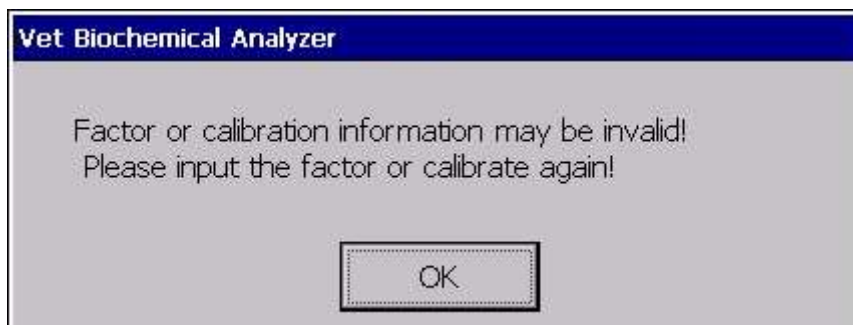


Figure 10-2

Step 5,Click**【QC】**button, show the following dialog to remind operator to select QC Materiel.

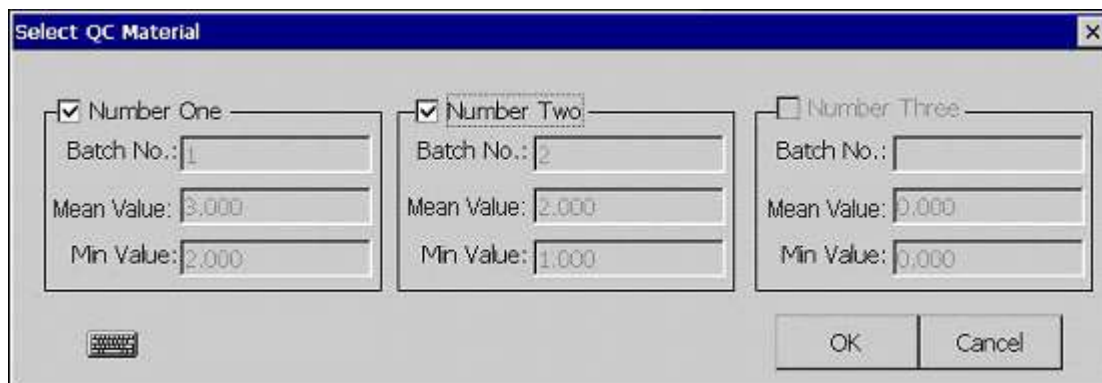


Figure 10-3

【Number One】 :Test the QC Materiel.1 if selected

【Number Two】 :Test the QC Materiel.2 if selected

【Number Three】 :Test the QC Materiel.3 if selected

【OK】 :Click to finish selecting, then return to the **【Test】** interface and begin testing.

【Cancel】 :Click to quit selecting, then return to the **【Test】** interface and quit testing.

Step 6: Selected QC Materiel(>0), **【Operation Notice】**shows “Please aspirate QC *** Blank”

in the **【Test】** interface, press **【ASP】** to aspirate the first QC Material Blank(See the reagent manual for preparing method),wait for testing. To avoid crossed pollution, please click **【Rinse】** or press **【Rinse】** key to aspirate distilled water for rinsing, click or press it again to finish rinsing.

While QC Material Blank is tested, system shows “Please aspirate QC Material***”, then press **【ASP】** to aspirate QC Material for testing. Test next QC Material if last test is finished. If all QC Material is tested, system will show the following dialog, click**【OK】**to finish the test.



Figure 10-4

【OK】 :Click to finish the test, and return to the **【Test】** interface.

Chapter 11 Sample Records

1. Summary

The analyzer can query the records by fuzzy or combination terms. It facilitates management of patient records in hospital by providing powerful capability of test records inquiry function and the report of the patients print functions.

2. Browse sample records

Click【Records】in main interface, or Click【Today Records】in 【Test】interface to show 【Sample Records】interface. System could inquire sample test records today automatically when the window is opened.

Seeing the following figure test records are displayed or printed. The system displays case-history in the order of dates. The later records are displayed on the top. The system can display the records less than 200.

No.	Sample SN	Master	Phone	Animal Name	Species	Tested Date	Test	Conc.	
1	1			Dog	Dog	2008-04-08	ALT	242.2	H

①Sample records list

②Test records list

Master Name: Animal Sex: Male
Phone: Animal Age: Y M
Animal Name: Tested Date: 2008-4-8

Figure 11-1

- ① Sample Records List: display what you query, select one or more records to print, browse or delete
- ② Test Records List: display the selected test records in **【Sample Records List】**, select one or more to delete.

【Query】: Click to display the dialog **【Input Test Records Conditions】** (Figure 11-2), and input the query terms, then click **【OK】** to query the records.

【Print】: Click to print the selected records.

【Preview】: Click to preview the effect of printing.

【Add Sample】: Click to display **【Add Sample Records】** interface on the following figure, then add a sample record.

【Delete Sample】: Select a sample in the **【Sample Records List】** interface to delete it.
Attention! Delete all relevant records if delete the sample.

【Master Name】: User can input or amend the master name here.

【Phone】: User can input or amend the master's phone number here.

【Animal Name】: User can input or amend the animal name here.

【Animal Sex】: User can input or amend the animal sex here.

【Animal Age】: User can input or amend the animal age here.

【Test Date】: Display the test date here and the date can't be amended.

【Edit】: Click it, then user can edit the sample info.

【Save】: Select a sample in the **【Sample Records List】** interface, and input the sample name, sex etc, then click **【Save】** to save.

【Add Test】: Click to display the **【Add Test Records】** interface on the following figure, then add a selected test record in the **【Sample records list】** interface.

【Edit Test】: Click to display the **【Edit Test Records】** interface on the following figure, then edit a selected test record in the **【Sample records list】** interface.

【Delete Test】: Click to delete a selected test record in the **【Sample records list】** interface.

3. Query sample records

Click **【Query】** in **【Sample Records】** to show the following window, and input the query terms ,then click **【OK】** to query ,the result is displayed in the **【Sample records】** windows.

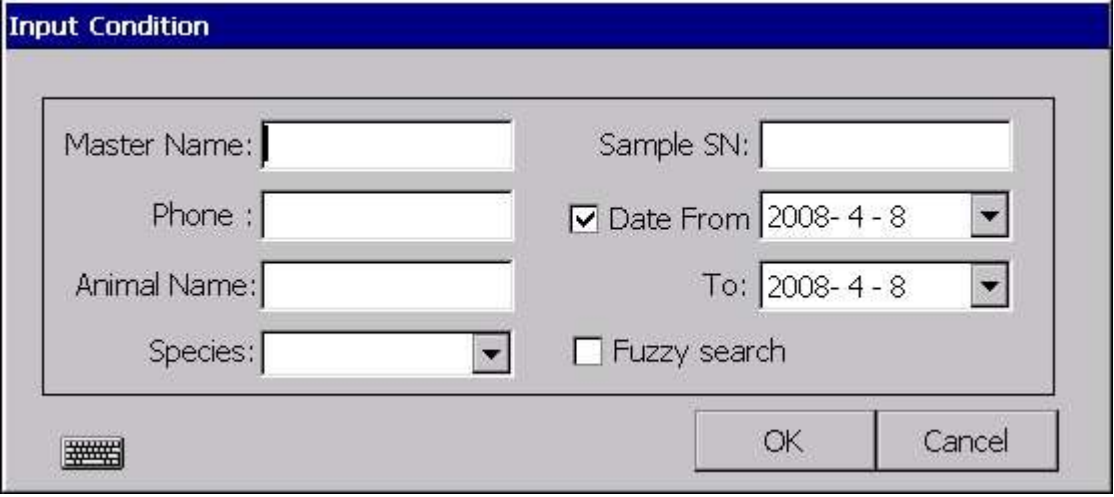
The image shows a software window titled "Input Condition". It contains several input fields: "Master Name:" with a text box, "Sample SN:" with a text box, "Phone :" with a text box, "Animal Name:" with a text box, and "Species:" with a dropdown menu. There are also date selection fields: "Date From" and "To", both with dropdown menus showing "2008- 4 - 8". A checkbox labeled "Fuzzy search" is present. At the bottom right are "OK" and "Cancel" buttons. A keyboard icon is at the bottom left.

Figure 11-2

【Sample SN】 :Input the sample SN that you query, the number can be part of the whole sample SN, and letters within 16.

【Master Name】 :Input the name that you query, the words can be part of the whole name, and letters within 16.

【Phone】: Input the phone number than you query, the words can be part of the whole number.

【Animal Name】 : Input the name that you query, the words can be part of the whole name, and letters within 16.

【Species】 : Set animal species.

【Date From】 :Input the prior date.

【Date to】 :Input the later date.

【Date Option】 ;Select to query according to the setting date

【Fuzzy Search】 :select to fuzzy search .For example, if input"1" as the Sample SN, the Sample SN according with the condition may be"1", "11", "21", or"31".

【OK】 :Click to query, and display the result in **【Sample Record】** interface.

【Cancel】 : Click to quit querying, and return to the **【Sample record】** interface.

The relation of all query terms is "and". For instance, "1"as the sample ID, "ALT" as the project name, "John" as the name, date from" 1/1/2006"to" 4/1/2007",if fuzzy query, the result is the sample records that the test date from" 1/1/2006"to" 4/1/2007",sample ID "1",and name including "John", and tested "ALT".

For a period of time, the record of data-base is increased, and then the query time will be longer. In addition, if the query term is too fuzzy, the result will be too much to display.

Please input the more complete term when you query.

4. Print sample records

Select the sample records in the **【Sample Records】** window ,and click **【Print】** to print the selected records.

If selected the external printer, there will be two print mode: continuous or discontinuous. (See the section “System setting” for selecting and setting the printer).


Continuous mode is to print more records on a sheet “A4” paper. (External printer, “A4” paper). Simultaneity, all project records of a sample will be printed on the same paper mostly. On discontinuous mode, different sample records can’t be printed on the same paper.

5. Add, edit, or delete sample records

In **【Sample records】** interface, browsing by sample, select the records in **【Sample Records List】** interface for operating. Sample records will be display in the edit textbox below **【Sample Records List】** , then input the information ,and click **【Save】** to finish editing the sample records.

Select the records in the **【Sample Records List】** interface for operating. Click **【Delete】** , **【Print】** ,or **【Preview】** to delete ,print or preview.

Click **【Add new sample】** , show as the following figure. Input the information , and click **【OK】** to add a new sample.

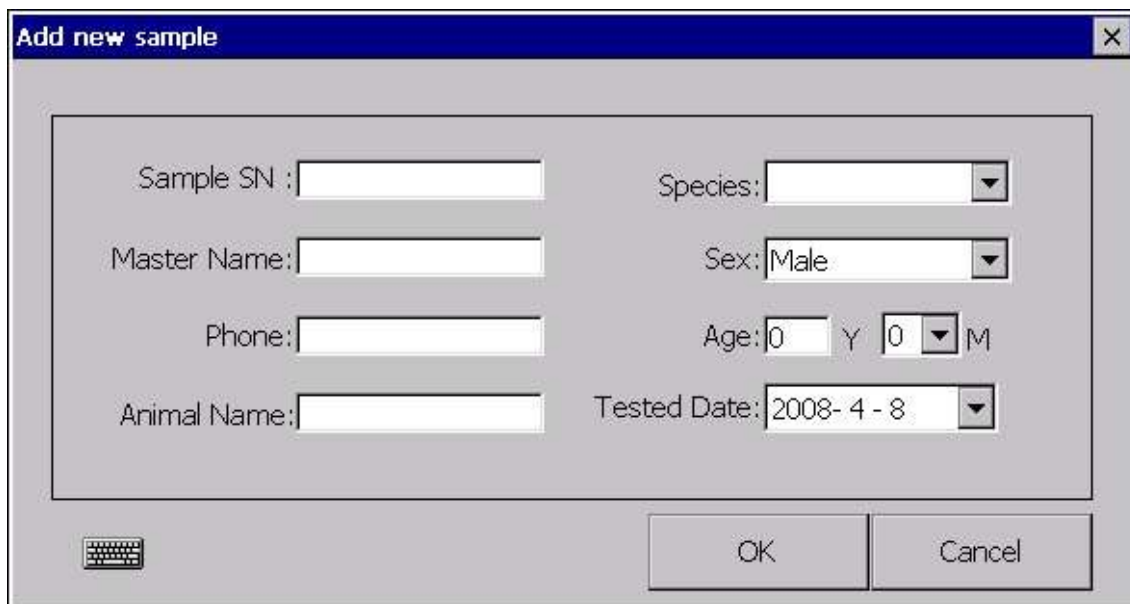

Add new sample

Sample SN : Species:

Master Name: Sex:

Phone: Age: Y M

Animal Name: Tested Date:




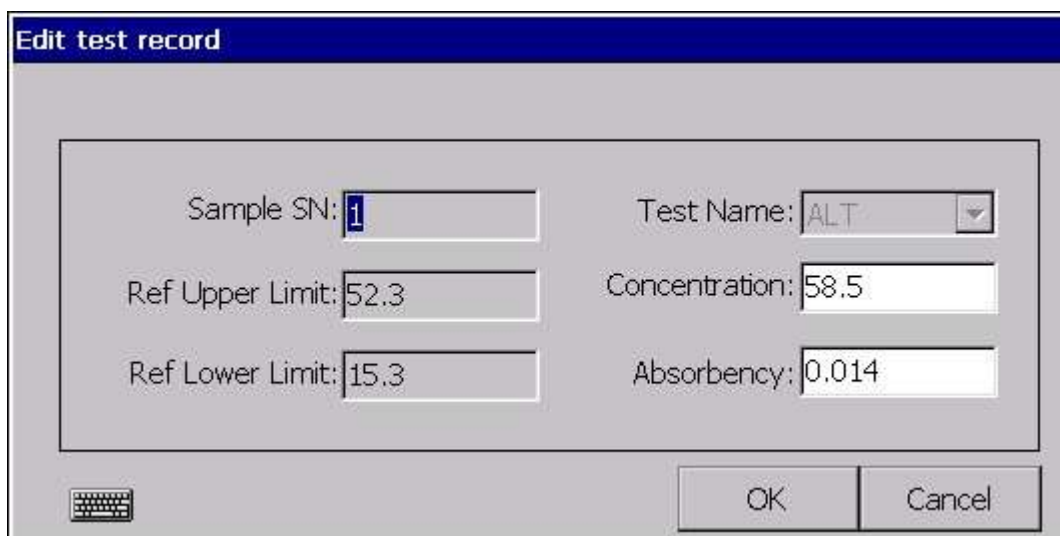
Figure 11-3

6. Add, edit, or delete test records

Browsing by the sample, in **【Sample Records】** interface, Select the records on the **【Sample Records List】** for operating. The **【Test Records List】** will display all the records of this sample. Select the test records in the **【Test Records List】** interface.

Click **【Delete Test】** to delete the test records.

Click **【Edit Test】** in **【Edit test result 】** interface (See the following figure). Input the information ,then Click **【OK】** to finish editing. Or click **【Cancel】** to quit editing.



Edit test record

Sample SN: Test Name:

Ref Upper Limit: Concentration:

Ref Lower Limit: Absorbency:


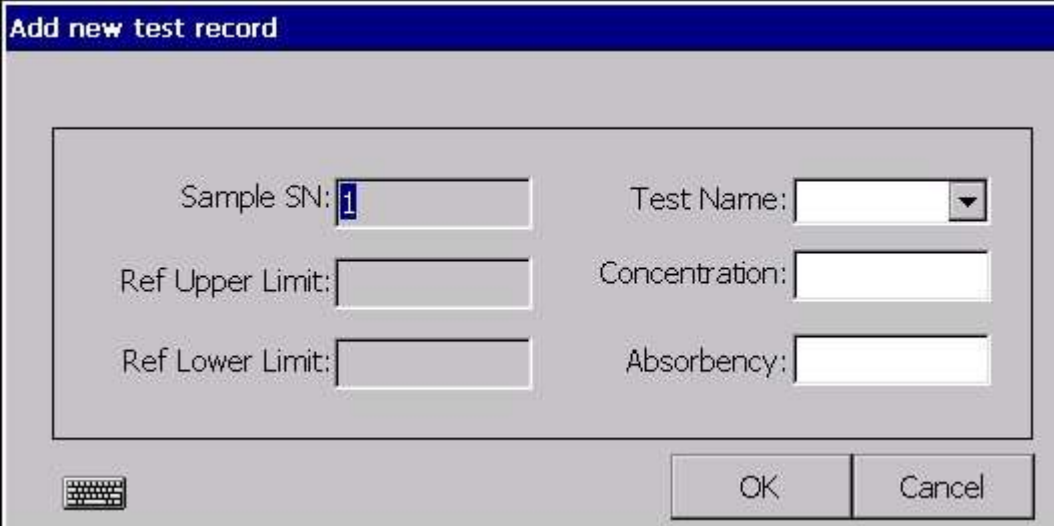


Figure 10-5

Click **【Add Test】** to show the following dialog, then input the information, and click **【OK】** to add a new test and return. Or click **【Cancel】** to quit and return.



The dialog box is titled "Add new test record" in a dark blue header bar. It contains a light gray rectangular area with the following fields:

- Sample SN:
- Test Name: (with a dropdown arrow)
- Ref Upper Limit:
- Concentration:
- Ref Lower Limit:
- Absorbency:

At the bottom left of the dialog is a keyboard icon. At the bottom right are two buttons: "OK" and "Cancel".

Figure10-6

Chapter 12 Calibration Records

1. Summary

The analyzer can be calibrated with single-point or multi-points mode.

2. Query and print the calibration records

In the main window, click **【System】**, then click **【Calibration records】** to show the following figure.

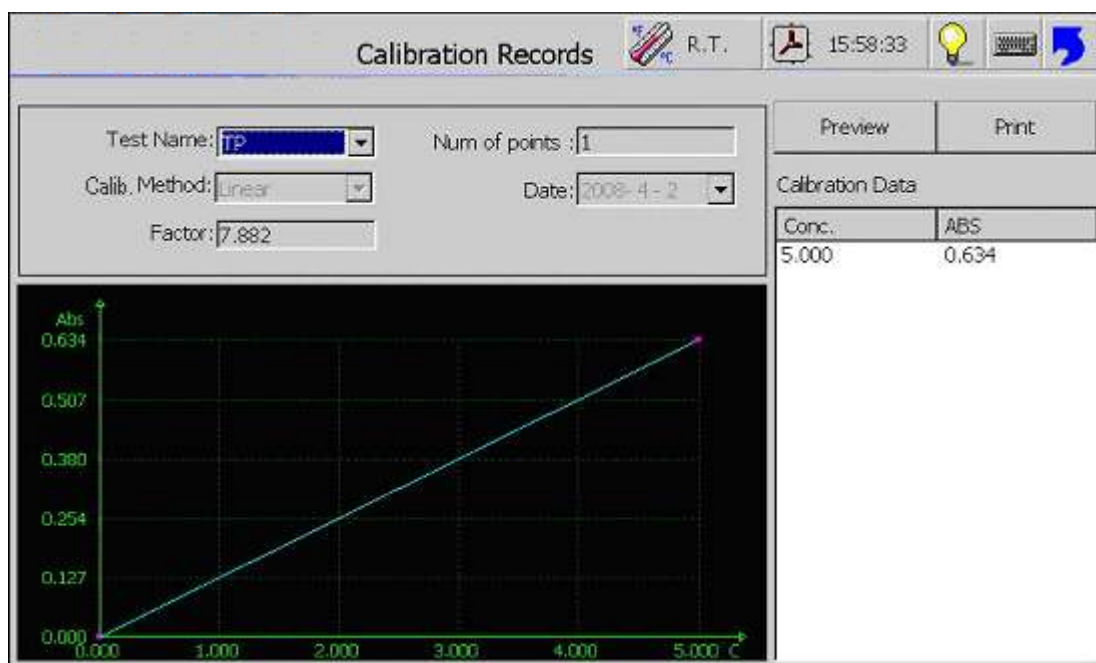


Figure 12-1

【Test Name】: Select test, then the system will look for the calibration records of this test, and display the information with two modes --data and figure. If the calibration records of the test has existed, the **【Print】**, **【Print Preview】** button can be clicked.

【Num of Points】: Number of the standard -concentration points.

【Calib. Method】: Calibration method, linear or non-linear.

【Date】: Calibration time.

【Factor】: Calibration factor, if non-linear, there is no factor.

【Preview】 :Click to preview the effect of printing.

【Print】 :Click to print calibration records.

【Calibration Data】 :Show the calibration data of the selected test.

Chapter 13 Quality Control

1. Summary

- 1) BS600V biochemical analyzer has a powerful quality control function; it is so convenient to operate which makes the QC in lab very easy.
- 2) Three QC materials can meet all test projects.
- 3) Calculate AVG, SD and CV of the QC in a certain time selected automatically.

2. Query and print QC records

Click【SYSTEM】in main window, select 【QC records】in pop-up window, then you can see the following interface (figure 13-1).

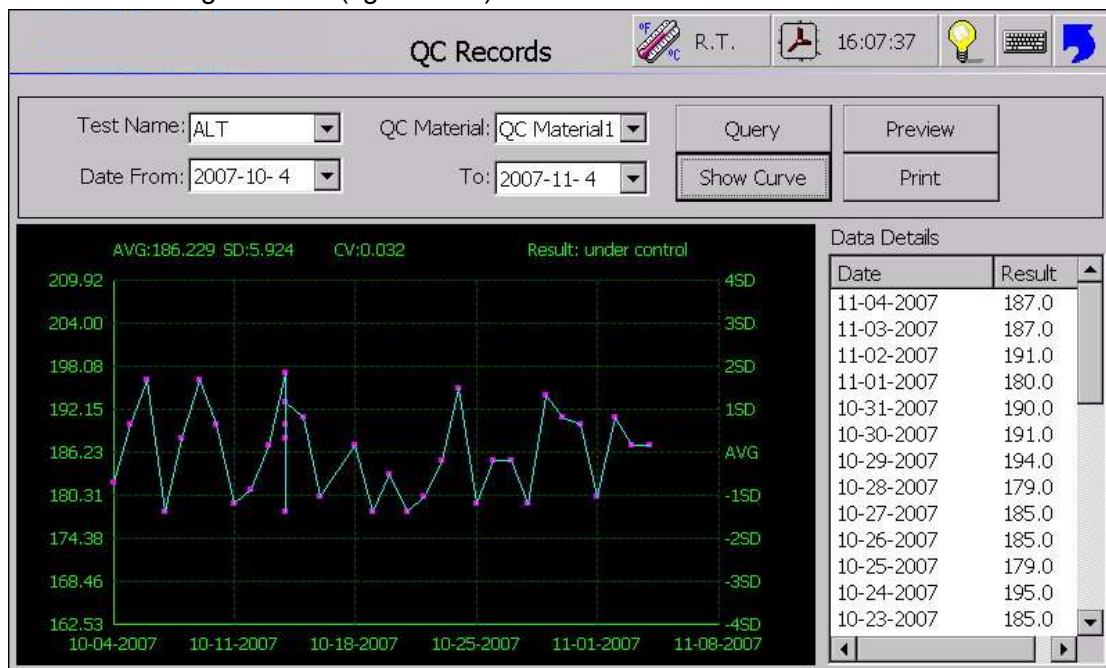


Figure 13-1

- 【Test Name】: Select the test need to query.
- 【QC material】: Select the QC material need to query.
- 【Date From】: Specify the start date need to query.
- 【To】: Specify the end date need to query.
- 【Query】: Click to get the results.
- 【Show Curve】: Click this item to show QC curve of the queried result, along with the AVG, SD and CV calculated automatically.
- 【Print Preview】: Click this item to preview the effect of the QC records.
- 【Print】: Click to print the QC records.

Chapter 14 Turn Off

1. Click **【EXIT】** in the main window, you can see the following dialog.

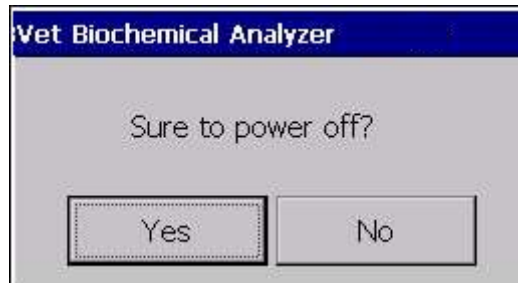


Figure 14-1

【YES】: Click to enter power off process.

【NO】: Click to return the main window.

2. When click **【YES】**, you can see following window:

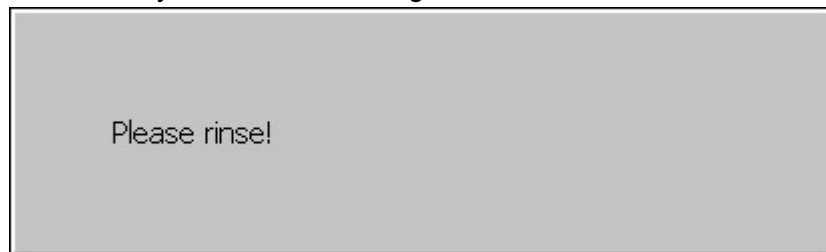


Figure 14-2

Click **【ASP】** OR **【Rinse】** to aspirate the distilled water, then the instrument start to rinse.

When rinse is finished, system prompt that you can turn off safely as follow, then you can turn off the power switch directly.

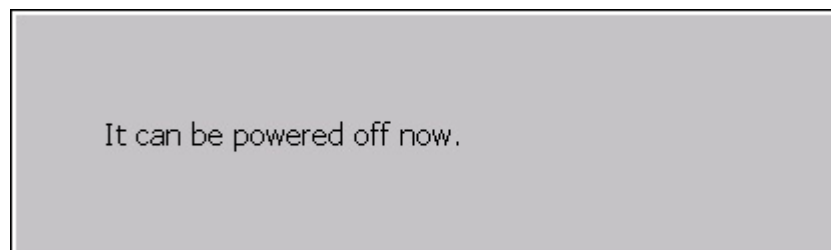


Figure 14-3

Hint:

To prolong the life of the instrument, make sure turn off the device according to steps above-mentioned.

Chapter 15 Software Maintenance

BS600V provide a powerful software system maintenance functions, in some cases, the need to safeguard systems, operating steps are as follows:

1. Start the process of system maintenance

If the state apparatus in the boot, make sure your system closed and the closure of power equipment. Press **【Enter】** on the keyboard instrument panel ,simultaneously, turn on power equipment, open up the key until the show interface shown in Figure 15-1 .



Figure 15-1

【Update】: Click to start the implementation of the updated, the system automatically updates search documents and complete the update process.

【Restore program files to the last edition】: Select the **【Restore】** option and click **【OK】** to restore a recent escalation of the version

【Restore factory system settings】: Select the **【Restore factory system settings】** option and click **【OK】** to restore factory system settings..

【OK】: Click to start the implementation of the selected option.

【Cancel】: Click to suspend the implementation of the selected option.

2. Update

Insert the SD or flash card storing the updated file, select **【Update】** and click **【OK】**, system shows the dialog box in figure 15-2.



Figure 15-2

【Yes】: Click to start the implementation of the updated, t system automatically search files and complete the update process.

【No】: Click to suspend the implementation of the updated and returned to the **【System Maintenance】** interface

3. Restore

You can restore the former version after updated. Select **【Restore】** and click **【OK】** in figure 15-1. Show the dialog box in figure15-3:



Figure 15-3

【Yes】: Click to start the implementation of recovery, the system will automatically recovery to last updated version before.

【No】: Click to suspend the implementation of recovery and return to the **【System Maintenance】** interface.

4. Restore factory setting

Select **【Restore factory setting】** and click **【OK】**, show the dialog box in figure 15-4:

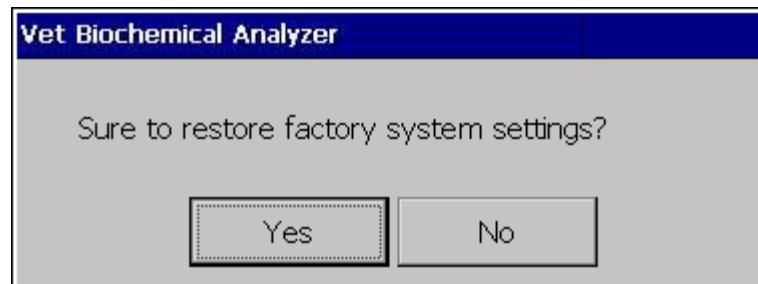


Figure 15-4

【Yes】: Click to start the implementation of recovery, the system will automatically restore factory setting.

【No】: Click to suspend the implementation of recovery and return to the **【System Maintenance】** interface.

Chapter 16 Routine Maintenance

Vet biochemical analyzer is a clinic precision analytical instrument. To make sure the instrument in a good state, routing maintenance is strongly required. Although the BS600V routing maintenance is very easy, you must be patient and careful.

1. Clean the instrument

Clean the exterior of the instrument with neutral cleanser or a piece of wet cloth; keep the environment of the instrument clean.

Notice:

Do not make the instrument contact with any organic impregnant, lipid and caustic matters.

2. Clean the flow cell

a) Clean the exterior of the flow cell

If the exterior of the flow cell is dirty, you can clean it with a piece of soft cloth moistened by absolute alcohol.

b) Clean the inner of the flow cell

Place the container with distilled water under the pipette, press **【Rinse】** key to start rinsing, again to stop rinsing. Generally 30 seconds is needed. You can also use quartz glass cleanser, press **【Rinse】** key first, press **【Rinse】** key again to stop, allow cleanser to stay in flow cell for several minutes, at last, rinse it with distilled water for 1 minute. If the flow cell is still not clean, please rinse it once again.

c) Make sure clean the flow cell before turn off the instrument.

Notice: Do not leave the remain liquid for a long time in flow cell !

3. Instrument maintenances

1) Replace fuse

- 1) Insert a tweezer into the groove and prize the fuse box out;
- 2) Take the fuses out and replace it (As figure 16-1).

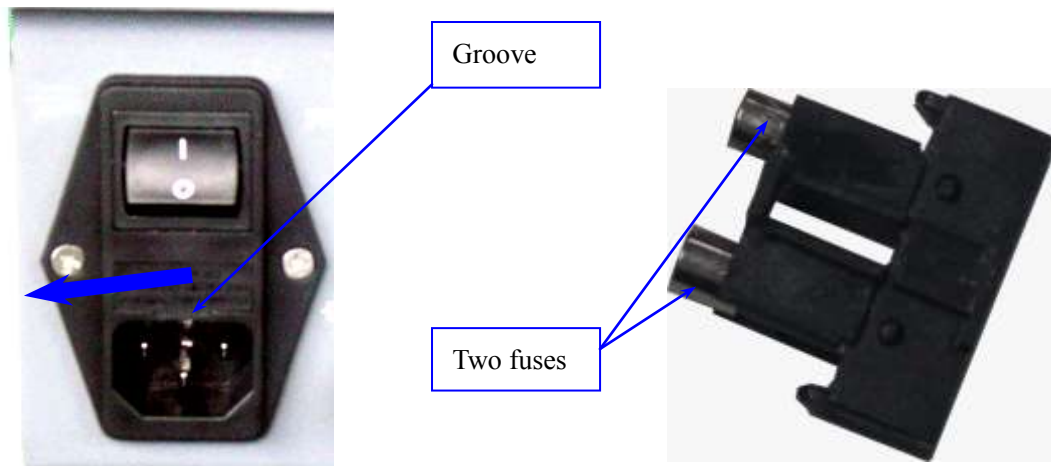


Figure 16-1

Fuse type 250V / 2A , $\phi 5 \times 20$



Notice:

The fuse of above type allowed only, make sure power wire is disconnected when replace the fuse. Operation with electricity is forbidden.

2) Change pipette

If the pipette or flow cell is jammed by any other substances, you can clean it with an injector. If the pipette is damaged or jammed badly, you'd better replace it with a new one.

The way is as follows

- Open the removable cover
- Pull up the flow cell
- Take the pipette off from the flow cell at the entrance, on one side of the new pipette, pull up the fixed pipe (the widest pipe), and make sure the fixed pipe and the pipette have a superposition for 1cm, then fix the fixed pipe onto the entrance of the flow cell.
- Pull the pipette through the fixed metal pipe.

3) Replace tube of the pump

- Press down the handle;

- Take off the screw as figure15-2 and take off the cover of the pump;
- Take out the tube as figure 15-3;
- Take off the pipe of the two sides as figure 15-4and repalce it.

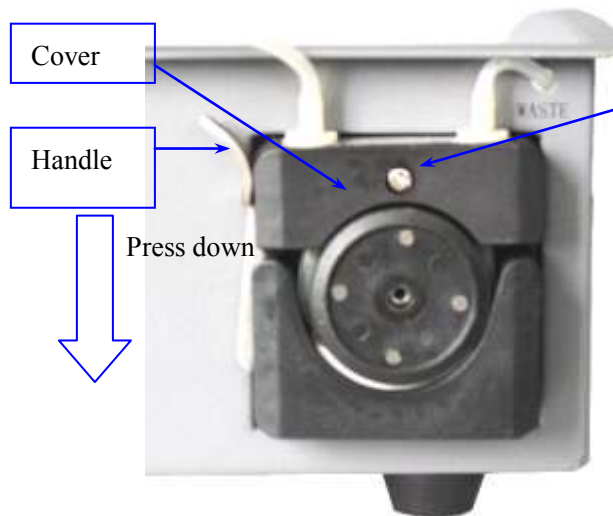


Figure 16-2



Figure 16-3



Figure 16-4

4) Replace lamp

- Take off the screw
- Disconnect the connection of the power supply
- the lamp part can be taken out.
- Replace it and tighten the screw and connect the power supply.

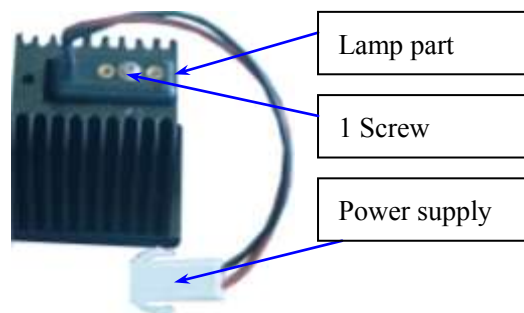


Figure 16-5

4. Trouble-shooting

Breakdown Description	Solution
No display on the screen after the unit is switched on	<ol style="list-style-type: none"> 1. Check power supply; 2. Check the power cord and plug; 3. Check if the fuse is broken.
The lamp of the instrument cannot be lightened	<ol style="list-style-type: none"> 1. Check the power supply before replace the lamp; 2. The lamp is damaged; replace it with a new one.
Printer does not work	<ol style="list-style-type: none"> 1. Check the power supply
No liquid in the flow cell	<ol style="list-style-type: none"> 1. Check if the pump is running properly; 2. Check the connect of the pipette and the flow cell 3. Pipette is either too long or too short; 4. Pipette may be jammed, clean it; 5. Flow cell is too dirty.
Abs of the water blank is too high	<ol style="list-style-type: none"> 1. Clean the flow cell; 2. Change the distilled water; 3. Check the lamp.
Repeatability is not good	<ol style="list-style-type: none"> 1. There are air bubbles in the flow cell, clean the flow cell; 2. Check the aspirating of the pump; 3. The quantity of the reagent is too little, increase the quantity of the reagent; 4. Change the lamp; 5. Reagent is polluted.
The quantity drew by the pump varies from time to time.	<ol style="list-style-type: none"> 1. Check if the pipette is jammed; 2. Change the pipe of the pump.
Quality control value not in target scope	<ol style="list-style-type: none"> 1. Check the period of validity; 2. Check if the parameters is correct; 3. Make sure the QC material is not polluted.

Chapter 17 Packing, Storage and Transportation

- 1、 Put the unit in protective poly foam and then put it in the wooden box.
- 2、 Take out the unit and power them on for four hours if the units have been put in stock for more than 6 months, and put them back according to the instructions marked outside the carton.
- 3、 Do not stack the units or let the units contact ground directly. Remember to leave some space between the products and walls and roofs.
- 4、 Shock absorption has been deliberately considered when the parking is designed and they are suitable for air, railway, road, and sea freight. Do not shock or place the products upside down and be free of rains or snows.
- 5、 Transport and Storage Environment: Temperature: 0℃~40℃; Humidity: ≤80%.
- 6、 Product Label and Nameplate: