**INVACARE® CANES
CANNES INVACARE®
BASTONES INVACARE®**

- Featuring a variety of grips, shafts and bases. Refer to page 2.
- Offrant une variété de gaines, de tiges et de bases. Se référer à la page 9.
- Con características variadas de agarradores, astas y bases. Refiérase a la página 16.

Orthopedic Grip Canes have a three (3) year limited warranty. All other Invacare Canes have a Lifetime Limited Warranty.

These canes provide assistance for an individual weighing up to 250 lbs (114 kg).

WOODEN CANE

- Constructed from durable hardwood
- Reliable support
- Long-lasting durability

ADJUSTABLE CANES

- A variety of comfortable handle and grip styles
- Durable, lightweight anodized aluminum shaft
- Locking adjustment collars
- Metal-reinforced rubber tips

CROOK

INVACARE GRIP - The flat top helps reduce stress on hand.

DUAL-CAM GRIP - Form-fitting gripping area positions line of force through the shaft and cane tip.


OFFSET

FOAM GRIP - Soft, spongy material resists "breakdown" over time.

ORTHOPEDIC GRIP CANES

- Ergonomically shaped grip spreads pressure evenly across the palm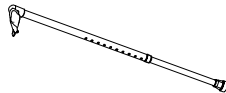

QUAD CANES

- Right- or left-hand use
- Adjustable

**MEDIUM/LARGE
BASE**


**SMALL
BASE**

FOLDING CANE

- Adjustable
- Folds to 11.5"
- Comfortable contour grip
- Anodized aluminum shaft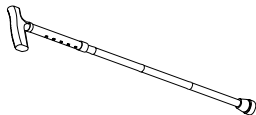

CANE TIPS

- For Invacare canes

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the safe operation of INVACARE CANES, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.

Invacare Canes can provide ambulatory assistance for an individual weighing up to 250 lbs (114 kg). The canes are designed to provide support, increased stability and assistance to the end user while walking. The canes are **NOT** designed to support the total weight of individuals.

ADJUSTABLE CANE/QUAD CANE WARNINGS

Single or Double-button **MUST** be fully engaged and protruding through height adjustment hole on cane handle to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place **BEFORE** using.

Make certain cam lock is secure at all times.

When using a Quad Cane, the cane should be placed on the walking surface with all four legs in contact with, and vertical tube perpendicular to, the walking service. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury. Extra care should be exercised when walking on a slippery or wet surface.

SAFETY SUMMARY

ADJUSTABLE CANE/QUAD CANE WARNINGS

If cane is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips do not twist on cane handle - otherwise damage or injury may occur.

For stability, ensure that longest/or flattest portion of base is closest to foot - otherwise, injury or damage may occur. It may be necessary to adjust Quad Cane for right-hand or left-hand use. Refer to **ADJUSTING QUAD CANES** in this instruction booklet.

FOLDING CANE/ORTHOPEDIC GRIP CANE WARNINGS

Ensure that the snap button fully protrudes through adjustment hole of the cane. This ensures that the cane is securely locked in position.

BEFORE using the Folding Cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.

WOODEN CANE WARNINGS

Wooden Canes **MUST** be cut by Invacare Dealers **ONLY**. Use common sense when cutting Wooden Canes with hand tools. Safety glasses should be worn and other precautions may be necessary. Read and follow all instructions accompanying hand tools **BEFORE** using.

PROPER CANE HEIGHT (FIGURE I)


NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction booklet.

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Perform one (1) of the following:

WOODEN CANE - Remove the cane tip. Turn cane upside down, as shown in FIGURE I. Mark cane for cutting at wrist length. Refer to **CUTTING CANES** in this instruction booklet.

FOLDING CANES, ADJUSTABLE CANES, ORTHOPEDIC GRIP CANES AND QUAD CANES - Adjust cane so top portion of handle touches inside of wrist where it meets palm. Refer to ADJUSTING CANE HEIGHT in this instruction booklet.

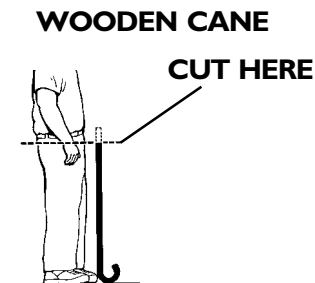FOLDING CANES, ADJUSTABLE CANES, ORTHOPEDIC GRIP CANES AND QUAD CANES

FIGURE I - PROPER CANE HEIGHT

CUTTING CANES

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction booklet.

NOTE: The following procedure should be performed by an Invacare Dealer ONLY.

NOTE: The following procedure applies to Wooden Cane ONLY.

WARNING

Cane tip MUST be removed BEFORE cutting cane. Cane tip is difficult to remove from cut portion of cane and injury may occur when handling sharp edge.

1. Remove cane tip.
2. Cut cane at position determined in PROPER CANE HEIGHT in this instruction booklet.

NOTE: Cut must be level and smooth. Avoid cutting on an angle, cane will be unstable.

3. Install cane tip onto end of cane.

NOTE: It may be necessary to tap end of cane on ground to ensure complete installation of cane tip onto end of cane.

ADJUSTING CANES (FIGURE 2)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction booklet.

CANE HEIGHT

1. Perform one (1) of the following:

FOLDING CANE AND ORTHOPEDIC GRIP CANE - Turn the anti-rattle collar **CLOCKWISE** to loosen.

ADJUSTABLE CANE AND QUAD CANE - Loosen cam lock by lifting **UP** on cam lever.

2. Press **IN** snap button or double button.
3. Slide extension up or down until snap button or double button protrudes through an adjustment hole on handle end of cane.

NOTE: An audible "click" will be heard when snap button or double button securely protrudes through the adjustment hole.

4. Repeat **STEPS 3-4** until cane is at height determined in PROPER CANE HEIGHT in this instruction sheet.
5. Perform one (1) of the following:

FOLDING CANE AND ORTHOPEDIC GRIP CANE - Turn the anti-rattle collar **COUNTERCLOCKWISE** to tighten.

ADJUSTABLE CANE AND QUAD CANE - Press **DOWN** on cam lever to secure cam lock.

ADJUSTING QUAD CANES FOR RIGHT OR LEFT HAND USE

1. Loosen cam lock by lifting **UP** on cam lever.
2. Depress the double-button and turn cane handle **180°** until double-button locks into same height adjustment hole on other side of cane.
3. When cane handle is firmly positioned at desired height and double-button protrudes through height adjustment hole on cane handle, press **DOWN** on cam lever to secure cam lock.

CARE AND MAINTENANCE

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction booklet.

1. Ensure snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged or worn items immediately.

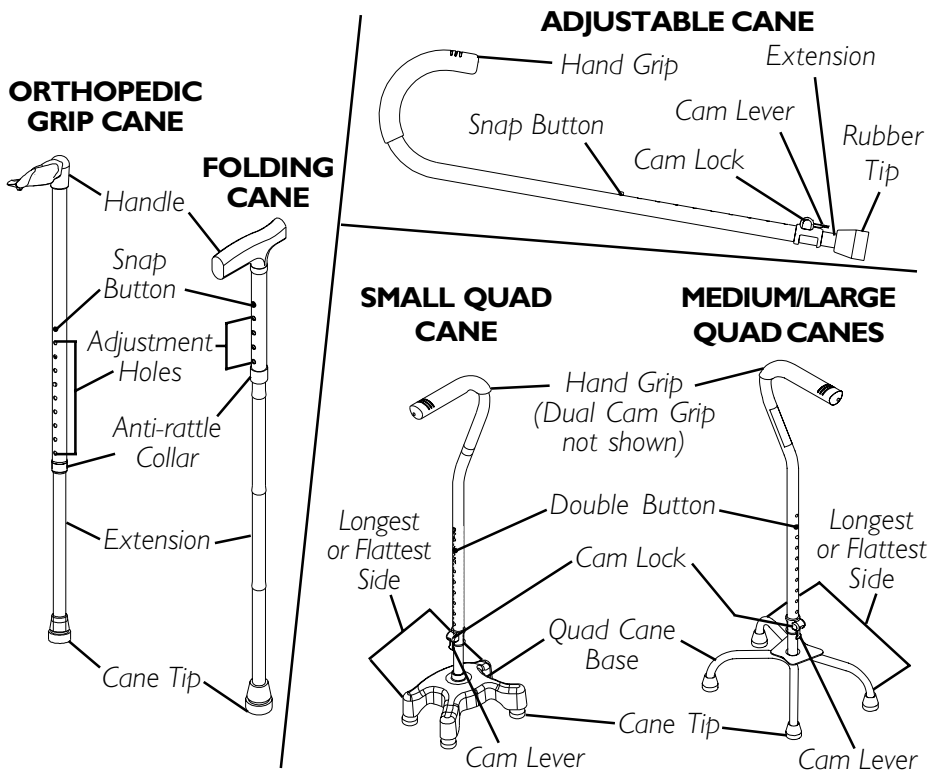
FIGURE 2 - ADJUSTING CANE HEIGHT

LIFETIME LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in material and workmanship for the lifetime of use by the original purchaser for the Adjustable Canes, Wooden Canes, Quad Canes, Folding Canes and for three(3) years for the Orthopedic Grip Canes. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare