

## Pre-Treatment Preparation


1. Confirm that the patient does not have any implanted devices, pacemakers. Refer to the Contraindication list in the User Manual. In any doubt, contact a physician.
2. Remove all metal objects or accessories (piercing, jewelry etc).
3. Clean the treatment area thoroughly with water and soap in order to remove lotion and makeup.
4. Take photos of the patient.
5. For body treatments only:
  - While patient is standing, mark the area to be treated.
  - Divide the treatment area into 10x10 square cm. Mark the boundaries or corners with a permanent marker.
6. Position the patient comfortably on the treatment bed.
7. Clean the Eclipse Firma treatment handpiece with 70% alcohol followed by a dry gauze. Make sure the handpiece is dry before starting a new treatment.

## Performing the Treatment

1. Apply gel provided with the Firma to the first treatment square/area.
2. Move the treatment handpiece on the selected square/area in a circular motion.
3. While in constant motion, push the trigger button to activate an RF pulse.
4. After completion of the pulse, measure the surface temperature using the laser thermometer.
5. If the temperature is  $< 40^{\circ}\text{C}$  – increase power by 2W and perform an additional pass.
6. If the temperature is  $> 42^{\circ}\text{C}$  - decrease power by 2W and perform an additional passes.
7. When at least 6 passes (for a face treatment) or 8 passes (for a body treatment) at temperature  $> 40^{\circ}\text{C}$  have been completed consecutively, move to the next treatment area and repeat the procedure.

## Setting Treatment Parameters

1. From the Main Menu, choose the desired application.
2. Choose the appropriate handpiece according to the application.
3. Choose the target area for treatment (1).
4. Modify the power settings (optional) (2).
5. Click the “Start” button (3) to confirm settings and enable handpiece trigger. The “Pause” button will start blinking.
6. The time counter (4) will indicate the time remaining to complete the pass.
7. The “Passes” counter (5) counts the passes performed. It can be reset by clicking the button.

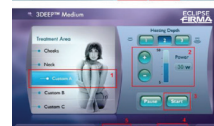

PRODUCT PROTOCOL  
FOR SUCCESS

# ECLIPSE FIRMA™

## Post - Treatment

1. Clean the treatment handpiece with 70% alcohol followed by dry gauze.
2. Make sure the handpiece is dry before starting a new treatment.

## Accessories

Multiple easy-to-use handpieces for specific target zones are available. Each unit comes standard with two handpieces. Selection available and uses can vary based on model. Please contact your representation with any questions.



## Pricing

It is suggested to provide Firma sessions as a bundle package. Many provide four sessions on average, but that may vary based on a patient's needs.

Estimated Patient Fees: Range \$3,500 to \$4,500 based on patient treatment plan.



PRODUCT PROTOCOL  
FOR SUCCESS



Eclipse Aesthetics, LLC.  
13988 Diplomat Drive, Suite 160  
Dallas, Texas 75234  
800-759-6876  
[www.EclipseAesthetics.com](http://www.EclipseAesthetics.com)

**Need Help?** The Eclipse Team is Here to Assist. (800) 759-6876

