

# **Body Shaping System V8plus User Manual**




# Contents

Chapter 1 System Illustration.....	4
1.1 Equipment Illustration.....	4
1.2 Handpiece Illustration.....	4
1.2.1 Handpiece 1.....	4
1.2.2 Handpiece 2.....	5
1.3 Parts of Host System.....	6
1.4 Technical Specifications.....	6
1.5 Working Conditions.....	7
Chapter 2 Safety.....	8
2.1 Optic Safety.....	8
Chapter 3 Operation.....	9
3.1 Parts Installation.....	9
3.2 To Install and Discharge Handpiece.....	10
3.3 To Install Earth Line.....	10
3.4 To Turn on the Equipment.....	10
3.5 Functions of Handpiece and Default Parameters.....	11
3.6 Operation Procedure.....	12
3.7 To Issue and Use Wireless IC Card.....	14
3.7.1 IC Card.....	14
3.7.3 To Make Equipment Have IC Function.....	14
3.7.4 To Issue Unblock Card.....	15
3.7.5 To Stop Function of IC Card.....	15
3.7.6 To Use Wireless IC Card.....	15
3.8 Warning for Handpiece Detection.....	16
3.9 To Turn off Equipment.....	16
Chapter 4 Maintenance and Troubleshooting.....	17
4.1 Protective Maintenance.....	17
4.2 To Clean and Maintain Handpiece.....	17
4.3 Trouble Elimination Directory.....	17
Chapter 5 Clinical Guide.....	19
5.1 Treatment Principle and Functions.....	19
5.2 Indications.....	19
5.3 Contraindications.....	19

- 5.4 To Set Course of Treatment..... 19
- 5.5 Complications..... 19
- 5.6 Notes..... 19
  - 5.6.1 Treatment on Face..... 19
  - 5.6.2 Treatment on Eyelid..... 20
  - 5.6.3 Treatment on Neck:..... 21
- 5.7 Treatment on Body..... 21
  - 5.7.1 Treatment on Belly and Waist..... 21
  - 5.7.2 Treatment on Back..... 23
  - 5.7.3 Treatment on Hip..... 24
  - 5.7.4 Treatment on Leg..... 24
  - 5.7.5 Treatment on Upper Arm..... 25
- 5.8 Notes after Treatment..... 25
- 5.9 Suggestions about Operating Modes for Beauty Salon..... 26
- 5.10 Questions and Replies about V8plus..... 26
- 5.11 Questions about Treatment on Face..... 28
- 5.12 Questions about Treatment on Eye..... 29

# Chapter 1 System Illustration

This chapter is a general illustration of Body Shaping System V8plus.

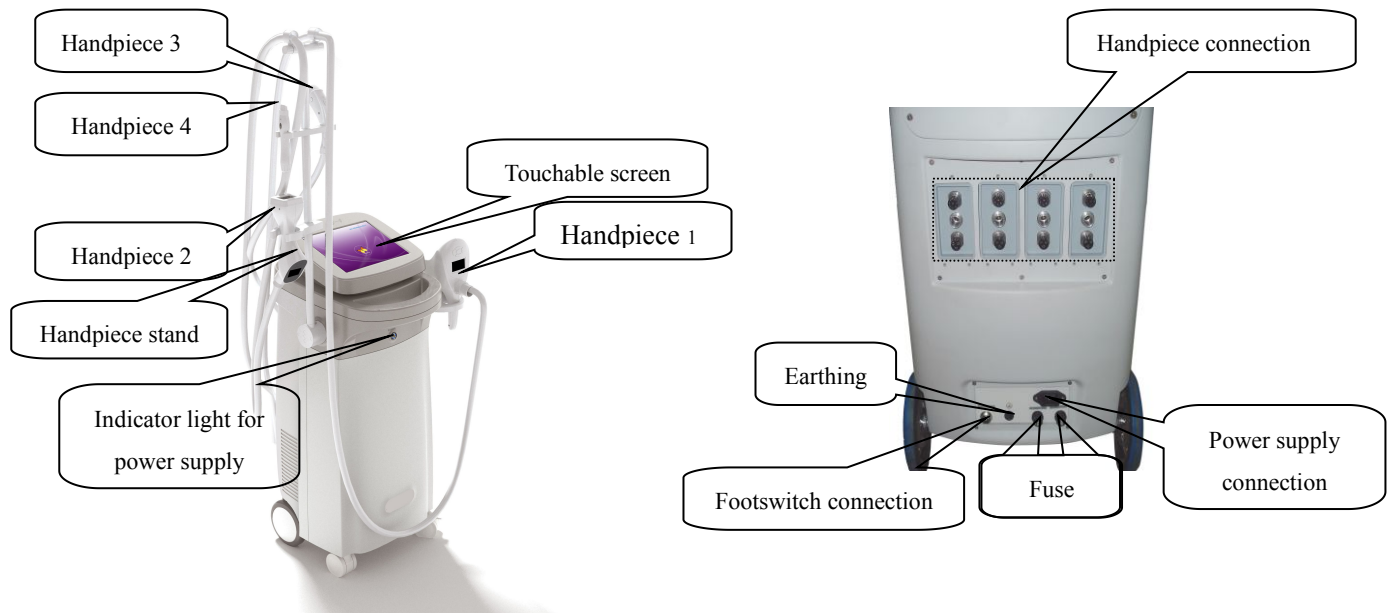


## 1.1 Equipment Illustration

- Controlling system: 12" TFT, touchable true color screen, power switch, micro-computer controlling system etc to control operation of whole system;
- Encermologie and pulse mechanical tissue manipulation;
- RF(Radio Frequency): High power output radio frequency system;
- Near-infrared laser system: 940nm semi-conductor laser technique;
- Wireless IC card controlling system: RFID identify technology makes you control the system easily.

## 1.2 Handpiece Illustration

### 1.2.1 Handpiece 1

- Operation illustration of handpiece 1:**
  - Press “▲”, “▼” to select specifications;
  - Press “◀”, “▶” to adjust parameters;
  - Then you press “Enter” to work;
  - You could do the above adjustment on system display screen;
  - Parameters could not be adjusted while working.

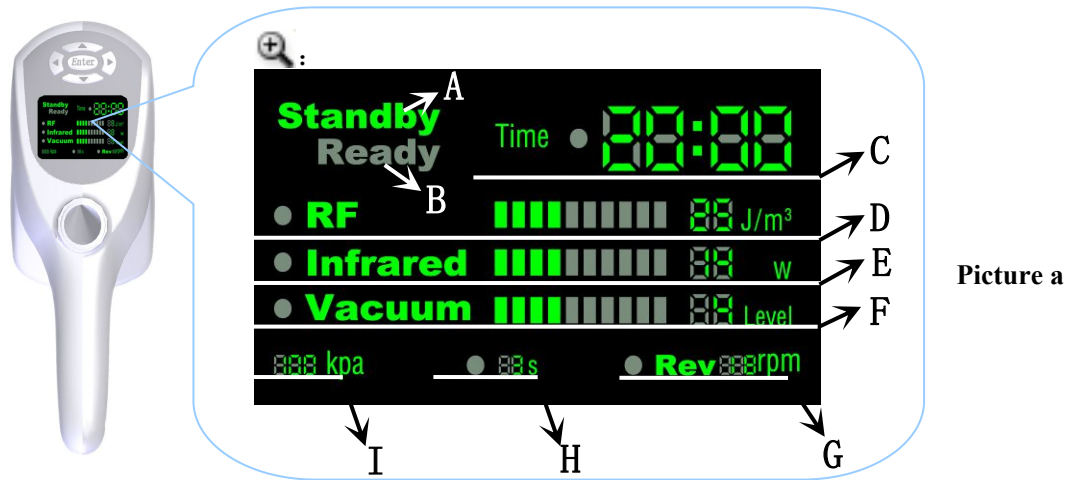


Picture a

2) Display for handpiece 1: (picture a)

- A: Standby;
- B: Preparation;
- C: Time;
- D: Energy of RF parameters;
- E: Near-infrared laser;
- F: Distinction of negative pressure parameters;
- G: Speed of roller;
- H: Pulse time;
- I: Absolute value

### 1.2.2 Handpiece 2

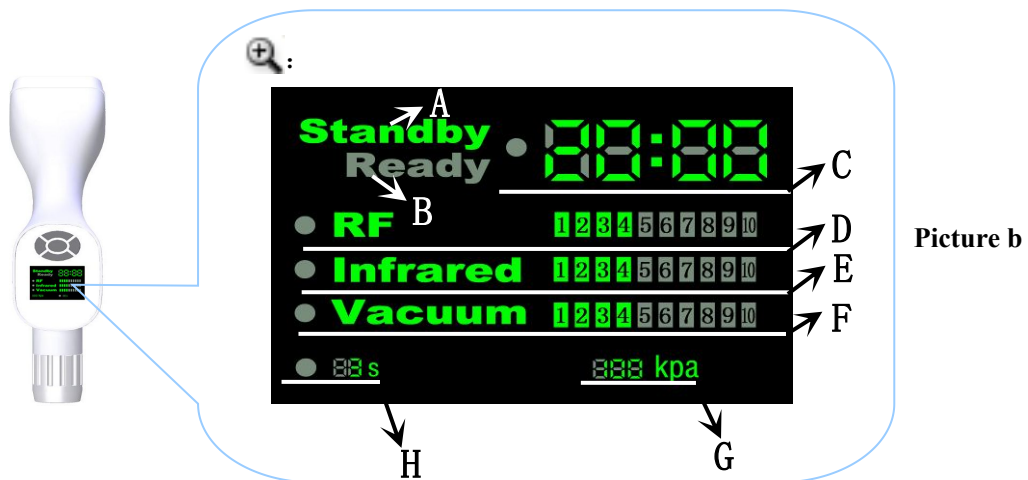

Picture b

1) The operation of handpiece 1 and handpiece 2 are the same.

2) Display for handpiece 2: (picture b)

- A: Standby;
- B: Preparation;
- C: Time;
- D: Energy of RF parameters;
- E: Near-infrared laser;
- F: Distinction of negative pressure parameters;
- G: Absolute value
- H: Pulse time;

### 1.3 Parts of Host System



Host system×1



Eyecup×1



Power line×1



Hand piece1×1



Hand piece 2×1



Hand piece 3×1



Hand piece 4×1



Footswitch×1



Earthing line

### 1.4 Technical Specifications

Items	Specifications
LCD	1) Display screen of host:12" TFT , touchable true color Screen  2)Display screen on handpiece:  i. Display screen on handpiece 1: 2.4" ii. Display screen on handpiece 2: 1.9"
Working Mode	Pulse  Pulse width: 0.5s-7.5s

Negative pressure	1. Absolute value: 90kPa -25kPa(68.4cmHg -19cmHg);  2. Relative value: 10kPa -75kPa(7.6cmHg -57cmHg);
Speed of roller	0-36 rpm
Working mode for roller	2 types
Safety checking	Real time on line
RF frequency	1MHz
RF energy density	Max: 60J/cm <sup>3</sup>
Laser wavelength	940nm
Laser power	20W (max)
Number of handpiece	4 (Auto Identification)
Treatment area	4mmx7mm/8mmx25mm/30mmx50mm/40mmx60mm
Management system	Wireless IC card
Rated input power	750VA
Mode of power supply	AC230V±10%, 50Hz±1Hz/ AC110V±10%, 60Hz±1Hz (Optional)
Net weight	50kg
Physical dimension	480mm×410mm×1015mm

### 1.5 Working Conditions

- 1) Environmental temperature for operation : 10°C—30°C;
- 2) Environmental temperature for keeping: 0—50°C;
- 3) Relative humidity: ≤70%;
- 4) Air pressure: 860hPa~1060hPa;
- 5) Environment request: avoid direct radiation of intense light ;
- 6) To avoid intense light.


Note: If the temperature in the room is too low, or the instrument have been not used for many days, the temperature of the whole system shall recover to 10 °C or higher before start.

## Chapter 2 Safety

This chapter shows the safety measures need to be considered when you use V8plus. The operators and maintain personnel should be familiar with the safety requirements and procedures when they use the equipment.


### 2.1 Optic Safety


- 1) Do not look directly into the delivery outlet of laser;
- 2) Be sure that rated input voltage written on the label is coincident with the supply in the location;
- 3) The maintain process can only be carried out ten minutes after the power has been shut off and the electric line has been unplugged. Otherwise, there is danger to the personnel or the machine;
- 4) There is dangerous high-voltage in the machine. So no one is allowed to disassemble any parts of the machine except the technical personnel appointed by our company;
- 5) Good earthing is very important to operation safety. This equipment has good earthing by yellow and green wires in power line. (Picture A).

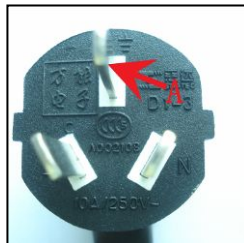

Picture A



### 2.2 Safety Designation Illustration

“” shows “ protection grounding ”

“” shows high voltage here is dangerous, and do not touch it.

“AC230V 50Hz”or “AC110V 60Hz” shows the identifier of input power supply. (optional)

“BGXP”“AC250V 10A”or“AC110V 15A” separately shows the type of the fuse and rating. (optional)


“” is sign for laser.  




## Chapter 3 Operation

### 3.1 Parts Installation

#### 1) To install handpiece stand

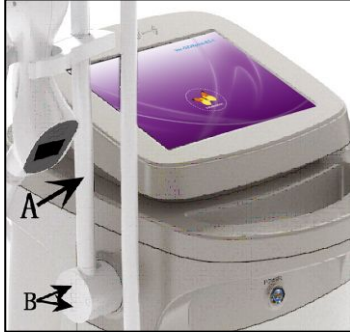


Picture 1




Picture 2



Picture 3

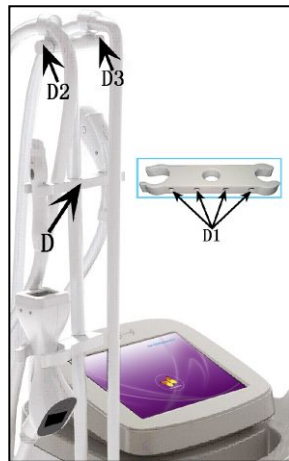

Picture 4

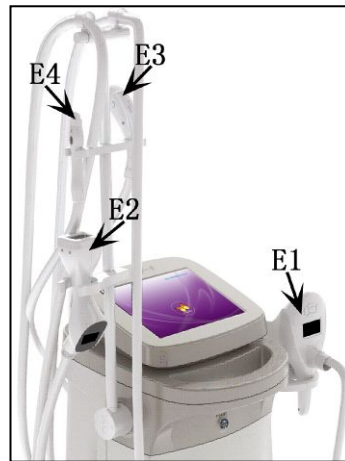


Picture 5

- Install A (in picture 1) in relative place like picture 1 shows and install two screws for fixing (B in picture 1);
- Install F (in picture 2) and install two screws for fixing (F1 in picture 2);
- Fix C and D on relative place of A (picture 1);
- Install screws (M4 × 12) in C1 (in picture 3) and install screws (M4 × 12) in D1 (in picture 4)
- Then place handpiece in relative space (E1\E2\E3\E4 in picture 5);
- Take down D2/D3 and fix the two rings of handpiece 1 and handpiece 2 (Picture 4)

### 3.2 To Install and Discharge Handpiece



Picture 6

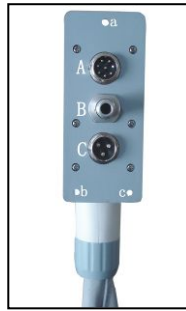

Picture 7



Picture 8

#### 3.2.1 To Install Handpiece

- 1) The number of handpiece shall correspond with the number “1, 2, 3, 4” on the equipment (picture 6);
- 2) “A, B, C” on handpiece (picture 7) shall correspond with “A, B, C” on the equipment (picture 6);
- 3) Installation order: handpiece 1, handpiece 2, handpiece 3, handpiece 4;
- 4) Fix 3 screws (a in picture 8) in a, b, c (picture 7).


#### 3.2.2 To Discharge Handpiece

Discharge the four handpiece like this:

- 1) Do turn off the equipment;
- 2) Take down 3 screws (a in picture 8) in a, b, c (picture 7) and take down handpiece directly.

### 3.3 To Install Earth Wire

Before using equipment, you must connect the earth wire as right picture shows. One side of the earth line shall be connected to equipment and the other side be connected to metal touching earth (such as radiator and water pipe) .

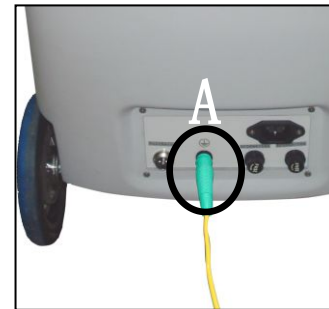

### 3.4 To Turn on the Equipment

- 1) Well insert the power line and press the power switch under display screen;
- 2) The display screen illumines after 1s. Then it enters initialization interface  
(Picture 9, version number: Ver-GZV8plus-E2-1) ;
- 3) As well as initialization, the system checks connection of the four handpiece in turn;  
After that the system enters main interface (Picture 10);  
4.2 in Chapter 4 is trouble elimination.



Picture 9



Picture 10

### 3.5 Functions of Handpiece and Default Parameters

#### Form 1: Functions of handpiece

Handpiece	Power of near-infrared laser	Speed of roller	RF energy	Negative pressure *Level	Pulse time *s
Handpiece 1	0~20W	0~36rpm	0~60J/cm <sup>3</sup>	1~9	1s ~8s
Handpiece 2	0~20W	No		1~9	1s ~8s
Handpiece 3	0~10W	No	0~50J/cm <sup>3</sup>	1~7	1s ~6s
Handpiece 4	No	No	0~40J/cm <sup>3</sup>	1~6	1s ~4s

#### Form 2: Default parameters

Handpiece	Power of near-infrared laser	Speed of roller	RF energy	Negative pressure *Level	Pulse time *s
Handpiece 1	14W	18rpm	25J/cm <sup>3</sup>	3	3s
Handpiece 2	14W	-	25J/cm <sup>3</sup>	3	3s
Handpiece 3	6W	-	25J/cm <sup>3</sup>	1	2s
Handpiece 4	-	-	20J/cm <sup>3</sup>	1	2s

#### Form 3: Default handpiece for different body parts

Body parts	Default handpiece	Body parts	Default handpiece	Body parts	Default handpiece
Belly	Handpiece 1	Calf	Handpiece 2	Face	Handpiece 3
Back		Waist		Neck	
Thigh		Arm		Eyelid	Handpiece 4
Hip					


### 3.6 Operation Procedure

Note: Default handpiece is handpiece 1;

The following procedure is taking treatment on back for example.

**Interface of main menu:** After initialization, the system enters main interface (Picture 11)

**Interface of main operation:** Press A in picture 11 and the system enters main operation interface (picture 12)

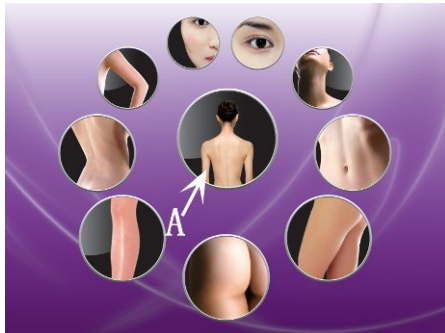



Picture 11 (main menu)



Picture 12 (main operation)

➤ **A in picture 12:**


This part shows treatment manner on human body;

Press “” to stop it; Press “” to start it.

➤ **B in picture 12:**

**Handpiece selection:** Press handpiece you want and E in picture 12 would show it;

**To set working time:** In main operation interface (default working time is 20 minutes):

Press “” and it turns green;

Press “+” or “-” in E to adjust it (adjustment scope: 0-60 minutes, 1 minute/press)


➤ **C in picture 12:**

**To adjust energy of infrared laser:**

In main operation interface, Press “”, the parameter would turn green;


Press “+” or “-” in E to adjust it.

**To adjust output RF energy:**

Press “” and the parameter would turn green;

Press “+” or “-” in E to adjust it.


**To adjust speed of roller:**

Press “” and the parameter would turn green;

Press “+” or “-” in E to adjust it.


➤ **D in picture 12:**

**To adjust vacuum:**

Press “” and the parameter would turn green;

Press “+” or “-” in E to adjust it.

**To set duration of pulse:**

Press “” and the parameter would turn green;

Press “+” or “-” in E to adjust it.

**To enter standby situation:**

Directly press “Ready” key to enter standby situation. Indicator light of selected handpiece is illuminated.

**To enter working situation:** Press A in picture 13 or step on footswitch;


**To exit working situation:** Press A in picture 13 or step on footswitch;


**To enter standby situation:**

Under ready situation, press “Standby” key. Indicator light of selected handpiece goes off.



Picture 13

**To exit main operation interface:** In main menu interface, press “” .

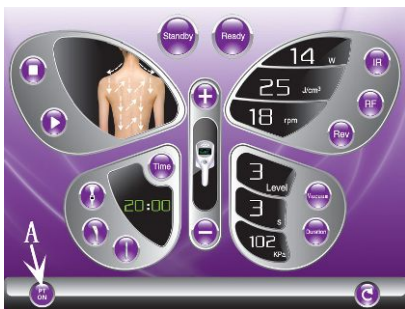


Picture 14



Picture 15

**Air pressure detection:** At the beginning, it is on (A in picture 14);

Press “”, it is shut off (A in picture 15);

**Note:** If there is something wrong with air pressure, you shall shut off air pressure detection. And contact with our company.

### 3.7 To Issue and Use Wireless IC Card

#### 3.7.1 IC Card

IC card contains key card, user card and unlock key card.



Picture 16

#### 3.7.2 To Issue User Card



Picture 17 (to issue card)



Picture 18 (succeed)



Picture 19 (failed)

- 1) After turning on the equipment, put key card on A in picture 16 and system shows picture 17;
- 2) To set issuing time: Touch “+” or “-” to adjust issuing time. The system adds or reduces 100 minutes after each press. Adjustment scope: 0-99900 minutes;
- 3) Put blank card on A in picture 16 and the system would issue user card automatically;  
If successful, it shows “Issue OK!” (Picture 18);  
Or it shows “Issue Error!” (Picture 19);
- 4) Repeat the above operation step and you could issue more card;
- 5) In issuing interface, the system would exit issuing or main operation interface automatically in 60s;
- 6) To finish setting: press “Back” .

#### 3.7.3 To Make Equipment Have IC Function



Picture 20



Picture 21



- 1) After turning on equipment, make issued user card touch A in picture 16 and could hear warning tone;  
Then A in picture 20 would be showed and the digital (such as “14400”) is the left time.
- 2) If there is no time left in IC card (Picture 21), after touching “Standby” in main operation interface, system could not enter “Ready” situation.

### 3.7.4 To Issue Unlock Card



**Picture 22 (to issue unlock card)**



**Picture 23 (succeed)**



**Picture 24 (failed)**

- 1) In the interface of issuing IC card, press “Unlock Card” key to issue unlock card (Picture 22);
- 2) Put blank card on A in picture 16 and the system would issue unlock card automatically;  
If successful, it shows “Issue OK!” (Picture 23);  
Or it shows “Issue Error!” (Picture 24);
- 3) Repeat the above operation step and you could issue more card;
- 4) In the interface (picture 22), the system would get back to main operation interface in 60s automatically;
- 5) To finish issuing: Press “Back” key to quit from issuing interface.

### 3.7.5 To Stop Function of IC Card

After turning on the equipment, you put unlock card on A in picture 16 and could hear warning tone;  
Then A in picture 20 would disappear (picture 21);  
Equipment with no IC card function could be used with no user card.

### 3.7.6 To Use Wireless IC Card

#### 1. Agent

- 1) Agent wants to rent the equipment and take charge according to time;
- 2) The equipment works for 8 hours per day, 30 days/month and the total time is 14,400 minutes (8 hours/day\*30 days\*60 minutes). The agents take charge once and take following measures to manage:
  - Enter interface of issuing card with key card and set time:14,400 minutes;
  - Put blank card or user card on A in picture 16;  
The sound you hear shows succeed in issuing user card;
  - Exit from issuing interface, make issued user card touch A in picture 16. Give user card and equipment to customer (The agent keeps the key card);
  - The user must insert user card into slot (on the top of the back of equipment) when he uses the equipment;

If there is no time left, the equipment is locked on;

- After user giving payment to agent, the agent would issue a user card with 14,400 minutes again and give it to user;
- If user wants to buy the equipment, the agent could give key card to user after receiving payment.

## 2. Beauty salon

- 1) The beauty salon take charge by renting card and taking count of treatment course and treatment times;
- 2) 10 times/treatment course and 30 minutes/time;

The beauty salon could take following measures to manage:

- Enter interface of issuing card with key card and set time:300 minutes (10 times \*30 minutes/time);
- Put blank card or user card on A in picture 16;

The sound you hear shows succeed in issuing user card;

- Exit from issuing interface, make issued user card touch A in picture 16. Give customer card and equipment to customer (The beauty salon keeps the key card);


- When use the equipment, must insert user card into slot (on the top of the back of equipment);

Set the time:30 minutes;

After 30 minutes, the equipment stops working and make two sounds;

- After treatment, the customer takes away the user card.


## 3.8 Warning for Handpiece Detection

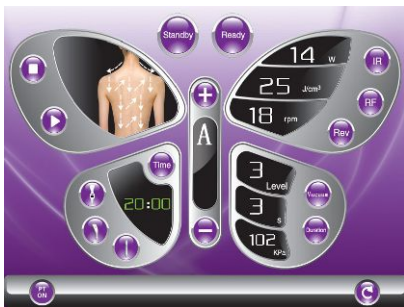

Picture 25



Picture 26

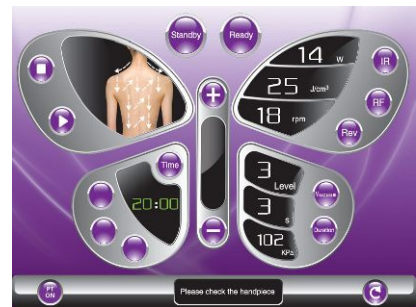


Picture 27

Main operation interface would show “Please check the handpiece” (picture 27) if following situations happens:

- 1) A in picture 25 (no handpiece is selected);
- 2) A in picture 26 (handpiece isn't installed).

## 3.9 To Turn off Equipment

Under standby condition, press “”, exit from main operation interface to main menu interface;  
Press the power supply button.



## Chapter 4 Maintenance and Troubleshooting

**Warning:** Ten minutes later turn off the equipment, you draw the pin of power supply to vindicate the machine. Do not leave the machine with the cover board opened up and no attendance. Or it can harm human being or the machine itself.

### 4.1 Protective Maintenance

The preventative maintains should be taken by our authorized staff.

### 4.2 To Clean and Maintain Handpiece

- 1) After each treatment, the handpiece shall be cleaned by ultrasonic cleaner;
- 2) Cleaning procedures:
  - a) Fill 2/3 capacity of ultrasonic cleaner with warm water (the temperature of water is about 30 Degrees Centigrade);
  - b) Put handpiece in ultrasonic cleaner;
  - c) After handpiece has been cleaned for 3 minutes, you dry the handpiece with tissue paper or warm wind.

### 4.3 Trouble Elimination Directory

Trouble	Reasons	Measures
Start-up failure	Trouble with power switch	Change power switch
	Didn't insert power line well	Insert power line well
	Blown fuse indicator	Change fuse
	Trouble with control board	Change control board
	Trouble with switch power supply	Change switch power supply
Key on handpiece has no response	Trouble with control system	1. Shut off power switch and start-up again; 2. Contact with authorized stuff
No response of touch screen	Trouble with touch screen	Contact with authorized stuff
Something wrong with negative pressure	Trouble with vacuum pump	Change vacuum pump
	Trouble with AC contactor	Change vacuum pump
	Trouble with electromagnetic valve	Change electromagnetic valve
	Trouble with 24V switch power supply	Change switch power supply
	Trouble with air connection	Change socket connector
	Trouble with handpiece	Check the handpiece
No RF	RF level is 0	Increase level
	Trouble with PA board	Change PA board
	Trouble with chip of barometer on control board	Change chip
	Trouble with control board	Change control board
	Trouble with relay for exchanging RF	Change interposer
	Trouble with inner parts of handpiece	Check the handpiece
No laser	Laser level is 0	Add to level
	Trouble with inner parts of handpiece	Check the handpiece
	Trouble with on control board	Change control board
Roller doesn't work	Speed of roller is 0	Add to level
	Poor contact in handpiece or trouble with motor	Check the handpiece
	Trouble with control board	Change control board

## Chapter 5 Clinical Guide

### 5.1 Treatment Principle and Functions

- a) Infrared laser reduces skin impedance by heating skin and RF energy penetrates deeply into connective tissue; The synergistic combination of infrared laser and conducted RF energies increase the oxygen intracellular diffusion by heating the skin
- b) Vacuum plus specially designed rollers manipulate leads RF penetration to be even 5-15mm. At the same time, vacuum and roller mechanical tissue manipulation nips and stretches fibrillar connective tissue, effectively breaks down subcutaneous fat as well as extruded capillary vessel, Increase lymphatic drainage, promotes metabolism and reduces or shrinks the size of the actual fat chamber and greatly improved body contouring effect.
- c) The technology that vacuum folds skin makes RF energy penetrate a specific folded skin, greatly improve effect and safety, even for the upper eyelid treatment.

### 5.2 Indications:

- 1) Black circle around eyelid removal, wrinkle around eyelid improvement and pouch improvement;
- 2) Upper eyelid lifting and wrinkle removal;
- 3) Body contouring after birth;
- 4) Body re-shaping, reduce waist;
- 5) Generalized obesity, localized adiposity, fat dissolving, skin tightening ( arm, leg, shoulder and back, mountaineering shoe, buttocks) ;
- 6) Alleviate arthritis pain and whole body physical therapy;
- 7) Stretch mark improvement

### 5.3 Contraindications:

- 1) Pacemaker and defibrillator in body;
- 2) Metal under treated skin;
- 3) With skin diseases or cancer history;
- 4) With other serious disease, such as heart disease or diabetes;
- 5) Gestation period and breast-feed;
- 6) With thermal stimulus history;
- 7) Openness wound;
- 8) Light-sensitiveness;
- 9) Bronzer skin because of solarization;
- 10) Menses.

### 5.4 To Set Course of Treatment:

- 1) Face: 30 minutes / treatment, 1 treatment /week,10 times/course;
- 2) Eyelid: 15 minutes / treatment, one treatment / week, 10 times/course;

- 3) Whole body : 45-60 minutes/treatment, local treatment:15 minutes  
Twice treatments / week, 10 times/course;

## 5.5 Complications

Normally there are no complications and discomfortableness.

## 5.6 Notes:

- 1) Things preparation:

Camera, facial treatment cleanser, essential oil for conditioning and alcohol swab

- 2) Preparation before treatment:

- To illustrate the function of the equipment and the feeling during treatment for customer;
- Don't use hormonal drugs one week before treatment (prednisone) ;
- Don't wear metal;
- Take photo to keep record and the background and light of background shall be the same

- 3) Treatment procedures:

First step: Use warm water and towel to clean skin to be treated;

Second step: Lay essential oil for conditioning on skin to be treated;

Third step: Turn on the equipment and adjust parameters;

Fourth step: Do the treatment

- 4) Treatment skills:

During the treatment, usually ask the feeling of customer, the strength, the heat received and comfort;

To observe whether the skin turns red and adjust parameters and energy according to response of customer for making sure what skin area can be treated repetitively;

You can reduce parameters when you treat on ribs and inguinal and thinner skin.

### 5.6.1 Treatment on Face

- 1) **Applications:** Face lifting/ skin tightening/wrinkle removal;

- 2) **Course of treatment:** once/week, 10 times/course

- 3) **Handpiece selection and parameters setting:**

Use handpiece 3;

RF: 25J/cm<sup>3</sup>~35J/cm<sup>3</sup>/ Vacuum: 1~3/ Pulse time: 1s~2s/ Infrared: 6W~10W

- 4) **Notes:**


Reduce 1-2 energy level (RF and vacuum) when you treat on forehead, cheekbone and cheen, because skin of these parts is too thin to touch head of handpiece completely.

- 5) **Treatment procedures:**

- a) Clean skin (Exfoliate it if necessary);

- b) Lay essential oil for conditioning on skin to be treated;
- c) Treatment sequence: Cheek----Forehead----Upper lip;

6) **Treatment manner** (Picture 1)

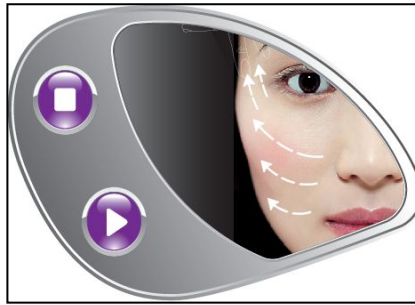

**Picture 1**

i) First step: Up-lifting

- a) Pull up head of handpiece according to dermatoglyph as picture 1 shows (picture 1);
  - b) To up draw skin with the other hand while treatment;
  - c) Repeat to treat on this part for 3-4 times
- ii) To get best result: repeat to treat on special parts, such as nasolabial folds, forehead lines, wrinkle and pores visible.

7) **Effect**

- a) Skin treated on is reddish;
- b) Wrinkle on face is improved;
- c) Skin treated on is up-lifted and tightened

**5.6.2 Treatment on Eyelid**

1) Applications:

Up-lifting (upper eyelid and lower eyelid) / tightening/ pouch removal/ black circle around eyelid removal;

2) Course of treatment: once/week, 10 times/course

3) Handpiece selection and parameters setting:

Use handpiece 4;

RF: 20J/cm<sup>3</sup>~25J/cm<sup>3</sup>/ Vacuum: 1/ Pulse time: 1s~2s

4) Treatment procedures:

- b) Clean this part;
- c) Lay essential oil for conditioning on skin to be treated

5) Treatment manner (picture 2)



**Picture 2**

i) First step: Up-lifting

- a) Move the head of handpiece as picture 2 shows;
- b) To up draw skin with the other hand while treatment;
- c) Repeat to treat on this part for 3-4 times

ii) To get best result: repeat to treat on upper eyelid, lower eyelid, wrinkle and pouch

6) Effect

- a) Skin treated on is reddish;
- b) Wrinkle on eyelid is improved;
- c) Eyelid is up-lifted and tightened

### 5.6.3 Treatment on Neck

1) Applications:

Up-lifting/ skin tightening/wrinkle removal

2) Course of treatment: once/week, 10 times/course

3) Handpiece selection and parameters setting:

Use handpiece 3;

RF: 20J/cm<sup>3</sup>~30J/cm<sup>3</sup>/ Vacuum: 1/ Pulse time: 1s~2s/ Infrared: 6W~10W

4) Treatment procedures:


- a) Clean this part;
- b) Lay essential oil for conditioning on skin to be treated;

5) Treatment manner:

- a) Move the head of handpiece as picture 3/picture 4 shows;
- b) To up draw skin with the other hand while treatment;
- c) Repeat to treat on this part for 2-3 times



Picture 3

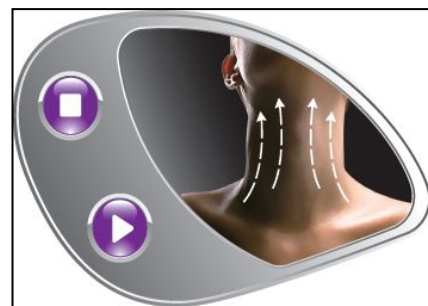

Picture 4

### 5.7 Treatment on Body

Applications: skin tightening (belly, back, waist, calf, thigh and hip) /body shaping;

Course of treatment: twice/week, 10 times/week

#### 5.7.1 Treatment on Belly and Waist

1) Preparation before treatment:

a) Consultation:


Beautician introduces to customer the applications, feelings during treatment and contraindication (beautician could refer to 5.10 in Chapter 5);

Ask customer take down metallic jewelry; customer shall tell beautician the feelings during the treatment in time to help adjust treatment manners and increase comfortableness.


b) Take photos: Take photos of different sides and keep record so as to make contrast;

To take photos under the same background with the same light

c) Take measurement and keep record (picture 5)

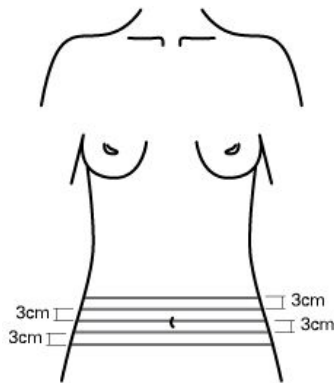

Picture 5

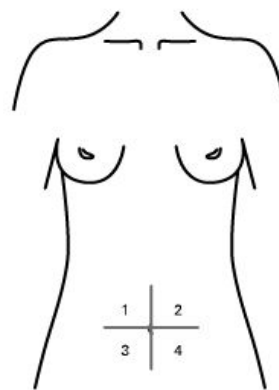

Picture 6

2) Handpiece selection and parameters setting:

Use handpiece 1;

RF: 40J/cm<sup>3</sup>~60J/cm<sup>3</sup>/ Vacuum: 3-8/ Pulse time: 2s~3s/ Infrared: 10W~20W/Speed of roller: 18rpm~30rpm

3) Notes:

a) During the treatment, ask customer about the feeling, strength, heat, comfortableness;

b) Reduce 1-2 energy level while treatment on waist, stomach and lateral rib;

4) Treatment procedure:


a) Clean skin with warm water;

b) Lay essential oil for conditioning on skin to be treated;


5) Treatment manner:

i) treatment on belly


three kinds of manners: picture 7,8,9

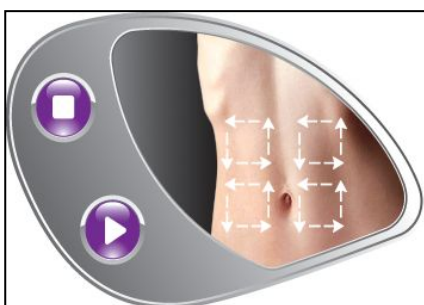

Picture 7

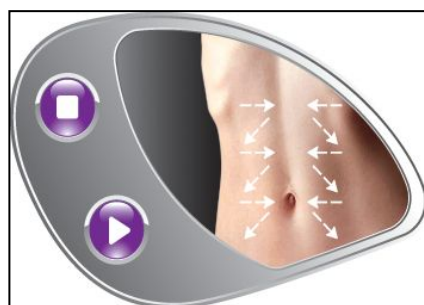


Picture 8

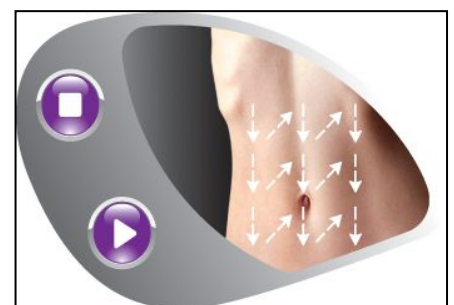

Picture 9

ii) treatment on waist (picture 10)

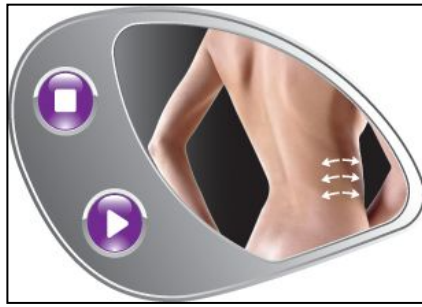

**Picture 10**

6) Notes:

Before treatment, make handpiece touch skin completely without air leakage;

Then step on footswitch or press key on handpiece to start the treatment;

If stop during the treatment, you could press key on handpiece or release the footswitch

7) Treatment skills: Divide the skin to be treated into four parts (picture 6) ;

After treating on one part, treat on next one;

Repeat to treat on the four parts to make belly absorb heat evenly

**5.7.2 Treatment on Back**

1) Handpiece selection and parameters setting:

Use handpiece 1;


RF: 40J/cm<sup>3</sup>~60J/cm<sup>3</sup>/ Vacuum: 4-8/ Pulse time: 2s~3s/ Infrared: 10W~20W/Speed of roller: 18rpm~30rpm

2) Treatment procedure:


a) Clean skin with warm water;

b) Lay essential oil for conditioning on skin to be treated;


3) Treatment manners (picture 11-14):

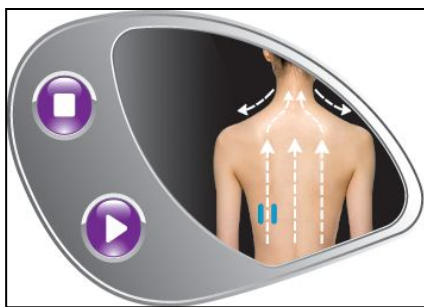


**Picture 11**

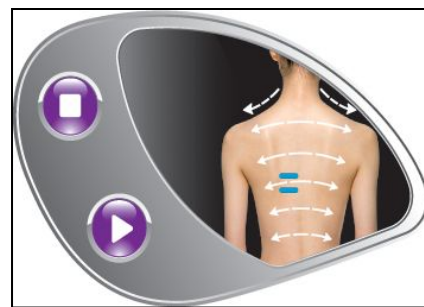

**Picture 12**

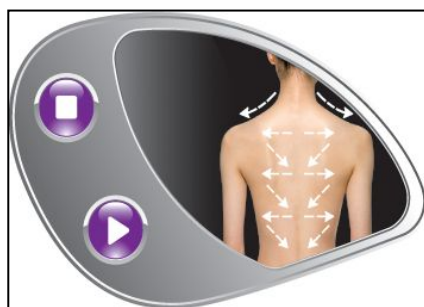

**Picture 13**

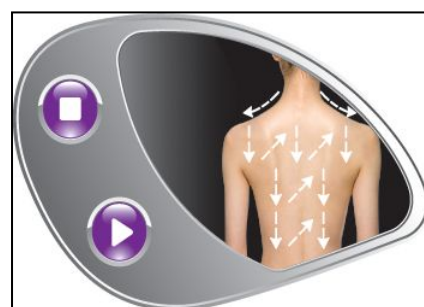

**Picture 14**

### 5.7.3 Treatment on Hip

1) Handpiece selection and parameters setting:

Use handpiece 1 or handpiece 2;

RF:  $40\text{J}/\text{cm}^3\sim 60\text{J}/\text{cm}^3$  (for handpiece 1) ;  $35\text{J}/\text{cm}^3\sim 55\text{J}/\text{cm}^3$  (for handpiece 2)/

Vacuum: 4-8 (for handpiece 1) ; 3-7 (for handpiece 2)/


Pulse time: 2s~3s/ Infrared: 10W~18W/Speed of roller: 18rpm~30rpm

2) Treatment procedure:


a) Clean skin with warm water;

b) Lay essential oil for conditioning on skin to be treated

3) Treatment manner:

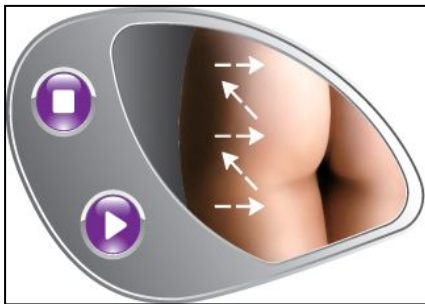

Picture 15

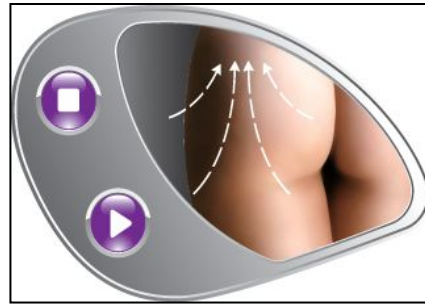

Picture 16

### 5.7.4 Treatment on Leg

1) Handpiece selection and parameters setting:

Use handpiece 1 or handpiece 2;

RF:  $40\text{J}/\text{cm}^3\sim 60\text{J}/\text{cm}^3$  (for handpiece 1) ;  $35\text{J}/\text{cm}^3\sim 55\text{J}/\text{cm}^3$  (for handpiece 2)/

Vacuum: 4-8 (for handpiece 1) ; 3-7 (for handpiece 2)/


Pulse time: 2s~3s/ Infrared: 10W~18W/Speed of roller: 18rpm~30rpm

2) Treatment procedure:


a) Clean skin with warm water;

b) Lay essential oil for conditioning on skin to be treated


3) Treatment manner for thigh:

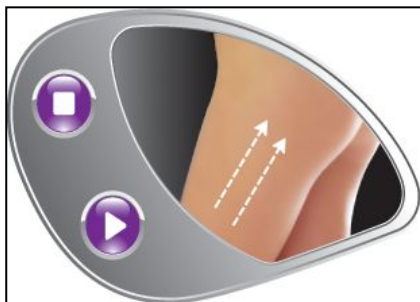

Picture 17

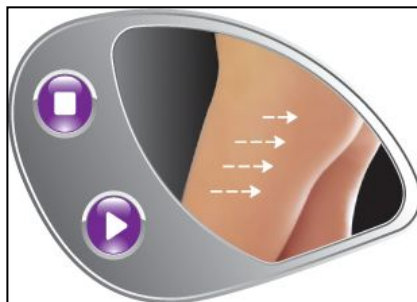


Picture 18

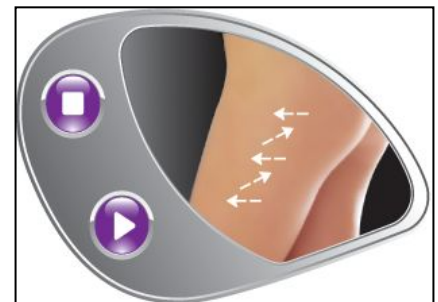


Picture 19




4) Treatment manner for calf:

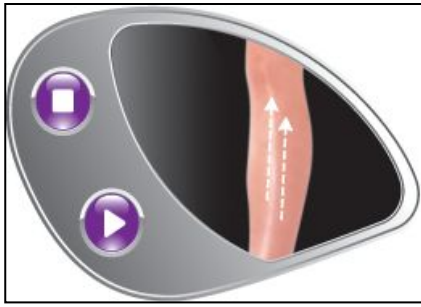

Picture 20

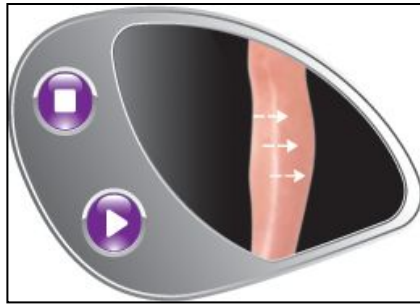

Picture 21

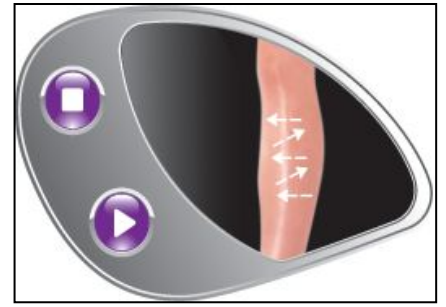

Picture 22

5) Note:

- Treatment begins from root;
- First treat on outside, then treat on inner part;
- The inner part is sensitive

### 5.7.5 Treatment on Upper Arm

1) Handpiece selection and parameters setting:

Use handpiece 2;

RF:  $35\text{J}/\text{cm}^3 \sim 55\text{J}/\text{cm}^3$ /Vacuum: 3-7/Pulse time: 2s~3s/ Infrared: 10W~18W/Speed of roller: 18rpm~30rpm

2) Treatment procedure:

- a) Clean skin with warm water;
- b) Lay essential oil for conditioning on skin to be treated

3) Treatment manner:



Picture 23

4) Note:

- First treat on outside, then treat on inner part;
- The inner part is sensitive

### 5.8 Notes after Treatment

- 1) Do not use cold water to clean treated area in 8 hours after treatment (except face);
- 2) Do not do massage on relative skin;
- 3) Avoid solarization in the first week after treatment;

- 4) Could lay milky skin care on treated area;
- 5) Do not eat raw, cold or spicy food in 8 hours after treatment

## 5.9 Suggestions about Operating Modes for Beauty Salon

- 1) Adopt card (course of treatment) to operate so that you could get persistent therapeutic effect ;
- 2) Could issue cards with different treatment items, such as physical therapy on body, relieve subhealthy, relieve ache and ankylosis of shoulder, neck, waist and belly.

## 5.10 Questions and Replies about V8plus

### 1. Customer: What's V8plus?

**Hairdresser:** V8plus is a kind of body contouring equipment. It adopts four kinds of technology: bipolar RF, near-infrared laser 940nm, endermologie, pluse mechanical tissue manipulation. The four kinds of technology could effectively promote metabolize of lipocyte and fiber cells so that we get result of body contouring and cellulite removal.

### 2. Customer: How to set course of treatment of the equipment?

**Hairdresser:** Body: twice per week, 30 minutes each treatment, one course of treatment contains ten treatments;  
Eye: once per week, 5 minutes each treatment, one course of treatment contains ten treatments

### 3. Customer: How long does one treatment take? When does the effect appear?

**Hairdresser:** Whole body : 45-60 minutes, local treatment:15 minutes  
One course of treatment contains ten treatments;

Distinct treatment effect would appear 1~2 months later. Tigh would decrease 1~5cm and waistline would decrease 2~6cm (normally circumference would decrease 2~3cm). The firming extent would get to 50-75%.

### 4. Customer: Is there fatalness when we use this equipment? What's the feeling of customer?

**Hairdresser:** Because this equipment is designed according to medical hairdressing technology, so it is safe to use it.

Customer would feel very comfortable during the treatment.

### 5. Customer: What's the condition of customer after treatment?

**Hairdresser:** After accepting a complete comfortable treatment, the customer's urinary output would increase. Since V8plus could promote metabolize of adipose tissue. The dissolved lipocytes are absorbed by lybph system.

### 6. Customer: Why does the customer have sense of hotness from inside?

**Hairdresser:** This equipment adopts bipolar RF technology. The working depth would be 5mm-15mm subcutaneous. Heat elimination is slower. If the operation is perfect, the sense of hotness would last half an hour to one hour.

**7. Customer : What's the notes after treatment?**

**Hairdresser:** Avoid intense sun bath.

**8. Customer: Shall customer during menses accept treatment?**

**Hairdresser:** Their feeling of pain, nerve is sensitive now and their resistance is weak. So they shall not accept treatment at this time.

**9. Customer: Any body could operate this equipment?**

**Hairdresser:** The operation of V8plus is very easy. So any person well trained could operate it.

**10. Customer: What's the effect after using this equipment?**

**Hairdresser:** 65% of the deeper tissue and the upper layers of the skin is removed or improved. Hip shrinks more than 3.2 centimeters. So many clinical cases proved that we could get body contouring effect soon after the very comfortable treatment.

**11. Customer: Why can this equipment treat deeper tissue and the upper layers of the skin?**

**Hairdresser :** Handpiece of this equipment adopts four kinds of technology: bipolar RF, near-infrared laser 940nm, endermologie, pluse mechanical tissue manipulation. When we use handpiece to help skin's movement, the deeper tissue and the upper layers of the skin is heated. Besides, The four kinds of technology could effectively promote metabolize of lipocyte and fiber cells so that we get result of body contouring and cellulite removal during comfortable relax.

**12. Customer: What's treatment principles of this equipment?**

**Hairdresser :** V8plus Shape&Smooth features revolutionary combination of Bipolar Radio Frequency, Near-infrared laser, Endermologie, pluse mechanical tissue manipulation. Subcutaneous tissue got heated and metabolism is promoted. As a result, we get the result of skin tightening, upper canthus lifting, black circle around eyes removal, body contouring and so on.

**13. Customer: What's the details about body contouring and skin tightening?**

**Hairdresser:**

A. Body curve:

Near-infrared laser 940nm could enhance flexibility of skin, rebirth collagen and spandex;

RF energy could heat subcutaneous fat to shrink lipocytes;

Endermologie add blood flux and promotes metabolize of fat.

B .To remove deeper tissue and the upper layers of the skin:

Trolley rubdown with endermologie could increase oxygen content of blood.

C .Face lifting:

Near-infrared laser 940nm could rebirth collagen and elastic fibers to improve wrinkle;

RF give heat to subcutaneous fat and drives lymph movements with negative pressure.

#### **14. Customer: What's the effect can this equipment give?**

##### **Hairdresser:**

- (1) Skin tightening, black circle around eyes and pouch removal;
- (2) Adiposis universalis, localized adiposity, wrinkle removal, lipolysis and body shaping ( arm, leg, shoulder, waist, twist and hip );
- (3) Renovation for post partum;
- (4) Improve deeper tissue and the upper layers of the skin, body contouring
- (5) Improve stretch marks;
- (6) Relax pain of muscle and whole body's physical therapy

### **5.11 Questions about Treatment on Face**

#### **1.Is there different effect according to different skin ?**

**Reply:** Normally it is.

Usually the effect of oily skin is more obvious than that of dry skin;

Fiber of dry skin shrinks badly. So make water replenishing to dry skin before and after treatment to heighten an effect.

#### **2.How long does the effect of once treatment on face last?**

**Reply:** Normally 3-7 days.

#### **3.Can customer with golden thread implantation accept the treatment?**

**Reply:** To try on small area and observe for a time

#### **4.Can customer with injectable fillers in face accept the treatment?**

**Reply:** These materials could not stand high temperature, so they shall not accept the treatment

#### **5.Can customer with silica gel fillers accept the treatment?**

**Reply:** No, they can't. Because silica is easy out of shape when it absorbs heat.

#### **6.Can customer with porcelain accept the treatment?**

**Reply:** It depends on the endurance capacity of customer.

**7.Can customer with metal tooth accept the treatment?**

**Reply:** It depends on the endurance capacity of customer;

Many people with strong endurance capacity could accept it when you put cotton piece on treated area.

**8.Can customer with allergic skin accept the treatment?**

**Reply:** They shall not accept the treatment.

**9.Why can't people with facial neuritis accept the treatment?**

**Reply:** Such people could not stand any simulation.

**10.Can customer accept treatment on face with this equipment and IPL treatment at the same time?**

**Reply:** It's better to accept treatment with this equipment 1 or 2 days before IPL treatment or 7days after IPL treatment.

**11.When can customer accept treatment after wound heals up?**

**Reply:** It depends on the depth of wound: normally one to three months after knife edge on belly; Though the epidermal wound heals up, the deeper muscle still needs a time to heal up.

**5.12 Questions about Treatment on Eye**

**1.What's feeling during treatment on eye?**

**Reply:** The eye has sense of hot. This sense would disappear in 20 minutes.

**2.What's the result after treatment on eye?**

**Reply:** Usually the black circle around eyes, wrinkle of canthus are improved. Canthus is lifted. Few people have mild edema. This is normal situation and it disappears in 2-3 days.

**3.What's the effect after treating congenital black circle around eyes?**

**Reply:** The effect is not obvious. It needs many times' treatment.

**4.What's the effect after treating pouch?**

**Reply:** Pouch contains fatty, edematous, slack ones. The effect of fatty pouch is more slowly and effect of edematous pouch is the fastest and most obvious. The obvious result appears after one to three times' treatment.

**5.Can we do treatment on wrinkle on upper eyelid?**

**Reply:** Yes, we can. This is the biggest advantage of this equipment. What's more, the effect is very good.