

HE-3000

HAND EDGER

User's Manual




LUXVISION

Notification

Dear Users,

Thank you for your purchase of HE 3000 Hand Edger Manual. Please take time to read our user's manual carefully before use.

This guarantees you to make full use of this unit and prolongs the operation life of this unit.

Precautions

If you have detected abnormal heat, smoke, noise or smell, immediately stop using the product.

In the event of an abnormality, turn off the power and disconnect the power plug from the power socket. Continuing to use the product may result in electric shock or fire.

Observe the instructions given below regarding the power cable:

- Be sure to use the supplied or specified power cable.
- Do not modify, forcibly bend, kink or pull the power cable.
- When disconnecting the power cable from the AC outlet, be sure to hold the cable by the plug.
Pulling the cable may cause wire breakage or short circuit, resulting in fire or electric shock.
- Do not connect or disconnect the plug of the power cable to/from the AC outlet using wet hands.
Doing so may result in electric shock.
- Do not touch the product with wet hands while the power cable is connected to the AC outlet.
Doing so may result in electric shock.
- If the product will not be used for a long period, disconnect the power cable from the power source. Leaving the cable connected to the power socket for a prolonged period will consume electricity and may result in heating.

Content

| | |
|----------------------------------|---|
| 1. Name of Parts | 4 |
| 2. Accessories | 4 |
| 3. Operation | 5 |
| 4. Precaution | 5 |
| 5. Common trouble Shooting | 5 |
| 6. Specifications | 6 |

1. Name of Parts

- | | | | |
|----|---|-----|------------------|
| 1. | Water can | 6. | Tools table |
| 2. | Water valve | 7. | Working lamp |
| 3. | Adjustable valve of water level | 8. | Motor switch |
| 4. | Three-in-one combinative grinding wheel | 9. | Lamp switch |
| 5. | Sponge | 10. | Low hose adaptor |

2. Accesories

1. one hose
2. one lamp
3. one piece of sponge
4. one oilstone stick



3. Operation

Open the carton and take out the foamed plastic cover. Then take the machine out carefully and place it on a big enough and quite stable desk. Please check the accessories, connect the exhaust hose, pour clean water into water can, adjust the adjustable valve of water level, open the water valve, turn on the working lamp and the motor switch. Then you can use it. This machine has one set “Three-in-one combinative diamond grinding wheel”, i.e. the left is rough grinding (180#), the middle is fine grinding (280#) and the right is V-fine grinding (400#). To get the best grinding result, you can select the different grinding wheel according to your requirements.

4. Precaution

To avoid blocking up the exhaust hose, please pour clean water into the water can. When the electric power can't reach the intensity demanded (voltage and frequency), don't use the machine. Don't use the machine for any other purposes except hand-grinding the edge of lenses.

5. Common Trouble Shooting

| Problem | Solution |
|---|---|
| 1. Motor doesn't rotate and the working lamp doesn't light up. | Check the power supply and see if the plug is connected with the socket and the voltage is stable. check the working lamp and see if it's useful. |
| 2. No water supply | Check the water level of the water can and see if the water valve is opened. |
| 3. Grinding fatigue | Use the oilstone stick to condition the wheel regularly. |
| 4. Water supply is available, but no water from the exhaust hose. | Open the machine, clear the waste thorough. |

6. Specifications

| | Dimensions (L*W*H) | Net.Weight |
|----------------------|--------------------|------------------------------------|
| Specification | 395*245*295mm | 7.8kg |
| | Timer | Power Requirement |
| | 1-5min | AC KXM20V/60HZ AC 200-240V/50Hz |

LUXVISION is not responsible or liable for indirect, special or consequential damages arising out of or in connection with the use or performance of the product or damages with respect to any economic loss, loss of property, loss of revenues or profits, loss of enjoyment or use, costs of removal or installation or other consequential damages of whatsoever nature. Some states do not allow the exclusion or limitation of incidental or consequential damages. Accordingly, the above limitation may not apply to you.

Every effort has been made to ensure the accuracy of this manual. However, LUXVISION, makes no warranties with respect to the documentation and disclaims any implied warranties of merchantability and fitness for a particular purpose. LUXVISION, Inc. shall not be liable for any errors or for incidental or consequential damages in connection with the furnishing, performance, or use of this manual or the examples herein. The information in this document is subject to change without notice.