

Features

- 100% Digital
- Multi-Memory
- Tele-Coil
- Volume Control
- Adaptive Noise Reduction
- Adaptive Feedback Reduction
- Dual Directional Microphones
- Tinnitus Masker
- Turn-on Delay
- Low Battery Indicator
- Multi-Memory Indicator
- Replaceable Thin-Tube

* Not all features may apply to your instrument.



P E R S O N A[®]
M E D I C A L



407 339 2422



170 N. Cypress Way • Casselberry, FL 32707



Info@PersonaMedical.com



PersonaMedical.com

UG / OTE2 / 1 © 14 Persona Medical

CE 0120

P E R S O N A[®]
M E D I C A L



Welcome back to the world of hearing™

Welcome back to the world of sound!™

A belief that in order to succeed, you must have a certain passion and desire to give your every endeavor your very best. For over 40 years, Persona Medical has forged this very commitment to excellence into every hearing instrument we design.


Open Fit Hearing Instruments




A rare combination of state of the art technology and custom craftsmanship brings you the best instruments available today. Currently, our products are distributed all over the world. We thank you for allowing us to assist with your hearing needs.

These instructions along with the guidance from your Hearing Healthcare Professional will help you attain the maximum benefit from your hearing instrument(s).


Cleaning & Changing Tubes

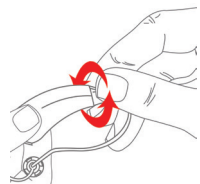


1. To remove tubes gently rotate the sound tubing holding the instrument firmly.

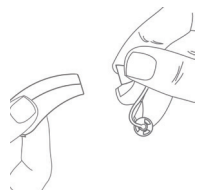

2. Pull sound tubing from the instrument by pulling directly opposite from the body of the Evok625.

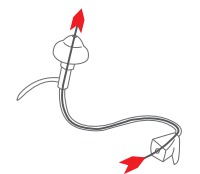

3. Insert the cleaning tool through the nose cone end. Thread all the way through the tubing & earbud.

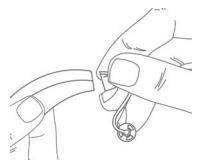

4. To reassemble or change sound tubing, simply line up and push tubing until it snaps into place.

Powering On/Off Your Evok625tm

Evok625 uses a hearing aid size 312 battery.

TO TURN ON - With your finger, gently open the door. Insert battery and close the door. Your Evok625 is equipped with a power on delay, you will be given several seconds to seat the device comfortably in your ear before it turns on.

When placing the battery in your device, position the battery with the positive (+) flat side up and insert it into the battery compartment section of the battery door. If the battery is inserted incorrectly or upside down, the door will not close. **Do not force the door to close.**

Before insertion, please clean your ear and make sure the battery door is closed properly.

CAUTION: Batteries are dangerous if swallowed! Keep batteries out of the reach of children. Never put batteries in your mouth for any reason they are easily swallowed. If a battery has been swallowed, seek immediate medical attention. For more information call the National Battery Hotline at 202-625-3333.

NOTE: Some irritation may occur due to becoming accustomed to wearing a new instrument. If swelling or redness occurs, discontinue use and visit your Doctor.

Inserting Your Evok625^m

- Hold the tubing with your thumb and index finger close to the ear dome and place the instrument over the top of your ear. (Fig. A).
- Gently insert the ear dome into your ear canal. (Fig. B)
- Move the retention lock into position by lifting and placing it into the bowl of your ear. The retention lock may be cut off with scissors if it bothers you or is difficult to use. (Fig. C)

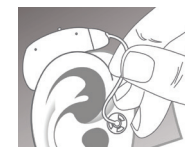


Fig. A



Fig. B

VOLUME ADJUSTMENT (Fig. 1 & 2)

Use your thumb and middle finger to hold each end. Use your pointer finger to toggle the volume control up/down (Fig. 1). The volume control rocks back and forth, always returning to the center position. EACH SHORT PRESS (1 sec.) will raise or lower the volume slightly. Pushing up increases the volume. (Fig. 2)

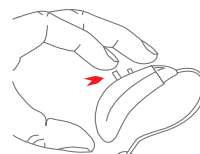


Fig. 1

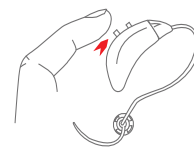


Fig. 2

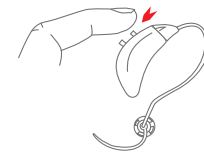


Fig. 3

Volume Adjustment (cont'd)

Pushing down lowers the volume. There are a maximum of ten volume settings. Every time you turn on the Evok625, you may need to adjust the volume to your preferred listening level if different than your prescribed setting..

TO REMOVE - Simply grasp the tubing close to the earbud or earmold and remove from ear canal. Lift instrument from the top of the ear.

TO TURN OFF - To turn your Evok625 off, simply open the battery door completely. Store in a safe dry place without the battery inserted.

PROGRAMS / MEMORIES (SEPARATE CONTROL)

After inserting the battery, the Evok625 starts in program one. Pushing the short button (most forward) will change the memory. (Fig. 3) The number of beeps correspond to the program. The programs cycle in sequence 1,2,3,4,1,2,3,4 etc....

Typical Programs:

- 1 - Speech - everyday use
- 2 - Noise - Restaurant, meetings, etc.
- 3 - **Optional Telephone**

Your hearing professional may choose a different configuration for your personal needs.