


PV 6542 AMP

HIDDEN NEEDLE COVER FOR USE WITH HUMATROPE® CARTRIDGES

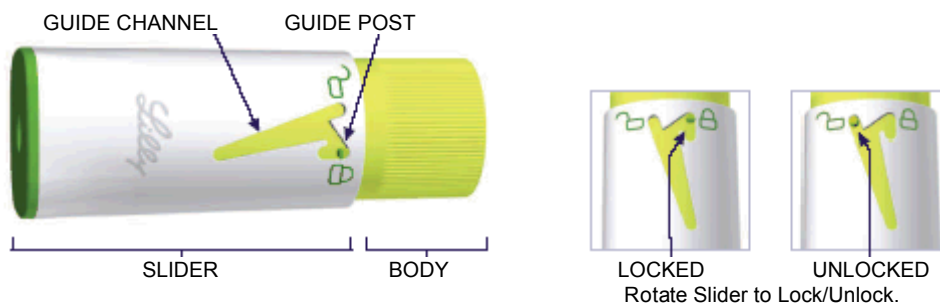


INSTRUCTIONS FOR USE ATTACHING AND USING THE HIDDEN NEEDLE COVER

The Hidden Needle Cover allows you the option to inject with the Needle hidden from view. Be sure the Pen is set up per the Pen user manual instructions.

- 1 Be sure the Pen Cap is removed. Attach a new Needle to the Cartridge and remove the Outer Needle Shield and Inner Needle Shield. Refer to the Pen user manual for detailed instructions on set-up. Follow your healthcare professional's instructions on the safe handling of needles.

- 2 Confirm the Hidden Needle Cover is in the locked position. If it is not locked, turn the slider so that the Guide post is in the Guide Channel, as shown. Do not push the Slider. The Hidden Needle Cover should now be in the locked position. This is important to avoid needle sticks in the next step.

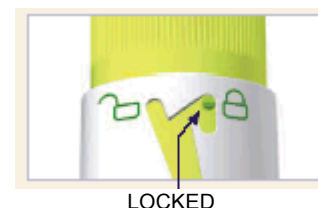

- 3 Insert the Pen with attached Cartridge and Needle straight into the locked Hidden Needle Cover until it stops. In case the Hidden Needle Cover did not get locked, do not block the opening of the Hidden Needle Cover while performing this step. Dial the Pen to the usual dose. Refer to the Pen user manual for detailed instructions on dialing the dose.




- 4 Unlock the Hidden Needle Cover by rotating the Slider clockwise until the Guide Post is located within the Guide Channel, as shown. To avoid Needle sticks, be careful not to push the Slider in after it is unlocked.



- 5** Inject the Humatrope dose as instructed by your healthcare professional. First insert the Needle at a 90° angle by pushing into the skin. The Slider will move over the Hidden Needle Cover Body as the Needle is inserted.

Then press the Injection Button. The Hidden Needle Cover will automatically lock as you remove the Needle from the skin.

Refer to the Pen user manual for detailed instructions on injections.

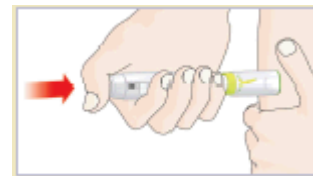

- 6** Confirm that the Hidden Needle Cover has returned to the locked position. Carefully pull the Hidden Needle Cover straight off and remove it from the Pen.

Remove and dispose of the Needle in a puncture-proof container. Follow your healthcare professional's instructions on how to do this safely.

Replace the Pen Cap.

Soiled parts can be cleaned with a damp cloth. DO NOT USE alcohol or other cleaning agents.



LOCKED

Made for Eli Lilly and Company
Pharmaceutical Delivery Systems
Lilly Corporate Center
Indianapolis, IN 46285, USA

CE 0344

Humatrope® is a registered trademark of Eli Lilly and Company.

© 2009, 2012, Eli Lilly and Company. All rights reserved.

Literature revised August 2, 2012

PV 6542 AMP

Lilly