



Drive Medical Ltd  
Ainley's Industrial Estate,  
Elland, UK HX5 9JP.

Doc Code (WSA006RT Iss 07/14)

## WALKING STICK ICE GRIP

The Walking Stick Ice Grip is suitable for most walking sticks and crutches with tubing or ferrule sizes between 19mm & 22mm diameter. It may help users when using their walking aids in snow or icy weather conditions.

The Ice Grip is covered by a 6-month comprehensive warranty covering design and manufacturing defects.

This does not include wear and tear, accidental damage or misuse. In the event of a defect, Drive Medical will supply replacement Ice Grips.

This warranty is supplied to the end user by Drive Medical Ltd and does not affect your statutory rights.

The Ice Grip comes with 2 pairs of screws.

The shorter screws are to install the Ice Grips on narrow diameter tubing.

The longer screws are to install the Ice Grips on wide diameter tubing.




To Install the Ice Grip, remove the two screws with a screw driver and position the Ice Grip around the bottom of the cane (as shown left).

Ensure the spikes are located at the bottom of the ferrule.

Secure the Ice Grip by attaching either the 2 short screws (for narrow tube) or the 2 long screws (for wide tube).

To flip up the Ice Grip when not in use, push the two white ends together and lift the Ice Grip up until it locks in position.

To flip down the Ice Grip to use, push the two white ends together then lower the Ice Grip until it locks in position.

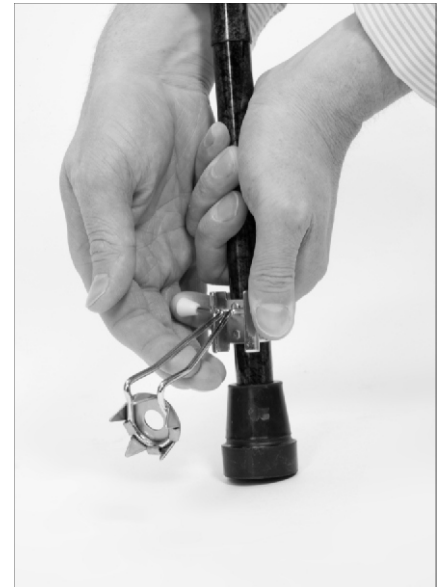

*Ice Grip shown in the UP position*

The Ice Grip can be locked in the up position when not in use (see prior instructions on how to do this).

**CAUTION:**  
**SHARP SPIKES**  
The Ice Grip features sharp spikes which could cause injury if the spikes come in to contact with the user or another person.

Ensure that the spike do not touch user or any other person.