

WARNING

1. Do not use sling if a patient suffers from back injury.
2. Please understand hoist operating instructions before using.
3. Check sling before use for signs of wear.


Hammock Sling Lifting Procedures

KH409N1—20-40kg

KH409N2—40-150kg

KH409N3—150-250kg

STEP 1: When the patient is lying in bed, roll them onto their side and place the sling under them similar to fitting a draw sheet, ie: roll the patient onto their side concertina the sling under the patients back. Align (A) with the base of the patients spine, then roll the patient onto the sling and centralise. If the patient is sitting up in bed or in a chair, lean the patient forward and slide the sling down the back of the bed/ chair. Try to tuck (A) as low as possible under the patient bottom.

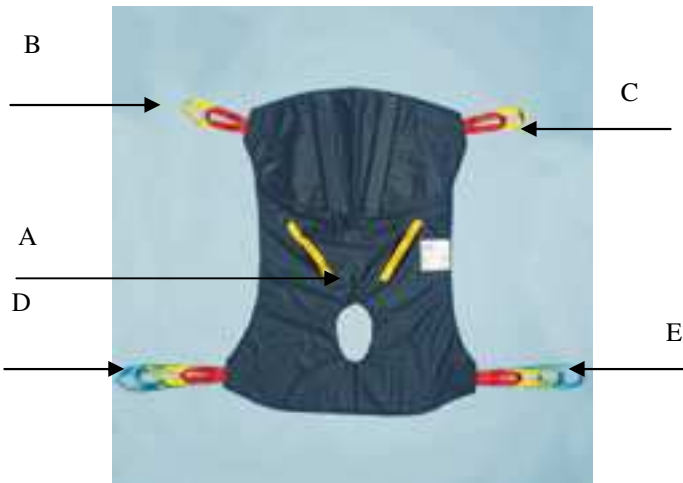

STEP 2: Bring the lifter into position, ensure hoist legs are gated and the hoist yoke is approximately over the centre of the patient.

DO NOT engage the brakes on the castors

STEP 3: Attach straps (B) & (C) to yoke first, then the leg straps (D) & (E)



Patients can be positioned 2 ways.

1. Sitting upright—(B) & (C) use short straps, (D) & (E) use long straps.
2. Reclined lift—(B) & (C) use long straps, (D) & (E) use short straps.



Note: 1. The operator/carers can steady the patient by holding the back handles.

Note: 2. To prevent patient discomfort ensure leg/thigh sections are not creased/folded under patient thighs prior to lifting

STEP 4. Start with the lifting process using the slow speed.
CAUTION: Hoist may move towards the patient when boom is raised
Position the patient over chair, bed or toilet and begin lowering. Make sure the patient feels comfortable and secure in the sling.

CARE INSTRUCTIONS

1. Sling Care

Protect from direct sunlight.

Do not puncture with any sharp objects.

Repair any tear immediately.

Do not lift a patient if there are any sling defects.

Washing and drying temperature must not exceed 60 degrees.

2. Spot Cleaning

The sling is machine washable.

Treat spills and stains as soon as possible.

Gently scrape away any soils or mop away liquid from surface of fabric with damp cloth.

Do not scrub with stiff brush.


Dry in shade, away from direct heat.

Dry thoroughly before use.



4 Halley Rd
Balcatta WA 6021
Tel: 08 9240 2944
Fax: 08 9240 1469
Email: info@kerryequipment.com.au
Website: www.kerryequipment.com.au

PATIENT RECORDS FOR MOUNTING POINTS USING KH409N1, KH409N2, KH409N3

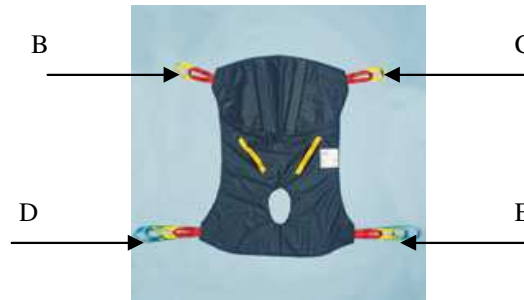[illegible]

USERS GUIDE

KH409N

HAMMOCK SLING



FEATURES

Mesh/netting option KH409NE.
Full body and head support.
Ideal for single or double amputees.
Commode aperture.
Positioning straps.
Colour coded attachment straps.
Machine washable.

