

## Warranty Details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

This does not affect your statutory rights.



### Drive Medical Ltd

Ainley's Industrial Estate, Elland,  
West Yorkshire, GB HX5 9JP  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

Doc Code: 12455 IFU (Issued 08/15)


## All Purpose Stool



[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

# Assembly

1. Secure part #1 ( Front frame) to part #2 (Back frame). Insert part # 3 (Foot pieces) into the Front and Back frame. Insure when installing part #3 (Foot pieces) into the frame, both buttons protrude through the notches in the Frame.
2. Insert part #4 (Backrest) into the Back frame and insert part #5 (Handle) into the Front/Back frame.
3. Use parts #6, #7 and #8 to tighten the Front/ Back frame.
4. Put part #9 (Seat) and insert with part #10 (Screws) to tighten the Seat.

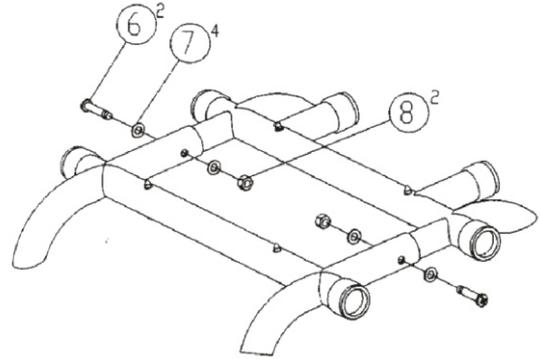

## Operation

Always set the legs at equal heights and at a height that is comfortable for the user. When the leg height is set correctly, the seat should be tilted slightly so the user is perched on the seat with their feet on the floor.

To adjust the leg height, press the push button on the stool leg and adjust to the correct height. Ensure that the leg locks in position with an audible click and is secure.

Do not exceed the weight limit of 21 stone (135kg)

Only use the stool on level surfaces only.

Clean the frame and upholstery with a damp cloth and mild detergent or disinfectant. Allow to dry before use.

