





JANAK
HEALTHCARE

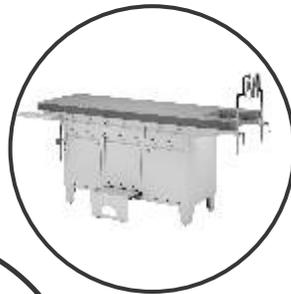

Examination Couches & Tables


User's Manual

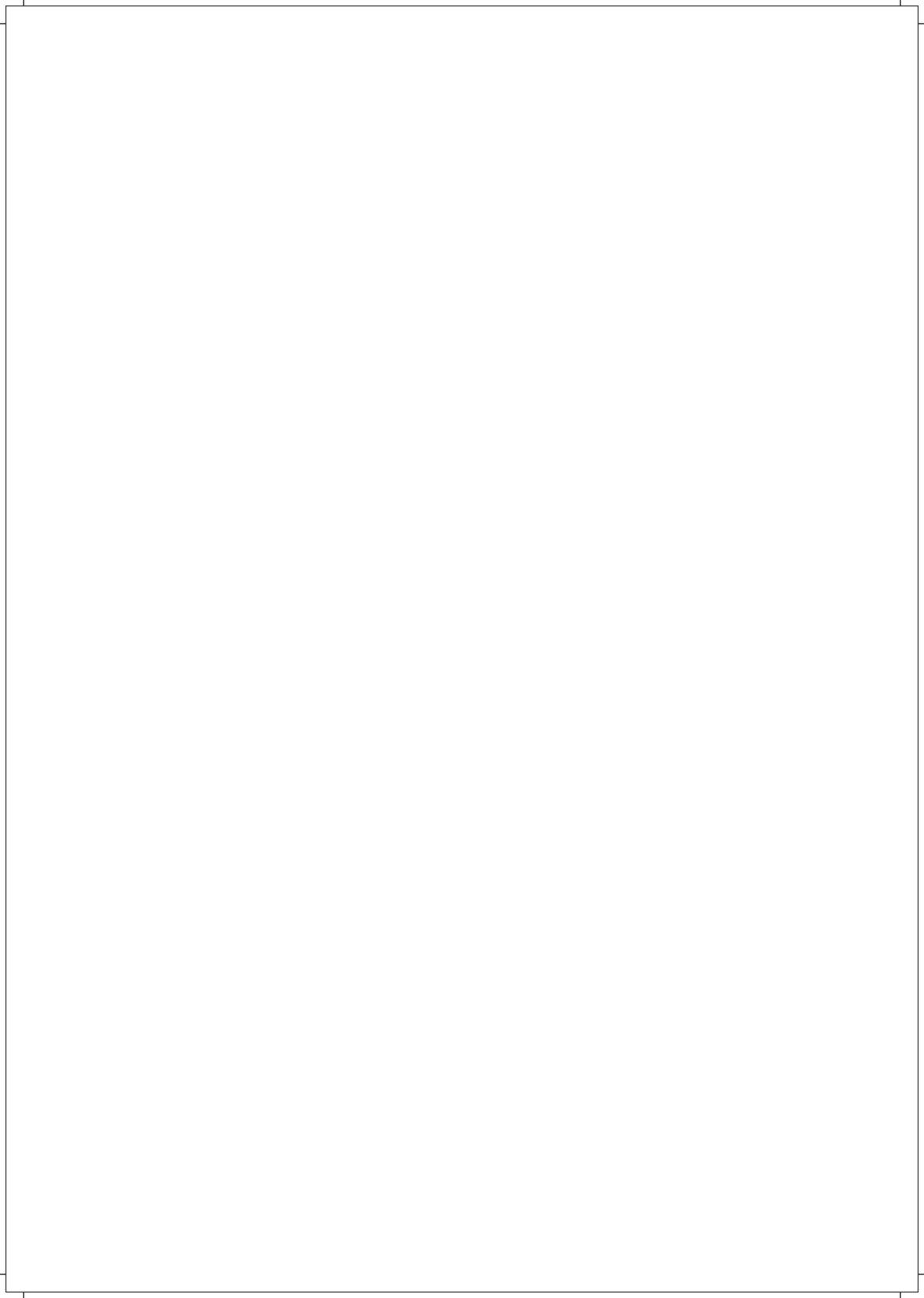

Index



1. Introduction	1
2. About this Product	1
3. Definitions	3
4. Safety	3
5. Environmental Specifications	5
6. Installation	6
7. Optical Accessories	
8. Operation	13
9. Maintenance & Cleaning	13
10. Warranty & Service	21

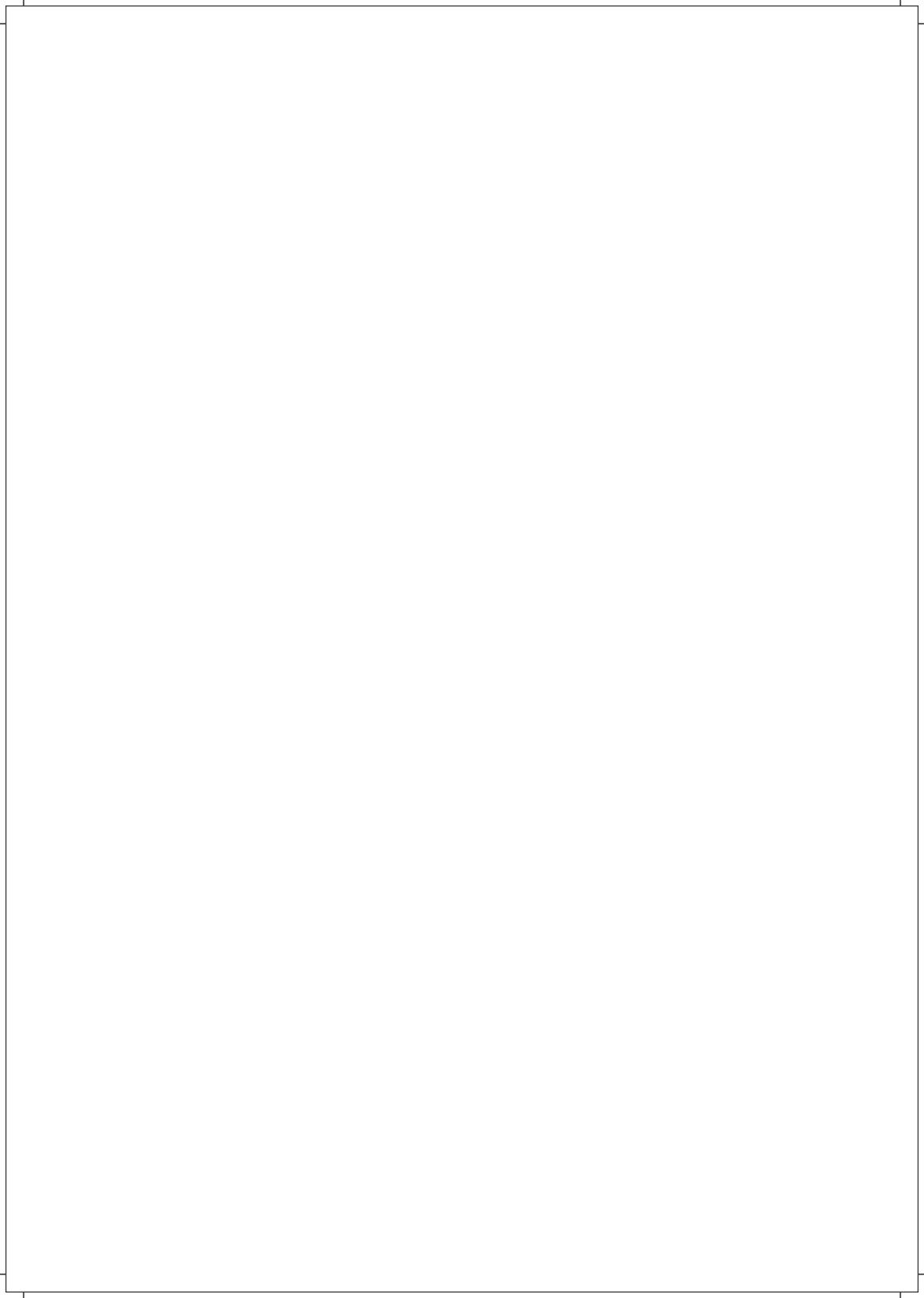



INTRODUCTION

About this User Manual

This User Manual contains instructions for the proper and safe use of Examination Couches & Tables, and for patient & user safety and comfort, which has to be strictly followed.

This User Manual has to be read and followed by every person using or handling these Examination Couches & Tables. In case of any query, kindly contact Janak Healthcare Pvt. Ltd.

ABOUT THIS PRODUCT

1803C

This Health Check Examination Couch is suitable for use in OPD and Exam Room, for carrying out general examinations like ENT, Ophthalmic, General Surgeon, Orthopedic Surgeon, etc (except Gyneac examination). The Headrest is adjustable for comfortable positioning of patients. The BP Apparatus Tray is adjustable for convenient positioning of BP Apparatus. This comes with Drawers and door for storage, and has an inbuilt Sliding Step Stool for easy mounting / dismounting of patients when required

1803M

This Health Check Single Top Examination Couch with 'U' cut is suitable for use in Gynaec OPD and Exam Room. The BP Apparatus Tray is adjustable for convenient positioning of BP Apparatus. This comes with Drawers and door for storage, and has an inbuilt Sliding Step Stool for easy mounting / dismounting of patients. U-Cut is provided for Lithotomic to help use of speculum and forceps. Lithotomy Rod covered with soft rubber is provided for lithotomic position.

1803L

This Health Check Single Top Examination Couch is suitable for use in OPD and Exam Room, for carrying out general examinations like ENT, Ophthalmic, General Surgeon, Orthopedic Surgeon, etc (except Gyneac examination). The BP Apparatus Tray is adjustable for convenient positioning of BP Apparatus. This comes with Drawers and door for storage, and has an inbuilt Sliding Step Stool for easy mounting / dismounting of patients when required

1807

This Health Check Examination Table with Upholstered Top is suitable for use in OPD and Exam Room, for carrying out general examinations like ENT, Ophthalmic, General Surgeon, Orthopedic Surgeon, etc (except Gyneac examination). The Headrest is adjustable for comfortable positioning of patients.

1808

This Health Check Examination cum Gynaec Table with upholstered Top & 'U' cut is suitable for use in Gynaec OPD and Gynaec Exam Room. The Headrest is adjustable for comfortable positioning of patients. U-Cut is provided for Lithotomic to help use of speculum and forceps. Lithotomy Rod covered with soft rubber for is provided lithotomic position. S.S. Bowl is provided for collection of Placenta. (Not to be used for storing other medical waste for safety and hygiene of User)

Examination Couches & Tables Features

1803C, 1803L - Please refer product catalogue **1803C-1803L** for salient features of these products.

1807 - Please refer product catalogue 1807 for salient features of these products.

1803M, 1808 - Please refer product catalogue **1803M-1808** for salient features of these products.

Technical Data

1803C, 1803L - Please refer product catalogue **1803C-1803L** for Technical Data of these products.

1807 - Please refer product catalogue 1807 for Technical Data of these products.

1803M, 1808 - Please refer product catalogue **1803M-1808** for Technical Data of these products.

Operator

Operator is the person appointed by the owner (i.e. clinic, hospital management, etc.) who will be responsible for the safe operation of these Examination Couches & Tables.

User

User is a person who is capable of assessing clinical condition of patient e.g. specialist medical personnel, doctors, nursing staff, attendants etc. On the basis of their training experience or through the instructions, they are entitled to use these Examination Couches & Tables on their own responsibility or to carry out work on it and are able to recognize and control possible risks to the patient as well as the Examination Couches & Tables.

Patient

The patient is a person who is ill, infirm, disabled or in need of medical assistance, who needs to be examined.

SAFETY

In this User Manual the following safety symbols are used



Danger: This symbol will appear wherever safety instructions are designed to protect people from physical harm. The symbol stands for imminent danger of death or serious injury.



Caution: This symbol will appear where situations are described, which might be dangerous, and which might inflict slight injuries.



Advice: This symbol will appear in front of additional helpful pieces of advice.

- A dot in front of the text implies the action that is required to be taken.

Safety Instructions

- Always use original accessories that have been designed and approved for use with the Examination Couch and Tables
- Ensure that the Examination Couches are assembled and installed in accordance with the instructions given in this User Manual (wherever required).
- Restrict child visitors without proper attendants. Supervision is necessary to avoid any mishap due to unintended movement of the Examination Couches & Tables.
- Keep some gap between walls and Examination Couches & Tables while in stationary position and also while moving from one place to another.
- Keep the User Manual handy with the Operator/User for ready reference.



Patient should be advised not to sit on edge of any section when it is in extended full length position.

ENVIRONMENTAL SPECIFICATIONS

Usage, Storage and Transport

Temperature	-10°C to 50°C
Relative Humidity	0% to 95% non condensing
Atmospheric Pressure	800 to 1060 hPa
Altitude	< 2000m

Installation and Operation of 1803C



1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each 1803C is supplied with the following components as per packing list:

A. Couch with body section lockers & drawers	1 No.
B. Headrest section	1 No.
C. BP Apparatus Holder with fixing bracket & star knob	1 No.
D. Hardwares	1 Pkt.



Notify if any damage has occurred during transportation either to the authorized agent or directly to Janak Health Care Pvt. Ltd.



Installation of the Examination Couches & Tables should be done only by Janak trained technicians.



Dispose the packing material waste in accordance with the local waste disposal regulations prevailing

Installation of 1803C



1. Place the headrest section on the main frame & fix it with the bolts
2. Fix the brackets with the bolts
3. Insert the BP tray holder Rod into the bracket & tighten the same with the help of star knob

Operation

Mechanical Adjustment:

- A. **Raising & Lowering Headrest:** Headrest section is adjustable on Gas Spring. Different positions can be achieved by pulling the headrest adjustment rod towards the headrest and moving the same to its desired position. Releasing the rod locks the headrest in its position.
- B. The BP Apparatus Tray can be adjusted to the desired position with the help of Star Knob
- C. The Sliding Step Stool can be pulled out for use & can be pushed back in.



Pushing the headrest adjustment rod in the downward direction (away from the headrest) can damage the headrest mechanism

Installation and Operation of 1803M




1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.


Each 1803M is supplied with the following components as per packing list:

- | | |
|---|--------|
| A. Couch with "U" Cut upholstery body section with drawers & lockers | 1 No. |
| B. Synthetic Rubber padded S.S. Lithotomy Rods with Rexine Ankle Straps | 2 Nos. |
| C. BP Apparatus Holder with fixing bracket & Star Knob | 1 No. |
| D. Hardwares | 1 Pkt. |

Installation

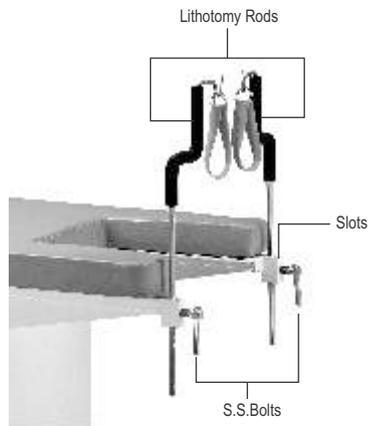

1. Insert the Lithotomy Rods into its Slots & tighten the same with the S.S. Bolts

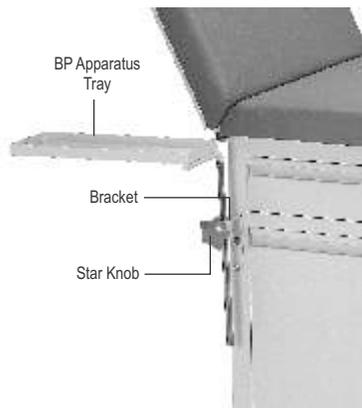

2. Insert the BP Tray Holder Rod into the bracket & tighten the same with the help of Star Knob

Operation

Mechanical Adjustment:

1. The Lithotomy Rods can be adjusted as per the desired height & can be locked with the help of S.S. Locks provided
2. The Sliding Step Stool can be pulled out for use & can be pushed back in.


Installation and Operation of 1803L



1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each 1803L is supplied with the following components as per packing list:

A. Couch with fixed upholstery top with drawers & lockers	1 No.
B. BP Tray Holder with fixing bracket & Star Knob	1 No.
C. Hardwares	1 Pkt.

Installation

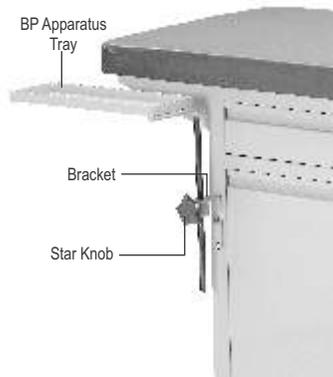

1. Insert the BP Tray Holder Rod into the bracket & tighten the same with the help of Star Knob

Operation

1. Sliding Step Stool can be pulled out for use & can be pushed back in.


Optional Accessories (For 1803C, 1803M & 1803L)

1. Paper Roll Holder (0175)



Paper roll holder to be fitted at head side of 1803L and 1803M, & at leg side of 1803C

Installation and Operation of 1807

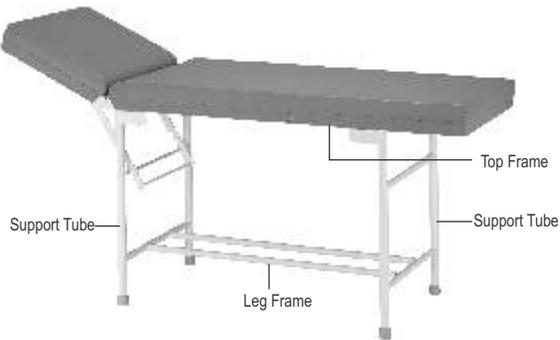

1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each 1807 is supplied with the following components as per packing list:


A. Top Frame with Ratchet flats	1 No.
B. Leg Frame	2 Nos.
C. Support Tubes	2 Nos.
D. Hardwares	1 Pkt.

Installation

1. Place the main frame upside down on the flat surface
2. Fix the support tubes in the welded sleeve
3. Bolt the leg frame after fixing the support tubes in the welded sleeve
4. Upright the table with the legs on the ground
5. Fix the Ratchet Rod with the bolts below the Headrest section


Operation

Mechanical Adjustment:

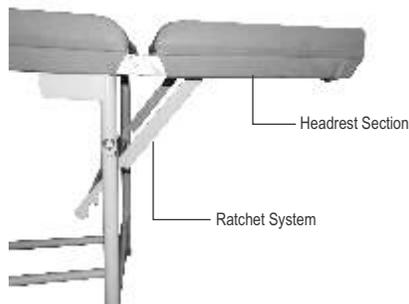

Raising & Lowering: Headrest section is adjustable on Ratchet system. Different positions can be achieved by adjusting headrest rod to suitable slot.

Installation and Operation of 1808

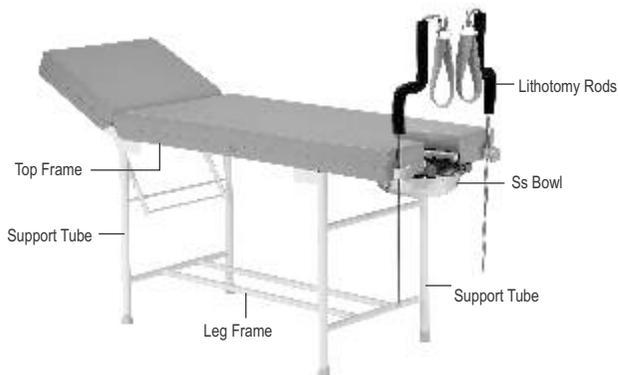

1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.

Each 1808 is supplied with the following components as per packing list:


A. Top Frame "U" Cut with Ratchet flats	1 No.
B. Leg Frame	2 Nos.
C. Support Tubes	2 Nos.
D. Synthetic Rubber padded S.S. Lithotomy Rods with Rexine ankle straps	2 Nos.
E. S.S. Bowl	1 No.
F. Hardwares	1 Pkt.

Installation

1. Place the main frame upside down on the flat surface
2. Fix the support tubes in the welded sleeve
3. Bolt the leg frame after fixing the support tubes in the welded sleeve
4. Upright the table with the legs on the ground
5. Fix the Ratchet Rod with the bolts below the Backrest section
6. Insert the Lithotomy Rods into its slots & tighten the same with the Star knob provided
7. Slide the S.S. Bowl at its location


Operation

Mechanical Adjustment:

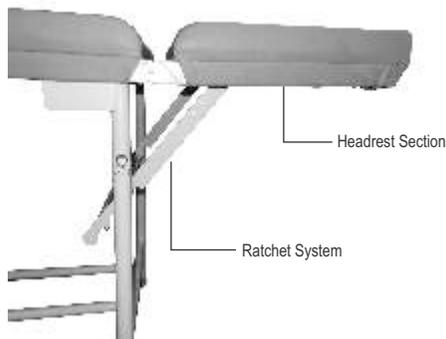


- A. Raising & Lowering: Headrest Section is adjustable on Ratchet System. Different positions can be achieved by adjusting backrest rod to suitable slot.

 Care should be taken to avoid fingers and hand getting entrapped between the ratchet and the ratchet rod in 1807 & 1808.

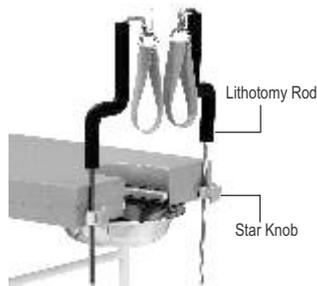

- B. The Lithotomy rods can be adjusted as per the desired height & can be locked with the help of Star knob provided.

Load Bearing Capacity (Product Specific)

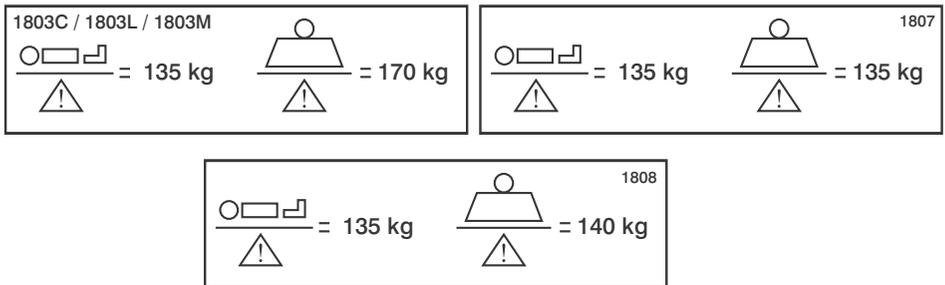

 Do not load the Examination Couches & Tables more than the safe working load mentioned above. Patient / user should not mount or sit or place any heavy load on the backrest section / leg section as it may cause toppling.

 With due consideration to environmental limitations, materials used to make the device are not affected by contact with any bio chemicals or bio waste. When used as per its intended purpose, all materials used in the product and accessories do not facilitate any breeding or growth of bio contaminants on their own.

MAINTENANCE & CLEANING

Maintenance

Each time before putting the Examination Couch & Table into service the user must check for -

- Lubrication of all the mechanical joints / pins / moving parts / tightening of the bolts are done properly, to avoid unnecessary play, sound and wear & tear of the Examination Couch & Table.
- Visual inspection for any mechanical damage (e.g. loose screws, etc.)
- If any damage or malfunction is observed, immediately report this matter to the operator.
- If the Examination Couch & Table is not in use, it should be stored in a dry and rodent free indoor environment on a flat surface with gradient not more than 5 degrees.
- If the Examination Couch & Table is not in use for long term, it is necessary to conduct a test to check for functioning and any mechanical damage that may have been caused.
- Depending on the usage, certain moving parts may require replacement due to wear & tear. We recommend timely replacement with original spares for safety and trouble free operation.



Mattress impermeability could be affected by needle sticks. Users should be instructed to avoid punctures caused by improper use of needle sticks. Failure to do so could result in cross – infection & patient injury. The upholstery should be regularly inspected for punctures, rips, tears or other such damage & replaced with suitable original mattress if required.



Smoking should be prohibited near the upholstery with / without patient, as this is a fire hazards, and mattress, sheets & pillows generally do not have flame retardant properties.

Cleaning & Disinfecting

- The Examination Couch & Table is not suitable for machine wash or steam cleaning.
- Do not use water jet for cleaning.
- The Examination Couch & Table cannot tolerate the influence of strong solvent, basic or alkaline liquids & caustic agents.
- The cleaning agents must not contain any substances which bring about any change in the structure or adhesive property of the plastic.
- The Examination Couch & Table should be properly cleaned and disinfected at regular intervals to avoid infection and stains.
- We recommend use of approved disinfectant and cleaning agents to clean the Examination Couch & Table. Take extra care to clean areas that may trap dust or dirt. All detachable accessories and its component can be cleaned separately.



Do not expose the Examination Couch & Table to excessive moisture as corrosion may cause personal injury or equipment damage.

Disposal:



Product and its defective parts especially elements made of plastic, steel, rubber material should be disposed off in accordance with the statutory rules and regulations prevailing for such disposal. The user is advised to contact Janak Healthcare Pvt. Ltd. for further assistance.

Routine Inspection Check List: Inspection By The User

(Recommended Frequency: As per usage and hospital policy but not later than one month)

Checks	Observation	OK	Not OK	Description of faults
Visual check of the mechanical components				
Main Body	Damage, Deformation			
Upholstery	Damage, Torn, Soiled			
Knobs, Screws and rivets	Loosened, Missing			
Performance check of all mechanical components				
Headrest	Smooth Movement			
Detachable Lithotomy Rod	Secure retention			
BP Apparatus Tray	Secure Retention			
Sliding Step Stool	Smooth Movement Drawer /			
Smooth Locking / unlocking	Door Lock			
Sign	Result of inspection		Date	

Troubleshooting Guide

Problem	Possible Causes	Remedy
Couch / Table shaking or uneven	<ul style="list-style-type: none"> • Check the Floor level & legs 	<ul style="list-style-type: none"> • Adjust / replace Rubber Shoes (1807/1808)
		<ul style="list-style-type: none"> • Adjust / replace Nylon Bush (1803C / 1803L/ 1803M)
Headrest not working (1803C / 1803L / 1803M)	<ul style="list-style-type: none"> • Check Alignment of Gas Spring • Gas Spring defective / damaged 	<ul style="list-style-type: none"> • Readjust the Gas Spring Bracket & Cover • Replace the Gas Spring
Headrest not working (1807 / 1808)	<ul style="list-style-type: none"> • Ratchet Mechanism is tight • Rivets loose 	<ul style="list-style-type: none"> • Adjust the Ratchet Mechanism • Tighten / replace rivets.



WARRANTY & SERVICE

Janak Healthcare Pvt. Ltd. warrants, solely to the end users of this product that for a period specified in the warranty certificate, such product shall perform as per its intended use. The foregoing warranties shall not apply if the Product has been subjected to: (a) physical abuse, misuse, abnormal use, or use not consistent with Janak Healthcare Pvt. Ltd.'s published directions, (b) any modification by anyone other than Janak Healthcare Pvt. Ltd. or third parties designated by Janak Healthcare Pvt. Ltd.; or (c) fraud, tampering, negligence or accidents, depending upon the nature of the products.

Limitation Of Liability: In no event shall Janak Healthcare Pvt. Ltd. be liable or obligated to the end user in any manner for any special, non-compensatory, consequential, indirect, incidental, statutory or punitive damages of any kind, including, without limitation, for injury or loss of life, lost profits, lost sales, lost revenue, loss of any software or data, or loss of use of any hardware, software or data, regardless of the form of action, whether in contract, tort, negligence, strict product liability, or otherwise, even if Janak Healthcare Pvt. Ltd. has been informed of or is aware of the possibility of any such damages in advance. Janak Healthcare Pvt. Ltd.'s total aggregate liability under this agreement and in relation to anything which Janak Healthcare Pvt. Ltd. has done or not done in connection with this agreement shall be limited to the price of the products purchased and paid for by the end user during the period in which such liability arises. The limitations set forth above shall be deemed to apply to the maximum extent permitted by applicable law and not with standing the failure of the essential purpose of any limited remedies.

In order to maintain functional safety and right to claim under the warranty only original replacements parts may be used. For ordering replacement parts, customer service request and further question, please contact our technical support department.

Disclaimer

Every effort has been made to ensure that the information in this user manual is valid. Janak Healthcare Pvt. Ltd. reserves the right to change the information in this document without prior notice.

MANUFACTURER'S ADDRESS

Corporate Office:

Janak Healthcare Pvt. Ltd.

Unit No. 12, 1st Floor, Kalpataru Point, Opp. Cinemax Theatre, Sion Circle, Sion (E), Mumbai 400022.

Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100

www.janakhealthcare.com

Registered Office:


Janak Healthcare Pvt. Ltd.

Janak House, Opp. Indian Oil Corp. Depot, Sheikh Misry Road, Wadala (E), Mumbai 400037, India.

Tel: +91 22 2413 0407/2418 0637 | Fax: +91 22 2413 9870

Toll Free No.: 1800 22 8020 | Email: technicalsupport@janakhealthcare.com



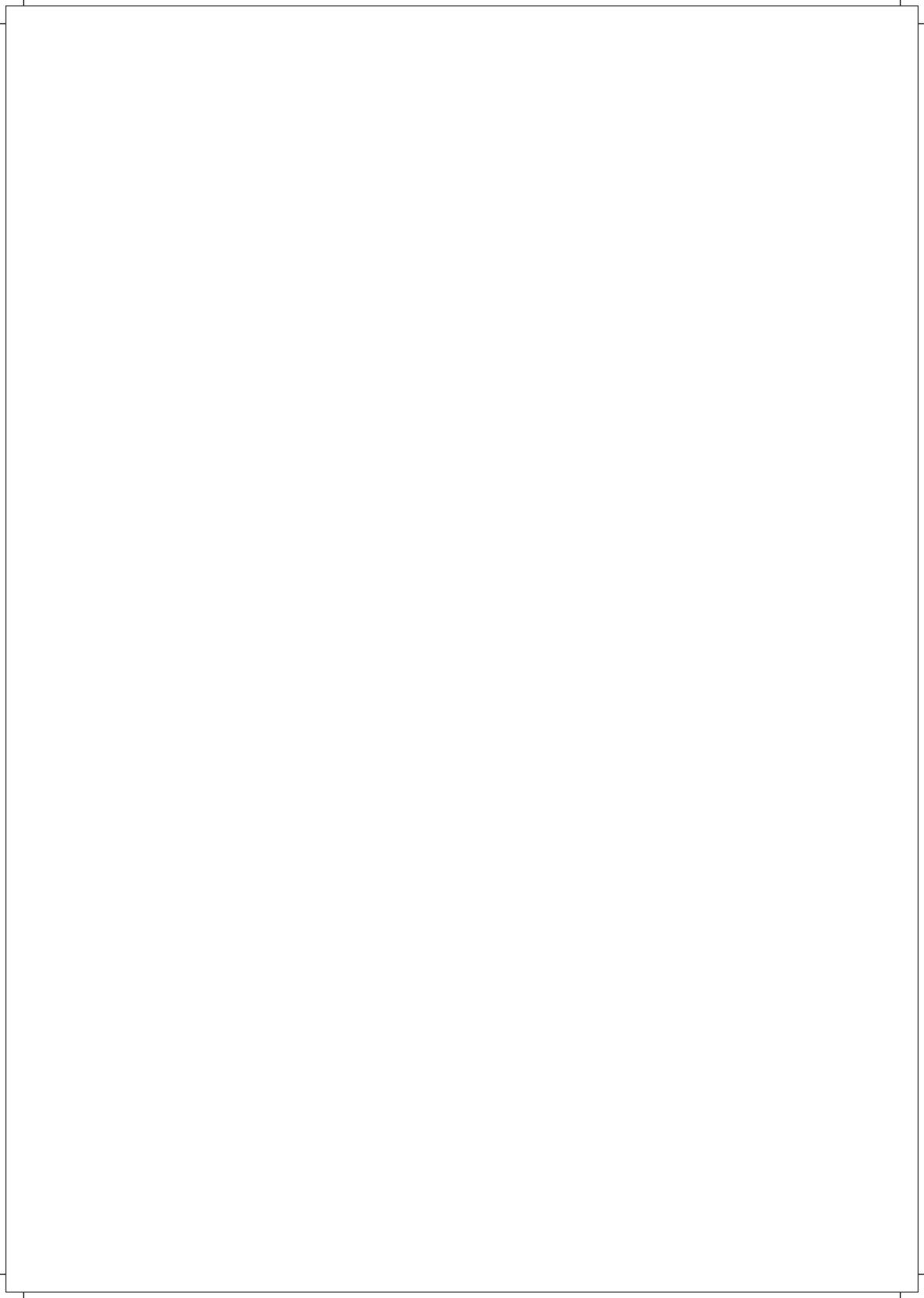



JANAK
HEALTHCARE

JANAK HEALTHCARE PVT. LTD.

Unit No. 12, 1st Floor, Kalpataru Point, Opp. Cinemax Theatre, Sion Circle, Sion (E),
Mumbai 400022. Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100
www.janakhealthcare.com