


# SLIMMING User Manual

## HKS306

  
**CERTIFICATE**  
of conformity with European Directives

Reference Number: LUM200704025

Product Name: INTENSE PULSED LIGHT

Applicant: AB HONGKONG BEAUTY&HEALTH EQUIPMENT MANUFACTURE CO., LTD.  
QINGLINWAN 18-804, HAISHU, NINGBO, CHINA

Manufacturer: AB HONGKONG BEAUTY&HEALTH EQUIPMENT MANUFACTURE CO., LTD.  
QINGLINWAN 18-804, HAISHU, NINGBO, CHINA

Brand Name: N/A

Model Number:

HKS-801	HKS-801B	HKS-807B	HKS-810	HKS-810B
HKS-810D	HKS-810C	HKS-811	HKS-811B	HKS-811C
HKS-816	HKS-803	HKS-800	HKS-800B	HKS-306
HKS-820				

Rating: 220V-240V, 800W

The submitted sample of the above equipment has been tested for CE marking according to the following European Directives:



- the EMC Directive 89/336/EEC amended by 92/31/EC & 93/68/EEC


Standard(s) used for showing compliance with the essential requirements of the specified directive(s):

Standard(s)	Test report No.	Issued by	Date
EN 55015: 2000+A1+A2	LUM200704025	Best EMC	24 April 2007
EN 61000-3-2: 2000+A2	LUM200704025	Best EMC	24 April 2007
EN 61000-3-3: 1995+A1+A2	LUM200704025	Best EMC	24 April 2007
EN 61547: 1997+A1	LUM200704025	Best EMC	24 April 2007

The referred test report show that the product complies with standards recognized as giving presumption of compliance with the essential requirements of the above listed EU Directive(s). Other relevant Directives have to be observed.

After preparation of the necessary technical documentation as well as the conformity declaration the CE marking as shown below can be affixed on the equipment.

SHENZHEN BESTEMC Co., Ltd  
  
Dick Newman  
Chief EMC Engineer  
Date: 24 April 2007

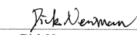Note: This certificate is part of the full report and should be read in conjunction with it.

  
**SAE TESTING CO., LTD**  
Http://www.sae.world.com.cn E-mail: ce@sae.world.com.cn

Reference No.: HRN0707016

Applicant: AB HONGKONG BEAUTY&HEALTH EQUIPMENT MANUFACTURE CO.,LTD  
Flat/Rm B 8/F Charming Bldg 72-74 Cheung Sha Wan RdK1

Manufacturer: AB HONGKONG BEAUTY&HEALTH EQUIPMENT MANUFACTURE CO.,LTD  
Flat/Rm B 8/F Charming Bldg 72-74 Cheung Sha Wan RdK1

Product: INTENSE PULSED LIGHT SKIN THERAPY SYSTEM

Model No.: HKS

Confirmation Letter

The following product, submitted by the applicant has been tested by us, and/or verified on the basis of the ROHS test report provided by the applicant, and found in compliance with the following directive:

2002/95/EC  
Restriction of the Use of Certain Hazardous Substance in Electrical & Electronics Equipment (RoHS)

  
Manager:   
Date: July

The statement is based on a single evaluation of one sample of above mentioned products.  
It does not imply an assessment of the whole production.  
The certificate holder is responsible for the consistent manufacturing of the product in compliance with the tested sample(s) or component(s) submitted to us.

2F, Chuangxin Building, No 673, Jiangnan Rd, Hi-Tech Park, NingBo315040, China  
Tel: +86 574 27721536 27721537 27721538 Fax: +86 574 27721538

## HKS306 SLIMMING FAST USER MANUAL



## PARTS OF MACHINE



The Induct Head of the Binding-energy Wave for Losing Weight: When in use, steadily coats enough quantity of conducting resin for losing weight on the slab of the binding energy wave, then fixes the spot where the user wants to lose weight with the bandage. Make sure the contact is good. The binding energy volume is generally adjusted to 3 . When the user feels the temperature is too high, then reduces the volume appropriately or reduces it to 0.




Emergence Switch:

With the humane special design, the user only has to press the top of red button gently when in operation, then the apparatus will be closed.



Spare Parts Kit:It is for storing uncommonly used spare parts and daily goods

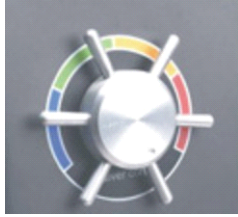

The Infrared Vibrating Fat Switch: When the ZK switch is on, turns the switch clock-wise; the infrared light first starts, then the rotating magnetic strength and frequency, and the long-wave vibration energy will simultaneously increasing to a certain degree. Attention: When revolving to the proper place, please do not use more efforts, or it will do damages to the regulator. when operation the user grasps lightly according to the peak red key, then entire machine closure



The self-adhesive patch: Pasting to a certain spot, the user cannot also use the patch together with the binding-energy wave for losing weight. Together with many different formula, the user can use it to increase the partial plasticity, improve clear the acupuncture channels and collaterals recuperation and buxom the chest




The Third Group of and Fourth Group' s Jacks for Losing Weight: It is interchangeable among induct heads. The user can adjust the quantity of the bioelectrical energy by adjusting the corresponding INT3 and INT4.



The Power Switch: the Fuse Plug; and the Power Line Plug; When replaces the fuse, the operator can push the fuse in by the screwdriver and then gyrate anticlockwise at the angle of 45 degrees. Then the fuse can spring out automatically.



The Installment of the Wing and Push-hand: matches the wing and the push-hand into the nail, and then screws the nail heads in the nails by hand. Do pay attention not to screw the nails with tools and not to screw the nails with great strength.

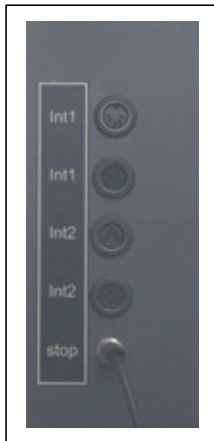

The First Group of and the Second Group Jacks of Self-adhesive Slabs


for Adjusting Acupuncture Points; STOP is the jack for emergency stop of the handle switch; When the users wants to adjust the intensity of the output of the bioelectricity, the user should adjust the INT1 and INT2 keys.



The Third Group of and the Fourth Group Jacks of Self-adhesive Slabs


for Adjusting Acupuncture Points; STOP is the jack for emergency stop of the handle switch; When the users wants to adjust the intensity of the output of the bioelectricity, the user should adjust the INT3 and INT4 keys.

## THE DISPLAY

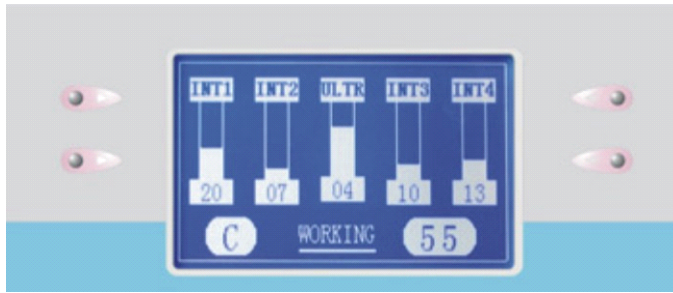

1. The First Group of Intelligent Muscular Bioelectrical Waves for Losing Weight (the DD Wave), the Display of the Strength of the Output (0~50MA)
2. The Second Group of Intelligent Muscular Bioelectrical Waves for Losing Weight (the DD Wave), the Display of the Strength of the Output (0~50MA)
3. The Intensity of the Binding-energy Wave for Dissolving Fat (0~8 Gears).
4. The Third Group of Intelligent Muscular Bioelectrical Waves for Losing Weight (the DD Wave), the Display of the Strength of the Output (0~50MA)
5. The Fourth Group of Intelligent Muscular Bioelectrical Waves for Losing Weight (the DD Wave), the Display of the Strength of the Output (0~50MA)
6. The working prescription choices demonstrate that different prescription choices have different effects. There are altogether 15 groups from A to T.
7. The operating time demonstrated, when chooses the good treatment prescription the computer can give a reference value, the opening establishment is 60 minutes, the American body teacher may adjust suitably, this demonstration value can the automatic countdown, when returns to the zero hour computer to be able to shut off the output knock off, and has the sound to remind the American body teacher.

## OPERATE PANEL

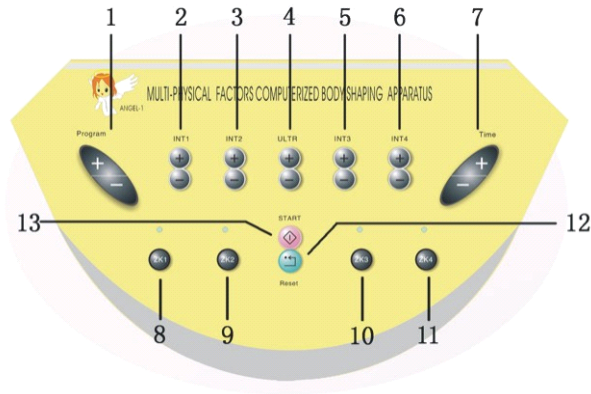

1. The formula prescription choice buttons (the A<sup>^</sup>T altogether 15 prescriptions) use when refers to the prescription function table.
2. first group of induct head can adjust the bioelectricity volume from 0 to 50 level. Generally speaking, the button adjusts to the volume until the user have the obvious feelings.
3. The second group of induct head can adjust the bioelectricity volume from 0 to 50 level. Generally speaking, the button adjusts to the volume until the user have the obvious feelings.
4. The ultra microwave volume control button control 8 groups of 16 ultra microwave generators and can adjust the binding energy wave to dissolve the fat from level 0 to 6; Generally speaking, the intermediate quantity is recommended; The higher the value is, the higher the heat is. If not used, the ZK switch for the spare induct head for losing weight is recommended to be closed.
5. The third group of induct head can adjust the bioelectricity volume from 0 to 50 level. Generally speaking, the button adjusts to the volume until the user have the obvious feelings.
6. The fourth group of induct head can adjust the bioelectricity volume from 0 to 50 level. Generally speaking, the button adjusts to the volume until the user have the obvious feelings.
7. The timing button: presses the button to adjust the operating time every 3 minutes
8. The first group induct head of the binding energy wave to dissolves fat; Infrared light coupling; The switch for the dynamic rotating magnetism and the long-wave vibration functions, presses the button, all functions in the first group of induct head except the muscles bioelectricity will be shut off or open.
9. The second group induct head of the binding energy wave to dissolves fat; Infrared light coupling; The switch for the dynamic rotating magnetism and the long-wave vibration functions, presses the button, all functions in the first group of induct head except the muscles bioelectricity will be shut off or open.
10. The third group induct head of the binding energy wave to dissolves fat; Infrared light coupling; The switch for the dynamic rotating magnetism and the long-wave vibration functions, presses the button, all functions in the first group of induct head except the muscles bioelectricity will be shut off or open..
11. The fourth group induct head of the binding energy wave to dissolves fat; Infrared light coupling; The switch for the dynamic rotating magnetism and the long-wave

vibration functions, presses the button, all functions in the first group of induct head except the muscles bioelectricity will be shut off or open.

12. Returns the key to zero to end the operation or have the middle stop.


## OPERATE program

1. Presses 13, the start button, then the screen will display 7 groups of data.
2. Keeps the emergency switch under the customer 's control so that the customer can stop the system automatically by controlling the button when needed.
3. Spreads evenly enough amount of south-conducting gel on the sheet metal for losing weight, and places the induct heads on the place where the user wants to lose weight. Generally speaking, the abdomen is often the part of body that the user wants to lose weight. For those users who have relatively small waists, 2 induct heads are enough. However, generally speaking, 4 induct heads are often used. then uses the elastic bandage to fix them so that they are close enough to the skin as not to leave any gap. If not, the idle binding energy would give off heat or the gap would conduct electricity, which would burn the user' s skin. Do pay attention not prevent the induct heads from touching one another, or the noises by friction will be generated in vibration
4. Presses button 1 to choose a prescription from the attached list 1-1 "Formula Prescription Table".
5. Presses buttons 2, 3, 4, 5, 6 to adjust the intensity of the muscular bioelectrical wave of the induct heads (the DD wave), the intensity is ranged from 0 to 50MA. Please pay attention to user' s feelings while in operation. Adjusts it to the degree that the user has obvious feeling about it and is acceptable. The highest degree for adjusting is the tolerance limit. Generally speaking, adjusts it from 5 to 10, the user can feel it. Pay attention not to adjust the induct head(s) to a higher degree if not in use.
6. Turn the switch of the infrared vibrating fat energy to adjust it to the most appropriate intensity of the long-wave rotating vibrating fat. Generally speaking, the orange area is appropriate. In this area, the user can feel the intensity of the muscular bioelectricity of the induct heads decrease, then adjusts it the intensity of the DD wave of the induct heads again. Generally speaking, a appropriate range should be from 15 to 35, until the user can feel contraction.
7. Presses button 4 to adjust the intensity of the energy-binding wave for dissolving fat. Generally speaking, gear 3 is the common choice; when the user feel too much heat, then adjusts it to a lower degree. The adjustment range is from gear 0 to gear 6.
8. Presses key 7 for timing to adjust the working time of the system for losing weight, please refers to the expression 1-1.

# OPERATE technique


## 1. Losing Weight on the Buttocks and the Shoulder

Commonly seen problems of the buttocks and shoulder: the piling up of the hypodermic fat; the muscles and the skin lose elasticity; the weary of the waist and back  
Suggested operation sequence: The massage of the induct head for losing weight + the sticking of acupuncture points

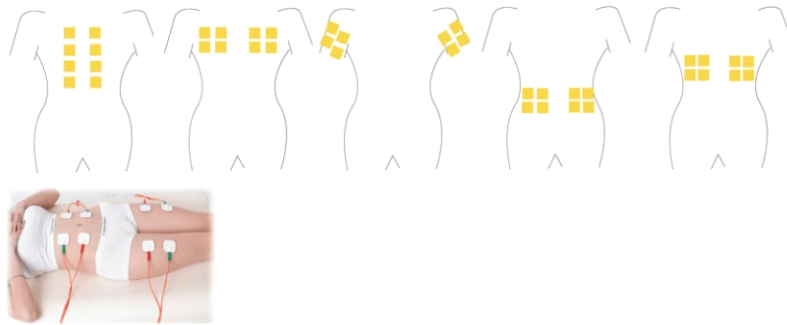

## 2. To Raise and Strengthen the Buttocks

Dermis-----The Binding-energy Wave for Losing  
Epidermis-----the Infrared Hot Coupling Effect  
Organism-----The Dynamic Rotating Magnetism Recuperation

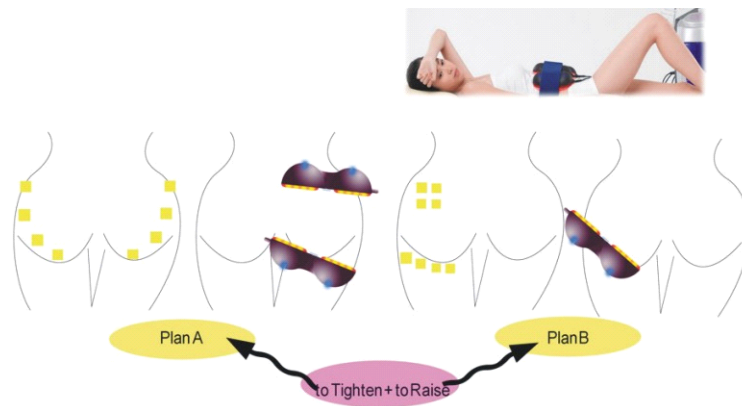

## 2. To Raise and Strengthen the Buttocks

Dermis-----The Binding-energy Wave for Losing  
Epidermis-----the Infrared Hot Coupling Effect  
Organism-----The Dynamic Rotating Magnetism Recuperation

Formula NO. A/B	Function	Reference time
A/B	The Partial Obese (Four Limbs)	45 minutes
C/D	The Whole-body Obese	60 minutes



E/F	The Whole-body Sagging Obese	60 minutes
G/H	Stubborn Obese	60 minutes
K	Changes the Partial Cedrat Organization	45 minutes
M	The Dropsy Obese	45 minutes
P	The Centripetal Obese	60 minutes
R/S	The Periphery Obese	60 minutes
R/S	The Periphery Obese	45 minutes
N	The Self-adhesive Acupuncture and the Bioelectricity Way for Buxom breasts	60 minutes

## Suits the People Group and Contraindication

Suits the People Group

It is quite effective for the 12~60 years old people and it fits the non-pathological obese

treatment. Especially for those who have meager breasts' muscles and the breast mal-

development, the user can see the good effect for the buxom breasts

Contraindication

Pregnant Woman; women' s abdomen in menstruation; open wounds and hemorrhagic wounds; people who have the high fever and tumor, people who have the serious heart disease and other serious diseases; the people whose heart is implanted with the actuator

## Pay attention to the following and the repair and maintenance methods

1. Read the manual carefully before operating the machine to get good knowledge of

the service voltage, the operating power etc technical parameters of the apparatus; Makes sure the power supply is correctly connected; When disassembly and assembly the spare parts of the apparatus, the operator should handle with care; Please refers to the installation diagram and procedures while installing the apparatus.

2. While operating the rotating magnetism, the guest is supposed to take off the wristwatch and othe metal accessories; Do not eat anything one hour before using the apparatus to lose weight.

3. Choose different formula prescriptions and the corresponding intensity of different physical factors according to user' s specific situations; Makes necessary adjustments at any time by observing the guest' s reaction.

4. Gradually adjusts the intensity of the muscular bioelectricity (INT) when in use, from small to big adjus. Pay attention not to enlarge the intensity suddenly as to avoid the stimulation the guest may receive.

5. There are altogether 16 ultra-microwave energy-changing pieces, and each induct head for losing weight has 2 ultra-microwave energy-changing pieces; Do switch off the ZK switch while the induct head is not in use in order to avoid idling and heat given off would burn out the appratus; When the user feel too much heat, the operator should switch off or reduce the intensity of the binding-energy wave.

6. Before fixing the induct heads, be sure to cleanse the guest' s skin to keep the skin surface under clean and moist condition, then spreads evenly the conducting resin for losing weight, and fixes the induct heads with special elastic bandages; Makes sure to leave enough space among induct heads to avoid poor contacts and point contacts, or it will cause electric burn (causes bubbles)on the user' s skin; If the treatment time is too long, adds some conducting resin for losing weight when necessary to avoid the uneven electric conduction because of the dryness of the conducting resin. For those users who have sensitive skin, the beautician should place a gauze between the skin and the induct head(s); Soaks the gauze in liquid medicine or water and pay attention to supplement the moisture and the conducting resin as necessary.

7. To guarantee the curative effect and the health, it is a must to maintain cleanness of the accessories; Uses the ethyl alcohol or other non-corrosive methods of disinfections to disinfect and places them in specific places.

8. Please covers the apparatus with clean cloth or protective cover while the apparatus is not in use; Makes sure the power source is switched off; moistens a piece of soft cloth with the tepid water of under 40C and wrings it out and cleans the apparatus with it gently. Pay attention not to use volatile materials (e.g. gasoline, illumination powder and so on), which can do damage to the machine sometimes; Do also pay attention not to spray water over the main engine, otherwise it can do damage to the main engine or cause an electric shock.

9. The in-use apparatus, in case disturbed by other high-frequency equipment, possibly causes the reposition of the apparatus; the instrument must be far away the strong electromagnetic field and the high heat sources. The instrument should not lay aside in open-air areas and environment of high moisture content and dusty places such as the kitchen and bathroom and so on.

10. The power sources of the users must do the grounding according to the country' s stipulation for safely using electricity ( Pay attention that the water pipe, the fuel gas pipe and the heat supply pipe and so on cannot be used as ground conductor), otherwise it would possibly cause safety accidents and fire.

11. Do not place heavy items on the apparatus; do not beat and hit the fuselage; Do not lay aside flammable, explosive, volatile and corrosive items. Do not disassemble and carry on transformation to the fuselage and the accessories

12. Check the Machine Frequently,

After long-term use, the operator must carry on inspections to the apparatus to check if there are the following troubles:

The patch and the plug are exceptionally hot, even with burnt smell;

When in use, the machine gives off strange sound or has unusual vibrations;

The outer covering carries electricity; or there are other breakdowns.

If there is/are trouble(s) as above, please stops using the machine. In order to prevent the occurrence of breakdowns or accidents, the operator should first switch off the power switch and asks an experienced electrical technician to inspect the machine. If it still does not work, please get in contact with the local dealers as soon as possible.

## Instrument Trouble Shooting

1. When there is something wrong with the instrument, please refers to the instruction booklet to carefully inspect carefully the operation failures and if the power supply and voltage are all right.

2. When there is something wrong with the instrument, the operator first should turn off the switch and ask an experienced electrician to check if each output wire has open circuit and short circuit phenomenon. If so, please weld the wire(s).

3. When the safety fuse is damage, please screws the nut off the safety fuse, of the same specification.

4. If there is something wrong with the inner devices, please open the instrument by turning the screws and check whether there is anything wrong with devices, which can be replaced by devices of the same specification.

5. Please do not disconnect the electrical wires at one' s own will. If there is/are still some problem(s), please contact the local dealer as soon as possible.