

User's Manual

to perform treatments

MN-7000

**How to make your
device work**



LOW TENSION AND EMC DIRECTIVES

GENERAL INSTRUCTIONS

Professional heating blanket for aesthetic use. It has been manufactured in order to wrap up the whole body, except for the head.


MN 7000 is composed by:

- 1 bag divided into three independent areas. Size: open 160x 200 close 70x200.
- Material: 100% vinyl treated with "VANQUISH"
- 3 extricable heating blankets. Each pad is managed by a 3 temperature switch (0-1-2-3-0).
- Size: 150x50
- Material: inside 100% polyester; outside 100% vinyl treated with "VANQUISH"

The slimming sauna MN 7000 is available with or without an automatic timer.



The 3 heating blankets are equipped with "ANTI-FOLD" FLEXIBLE STAVES. This system prevents the heating blanket from getting caught between the joints of the bed, causing the blanket to dangerously overheat.

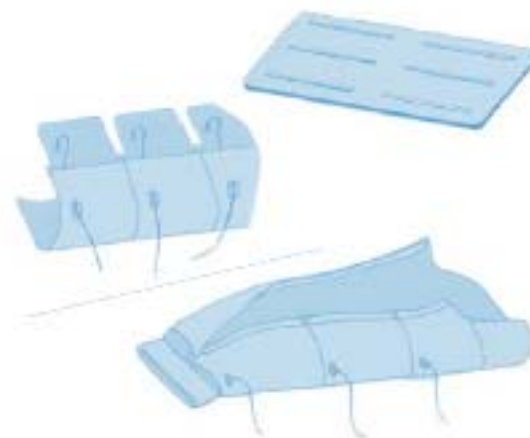

TECHNICAL CHARACTERISTICS

TECHNICAL CHARACTERISTICS

230V~50 Hz 90 Watt x 3 (Total: 270W)

Conector con posición de tres temperaturas (0-1-2-3-0) con piloto de luz.

Conector provisto de fusible A 0,85.

Resistencia eléctrica con cable de protección.

Este producto se ha fabricado en conformidad con los estándares de la CEE: 89/336/CEE (Compatibilidad electromagnética), 73/23/CEE (Baja tensión, seguridad eléctrica).



BEFORE USING THE BLANKET OPERATION MODE

BEFORE USING THE BLANKET, make sure that:

1. The network tension corresponds to the one indicated on the MN-7000 label and in the instruction manual.
2. That the three switches are turned to zero.
3. The heating subsections of the interior of the MN-7000 are placed in a correct position: **well stretched and without foldings.**

IMPORTANT: IN ORDER TO AVOID DAMAGES AND IRREPAIRABLE DEFECTS, NEVER FOLD THE BLANKET IN TWO, DO NOT PUT WEIGHT ON TOP OF IT AND DO NOT LEAN ON IT WHEN IT'S FUNCTIONNING OR WHEN IT'S HOT



Fig. 2

- Put the MN-7000 on the bed, leaving it **open** and making sure it is well **extended, without any folding.** (Fig. 2).
- Insert the connection cables in the plug, making sure the input wires aren't twisted or rolled up.
- Accomodate the client on the bed and close the MN-7000.
- Put the switches at position nº3 for about 10 minutes in order to acquire the maximum temperature. Then change the position of the switch to nº2 in order to maintain an average temperature. In position nº1, a minimum temperature is maintained.
- In order to avoid a temperature dispersion, one can cover the sauna blanket with other (**NON-ELECTRICAL**) sauna blankets in case the professional considers it necessary.
- At the end of each treatment, remember to turn off the sauna blanket MN 7000, turning the three switches to zero. In the case of centers with a large work volume, we recommend using a correctly programmed timer that assures the device will be switched in case of oblivion.

REMEMBER

- The sauna blanket must be well spread.
- Please assure the current cable is not intertwined or twisted.
- In order to avoid losing heat, you can cover the sauna blanket with another blanket (non electrical).
- In order to make the heating blanket obtain its maximum temperature, position the temperature control at 3 for about 15 minutes. Then switch to position 2 and afterwards to position 1 in order to maintain the minimum temperature.
NEVER HEAT THE BLANKET WITHOUT A PERSON INSIDE.
- In order to avoid interruptions and irreparable damages, never fold the sauna blanket and never put weight on top of it.

CLEANING

Before undertaking any cleaning or maintenance action, switch the temperature control to zero and disconnect the device from the general energy subministration network.

In order to guarantee a maximum and durable efficiency of your sauna blanket, it is important to be as careful as possible when cleaning and to respect a maximum of hygiene. Use a cloth moistened with a non-alcoholic cleaning product or just water and soap to take out stains. **DO NOT USE DETERGENTS OR CHIMICAL PRODUCTS.**

The professional beautician has the obligation to wrap up the client in a disposable plastic blanket before starting the treatment, because of reasons of hygiene as well as maintenance of the MN-700.



www.ema.es



ENGLISH

MN-7000

User's Manual

to perform treatments

ATTENTION

The heating blanket MN 7000 shall be used under the supervision of qualified personnel only.

Do not use it for children, invalids or people who are insensitive to heat.

Before connecting the equipment to the main supply, please check whether it's compatible with the electrical features of the device. Electrical features are detailed on the label.

Never place the equipment in water. The device must be used only when completely dry. Never touch the electric plugs/sockets with wet or damp hands.

Pay attention not to insert needles or pins or other sharp elements.

Never open the device or tamper with its accessories. If any repairs are necessary, please contact the technical service. Failure to observe these instructions will compromise the safety of the equipment.

- Between one treatment and the following, the temperature of the blanket can be maintained at position 1, but never longer than 5 minutes. The sauna blanket should be kept UNFOLDED on the bed. Make sure it is correctly stretched out. THE BLANKET CAN NEVER BE FOLDED, because of the risk of burning the MN-7000 itself.
- Always remember to turn off the sauna blanket at the end of the day, switching the temperature to position 0 and disconnecting the blanket from the power network.

ATENCIÓN

It is forbidden to remove the label located on the cover where the serial number can be found. Ignoring this instruction automatically nullifies the warranty.

Immediate risk of breaking down and burning:

The following circumstances have proven to be the most likely causes of defects in the MN-7000. We remind the professional that he or she should never perform the following actions:

1. **The heating blanket is left on without a person inside.**
2. **The beautician forgets to disconnect the blanket once the treatment is finished.**
3. **A heavy blanket is put on top of the blanket while the blanket is still functioning and warm.**
4. **The blanket is folded and left on.**
5. **The internal thermosections are incorrectly positioned (for instance, by folding them).**
6. **The blanket is pre-heated while being closed.**

We remind you that these actions are dangerous and that they can cause the MN-7000 to burn.

SUBSTITUTION OF THE EXTRACTABLE SUBSECTIONS

In case of SUBSTITUTION OF THE EXTRACTABLE SUBSECTIONS of the MN 7000, we hereby detail the procedure to be followed by the client:

1. Turn of the MN 7000, turning the switches to zero.
2. Disconnect the three switches of the current input.
3. Spread out the MN 7000 on a flat surface, as indicated in Figure 3.

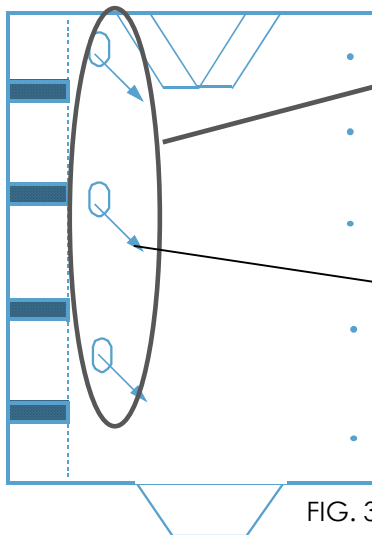


FIG. 3

Take a look at the MN 7000 and you should observe the three outlet holes of the input cable

Input cable with switch

5. Slide the input cable of the defectuous subsection in the interior of the bag through the lining.
6. Remove the heating subsection that requires substitution. Figure 4.


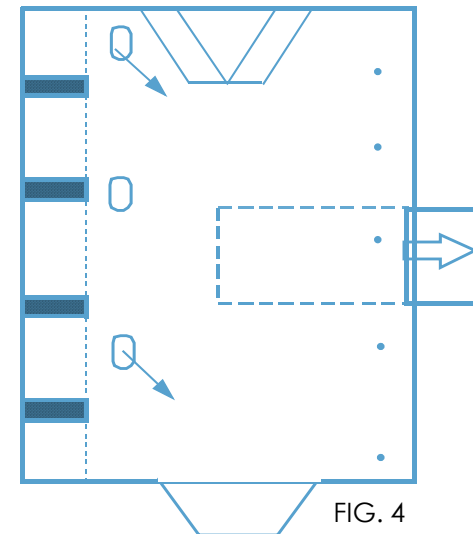



FIG. 4

SUBSTITUTION OF THE EXTRACTABLE SUBSECTIONS


7. Take the new heating subsection and position it as shown in figure 5. (When looking at the blanket you should see the outlet of the input cable and the labels).

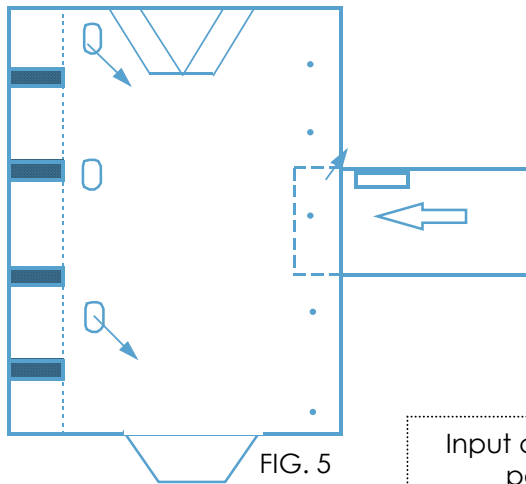

Input cable in this position!

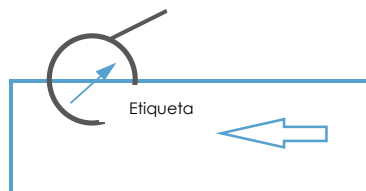

8. Insert the subsection, letting the input cable dangle outside the lining. **IMPORTANT: PLEASE ASSURE THE SUBSECTION IS WELL SPREAD INSIDE THE BAG. NOT THE SLIGHTEST PART SHOULD BE FOLDED.**
9. Close the subsection with the corresponding button and turn the MN 7000 so it can be used.

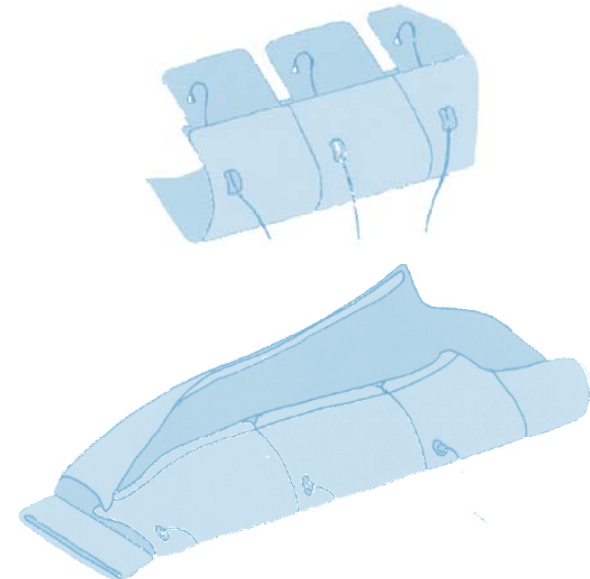

WARRANTY

The **WARRANTY** is valid for the period of a year starting from the date of purchase and for the product with all of its components. The date of purchase reflected in the invoice bill counts as the date of initiation.

The WARRANTY covers the free repair of all the damaged parts or parts presenting defects before the purchase date.

The WARRANTY does not cover:

- Irregularities in the functioning or defects derived from an incorrect electrical installation.
- Defects or damages produced because of an installation different from the one indicated in the instruction manual.
- Defects derived from the fact that the product has been used with or connected to accessories different from the ones delivered or formerly authorised by the manufacturer.
- Thefts, drops, extreme thermal or environmental conditions.
- Modifications, connections, manipulations or repairs made by non authorised personnel.
- Natural wear and tear of the product.
- Device of which the serial number has been removed, manipulated or changed or has become invisible.
- Damages caused by transport.

The manufacturer cannot be held responsible for damages inflicted upon objects or persons as a consequence of the non-observation of all the warnings and instructions detailed in the user's manual.

Changes: no changes can be made without the the manufacturer's previous authorization. After receiving the damaged device, the product will be revised before accepting the warranty obligation. The merchandise should be send prepaid.

The WARRANTY is only valid if the product comes with the present document duly signed, and duly filled in by the distributing company and accompanied by a fiscal document accrediting the purchase date.

In case of controversy, only the Spanish courts will be considered competent.

Buyer

Salesman's stamp

Purchase date

Serial number

DECLARACIÓN DE CONFORMIDAD

DECLARATION OF CONFORMITY

DECLARACIÓN DE CONFORMIDAD

DECLARATION OF CONFORMITY
DECLARATION DE CONFORMITÉ
89/336/CEE
73/23/CEE

MARZO 2005
MARCH 2005
MARS 2005



Nombre del Fabricante: BIOINSIDE INVESTIGACIÓN Y TECNOLOGÍA, S.L.

Manufacturer's name:

Nom du fabricant :

Dirección del Fabricante:

Manufacturer's address:

Adresse du fabricant :

*Sant Martí de l'Erm, 22
E 08970 – Sant Joan Despí
BARCELONA - SPAIN*

Declara que el producto:

Declares that the product:

Déclare que le produit :

Electrocosmético

Electrocosmetic

Electro-cosmétique

Nombre del producto:

Name of the product:

Nom du produit :

MN 7000

Marca:

Brand:

Marque :

EMA

Cumple con la normativa:

Complies with the norm:

Est conforme à la norme :

EN 55014 (1994)
EN 55104 (1995)
EN 61000-3-2 (1995)
EN 61000-3-3 (1995)
CEI 62-39 (1992)

Y por lo tanto responde a los requisitos esenciales de la Directiva:

And therefore complies with the essential requirements of the Directive:

Et ce pour quoi accomplit les conditions essentielles de la Directive:

DIRECTIVA 89/336/CEE (Compatibilidad Electromagnética)

DIRECTIVA 73/23/CEE (Baja tensión –Seguridad eléctrica)

Manufacturer's suggestion:

We care for your opinion. Please feel free to send us your comments and ideas so we can offer you a better service day after day. You can contact us by sending your letters to the following address:

Bioinside Investigación y Tecnología, S.L.

Sant Martí de l'Erm, 22
Polígono Industrial Fontsa
08970 Sant Joan Despí
Barcelona
Spain

National phone number: 93 444 42 62

International phone number: + 34 93 477 42 63

E-mail: info@ema.es

GET INFORMED ABOUT OUR NOVELTIES

WE ARE AT OUR CLIENTS SERVICE AND WE WILL ALWAYS HELP YOU

1. BEFORE ANY DOUBT, PLEASE FEEL FREE TO CONTACT US, FOR TECHNICAL QUESTIONS REGARDING THE USE OF THE BEAUTY DEVICE OR FOR NO MATTER WHAT KIND OF QUESTION REGARDING THE PERFORMANCE OF TREATMENTS.
2. WE CONSTANTLY OFFER FORMATION IN OUR PREMISES AND WE CAN HELP YOU TO ENLARGE YOUR KNOWLEDGE AND TO MASTER THE TREATMENT TECHNIQUES WITH EASE. ASK FOR OUR FORMATION CALENDER.

EMA

Bioinside Investigación y Tecnología, S.L.
Polígono Industrial Font Santa
Sant Martí de l'Erm, 22
08970 Sant Joan Despí
Barcelona (España)

Tel. 902.31.51.71
info@ema.es