


Minnow Bath Support

User's Manual – September 2006

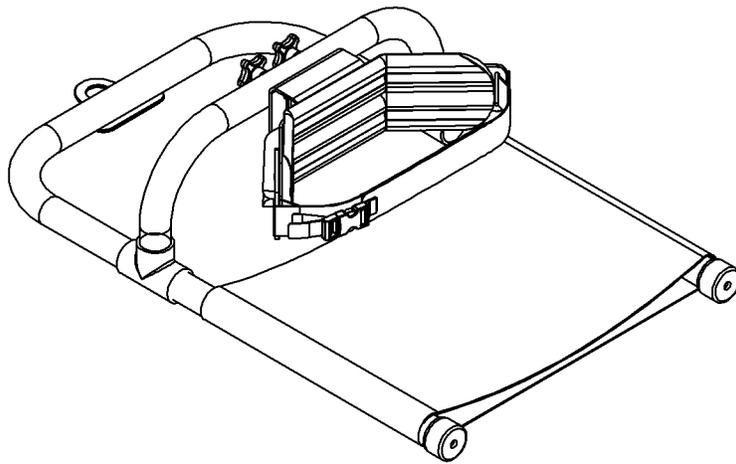

By

SNUG SEAT®

INTRODUCTION

Thank you for purchasing the Minnow Bath Support from Snug Seat. When used properly the Minnow should provide years of convenient bathing assistance.

It is very important to read all instructions before using this product. Keep these instructions for future reference, as they contain vital safety and use information.



WARNINGS

- To prevent burns, set your hot water heater to 120 degrees Fahrenheit or lower. Always check the water temperature before placing your child in the tub. Swirl the tub water around with your hand to eliminate any hot spots.



THIS PRODUCT IS NOT A SAFETY DEVICE!


- Children have **DROWNED** while using infant bathing devices. Since children can drown in as little as 1 inch (2.5 cm) of water, **ALWAYS** bathe child using as little water as necessary.
- **ALWAYS** keep child within adult's reach.
- **DO NOT** allow other children to substitute for an adult.
- **DO NOT** carry child in this product.
- **ALWAYS** test water temperature before placing child in this product.
- **DO NOT** use this product if any parts are missing or broken.

PARTS INVENTORY



WARNING

Before each use please inspect this product for damaged hardware, loose parts, missing pieces, or sharp edges. Do not use this product if any parts are missing or broken.

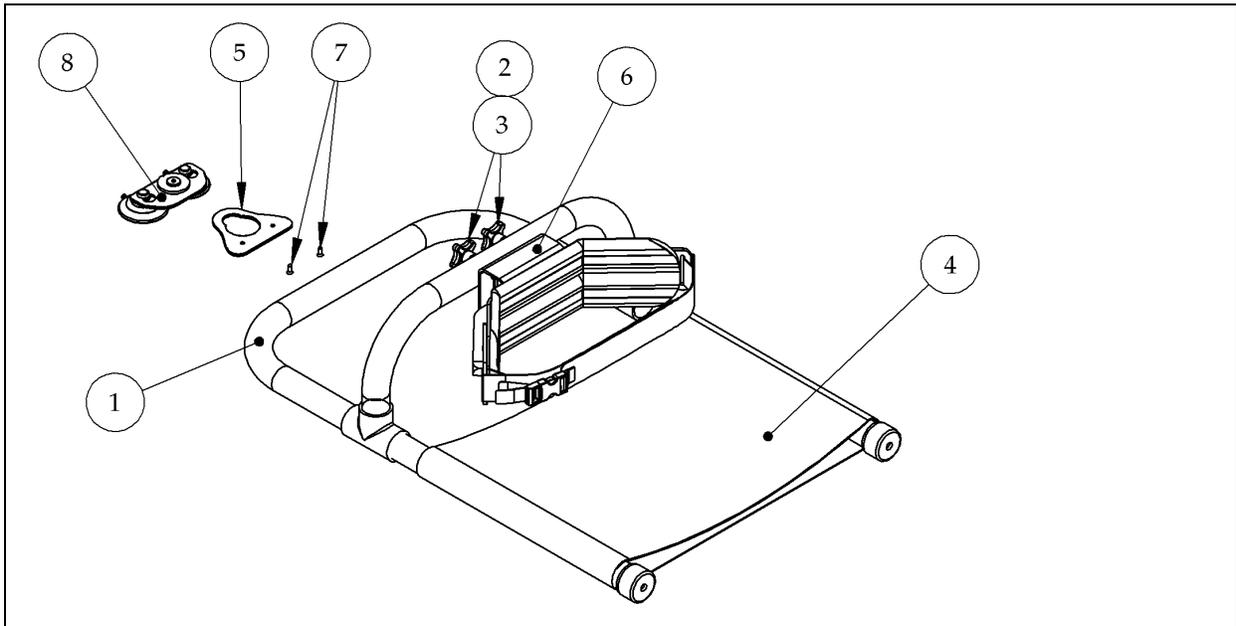


Item Number	Quantity Provided	Description
1	1	Bath Support PVC Frame
2	2	1/4-20 x 2.25" SS Carriage Bolt
3	2	Four-Arm Knob, Blue
4	1	Bath Support Pad Wrap
5	1	Frame Hanger Plate
6	1	Back Support Assembly Complete
7	2	#8 x 1/2" SS Sheet Metal Screw
8	1	Bath Support Hanger Wall Plate Assembly

Bath Support Hanger

Assembly

The Minnow Bath Support comes with a hanger assembly for easy drying of the unit and convenient storage when not in use. Before using the Minnow, you must assemble the frame hanger plate (5) to the pre-drilled PVC frame (1) using the stainless steel screws (7). This assembly requires the use of a Phillips head screwdriver.

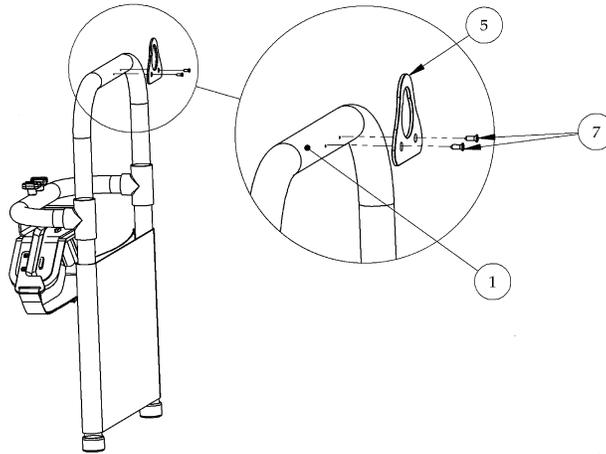


Use of Hanger

The Minnow Bath Support hanger wall plate assembly is provided with two large suction cups that adhere to many bathroom surfaces. Hanging the Minnow is a convenient way to speed drying of the unit and to store it when not in use.



WARNING


Pooled water can cause slippery surface conditions and increase the chances of falling in the bathroom. Never hang the Minnow in an area where water can accumulate and pool beneath it.

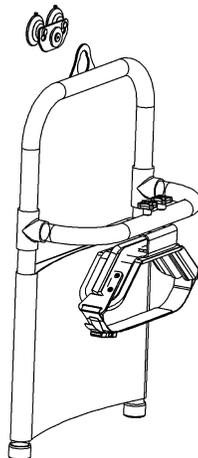

Back Support Assembly

Adjustment

The Minnow Back Support assembly can be height adjusted for optimal fit. Simply loosen the blue plastic knobs (3), adjust the back support as needed, and tighten the blue knobs.

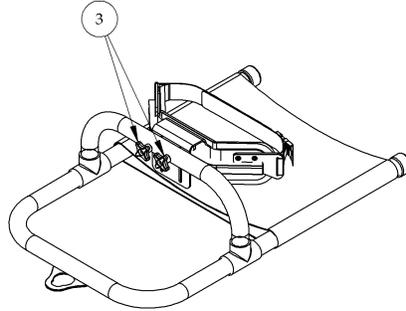


Safety Trunk Strap

The Safety Trunk Strap is provided to secure the user's trunk in the Minnow. The Safety Trunk Strap uses a hook and loop fastener for quick and easy closing and a safety buckle for added security.




WARNING - Always secure the safety buckle!

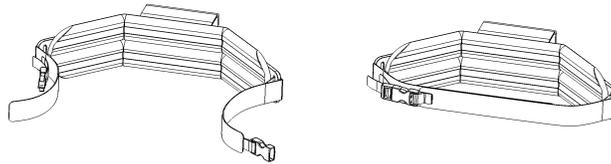


Adding An Additional Back Support

A second Back Support can be easily added to the Minnow to accommodate taller users.

Assembly - The Back Support is independently height adjustable and can be attached to the Minnow in-line with the standard back support (upright seating), or slightly behind the standard back (recline seating).

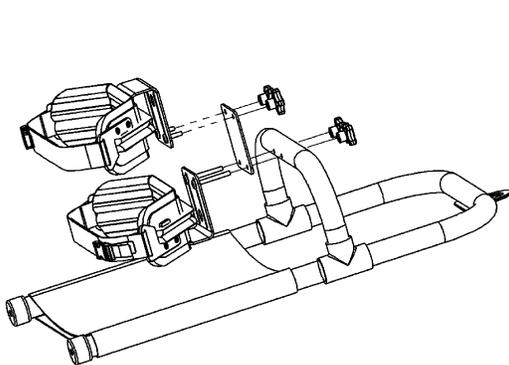

Upright Seating

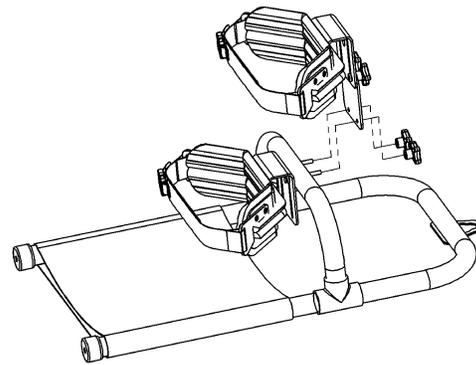

Recline Seating

Bath Support

Cleaning and Care

Removable mesh seat pad

Machine wash warm and tumble dry. Use non-chlorine bleach as needed. Bleach will yellow the cloth and accelerate deterioration especially if soaked.

PVC Frame and trunk pad

Use a mild detergent.

Measurements

	Single Back Support	Dual Back Supports
Seat width	14"	14"
Back Support, Inside Width*	12"	12"
Back Support, Height Range	7½" to 10½"	7½" to 13¾"
Frame Width	17"	17"
Frame Depth	27½"	27½"
Product Weight	4.5 lb	5.4 lb

*The back support is made of aluminum and can be mildly bent to adjust to different torso widths.