

Ultrasonic Liposuction Equipment (Desktop)



User Manual

SUMMARIZE

Cellulite is one of medicinal social problems in the developed countries. In china,

there exist about 0.5 billion persons potential on body slimming. Who consumed on it 6 to 10 billion RMB yuan. One third people in Beijing, Shanghai, Guangzhou died of the heart and blood vessel diseases, which is much more than natural aged death. Basis on it, all kinds of slimming ways in the market, slimming brands quickly popular. But many unscientific, immature thoughts and technics make most of people's weight reduction, then turn fat, again and again, as well as unreasonable equipments make people's bodies serious damage.

WHAT DOES THE OBESITY MEAN?

The obesity means a certain extent overweight and fatness layer overdepth, it is fatness in the body, especial is caused by the triglyceride accumulation too much. The obesity can be divided into two, Single Obesity and Secondary Obesity. Normal we see the former. Single Obesity occupies 99% in the obesity, we cannot find what causes the single obesity. In medicine, it is also named Primary Obesity. It maybe relevant to Descendibility, foods, sporting habits. So called S Secondary Obesity, that is caused by some healthy problems, that is to say, we can source the reasons of Secondary Obesity. It occupies 1% in the obesity, can be classified into Hypothalamus Obesity, Pituitary Obesity, Hypothyroidism Obesity, Cushing's Syndrome Obesity, Gonad Functional Retardation Obesity etc, separately caused by the thalamencephalon, pituitary, hypothyroid and gonad diseases. Among them. Mostly Cushing's Syndrome Obesity, Hypothyroidism Obesity happen to the adults, Hypothalamus Obesity always to the children, which is caused by the craniopharyngioma

WORK THEORY

The equipment takes the strong ultrasonic heads to release 40,000Hz powerful ultrasonic, speedy vibrates fatness cells, produces countless vacuum air pocket in and out of cells, mightily impacts the fatty cells, makes the cell membrane produce the inner cracking, so as to dissolve the triglyceride to be the glycerin and free fatty acid. Then then takes the power of RF wave with 0.5MHZ, to do deep-seated skin Diathermy, supply the skin oxygen and organic nutrition, strengthen cells function, advance the blood and lymph circulation, activate metabolism, remove and soften the cellulite, so get the body slimming and beauty effects.

FUNCTION AND THERAPEUTIC THEORY

1. Mightily cracking the cellulite, remove the fatness.

It uses the strong ultrasonic head to shoot 40,000Hz ultrasonic, after entering the body, will make the fatty cells produce mighty cracking effect and fatty cells friction,

effectively exhaust the heat energy, exhaust cells water, let fatty cells shrinking. Moreover, when the ultrasonic vibrates, there will bring mighty impaction among cells, fatty cells will be less, so get effect of fatness removal.

2. Dissolve the fatness, do lymph drainage, skin tightening, enhance skin elasticity.

It takes the most advanced RF technique and energy, directly penetrate into deep-seated cellulite. With targeted oriented RF output, it makes the fatty cells in quick active state, produces heat friction, higher the partial temperature. Through the sweat gland, liver sausage circulation and lymph, exhaust the redundant fatness and toxins out of the body, so get the effect of cellulite dissolving.


TECHNICAL ADVANTAGES

1. Adopt the top ultrasonic liposuction technique in the world.
2. Suitable for all kinds of skin.
3. Comfortable, painless, woundless during the treatemnt.
4. Easy operation, easy study.
5. Non- exhaustion, low cost investment and quick high returns..

FEATURES

1. Free operation, . non-anaesthetic.
2. No ruggedness.
3. No bleeding, tumidness and bruise.
4. No side effects, good effects, no rebounding phenomenon.
5. Woundless, will not influence the nomal working and living.

SKETCH of the CONTROL PANEL

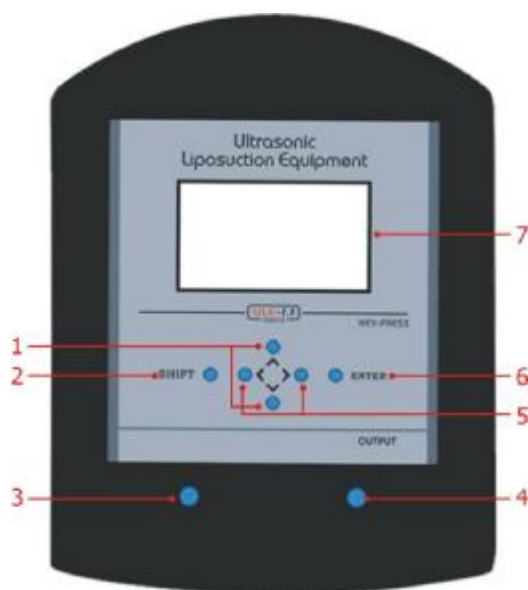

1. The button to choose.
2. Shift button, to reverse the original status.
3. The socket of Strong ultrasonic head
4. The socket of Strong RF head
5. Left and right button, Upper and lower buttons: to adjust languages, different modes and intensities etc.
6. ENTER, to make sure and enter the next menu.
7. LCD

DIAGRAM of the ACCESSORIES



Strong ultrasonic head

RF head

PACKING LIST

Description	Qty
Strong ultrasonic head	1pc
RF head	1pc
Power supply wire	1pc
User manual	1pc

SPECIFICATION

Voltage: 110/220V

Power:110W

Package:

G.W./N.W.:

COURSE OF CELLULITE CRACKING

1. **Ultrasonic liposuction:** Throught the strong ultrasonic, direct into the fatty layer, speedily vibrate deep-seated cellulite, produce numberless vacuum air packets, mightily strike the fatty cells, let them produce inner cracking, and dissolve to be the free fatty acid.
2. **RF dissolves the fatness:** Through high-effect RF energy, repromote the fatness dissolving, promote the body metabolism. Through liver sausage circulation exhaust the dissolved fatty acid out of lymph acid.

OPERATION STEPS


1. Put throught the power, switch on the machine.
2. Apply proper slimming products on the treated part. Suggestion with the cream products.
3. LCD shows choices of languages, click the left or right button to get chinese or english, then make sure by Enter clicking.
4. Enter the main menu, clic the upper or the lower button to choose the treating course, then make sure by Enter clicking.
5. Enter the **Ultrasonic liposuction** course, click the upper or lower button to choose the modes, click the left or right button to choose the working time, wave type and intensity. Then make sure by Enter clicking, LCD will show 'Working', that means the machine starts working.

- (1) Time is adjusted from 5 to 8 minutes.
 - (2) According the customer habitus, to choose the proper wave type, like continuous wave, pulse wave(also called discontinuous wave). The continuous wave can penetrate skin surface layer, the discontinuous wave can penetrate the deep-seated skin layer.
 - (3) According to the customers' endurance, to adjust the intensity.
6. Enter the **RF dissolving fat** course, click the upper or lower button to choose the mode, click the left or right button to adjust the working time, intensity, then make sure by clicking Enter. LCD will show 'Working', that means the machine starts working.
- (1) Time is adjusted from 15 to 20 minutes.
 - (2) According to the customers' endurance, to adjust the intensity.
7. After finish, click Enter to pause working, then power off, clean and disinfect the accessories, and place them in original well.


Gesture suggestion

On the belly

1. Take the Strong ultrasonic head, do circling gesture, from both sides of the waist, pull to the middle. Refer Pic A

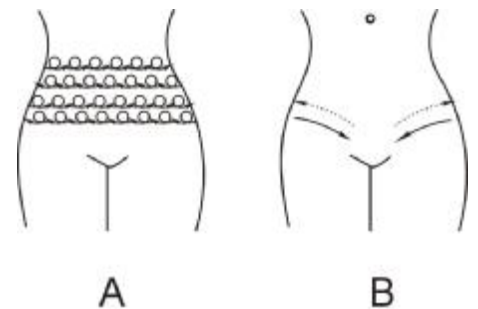


2. Gesture 1: Take the RF head, with circle gesture between the chests and slanting pull at the groin, to open the points. Refer Pic B

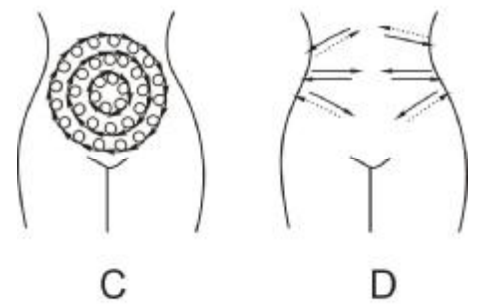

3. Gesture 2, To navel as the center, from outer to inner, to do small circles to dissolve redundant fatness. Refer Pic C

4. Gesture 3, Slanting pull the fatness between both sides of the waist, lastly pull to the groin, make the fatty acid exhausted with lymph drainage. Refer Pic D

5. Gesture 4, Slanting pull the fatness under the ribs to the groin, then return it under the ribs, repeat it some times. Refer Pic E

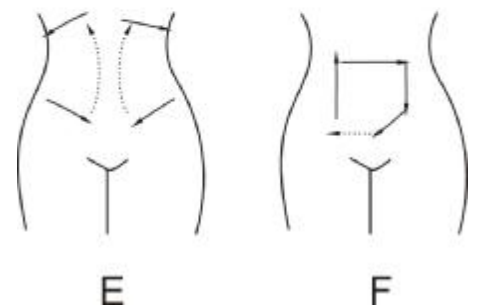

6. Gesture 5, Following the Ascending colon, Transverse colon, Descending colon, Recta anus, massage and promote the intestines peristalsis, evacuate the bowels. Refer Pic F

P.S.: the real lines ask the gesture with soft power, but the broken lines ask the gesture with out power. On the legs and arms, mostly suggest to do circling or slant pulling gesture, to pull the obesity to the lymph place.

ADVERTION & CAUTION

1. Who with serious diseases, like heart diseases, gastric ulcer, serious tummy bug, duodenum ulcer, hypertension, diabetes, please don't take the treatment.
2. Pregnant women, women during lactation, whose belly is operated not over three months, who in menses, please don't take the treatment.
3. Avoid foods with high protein, high fatness, high energy, please eat foods with light, not greasy or strongly flavored.
4. If who is having some slimming medicines, please stop it and do the treatment after one to two months. If not, we have to suggest to prolong the period of treatment.
5. Be sure not to use the machine with limosis. once eating foods, suggestion is to take the slimming treatment after one hour.
6. During treatment, please avoid leaking the essence oil to the control panel, it maybe induce the panel broken.

POST SERVICE AND MAINTENANCE

Post service

We will guarantee the quality of our equipment, normally there is no quality problem

1. If there is a deficiency, please email or call us on the reasons, it would be best if this can be dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician, our technician will give a satisfactory answer within 1-7 working days
3. If the problem comes from the spare part, we could consider in changing the spare part for our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
4. If the problem comes from the main device, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new one to our client. If the cost of posting the problematic device back to JIMY is more than the value of a new device, JIMY would take it into consideration, according to the actual status; apply to re-send a completely new device back to our client.
5. If the inferior equipment very big or expensive, not convenient to send back by post office, we will give you an appointed Hongkong place, you can send Hongkong first. Jimy will ask a cooperated forwarder to take it back. Jimy will pay all charges from Hongkong to our office. Once we check the equipment is the quality inferior, we also agree, in your next order, to deduct the freight from the

customer's office to Hongkong if you fax or email us relevant shipping receipt and invoice.

6. Please make sure, if you send the inferior one to us by air etc. without any advanced notice, Jimmy will not pay relevant air freight and custom fee.

All in one, we will try our best to serve you and also need your fully cooperation.
Thanks.

Remarks

1. The term of maintaining guarantee is a year from the date of purchasing according to the receipt. (Expendable is not included in it.) Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchasing the parts and easily expendable products.
2. We won't supply maintaining guarantee if troubles as below:
 - a) Disassembling the machine by self;
 - b) Fall it without caution;
 - c) Lack of reasonable keeping and protect;
 - d) Not operate according to the user manual.
 - e) Expendable reasons

If changes of the design and specification of the machine, we will not notice you in addition.

If any queries, please feel free to contact with the local marketing company.