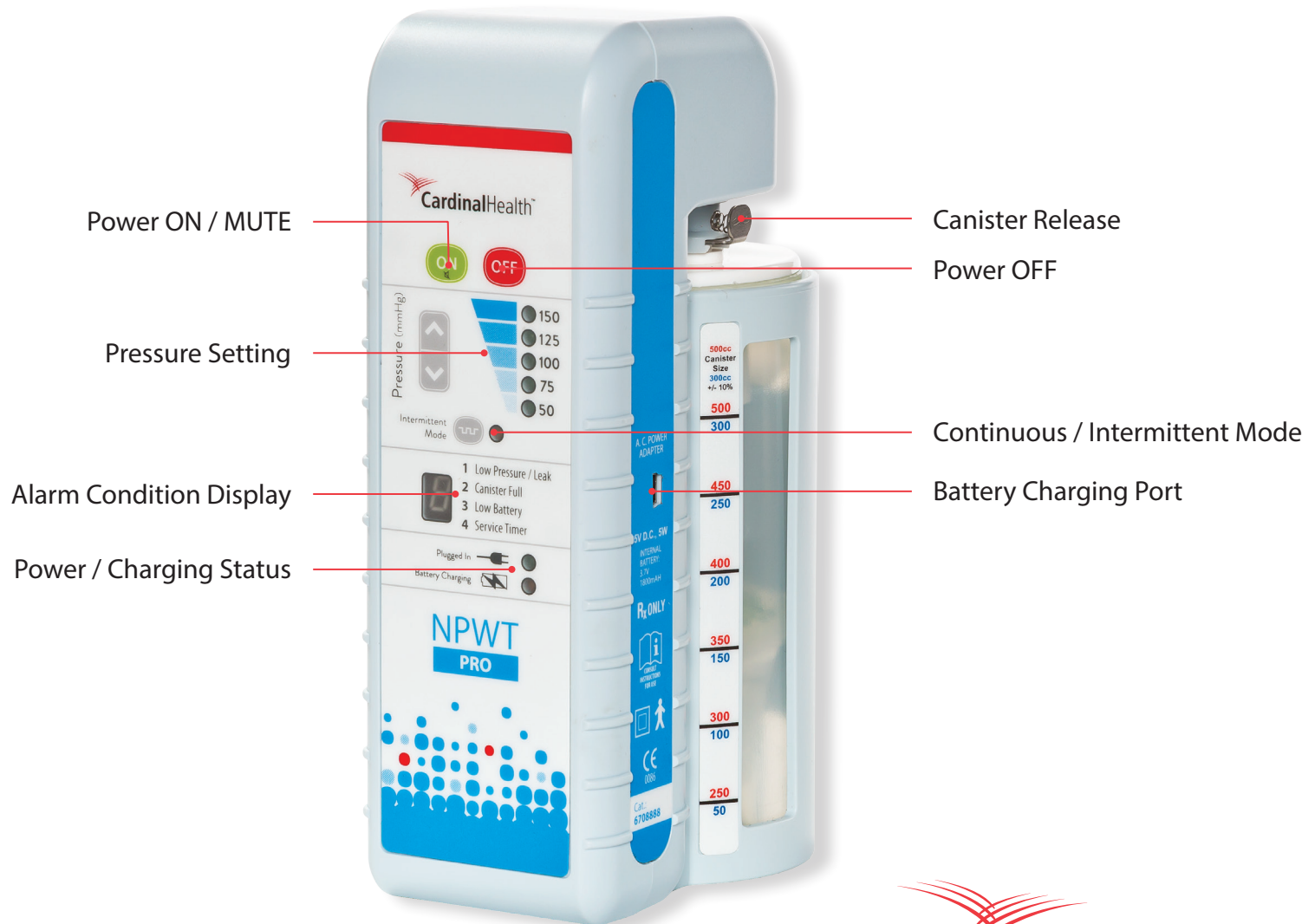


Cardinal Health™ NPWT PRO / PRO to GO / PRO at HOME Clinician Quick Reference Guide



NOTE: Please refer to the Cardinal Health™ NPWT PRO / PRO to GO / PRO at HOME Clinician User Manual for detailed instructions and review the Precautions and Contraindications before use.


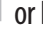

Insert Canister

Insert canister in holder by sliding it up into bottom back side of device. Line up the two short ports with the receptacle and press until it “clicks” into place and locks. To remove canister, press canister release button located above canister viewing window, grasp canister at bottom and pull downward.

Connect the Dressing

Connect the blue end of the SpeedConnect™ tubing to the open port of the canister. Gently twist and push connector on just enough to seal it. Make sure that the clamp on the SpeedConnect™ tubing is open. **NOTE:** Please refer to the Cardinal Health™ NPWT PRO / PRO to GO / PRO at HOME Clinician User Manual for instructions on how to properly dress the wound and troubleshoot issues.


Turning the Device ON

Press the ON  button to start therapy. Select desired Pressure Setting by pressing the UP  or DOWN  arrow button. A green light will begin flashing next to the selected pressure setting. The light will stop flashing and remain lit when the set pressure is reached. The light will flash if the pressure cannot be maintained.



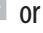
Therapy Lockout

To lock therapy setting, press and hold the ON  button for three seconds. The device will beep three times indicating therapy setting is locked. Repeat process to unlock therapy setting.

Intermittent Mode

To operate the device in intermittent mode, press the INTERMITTENT MODE  button. An amber light will indicate the device is operating in intermittent mode. Repeat process to return to continuous mode.

Adjust Alarm Volume

Volume may be adjusted by pressing and holding the ON  button and tapping the UP  or DOWN  arrow button. The numbers 1-5 will appear in the display. 1 is the lowest, 5 is the highest volume level.

Troubleshooting Alarms

If the device begins to alarm, refer to the troubleshooting table on the back to identify and correct the alarm condition.

Turning the Device OFF

To turn device off, press the OFF  button. The display will flash “o-f-f” three times while the device is preparing to shut down.

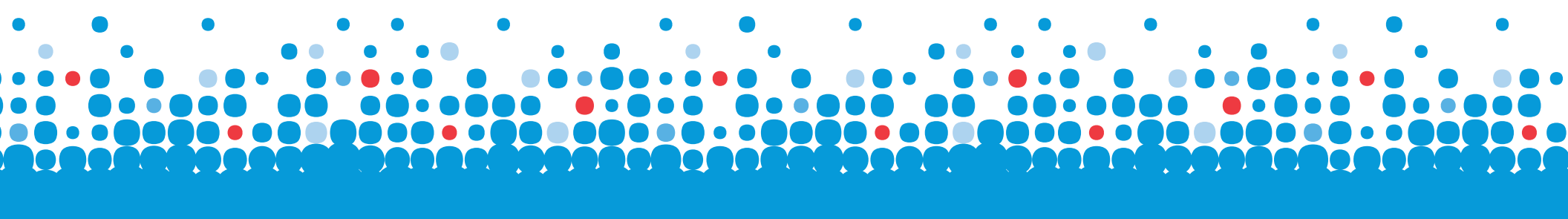
For questions or information, contact Cardinal Health at 1.866.484.6798




Notice to Users: As with any prescription medical device, failure to follow product instructions or adjusting settings and performing therapy applications without the express direction and/or supervision of your trained clinical caregiver may lead to improper product performance and the potential for serious or fatal injury.

Defibrillation: Remove the wound dressing if defibrillation is required in the area of dressing placement. Failure to remove the dressing may inhibit electrical current transmission and/or patient resuscitation.

Magnetic Resonance Imaging (MRI): The Cardinal Health™ NPWT PRO family of devices is not MRI-compatible. Do not take into the MRI area.

Hyperbaric Oxygen Therapy (HBO): NEVER allow a device—whether on or off—inside a hyperbaric chamber. The device must be disconnected from the patient prior to HBO treatment.



What you see or hear	Problem	What to do	More Information
FLASHING “1” “Low Pressure/Leak” Single beep. Device is making more noise.	There is an air leak in either the dressing or the tubing connections.	<ul style="list-style-type: none"> Clamp the tubing. If Low Pressure/Leak flashing “1” and audible alarm resets, there is a leak below the clamp – often in the dressing. Reopen the clamp before addressing the leak. Gently press around drape to check for leaks. If leak is found, patch with extra drape material. If Low Pressure/Leak flashing “1” and audible alarm continue, there is a leak above the clamp. Check tubing connection at the canister. Check to ensure the canister is fully seated and locked. Check for cracks in the canister or lid separation. If found, replace the canister. Open the clamp. 	<p>The alarm will reset, the pressure light will stop flashing, and the pump will become quiet after you find and seal the leak.</p> <p>Leaks often occur over areas of moist skin, creases or folds in skin, and wrinkles in the drape. They can occur if the drape snags on clothes or bed sheets.</p>
FLASHING “2” “Canister Full” Two-tone beep.	The canister is full.	<ul style="list-style-type: none"> Clamp the tubing. Turn device off by pressing the OFF  button. Press the canister release button above the canister and slide the full canister out. Cap and dispose of properly. Slide new canister in, align the short ports and click into place. Open the clamp and press the ON  button to resume therapy. 	<p>The canister full alarm begins when the canister is 90% full, but the device will continue to work until the canister completely fills.</p> <p>If the Cardinal Health™ PRO family of devices are placed on its front, fluid entering the canister will cause a false canister full alarm and the canister must be changed.</p>
FLASHING “3” “Low Battery” Three-tone beep.	The battery is low and will run out in about 30 minutes.	Plug in the device.	<p>Use only the power cord that came with the Cardinal Health™ PRO family of devices.</p> <p>When the device is getting power, a green light will illuminate on the front of the device.</p> <p>A yellow light below the green light will show that the battery is charging. It will turn off once the battery is fully charged.</p>
FLASHING “4” “Service Timer” Four beeps every 10 seconds.	Device is ready to be checked and serviced.	Return device to your representative for service.	This alarm cannot be Muted or manually reset by cycling power.
Pressure Setting will not change.	Pressure lock-out is engaged.	Unlock the device.	<p>To lock therapy setting, press and hold the ON  button for three seconds.</p> <p>The device will beep three times indicating therapeutic setting is locked.</p> <p>Repeat process to unlock.</p>
Device is quiet and fluid is not moving in the tube.	This is NOT a problem.	No action needed.	<p>When the dressing has a good seal, fluid may be removed from the wound and stay in the tubing. The foam will be compressed normally and the device will be quiet.</p> <p>You can use intermittent mode to move fluid in the tube to the canister.</p>
An amber light is showing on the front of the device below the pressure numbers.	This is NOT a problem. The device is operating in intermittent mode.	No action needed.	Intermittent mode maintains target pressure for five minutes and decreases to 25 mmHg for two minutes.