

Restore your back - improve your life

MDcureTM

USER MANUAL

INSTRUCTIONS FOR USE

MDcure – Electromagnetic Lower Back Therapeutic Massager

www.md curebackpain.com



Restore your back - improve your life



CONTENTS

	Page
1. Introduction	3
2. Intended Use	3
3. Features	4
4. Positioning of Pad	5
5. Operating Procedure	6
6. Recommendations for Use	7
7. Safety Instructions and Warnings	8
8. Troubleshooting	9
9. Battery	10
10. Care Instructions	10
11. Limited Warranty	11
12. Consumables & Servicing	12
13. Contact Details	12

1. INTRODUCTION

MDcure is a device for the relief of lower back pain. By producing therapeutic electromagnetic fields, this innovative patent-pending device has massaging effects including increased blood flow and perfusion, muscle relaxation, pain reduction and consequently, improved range of motion.

2. INTENDED USE

Lower back pain associated with:

- Muscle strain, pull or tear
- Muscle overuse, repetitive motion injury
- Poor posture, prolonged sitting
- Weak core muscles

3. FEATURES

1. Treatment pad – placed on the treated area
2. Control panel
3. ON power button
4. Buzzer for audible alerts
5. OFF power button – the unit shuts off automatically; the OFF button is used if treatment needs to be stopped
6. Battery compartment (back of device)
7. Power LED – Green LED lights up when unit is on
8. Status LED – Yellow LED flashes when treatment is being conducted
9. Connection of treatment pad to device



4. POSITIONING OF PAD

- Treatment pad (either side) should be placed on the lower back.
- Pad can be placed over clothing; there is no need for skin contact.
- Place the pad on the lower back gently; do not fold it.
- Use the provided fabric belt to tie the pad around your waist, keep it in position, and free your hands.
- The pad can be used in the sitting, standing or walking position.
- Do not remove the pad cover when treating your back.



5. OPERATING PROCEDURE

1. Connect pad to device.
2. Press and hold the ON power button until the green light indicator turns on.
3. After several seconds, the unit will beep once and the yellow light indicator will start flashing. The light will flash while treatment is conducted.
4. The device will shut off automatically at the end of the treatment after 30 minutes, the unit will beep once.

6. RECOMMENDATIONS FOR USE

For back pain it is recommended that you use the device 2 to 3 times a day, for at least 2 weeks, or as recommended by your health professional. If symptoms persist after 2 weeks, continue therapy either until symptoms end or 6 weeks, whichever is first.

The device has a cumulative effect. Therefore, it is recommended that you complete the course of treatments, even if you feel immediate improvement of your symptoms.

If your symptoms still persist after 6 weeks or worsen during treatment, consult your health care professional.

It is essential to follow the instructions in this user manual and recommendations made by your health professional.

7. SAFETY INSTRUCTIONS AND WARNINGS

Warning: Carefully read all ***Warnings and Instructions for Use*** prior to using the device. Failure to read and follow all instructions for use, or failure to observe all stated warnings, could diminish the effectiveness of the treatment.

Do not use MDcure:

- If you have a pacemaker or a defibrillator
- If you are pregnant
- If you are a patient treated for any type of cancer
- To mask or relieve undiagnosed pain
- Other than as instructed in this manual or directed by your health care professional

* Not to be used on infants or children

If you are in doubt about any of these warnings, please consult your health care professional.

8. TROUBLESHOOTING

Alert Signal	Explanation
The MDcure unit beeps three times and then turns itself off	Pad is disconnected
Green LED light is flashing	Battery is low
Green LED light is flashing with continuous beeping	Battery needs to be replaced; use 9V Alkaline battery

If your problem is still not resolved, contact Aerotel or your local dealer (contact details on back cover) for advice. If you are advised to send the device for maintenance, please ship (postage pre-paid) to the address specified on the back cover.

9. BATTERY

Be sure to use the specified battery. Specifications: 9V Alkaline Battery.

Contact your supplier to place orders.

Avoid discarding used batteries as ordinary waste.

10. CARE INSTRUCTIONS

The zipped-on pad fabric cover can be hand washed in lukewarm water and gentle soap.

Do not wash in a washing machine.

Do not wet the pad itself.

Do not fold the treatment pad.

Do not disconnect the treatment pad when in use.

Disconnect the treatment pad from the device when not in use.

11. LIMITED WARRANTY

Aerotel Ltd. devices are warranted against defective material and workmanship (excluding batteries) for one year following delivery of the device to the original purchaser. If any device or any part thereof in the judgment of Aerotel Ltd. is proven to be defective in material or workmanship within one year from the date of the original purchase, such defects will be repaired or replaced (at the company's discretion) free of charge for parts and labor. This warranty does not apply to any product which has been damaged by accident (dropped, waterlogged, used without authorized attachments and cables), or which has been misused, abused, altered or repaired by anyone other than Aerotel Ltd. or its representatives, or which has been damaged by uninsured shipping or inappropriate repacking for return. This warranty is in lieu of all other warranties expressed or implied including any implied warranty of merchantability or fitness for a particular purpose and no person is authorized to assume for Aerotel Ltd. any other liability in connection with the sale of this product. Aerotel Ltd. shall not be liable for any indirect, consequential incident, special or punitive damages arising out of or in connection with the operation of the device. To obtain the warranty, please complete the registration form enclosed with the device and post to the address listed on the back cover.

12. CONSUMABLES AND SERVICING

The following accessories are available from your supplier:

- Pad fabric cover
- Treatment pad
- Carry bag
- Batteries

CONTACT DETAILS

Aerotel Ltd.

Hamashbir 19, Holon, 5885325, Israel

Phone: +972-3-5585515

Fax: +972-3-5585516

Email: info@mdcurebackpain.com

Website: www.mdcurebackpain.com