

Operation manual Service manual

HL 250-eco X

250 Watt-Halogen lightsource with integrated CCD-Colour-Camera and two outlets for endoscopic applications



Table of contents

GENERAL ADVISES	3
Data of the equipment	
Warranty	
Service, Repairs and Modifications	
Disposal	
SAFETY REFERENCE / PLACE THE EQUIPMENT	
Normal use	
User qualification	
Unpacking / items included	
Safety precautions of installation	
Storage and operation conditions	5
SIGNS & SYMBOLS	6
Symbols	6
Signs	
DESCRIPTION OF THE EQUIPMENT	7
OPERATING ELEMENTS	8
Operating elements on the frontpanel	8
Operating elements on the rear	
CONNECTING AND OPERATING THE EQUIPMENT	11
Connecting the equipment	11
Operating the equipment	
SERVICE MANUAL	
General maintenance and repair advice	
Exchange of the mains fuses	
Selection of the line voltage setting	
Exchange of lamp	
Troubleshooting	
WIRING DIAGRAM	16
TECHNICAL DATA	17
Spare parts	
TABLE 'TECHNICAL SERVICE-INFORMATION'	
APPENDIX	
Flectromagnetic Compatibility (FMC)	10

General advises

Like all of our other products, this product is the result of years of experience and great care in engineering and manufacture.

This manual is destined to learn you understanding the function and the operation of your equipment. Before you switch on the equipment for the first time, please thoroughly read this manual and pay special attention to all safety instructions, so that endangering for the user and the patient is precluded. Please always store this manual with the equipment.

Before use, read this manual thoroughly.

An insufficient understanding of the dangers, warnings, cautions, and informations in this manual can result in death, serious injury, or equipment damage.

This product complies with the requirements of Directive 93/42/EEC concerning medical devices.



Data of the equipment

The type label (rear of unit) contains technical data, type and serial number of your unit. Please always indicate these data when ordering spare parts or in case of any question. Please enter here the technical data of your device!

Serial No.:	·	
Type:		
Date:		
Class:		
Hz:		
Amn ·		
•		
	Type: Date: Class:	Date: Class: Hz:

Rights

All rights on this user manual, especially the rights of duplication and publication and the rights on translation are reserved. No part of this user manual may be reproduced by any means (by photostatic copy, microfilm or other methods) without preceding written consent of the manufacturer or be reproduced, multiplied or published by means of electronic data processing. The information given in this user manual can be changed or extended without notice and do not represent any liability. Errors and technical changes excepted.

Warranty

1 year according to our warranty conditions.

Opening the equipment or performance of any repairs or modifications of the equipment by unauthorised persons shall relive the manufacturer of any liability for its performance. Any such opening, repair or modification performed during the warranty period shall void all warranty.

Wear parts are not included in the warranty.

The firm of the manufacturer shall be liable for failure or deterioration in the safe operation, operational reliability and performance of this equipment only subject to the conditions, that all assembly operations, system expansions, readjustments or repairs to same have been performed by a person or persons duly authorised by the manufacturer, that all electrical installations at the location of us meet applicable national and local electrical codes and that the instrument has been used in accordance with its operating instructions at all times.



Service, Repairs and Modifications

In conformity with the international safety regulations valid for medical devices, all activities such as check ups, repairs, modifications, calibrations etc. may only be carried out by the manufacturer or by explicitly authorized personnel.

All services carried out must be entered in the "Technical Service Notes" at the back of the user manual.

Liability

As manufacturer of the device, we only consider ourselves liable for safety, reliability and performance of the unit, if

- assembly, re-adjustment, modifications or repairs are performed by persons authorised by us
- the electric installation of the respective room corresponds to the regulations of VDE 0107 the instructions found in the user manual are strictly observed when operating the unit.

Disposal

In accordance with European Directive 2002/96/EC on Waste Electrical and Electronic Equipment, this symbol indicates that the product must not be disposed of as unsorted municipal waste, but should be collected separately.

Refer to your distributor resp. the manufacturer for return and/or collection system available in your country.



Safety reference / Place the equipment

Normal use



The equipment may only be used with accessories, wearing parts and disposable items, which have been designated by the manufacturer suitable for the instrument or the safety use of which is proven.

User qualification



The equipment may only be used by persons, who have a corresponding specialised qualification and who have been instructed in use of the equipment.

It is the user's responsibility to make sure, the equipment is safe and operates properly before using the equipment.

Unpacking / items included

Carefully unpack the equipment and accessories and remove it from their packing.

Check for missing items and evidence of shipping damage.

File any complaints with the manufacture or supplier immediately.

Retain the original packing materials for later use. These can come in handy, when the equipment must be transported.

Please verify immediately after having unpacked the equipment, whether the delivery is complete. The standard extent of delivery includes of the following:

- control unit
- > watertight camera head with 3m cable
- power supply cord
- ➤ BNC / Y/C cable
- > manual



Safety precautions of installation

Always place the equipment on a solid base.

The equipment may be used only in rooms having electrical installations conforming to applicable national, state and local electronical codes.

The unit must be joined to the central potential equalisation of the operating theatre or of the equipment trolley by means of a grounding cable.

The device must be connect to line voltage using the delivered protectively earthed power supply cord.

Storage and operation conditions

Storage temperature: -20°C to $+60^{\circ}\text{C}$ Operation temperature: $+10^{\circ}\text{C}$ to $+40^{\circ}\text{C}$

rel. humidity: storage: 10% to 90%

operation: 30% to 75%

Air pressure: storage: 600 mbar to 1300 mbar

operation: 700 mbar to 1060 mbar

Signs & symbols

Symbols

Attention, important note!



Safety note



Service



Signs

Please read the enclosed instructions!



Unit model BF



Beware of dangerous electrical voltage!



Connection for ground potential



Alternating voltage



Description of the equipment

The equipment you have purchased features a state-of-the-art colour CCD camera, which was especially designed for use with endoscopic applications. In combination with the built-in halogen illumination the device offers the user a compact unit for optimal image quality in nearly every endoscopic discipline.

The lightsource comes equipped with a built-in insuflation pump for use with gastroscopes and other flexible endoscopes. The pump can be switched on/off on the frontpanel.

There are adapters available for gastroscopes made by either Olympus, Fujinon or Pentax. The pump can be separately switched on and off.

CCD camera

Full digital signal processing ensures true colour reproduction. The camera consists of a control unit and a complete watertight camera head.

Located at the front of the ergonomically shaped camera head, there is a C-Mount-Thread adaptation, where you can connect usual lenses, vario-focus TV-couplers or other couplers to connect the endoscope.

Located inside the camera head is the CCD-chip, which takes the image and converts it into electric signals. These signals are conducted through the camera cable to the control unit.

Inside the control unit, the image data is processed and is provided either as Standard-VBS-Signal (BNC-connector, VHS-standard) or as Hosiden-Signal (Y/C-signal, Mini-DIN-connector, S-VHS-standard) on the rear panel for displaying on the monitor and/or other devices, such as videoprinter, videorecorder, etc.).

The image-brightness is controlled by an electronic shutter, which is working with a shutter-cycle at minimum of 1/100.000 sec., so that even when the image is totally irradiated a well-balanced video picture is delivered to the monitor

If the image is extremely dark illuminated, the shutter cycle becomes so high, that it is impossible to display a flicker-free picture. In this case, the videosignal is emphases electronically by max. +30dB.

This electronic emphasis is called AGC (Automatic Gain Control). It has to be taken into account, that the noise on the picture is increased, when the AGC is working.


Halogen lightsource

The lamp designed is optimised for endoscopic applications. The colour temperature equals daylight quality. The light source is equiped with two outlets for different applications, or adapters.

The equipment is desinged by the latest standards of security for medical equipment and complies with the demands of CE.

Operating elements

Operating elements on the frontpanel

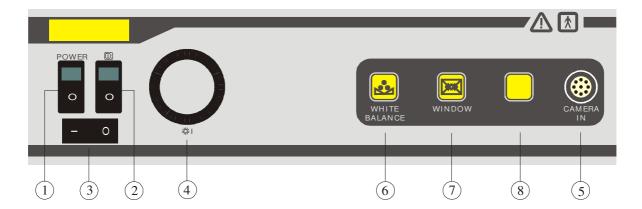

- 1) Mains switch
- 2) Switch camera
- 3) Switch Air pump
- 4) Connector for fibrescope
- 5) Connector for camera cable
- 6) Push button: WHITE BALANCE
- 7) Push button: WINDOW
- 8) Push button: BOOST

Mains switch

The control unit is turned on by switching the mains switch.

The mains switch has two different switching positions:

I switched on O switched off

When the control unit is switched on, this is indicated by the up-light green lamp inside the switch.

NOTE! It is only possible to switch the Halogen light source ON, if the CCD camera is switched OFF. On the other hand it is possible to switch the CCD camera ON, if the Halogen light source is switched ON or OFF.

Connector for fibrescope

For lightguide connection, there are interchangeable adapters available for all brands of lightguides. For gastroscopic applications, the lightsource comes equipped with a special adapter with integrated air supply for your gastroscope brand.



Using the wrong lightguide or adaptor can damage the lightsource and the accessories and threats the patient.

Pump switch

The integrated pump is switched on and off by using the corresponding switch.

Switch on the pump only right before the use of it and switch it off right after.

The pump is maintenance-free. Make sure, no dirt and liquids can get into the adaptor.

Connector for camera cable

This connector is used for attachment of the plug of the camera cable.

The plug can only be plugged in a certain position. This is achieved by a small nose on the top of the plug.

The red mark on the 19-pin-plug must correspond with the red mark on the socket to install the plug.

The plug fastens automatically when completely plugged in.

To unfasten the plug, grip the knurled part of the plug and pull the plug out of the socket.

Push button: 'WHITE BALANCE'

The WHITE BALANCE is done by pressing the corresponding push button.

Therefore, aim the camera head on a homogeneously illuminated white sheet of paper or a well illuminated multi-colour object. Simultaneously, press the push button once. The message "WHITE BALANCE OK" will be appear on the screen when WHITE BALANCE is finished. The current WHITE BALANCE is stored in memory and is also saved, when the camera is switched off.

Take notice, that the paper-sheet or the multi-colour object is not irradiated, that means, e.g. that the grain-structure of the paper is visible on the monitor.

If WHITE BALANCE is done correctly, a natural colour reproduction is guaranteed.

Push button: 'WINDOW'

The WINDOW-function is turned on and off by pressing the corresponding push button.

The WINDOW-function carries out a light measurement that only affects a central window of the picture. That means, that in contrast to standard measurement, where the average brightness is integrated over the entire picture size, only a centre part of the image is taken to evaluate the average brightness, outer margins are disregarded.

To activate the WINDOW-function, push the corresponding button. If the WINDOW-function is activated, a short message appear on the screen and a light-grey rectangle appears on the monitor picture for a short time, that represents the area, that is taken for brightness evaluation.

If the object to be regarded is in the centre of the picture, you should activate the WINDOW-function. The object itself is displayed correctly illuminated on the monitor and the outer margins of the screen are displayed proportionately darker or brighter.

Anewed pressing of the button deactivates the WINDOW-function.

Push button: BOOST-function

For working under bad illumination conditions, the camera is equipped with a Boost-function to increase image brightness. Press the push button to activate the function. Under good illumination conditions you'll see no effect on the screen.


Setting the brightness amplification

- pushing the Boost-button once show the current brightness amplification
- it's possible to changed between three attitudes, by pushing the Boost-button renewed, within one second: MIN,

MID, MAX

The steps will be indicated by a short message on the screen.

Operating elements on the rear

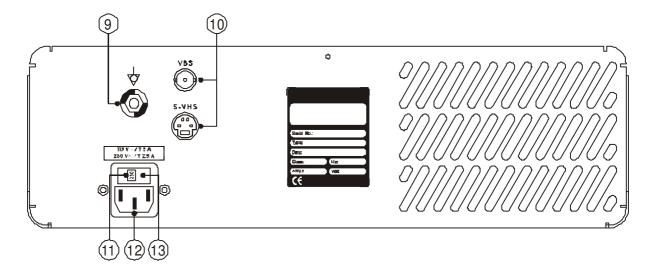

- 9) Potential ground connector
- 10) Video output terminal (VBS / S-VHS)
- 11) Voltage selector
- 12) Mains terminal device
- 13) Mains fuses

Video output terminals

You can attach external output-devices such as monitor, videorecorder or videoprinter to the VIDEO-OUT connectors.

The videosignal is provided in two different standards.

- 1. The BNC-terminal provide the videosignal as VBS-signal (composit-signal, VHS-standard).
- 2. The 4-pin-Y/C-terminal provides the videosignal as Hosiden-signal (separate conductors for chroma- and luminance-signal, S-VHS-standard).

Both video outputs are available simultaneously, so that both standards can be used at one time.

You should better use the Y/C-signal than VBS-signal if possible, because you will achieve a much better image quality in contrast to standard VHS-standard.

Terminal device for potential equalisation conductor

Basically, the control unit is protectively earthed by the 3-pin power supply cord when it is connected to a protectively earthed wall socket, as prescribed.

When running the equipment in rooms which comply to class 1 or 2E according to MedGV, the control unit must be joined to the central potential equalisation of the operating theatre or of the equipment trolley by means of a grounding cable.

Mains terminal device

The plug of the power supply cord is connected to the mains terminal device. Use only the delivered supply cord!

Mains fuses / Voltage selector

This little drawer right above the mains terminal device contains the mains fuses. The window inside the drawer shows the currently selected mains voltage. You have to control whether your mains voltage corresponds with the selection shown in the window.

Connecting and operating the equipment

Connecting the equipment

Before you connect the mains plug, check on the back of the equipment that the voltage indicated (in the square panel above the mains socket) is the correct one.

voltage = 230VAC → indicator '230' voltage = 115VAC → indicator '115'

If the incorrect voltage is indicated then the equipment may under no circumstances be connect. Before connecting a light cable into a light source adapter of the equipment, please ensure that you have the correct plug and adapter. A plug that is too long or too thin can for example be pushed too far into the equipment, and can damage the sensitive diaphragm or lens inside. This will result moreover in considerable loss of light, in the same way as a plug that is too short, or a plug of the wrong diameter, because of the wrong position of the contacts at the light entry.

All wiring has to be done before switching on the equipment!



After switching on a light source and the ensuing ignition of the lamp, the equipment should remain switched on for at least ¼ hour. A shorter shining period will considerably shorten the life expectancy of the lamp! After switching off a light source however, it may be immediately switched on again. A waiting period or cooling off period is not necessary.

CONNECT THE POWER SUPPLY CORD!

Use the delivered protectively earthed power supply cord to connect the control unit to the mains. Please check before, whether the current voltage selection matches your local mains voltage. Attach the equipment only to a grounded protective contact socket.

CONNECT THE CAMERA CABLE!

Install the plug of the camera cable in the corresponding socket on the frontpanel as described above. You can connect all lenses and endoscope couplers to the camera head, which are equipped with a C-Mount-Thread or CS-Mount-Thread. When you use objectives with CS-Mount-Thread, an additional intermediate ring is necessary.

Your dealer supplies various lenses and couplers for your special requirements.

CONNECT THE EXTERNAL OUTPUT DEVICES!

As described before, there are several standards of videosignal available on the rearpanel.

Here you can connect external output devices such as monitor, videorecorder, videoprinter, etc.

If you are using standard VBS-signal (BNC-terminal), please always apply high-quality 75 Ω -coaxial cables.

If the external output device has a switchable 75Ω -termination resistor, you should switch this on.

If you connect several devices in a row only the last device in the line needs to be terminated by the termination resistor.

If the external device does not provide such a termination resistor, you should connect the coaxial cable via a T-connection-adapter. The open end of the T-connection adapter is then terminated with the 75Ω -resistor. We recommend the use of the delivered S-VHS-cable.

CONNECT THE POTENTIAL EQUALISATION CONDUCTOR!

Join the terminal device for potential equalisation on the rearpanel with the central potential equalisation of the operating theatre or of the equipment trolley.

CONNECT A FIBRESCOPE!

Verify, whether the installed adapter matched the fibrescope you want to connect. Plug in the light plug of the fibrescope. Thus the air connection is automatically fixed. A wrong plug can damage the sensitive diaphragm or lens inside. The wrong plug will always cause a loss of light-intensity. Connect the water bottle to the fibrescope (if necessary). There is a holder for the water bottle installed at the left side of the housing.

Operating the equipment

Operating the camera

If all other devices are switched on and the control unit are in working modus by pushing the power switch, a picture appears on the monitor.

Now proceed with the WHITE BALANCE. Therefore, aim the camera head on a homogeneously illuminated white sheet of paper or a well illuminated multi-colour object. Simultaneously, press the push button. The current WHITEBALANCE is stored in memory and is also saved, when the camera is switched off. Take notice, that the paper-sheet or the multi-colour object is not irradiated, that means, e.g. that the grain-structure of the paper is visible on the monitor.

Now the regarded image is displayed correctly illuminated and true-coloured on the monitor, if the colour-adjustment of the monitor is set to a neutral value.

Operating the Lightsource

Before connecting a light cable into a light source adapter of the equipment, please ensure that you have the correct plug and adapter. A plug that is too long or too thin can for example be pushed too far into the equipment, and can damage the sensitive diaphragm or lens inside. This will result moreover in considerable loss of light, in the same way as a plug that is too short, or a plug of the wrong diameter, because of the wrong position of the contacts at the light entry.

After switching on a light source and the ensuing ignition of the lamp, the equipment should remain switched on for at least 1 hour. A shorter shining period will considerably shorten the life expectancy of the lamp! After switching off a light source however, it may be switched on again after 15 minutes waiting time.

To spare the lamp life, the lamp starts with time delay. The lamp will ignite approx. 10 sec. after starting the unit. Restart after 60 sec.

Now you can start the examination. There are no further adjustments necessary during operation. If the image is too bright, activate the window feature.

If you have finished examination, please switch off the control unit right afterwards, in order to allow a maximum life expectancy of the lamp.



Attention: When you switch off the light source, there is a 60 seconds delay, before the lamp will start again! This avoids lamp defects and expands the lifetime of the lamp!

Attention: At overheating the unit will automaticly switch off. In this case you have to switch off the unit by the mains switch. After approx. 60 sec. the unit may be switch on.

Service manual

General maintenance and repair advice

The instructions and information given in this chapter are only for instructed personnel, who are aware of the safety precautions necessary for repair and maintenance of medical electronic devices.


The manufacturer refuse any liability for unauthorised repair and modification.

The manufacturer will provide those circuit diagrams, itemised parts listings, descriptions, sets of adjustment instructions and other items of available documentation to suitably qualified user personnel duly authorised by the manufacturer for their use in repairing those components of the equipment that have been designated by their respective manufactures as reparable.

Only the supply of such technical documentation relating to the equipment shall not be construed as constituting manufacturer's authorisation of user's personnel, regardless of their levels of technical training, to open or repair the equipment.

Explicitly exempted here from are those maintenance and repair operations described in this manual.

Exchange of the mains fuses

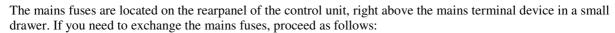



- ➤ Loosen the drawer by unfastening the two clamps located to the left and to the right of the drawer with a peaked tool and pull out the drawer.
- Take out the fuses.
- > Check the fuses. A blown fuse is indicated by the blackened glass cylinder or the visibly melted fuse conductor. If necessary, check the fuse with an ohmmeter.
- Install the corresponding fuses (declared on the label).
- Re-install the fuse-drawer.

Switch on the equipment again. If you have exchanged a defective fuse against a new one and the fuse blows again, the unit has an error. In this case, you must return the device to your dealer for testing and repair.



Selection of the line voltage setting

To set the correct voltage proceed as follows:

- PULL OUT THE MAINS PLUG!
- Using a small screwdriver or other sharp instruments, the black rectangular board above the socket can be lifted out.
- The white panel with the fuse can be taken out and turned 180°, and then put back into the fuse container.
- After that the rectangular fuse board can be put back (the small nose of the board facing downwards) and firmly pressed until it has completely snapped in.

Now the small white panel should show the correct voltage.









Exchange of lamp

The only part subject to wastage in the equipment are the lamp. The quoted light wattage reduces drastically and there may be difficulties in igniting. In order to secure safe operation of the equipment, the lamp should be changed after expire of this operation time.


The lamp must be changed for a new one as following:

- PULL OUT MAINS PLUG!

- Loosen the screws. Take off the cover and place it beside the unit.
- Attention: If the machine has been operating shortly beforehand the lamp could be very hot.
- All remaining current in the electronics will be, in a few seconds after switching off the equipment, completely safely discharged, so there is no danger in handling the equipment here.
- Remove the lamp cover.
- Hold fast the reflector of the lamp and remove the cable from the lamp.
- Pull out the lamp upwards out of its holder.
 Insert a new lamp into the holder and push it downwards.
- Re-install the lamp cable.
- Re-install the lamp cover
- Re-install the cover and tighten it with the screws. **Observe the grounding cable!**
- Switch on the unit to check function.

Further servicing, which should be done in conjunction with lamp replacement

Dust should be removed from the equipment with a vacuum cleaner using a small nozzle attachment. Dust can sometimes cling quite firmly to the ventilator blades. This should then be cleaned with a cloth and a little alcohol/spirits. Likewise dust settles on the heat protective filter (with lens, optional), this should also be cleaned with a soft cloth or blotting paper and pure alcohol or spirits.

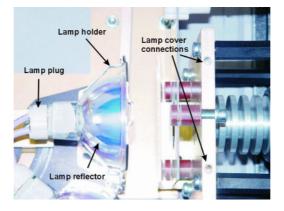



Cleaning / Disinfecting

NOTE: PULL OUT THE MAINS PLUG!

All parts of the outer surfaces of the equipment are totally insensitive to all the usual cleaning and disinfecting materials, so that you can use any of these without limitation. Apply liquids using a soft cloth or soft blotting paper, in order to avoid scratches on the surfaces and in order to be able to control the amount of liquid. With flammable liquids like alcohol especially, you should apply with a cloth. Do not let any liquid get into the equipment. After cleaning with flammable liquids, leave the equipment to dry for one hour, before it is switched on again. There is danger for example that an alcohol-air explosive mixture could form after cleaning. Improper and/or incomplete reprocessing can cause infection of the patient and/or medical personnel.




The camera-cable must not be autoclaved!

Troubleshooting

In any case of malfunction, you should check the wiring at first. Most errors are based on wrong wiring. The last column shows the referring chapter.

Malfunction	Possible reason	Remedial measure
Equipment doesn't work main switch doesn't shining	Line cable not connected	Connect line cable
	Main switch off	Turn main switch on
	Both line fuses defective	Check line fuses / exchange
	Wrong line voltage adjusted	Adjust right line voltage with fuse drawer
Equipment doesn't work main switch shining	Lamp after changing wrong connected	Check connector of the lamp
	Lamp defective	Change lamp
	One line fuse defective	Check line fuses / exchange
	Wrong line voltage adjusted	Adjust right line voltage with fuse drawer
No picture on the monitor	Mains supply cord not installed	Install mains supply cord
_	Mains switch off	Switch on mains switch
	Mains fuses blown	Check/exchange mains fuses
	Camera cable not installed	Install camera cable
	No mains voltage	Check mains voltage
	Video cable not installed/defective	Install/exchange video cable
	Monitor switched off	Switch on monitor
	Malfunction inside CCU	Check voltages::mains voltage: as shown on the voltage selector
Torono to a doub	Window fortune to make a second	Output voltage of power supply: +12VDC
Image too dark	Window feature turned on	Turn off window feature if not required or change window settings (use osd)
	Termination resistor on an external output device switched on	Switch off/remove termination resistors if not required
	Monitor settings not in neutral position	Set monitor settings to neutral values
Image too bright	Monitor settings not in neutral position	Set monitor settings to neutral values
	Shutter not working	Return device to manufacturer for repair
Poor image quality	Bnc-terminal in use	Use y/c-terminal for better quality
	Illumination too dark	Improve illumination conditions
Bad colours	White balance not done	Do white balance
	Monitor chroma settings misplaced Y/C-terminal or cable	Set chroma to a neutral value Check cable and Y/C-terminal
	defective/broken	
	Monitor chroma settings misplaced	Set chroma to a neutral value

Wiring diagram

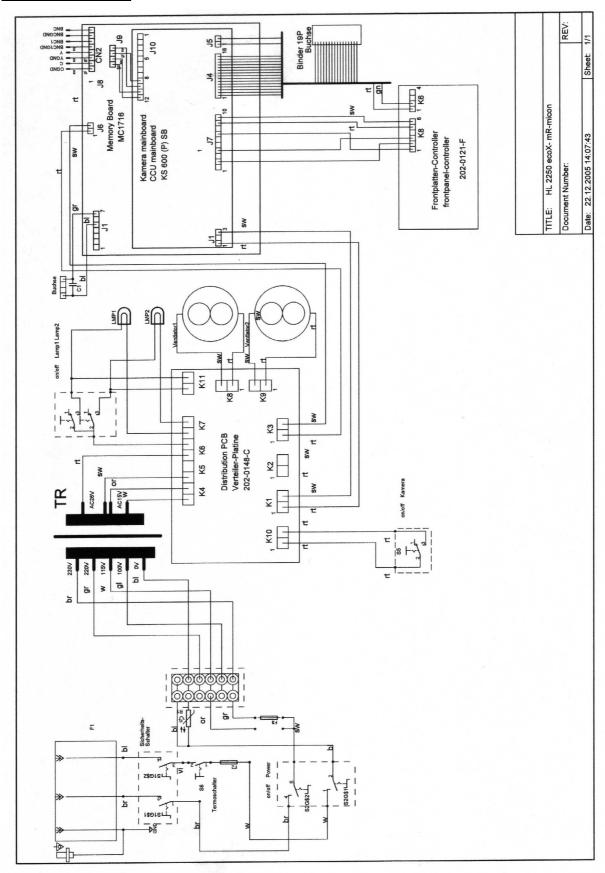

Technical data

TV-system PAL

Resolution active pixels:

752 H x 582 V

TV resolution 470 lines

Video-output $2 \times VBS (1Vpp/75\Omega), BNC$

1 x Hosiden (Y/C), Mini-DIN-connector

WHITE BALANCE automatically , with data-hold Shutter automatically, shutter-speed from

1/50 to 1/100.000 sec.

AGC automatically, max. emphasis +30dB

Light measurement Integral (Standard)

WINDOW

Power supply \bigcirc 230 VAC \pm 10%

② 115 VAC ± 10%

Power consumption approx. 280 W
Mains fuses fine fuses, 5x20mm

① 2x T1,6 A ② 2x T3,2 A

Lightguide connector all lightguides connectable using

interchangeable adapters

Lamp 2 halogen lamps

with reflector

Dimensions control unit 355x110x320 mm (WxHxD)

camera head Ø 29mm, length 56 mm

Weight control unit 5,5 kg

camera head approx. 70 g (without cable)

Class / Type 1 / BF

Manufactured and IEC 601-1 / CE; EN 60601-1; 93/42/EEC

Tested acc.

Spare parts

Lamp Halogen reflector lamp 24V, 250Watt OSRAM HLX 64653

Mains fuses fine fuses, 5x20 mm

230 VAC: 2x T 1,6A 115 VAC: 2x T 3,2A

Table 'Technical service-information'

DATE	СНЕСК	SIGNATURE

Appendix

Electromagnetic Compatibility (EMC)

Precautionary measures

Electromedical devices are subject to special precautionary measures concerning electromagnetic compatibility (EMC). This device is to be used for the purposes described in the operation manual and has to be installed, set up, and operated in compliance with the EMC guidelines.

Impact of mobile and portable RF communication devices

The emission of high frequency by mobile communication devices may impact the function of the electromedical device. Operating such mobile communication devices (e.g. cell phones, GSM phones) in the proximity of the electromedical device is prohibited.

Electrical connections

Connections between such plugs and sockets may not be established without first implementing ESD precautionary measures.

ESD precautionary measures

The following are ESD precautionary measures:

- Connect all electrical equipment to be connected to the device to a potential equalisation system (via PE).
- Use only the equipment and accessories mentioned in the operation manual.

The staff have to be informed about and trained in ESD precautionary measures.

The HL250-ecoX should be operate in an environment as stadet below. The user have to make sure that the HL250-ecoX will be operate in such environment.

Manufacturer declaration – Electromagnetic application

Noise emmission measurement	Correspondence	Electromagnetic enviroment - Guide
HF- Emission acc. to CISPR	Group 1	The HL250-ecoX used HF-energy solely for internal functions. Therefore, the HF-emission is very low and interferences with adjacent electronical equipment are improbable.
HF- Emission acc. to CISPR	Class B	The HL250-ecoX is adapted for use in all
Harmonics acc. to IEC 61000-3-2	Class A	facilities including living areas and such facilities with public utility provider.
Voltage fluctuations / Flicker acc. to IEC 61000-3-2	unapplicable	1 31

Manufacturer declaration – Electromagnetic noise immunity

Noise immunity tests	IEC 60601 – test level	Correspondence level	Electromagnetic enviroment - Guide
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge	± 6 kV contact discharge	Floors should be consist of concrete or wood or furnished with ceramic tiles. If the floor furnished with synthetic
	± 8 kV air discharge	± 8 kV air discharge	material, the relative air humidity should be 30 % average.
Quick transient electrical disturbances / Bursts acc. to 61000-4-4	± 2 kV for mains lines ± 1 kV for Input and output lines	± 2 kV for mains lines ± 1 kV for Input and output lines	The quality of mains voltage should be comply typical mains voltage of business or hospital environment.

Surge voltage acc. to IEC 61000-4-5	± 1 kV push-pull voltage ± 2 kV push-push voltage	± 1 kV push-pull voltage ± 2 kV push-push voltage	The quality of mains voltage should be comply typical mains voltage of business or hospital environment.
Acc. to IEC 61000-4-11	< 5% U _T (> 95% U _T collaps) for ½ period < 40% U _T (> 60% U _T collaps) for 5 periods < 70% U _T (> 30% U _T collaps) for 25 periods < 5% U _T (> 95% U _T collaps) for 5 sec.	< 5% U _T (> 95% U _T collaps) for ½ period < 40% U _T (> 60% U _T collaps) for 5 periods < 70% U _T (> 30% U _T collaps) for 25 periods < 5% U _T (> 95% U _T collaps) for 5 sec.	The quality of mains voltage should be comply typical mains voltage of business or hospital environment. If the user of the HL250-ecoX demand continued functions also by appearance of interrupts, it will be recommended to operate the HL250-ecoX by non-interruptable power supply.
Magnetic field by supply frequency (50 / 60 Hz) acc. to IEC 61000-4-8	3 A / m	3 A / m	Magnetic fields by line frequency should be correspond to typical values you will find in business and hospital environments.

Note: U_T is the line AC voltage before using the test level.

Manufacturer declaration – Electromagnetic noise immunity

Noise immunity test	IEC 60601 – test level	Correspondence level	Electromagnetic enviroment - Guide
			Portable and mobile radio transmitter should not be use within the recommended safety distance to the HL250-ecoX (including the lines)! The recommended safety distance will be calculate by the appropriate transmitting frequency formula. Recommended safety distance:
Guided HF-disturbances acc. to IEC 61000-4-3	3 V _{eff} 150 KHz to 80 KHz	3 V _{eff}	d=1,2 √P
Radiated HF-disturbances acc. to IEC 61000-4-3	3 V / m 80 MHz to 2,5 GHz	3 V / m	d=1,2 √P 80 MHz to 800 GHz
			d=2,3 √P 800 MHz to 2,5 GHz
			P – transmitter wattage rating (W) acc. to manufacturer (transmitter)data d – recommended safety disdance in meters (m) The electrical field strength of fixed radio
			transmitter is lower than the correspondence level ^b according to a fieldwork ^a .
			In the environment of equipment which bore the following pictograph, it is possible to appear interferences.

3 V/m

Note 1: At 80	Note 1: At 80 MHz and at 800 MHz the higher result is valid.			
_	Note 2: This guideline may not be the case for all situations. The spread of electromagnetic waves will be affect by absorption and reflexion of buildings, objects and humans.			
a	The electric field strength of fixed transmitter, like wireless telephones, mobile radio services, AM- and FM broadcast transmitter and TV transmitter, couldn't be predetermine correctly. In order to ascertain the electromagnetic environment at fixed HF-transmitter, it is commendable to inspect the location. If the ascertained field strength at the HL250-ecoX location exceed the specified correspondence level, it's may be necessary to adopt other means, e.g. reorientation or removal the HL250-ecoX.			
b	By means of the frequency range from 150 kHz to 80 MHz the field strength is less than			

Commendable safety distances between portable and mobile communication appliances and the HL250-ecoX

The HL250-ecoX is intendet for operation in electromagnetic environments, where radiated HF-disturbances will be controlled. The customer resp. the user of the HL250-ecoX can help to prevent electromagnetic disturbances by compliance minimum distances between portable and mobile HF communication appliances (transmitter) and the HL250-ecoX.

Commendable safety distances acc. to the maximum power output of the communication appliance:

W 44 0.1	Safety distance acc. to transmitting frequency (m)			
Wattage rating of the transmitter (W)	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2,5 GHz	
	d=1,2 √P	d=1,2 √P	d=2,3 √P	
0,01	0,12	0,12	0,23	
0,1	0,38	0,38	0,73	
1	1,2	1,2	2,3	
10	3,8	3,8	7,3	
100	12	12	23	

For transmitter, whose wattage rating isn't stated above, the safety distance can be calculated by using the equations in the respective column.

P is the wattage rating of the transmitter acc. to the specifications of the transmitter manufacturer.

Note 1: To calculate the commended safety distance of transmitter, with a frequency range from 80 MHz to 2,5 GHz, was an additional coefficient of 10/3 used. The additional coefficient should reduce the probability of disturbances by portable / mobile communication appliance, which are unintentional brought in the patient range.

Note 2: This guideline may not be the case for all situations. The spread of electromagnetic waves will be affect by absorption and reflexion of buildings, objects and humans.