

# Hematology reagents

## Erba Clenz

Enzymatic cleaner reagent for automated analysis



IVD

Name	Order No.	Package volume
Erba Clenz	50004375	1 L

For *in vitro* diagnostic use only

### INTENDED USE

Erba Clenz enzymatic cleaner reagent is a stabilized and micro-filtered proteolytic enzyme solution for regular automated cleaning, rinsing and washing of hematology analyzers capillaries, tubing and chambers, removing blood component precipitates and lipoprotein deposits on ELite hematology analyzers.

Erba Clenz enzymatic cleaner reagent should be used only with Erba reagents. Erroneous results may be obtained if the reagent is applied with third party reagents. Please refer to the instrument User Manual for further information.

### INGREDIENTS

Sodium chloride < 1.0%  
Proteolytic enzymes < 1.0%  
Preservative < 0.5%  
Stabilizers < 2.0%  
In ion-free water

### HANDLING PRECAUTIONS

- Erba Clenz enzymatic cleaner is environmental friendly, azide-free reagent, does not contain harmful ingredients.
- Avoid contact with eye, skin and clothing.
- In case of eye or skin contact flush eyes with copious amounts of water for several minutes or wash skin area with water.
- Keep the reagent container closed when not in use.
- Wear laboratory gloves when handling the reagent.
- All body fluid samples should be considered potentially infectious materials. Treat all blood and other potentially infectious materials with appropriate precautions. Use gloves, masks and gowns if blood exposure is anticipated.
- Please refer to the MSDS associated with the reagent.
- Use Good laboratory Practices (GPL) when handling the reagents.

### SPECIMEN COLLECTION AND PREPARATION

Erba Clenz reagent is intended for use with blood specimens collected by vein puncture in EDTA anticoagulant. Specimens for hematological analysis may be stored for up to 8 hours at 15-30°C or up to 24 hours after collection when refrigerated (2-8°C).

### REAGENT PREPARATION

- This reagent is ready for use and can be applied straight from the container, no special reagent preparation is necessary.
- Leave the reagent at room temperature for at least 12 hours.

### REAGENT REPLACEMENT

- Person installing the reagent must be a trained laboratory professional.
- Remove the inlet cap from the reagent container to be replaced. Connect the reagent inlet to the new reagent container.
- Be sure that the color on each tube, reagent container label and connector in the back of instrument match.
- Avoid any dust or microbial contamination of the tubing and reagents. Do not pour and mix the remains of a reagent from a container into other one.
- Prime thoroughly the new reagent and measure backgrounds according to the instrument User Manual.
- When installing a new lot of reagent, recalibrate the instrument as specified in your User Manual.

### REAGENT STORAGE, STABILITY AND DISPOSAL

Store Erba Clenz enzymatic cleaner reagent between 15 – 30°C. The shelf life of Erba Clenz is 24 months from the date of manufacture, if stored at the prescribed temperature range. Do not use reagent beyond the expiration date printed on the container label. Discard opened container after 120 days. Do not use reagent once frozen. Dispose of waste product, unused product and contaminated packaging in compliance with local regulations.

Date of last revision: 9. 9. 2013