


Operating and Maintenance Manual

Flowmeter



Models

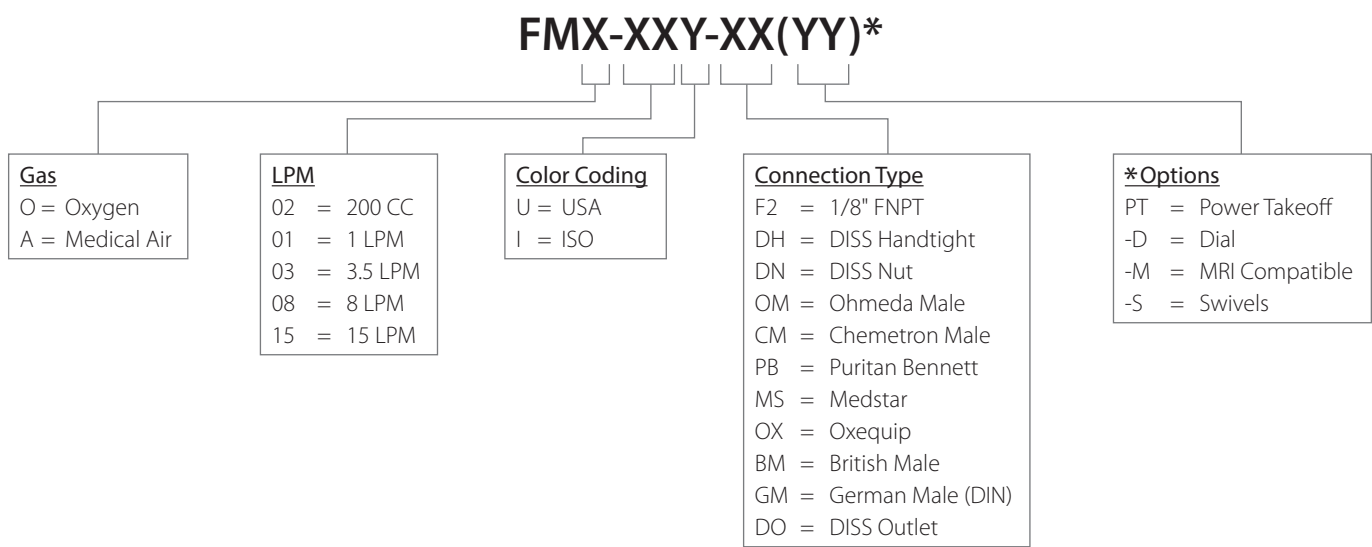

CAUTION: United States federal law restricts this device to sale by or on the order of a physician.

Read All Instructions Before Using

This manual instructs a professional to install and operate the Flowmeter. This is provided for your safety and to prevent damage to the Flowmeter. If you do not understand this manual, **DO NOT USE** the Flowmeter and contact your provider.

Safety Information - Warnings and Cautions



WARNING: Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



CAUTION: Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury.

CAUTION: Used without the safety alert symbol () indicates a potentially hazardous situation, which, if not avoided, may result in property damage.



CONSULT ACCOMPANYING DOCUMENTS



Symbol for “USE NO OIL”

Receiving Inspection

Remove the Amico Patient Care Corporation Flowmeter from the packaging and inspect for damage. If there is any damage, **DO NOT USE** and contact your provider.



WARNING: To minimize the risk of explosion or fire:

- **DO NOT** use or store oils, greases, organic lubricants or any combustible materials on or near this Flowmeter
- **DO NOT** use near any type of flame or flammable/explosive substances, vapors or atmosphere
- **DO NOT** smoke in an area where oxygen is being administered

Intended Use



WARNING: Use Flowmeters only for their **Intended Use** as described in this manual.

Specifications

The Flowmeter is intended for use by physicians, respiratory therapists and other authorized hospital personnel to administer selected doses of medical gases to a patient.

Flow Range	Increments	Accuracy	Flood
0 - 200 CC	25 CCM (Starts at 25 CCM)	± 20 CCM	500 CCM MAX
0 - 1 LPM	0.1 LPM (Starts at 0.1 LPM)	± 0.1 LPM	2.5 LPM MAX
0 - 3.5 LPM	0.125 LPM (From 0.125 to 1 LPM)	± 0.125 LPM or	45 LPM MAX
	0.25 LPM (From 1 to 3.5 LPM)	± 10% of Reading (whichever is greater)	
0 - 8 LPM	0.5 LPM (Starts at 0.5 LPM)	± 0.5 LPM, or ± 10% of Reading (whichever is greater)	50 LPM MIN
0 - 15 LPM	0.5 LPM (From 1 to 5 LPM)	± 0.5 LPM, or ± 10% of Reading (whichever is greater)	50 LPM MIN
	1 LPM (From 6 to 15 LPM)	± 0.5 LPM, or ± 10% of Reading (whichever is greater)	

Flood is the output of the Flowmeter when the flow indicator is beyond the highest calibrated graduation. The Flood range is as indicated on the Flowmeter labeling.

Transport / Storage Requirements: -40°F to 140°F (-40°C to 60°C)

The gas and inlet pressures are indicated on the Flow Tube or Flowmeter body.

Note: Storage or transport outside the specified range may cause damage to the Flowmeter.

The above Flowmeter models are calibrated at a specified inlet pressure of 70°F (21°C) standard atmospheric pressure. International models are calibrated per specifications marked on Flow Tube or Flowmeter body.



MRI WARNING: This product contains a magnetic, ferrous material that may affect the result of an MRI.

Operating Instructions



WARNING: Read this User Manual before installing or operating the Flowmeter.



CAUTION: Flowmeters must be operated with the Flow Tube in a vertical, upright position.



CAUTION: Only personnel instructed and trained in its use should operate this Flowmeter.

CAUTION: The Pediatric Models including 0-200 CC and 0-1.5 LPM contains a glass Flow Tube which is fragile. Special care should be observed to avoid breaking the Flow Tube.



CAUTION: Inspect the Flowmeter for visual damage before use, **DO NOT USE** if damaged.

1. Turn Flowmeter off by turning knob fully clockwise.



CAUTION: Overtightening the knob when turning the Flowmeter off will cause damage.



WARNING: The Flow Tube specifies the gas and pressure required. The accuracy will be affected if a different pressure, other than the one specified, is used.

2. Connect the Flowmeter to the appropriate gas source. The appropriate gas and pressure are specified on the Flow Tube or Flowmeter body.



CAUTION: Ensure all connections are tight and leak free.



CAUTION: Only use an oxygen-safe leak detector.



CAUTION: Attaching accessories to the outlet (which may increase resistance to outlet flow) may change indicated flow but will not affect the accuracy of the flow.

3. Verify that the Float Ball is at the very bottom of the Flow Tube.

NOTE: If the Float Ball is not at the bottom of the Flow Tube, the Flowmeter could be leaking. Please contact Amico Patient Care Corporation.

4. Adjust Flow:

To **increase** - Turn Knob **counterclockwise**

To **decrease** - Turn Knob **clockwise**

5. Set flow by aligning center of Float Ball with indicator lines on the Flow Tube.



WARNING: ALWAYS confirm prescribed flow before administering to a patient and monitor the flow on a frequent basis.

6. Adjusting the flow beyond the last calibrated indicator line will result in an undetermined flow.

7. To obtain maximum flush flow, turn knob fully counterclockwise.

NOTE: Any flow beyond the highest calibrated line on the Flow Tube with unrestricted flow is Flood/Flush flow.

Cleaning Instructions



CAUTION:

- **DO NOT** autoclave
- **DO NOT** gas sterilize with EtO (Ethylene Oxide)
- **DO NOT** immerse Flowmeter in any kind of liquid

1. Disconnect all connections before cleaning.
2. Clean exterior surfaces of the Flowmeter with a cloth dampened with a mild detergent and water.
3. Wipe dry with a clean cloth.

Maintenance and Prevention

Before and after each use, **inspect the flowmeter for damage.**




WARNING: When changing connectors on the Flowmeter for service or replacement, NEVER attach connectors of a different gas. Doing so may result in patient injury or damage to the equipment.



CAUTION: Disconnect Flowmeter from gas supply before servicing.

Flowmeter Assembly Drawing

Item #	Part #	Description
1	FMX-HOOD	Hood
2	FMX-ORG-104V	Flow Tube Top O Ring
3	FMX-15L-BLS	Stainless Steel Ball (15 LPM)
	FM-BALL-08	Silicon Nitride Ball (8 LPM)
4	FMO-15-TUBE	Flow Tube Oxygen (15 LPM)
	FMO-08-TUBE	Flow Tube Oxygen (8 LPM)
	FMA-15-TUBE	Flow Tube Medical Air (15 LPM)
5	FMX-ORG-010V	Flow Tube Bottom O Ring
6	FMX-ORG-015MV	Hood Bottom O Ring
7	FMO-LBL-15U	Label (15 LPM Oxygen USA)
	FMO-LBL-15I	Label (15 LPM Oxygen ISO)
	FMO-LBL-08U	Label (8 LPM Oxygen USA)
	FMO-LBL-08I	Label (8 LPM Oxygen ISO)
	FMA-LBL-15U	Label (15 LPM Medical Air USA)
	FMA-LBL-15I	Label (15 LPM Medical Air ISO)
8	FMX-BDY-STB	Flowmeter Body
9	FMX-FLTR	Hydrophobic Filter
10	FMX-NPT-M2F2	NPT Fitting 1/8" MNPT to 1/8" FNPT
11	FMX-ORG-011V	Needle Valve Adapter O Ring
12	FMX-VLV-ADB	Needle Valve Adapter
13	FMX-ORF-LRG	Orifice (Large)
14	OR-006-VIT	Needle Valve O Ring
15	FMX-NDL-LRG	Needle Valve (Large)
16	FMO-KNB-GRN	Knob (Oxygen/Green)
	FMO-KNB-WHT	Knob (Oxygen/White)
	FMA-KNB-YLW	Knob (Medical Air/Yellow)
	FMA-KNB-BLK	Knob (Medical Air/Black)

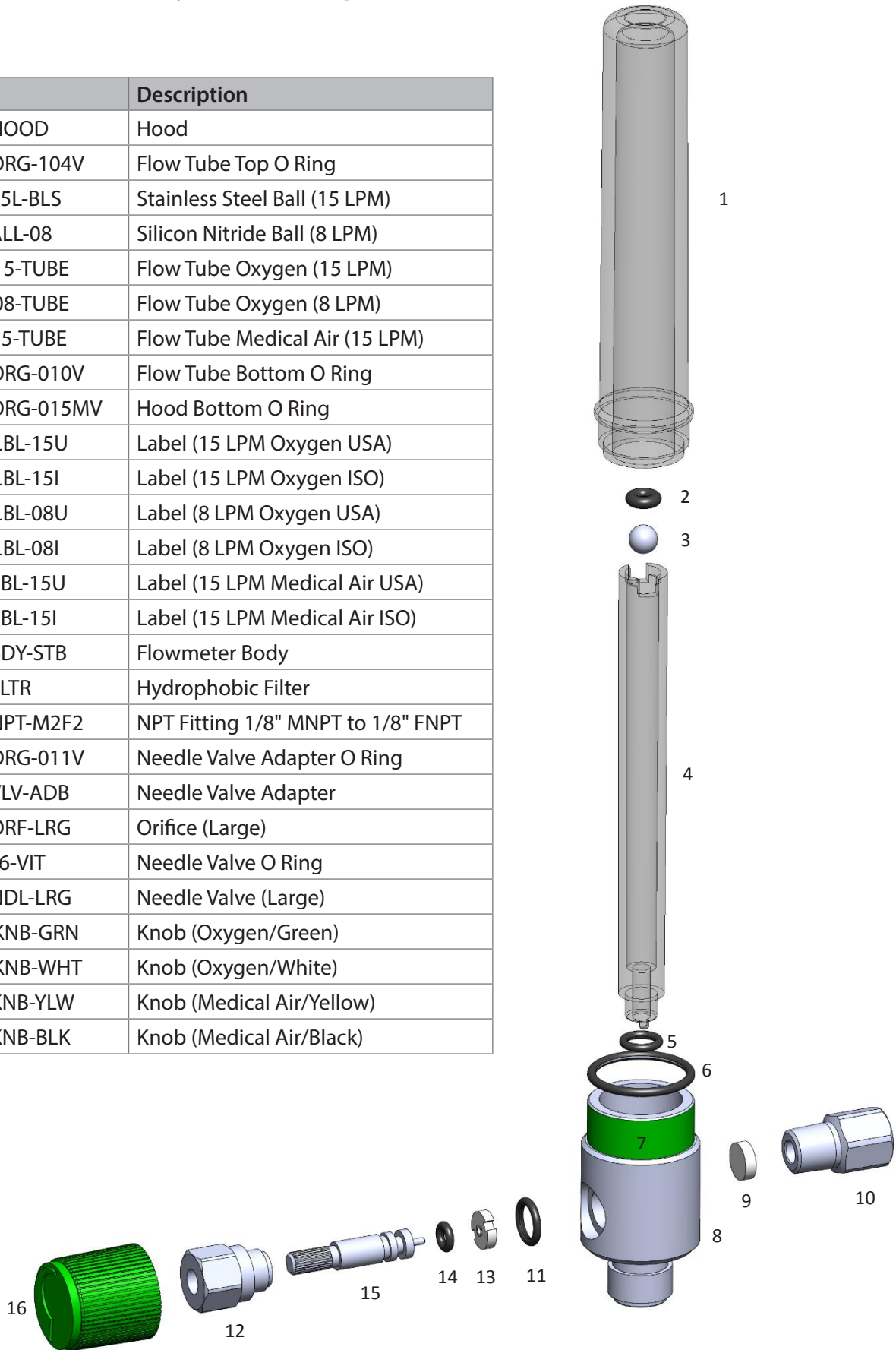



Warranty Policy - Flowmeters

This product is sold by Amico Patient Care Corporation, a Richmond Hill Corporation (the "Company") under the express terms of the warranty set forth below.

For a period of three (3) years (or for a period of five [5] years in North America only) from the date the Company ships this Product to the customer, this Product is warranted to be free from functional defects in materials and workmanship and to conform in all material respects to the description of the Product contained in the operation manual, if this Product is properly operated under conditions of normal use, regular periodic maintenance and service is performed and repairs are made in accordance with the operation manual.

This warranty shall not apply if the Product has been repaired or altered by anyone other than the Company or an authorized dealer or if the Product has been subjected to abuse, misuse, negligence or accidental damage. The Company reserves the right to stop manufacturing any product or change materials, designs or specifications without notice.

This warranty is extended only to the initial customer with respect to the purchase of this Product directly from the Company or from an authorized dealer as new merchandise. Dealers are not authorized to alter or amend the warranty of any Product described in this agreement.

This warranty is expressly in lieu of any other warranties, express or implied, including any warranty of merchantability or fitness for a particular purpose. The company shall not be liable for incidental, collateral, consequential or special damages including, but not limited to: lost profits or loss of use. The company's liability, in the aggregate, shall not exceed the purchase price of the product.

In order to file a warranty claim, the customer is required to return the Product prepaid to the Company at 20 Sims Crescent, Unit 1, Richmond Hill, Ontario, L4B 2N9, Canada. As determined at the sole discretion of the Company, Products which qualify under the warranty will be repaired or replaced, at the Company's option, and returned to the Customer via ground delivery at the Company's expense.

All claims for warranty must first be approved by Amico Patient Care Corporation's Customer Service Department at : apc-csr@amico.com or 905.764.0800. Upon approval, the Customer Service Department will issue a Return Goods Authorization (RGA) number. An RGA must be obtained prior to commencement of any warranty claim.

www.amico.com

Amico Patient Care Corporation | www.amico.com

122 East Beaver Creek Road
Richmond Hill, Ontario
L4B 1G6 Canada

Toll Free Tel: 1.877.462.6426
Tel: 905.764.0800
Fax: 905.764.0862
Email: info@amico.com